



ENDERMOLINK™ INSTALLATION GUIDE

CELLU M6 INFINITY®



**INNER POWER.
BETTER LIFE.**

ENDERMOLINK™ APPLICATION COMPATIBILITY

MINIMUM SOFTWARE VERSIONS REQUIRED:

- iOS: **iOS 17**
- Android: **Android 12**

Previous versions may limit the user experience and restrict access to certain features. Please note that the availability of updates to Android 12 may vary depending on manufacturers and models.

We recommend checking each device's specifications or consulting the manufacturer to confirm compatibility.



01

ENDERMOLINK™ APP

ANDROID VERSION



endermolink™ application


Cellu M6 Infinity®




**INNER POWER.
BETTER LIFE.**

Agir de l'intérieur. Pour le meilleur

CONNECT THE TABLET TO YOUR WI-FI NETWORK

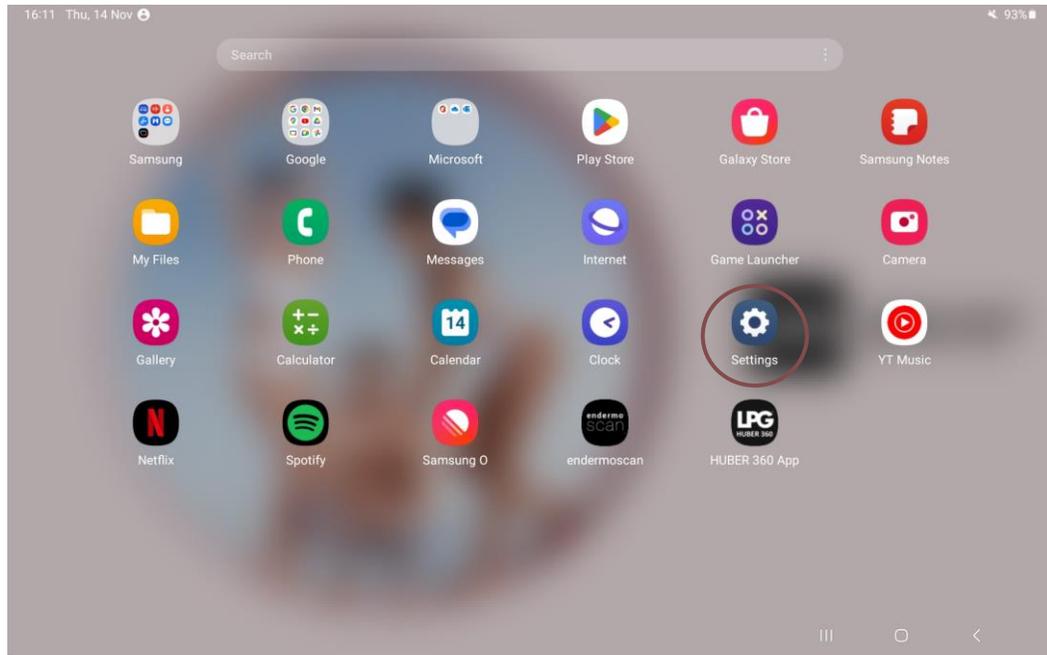


Start the **"Settings"** application available on the home screen of your tablet.

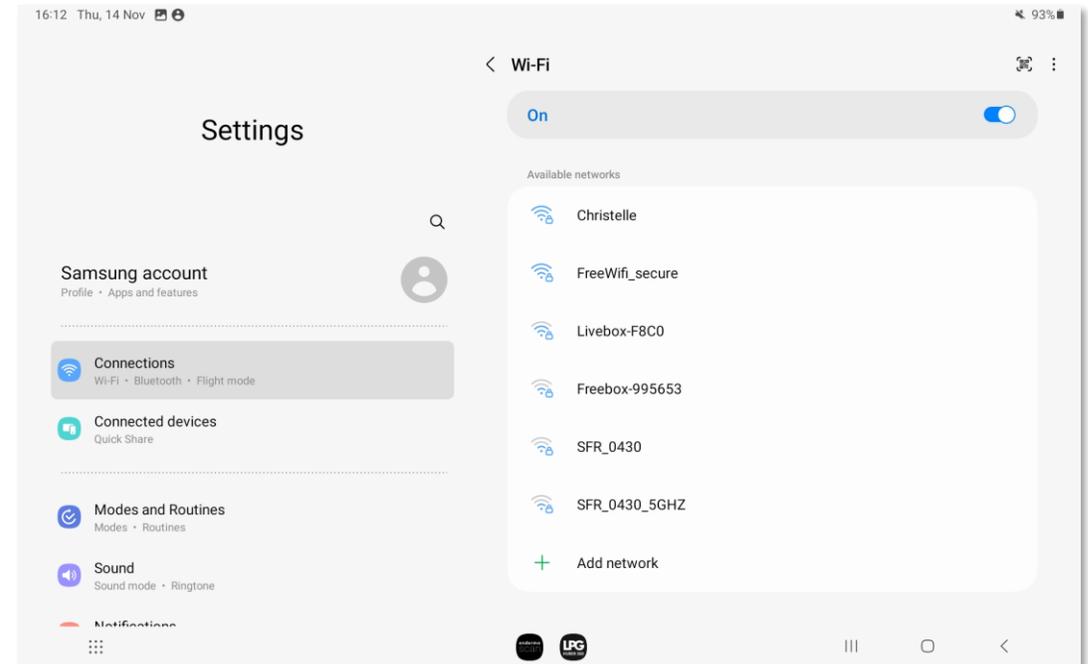


Select the **"Connections"** section, then click on **"WI-FI"**. Select your **WI-FI** network.

PLAY STORE ACCESS SIGN IN TO YOUR GOOGLE ACCOUNT

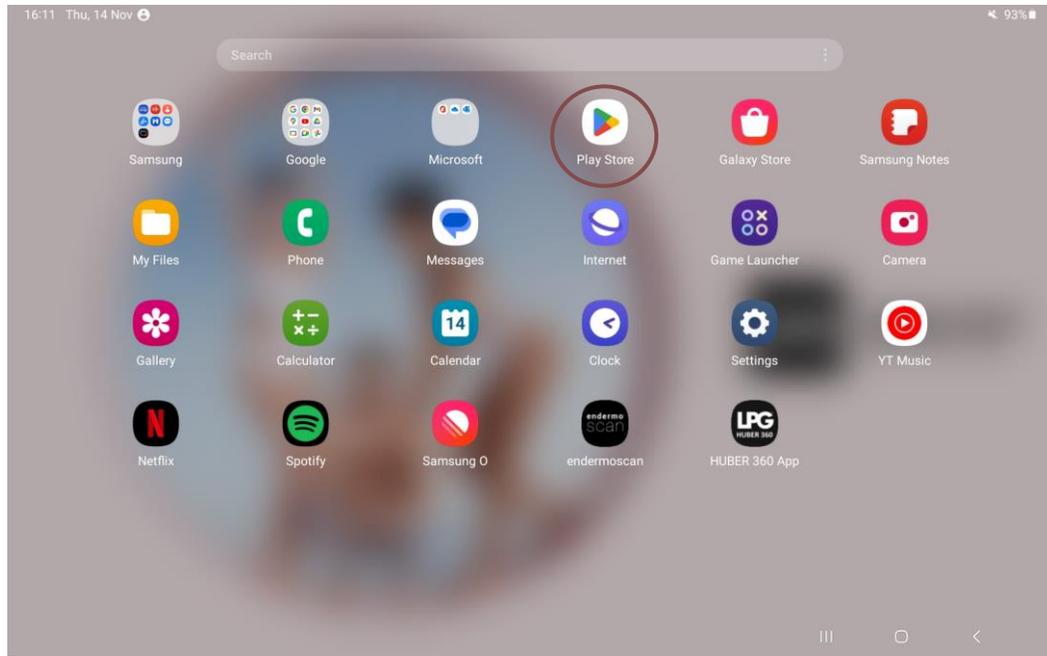

Then start the **"Play Store"** application on your tablet's home screen.

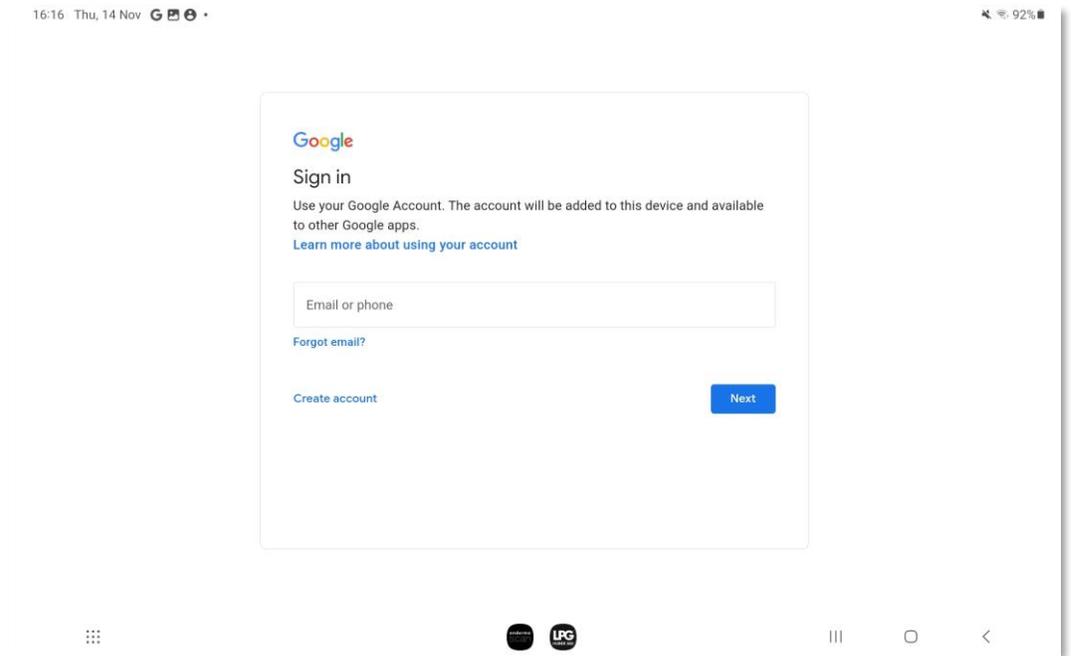

Sign in to **your Google Account**.
Enter **your username** (personal email address) and password.


endermolink™ application

Cellu M6 Infinity®




INNER POWER.
BETTER LIFE.

PLAY STORE ACCESS SIGN IN TO YOUR GOOGLE ACCOUNT

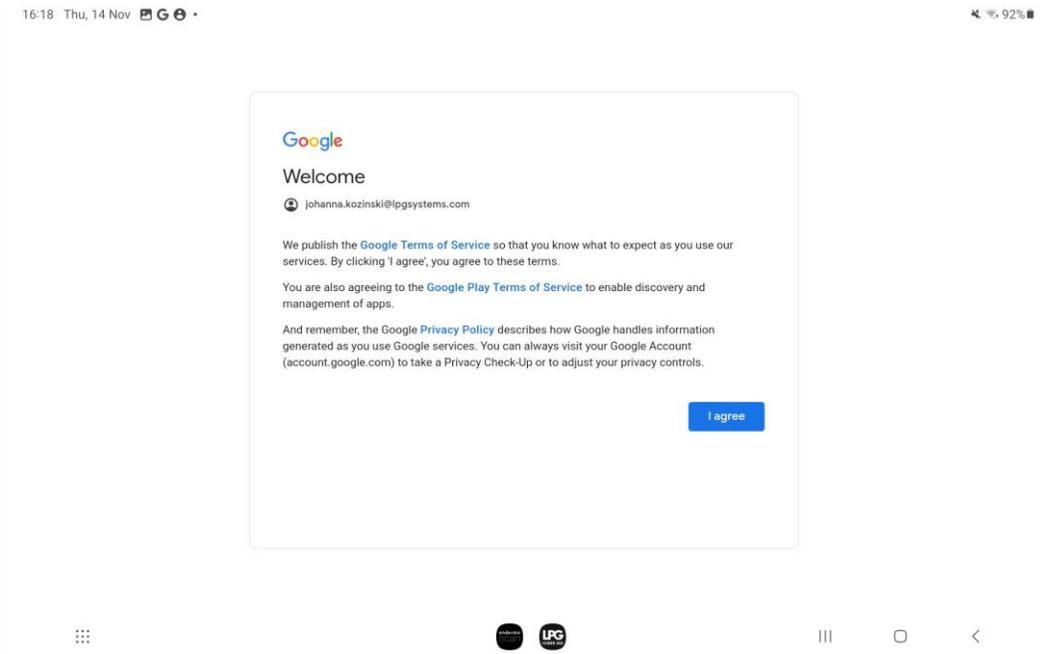

You are now **logged in.**

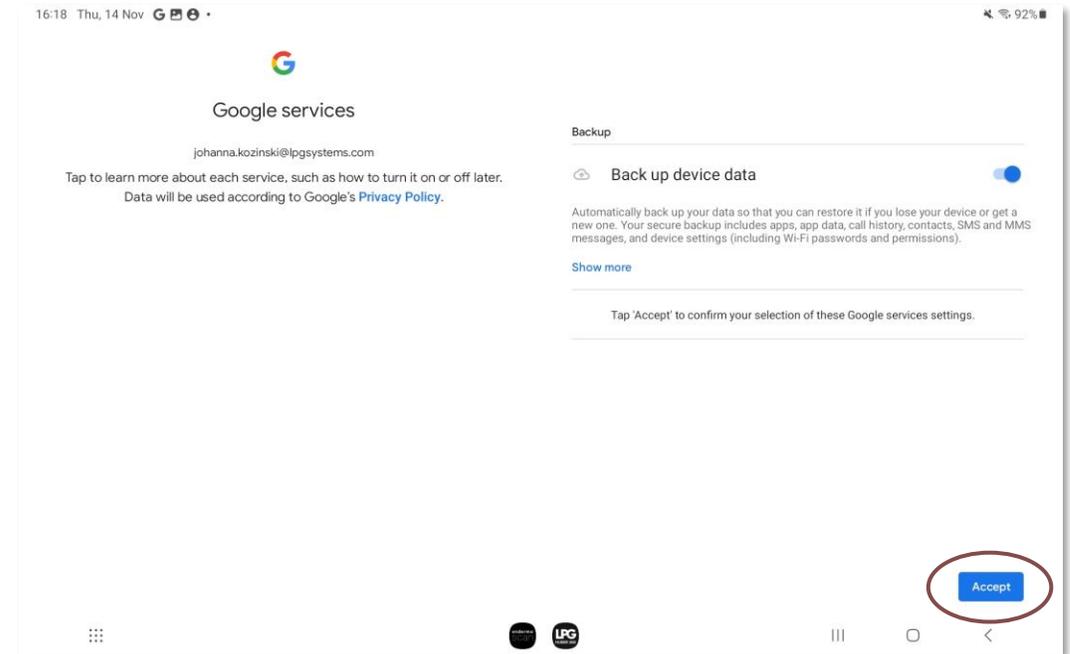

You can save the data on your device by clicking on **Accept.**


endermolink™ application

Cellu M6 Infinity®




INNER POWER.
BETTER LIFE.

DOWNLOAD THE endermolink™ application

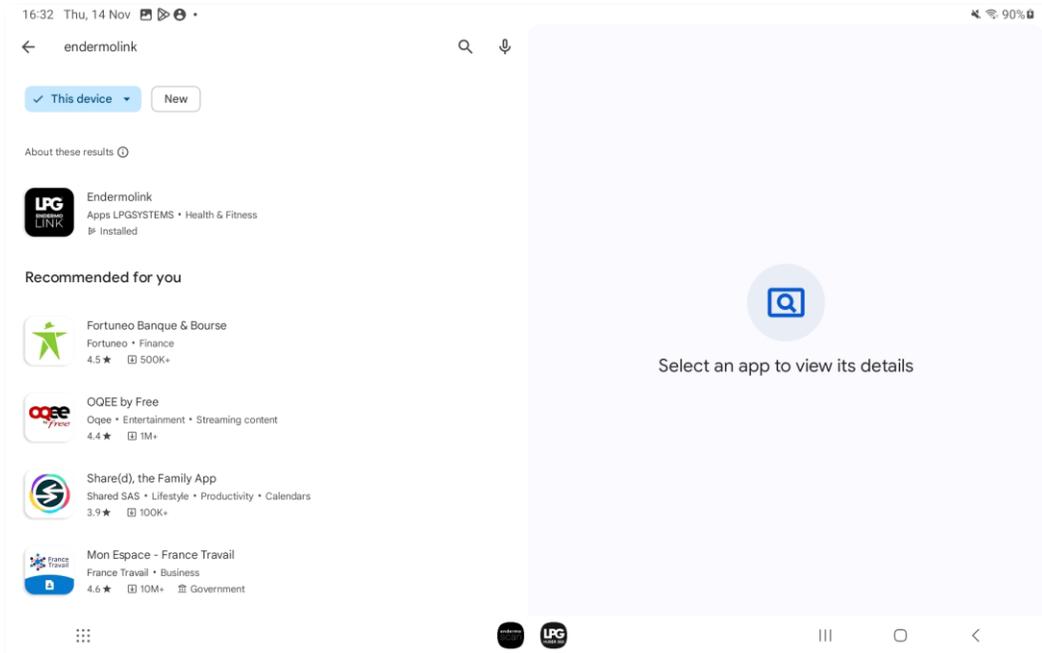

Start a search for the **Endermolink™** app in the **Play Store search bar**.

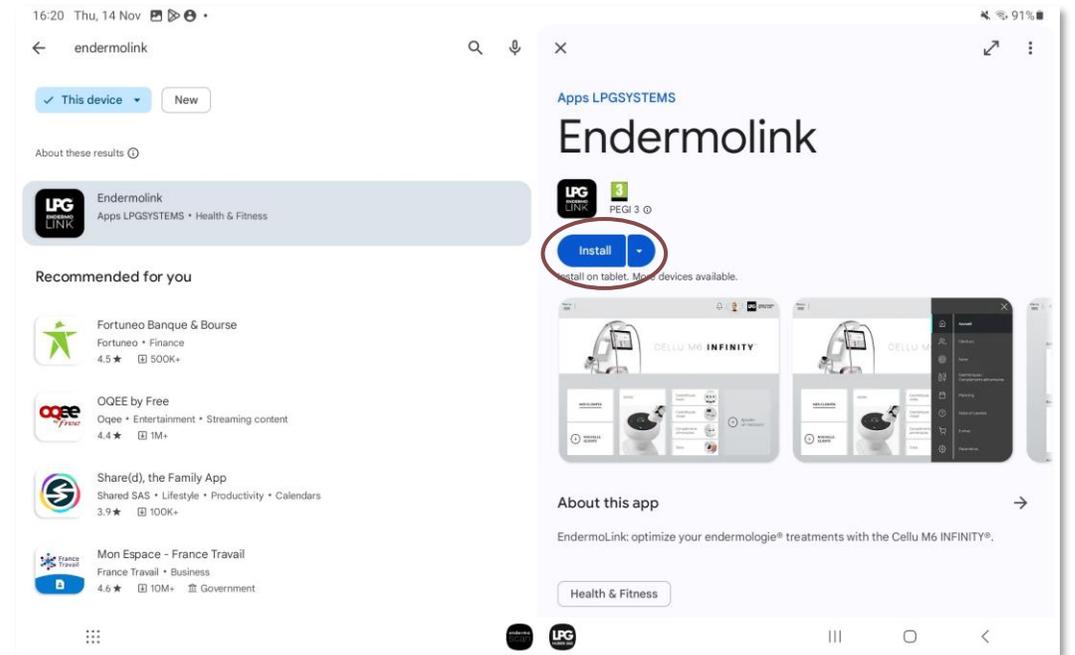

The **Endermolink™** app appears, click **Install**.


endermolink™ application

Cellu M6 Infinity®




**INNER POWER.
BETTER LIFE.**

DOWNLOAD THE endermolink™ application

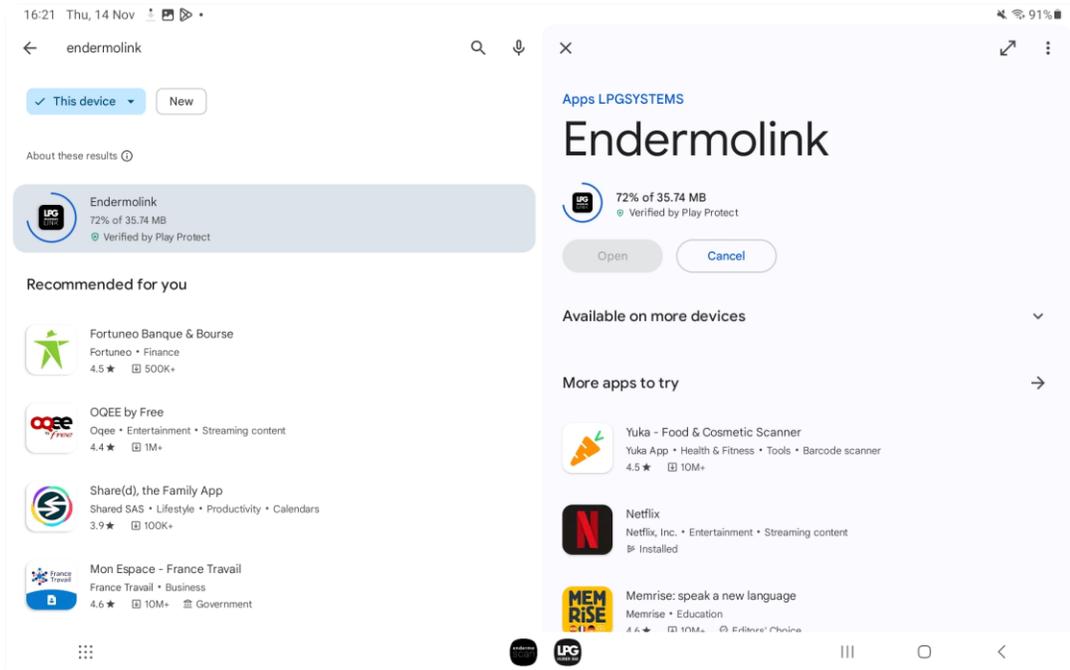

The **Endermolink™** app is being downloaded.

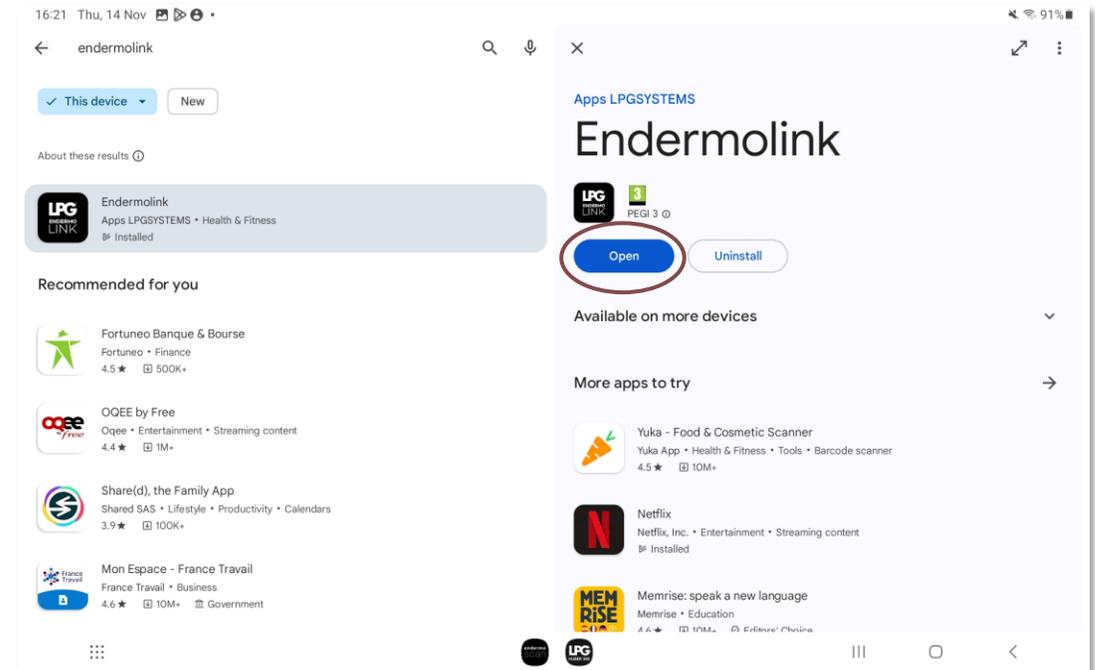


The **Endermolink™** app is now downloaded, click **Open**.

endermolink™ application


Cellu M6 Infinity®



LAUNCH THE endermolink™ application

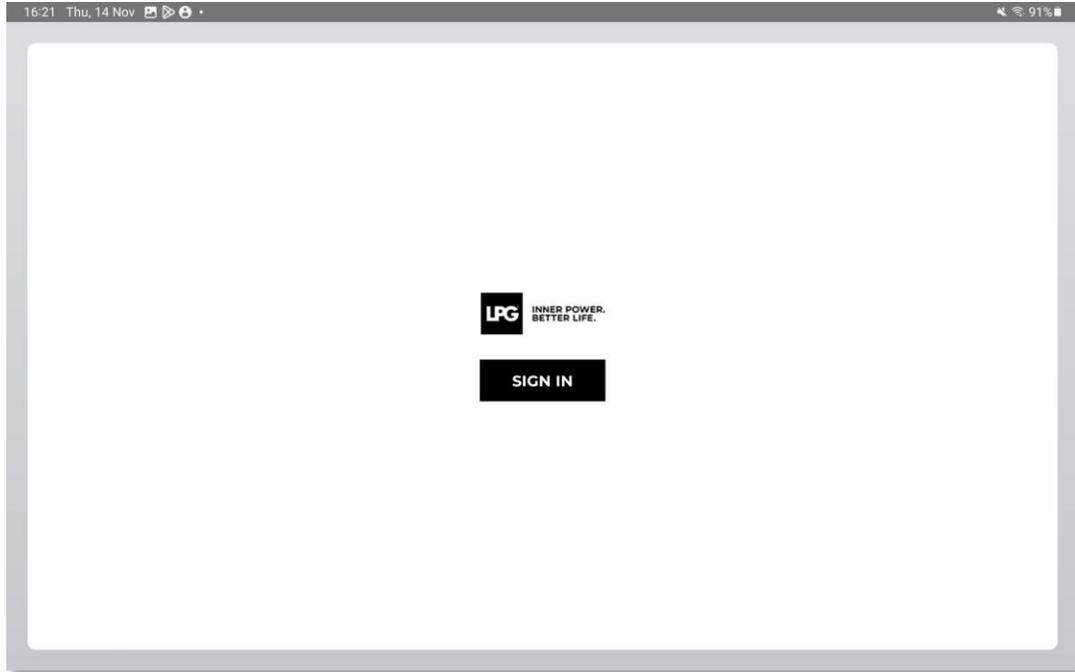

To open the **Endermolink™** application, click **SIGN IN**.

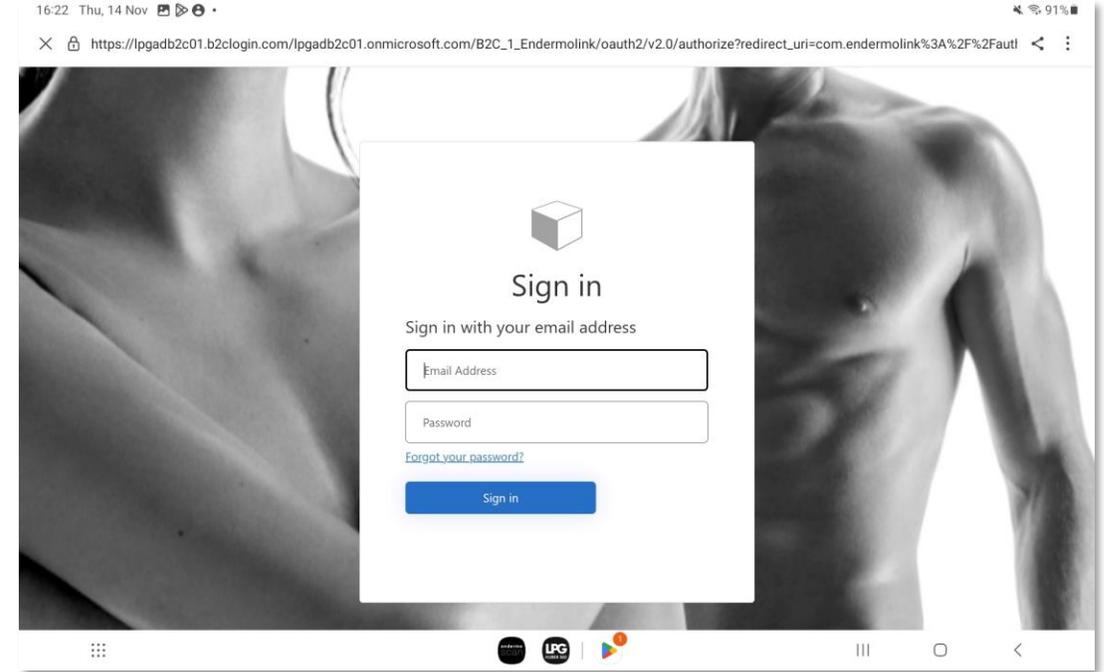

Enter your username (email address associated with your LPG® account) and password (existing if you are already an LPG® customer or chosen by you).

! If you have forgotten your password, click on "Forgot your password?" or on the following link: <https://portal.lpgsystems.com/password> to reset your password.


endermolink™ application

Cellu M6 Infinity®




**INNER POWER.
BETTER LIFE.**

LAUNCH THE endermolink™ application

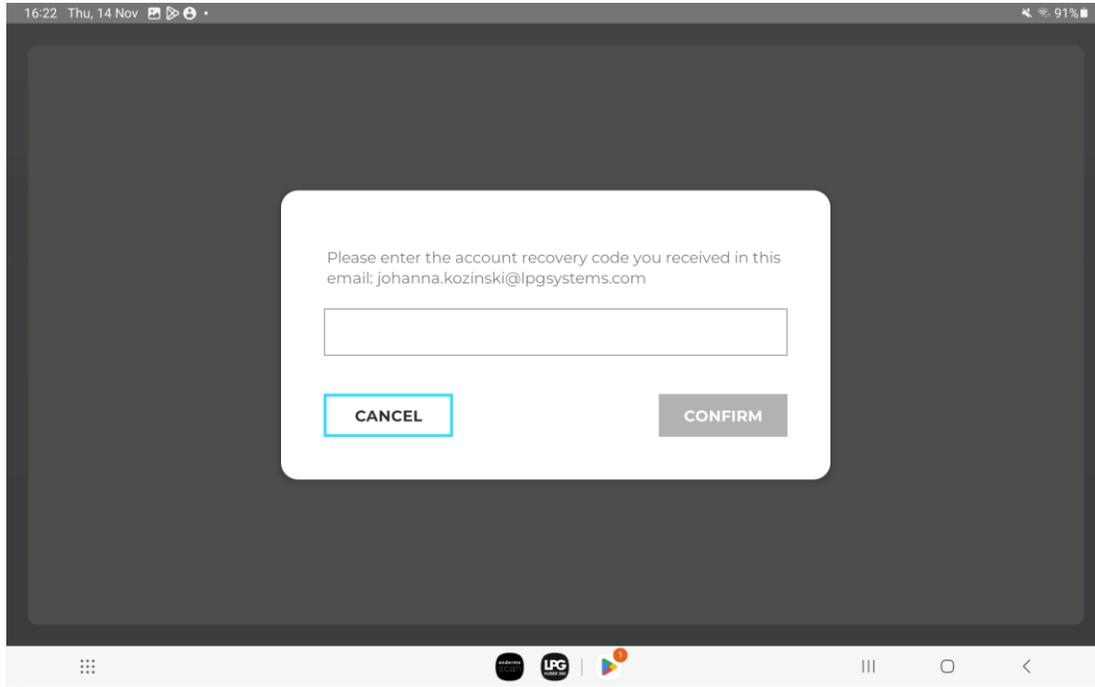

You will receive **an account creation code** by email. Enter the code and click **CONFIRM**.

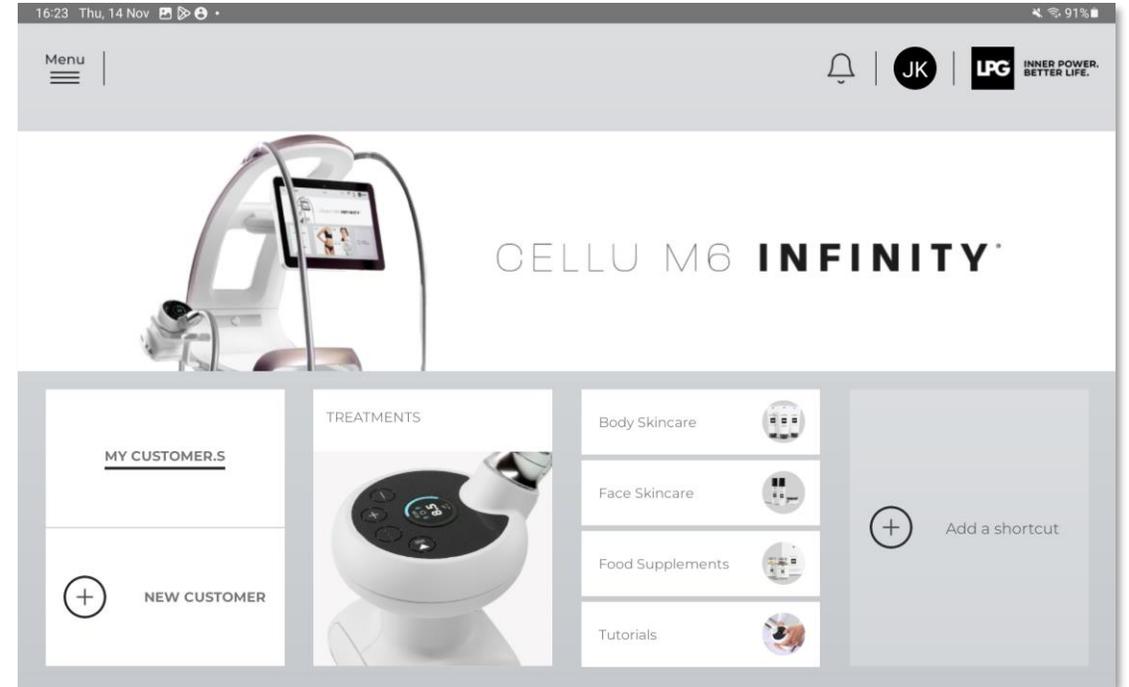

Welcome to the **Endermolink™ application!** You can now use the application and enjoy **its many benefits.**

endermolink™ application

Cellu M6 Infinity®



02

ENDERMOLINK™ APP

iOS APPLE VERSION




endermolink™ application

Cellu M6 Infinity®




INNER POWER.
BETTER LIFE.

CONNECT THE TABLET TO YOUR WI-FI NETWORK

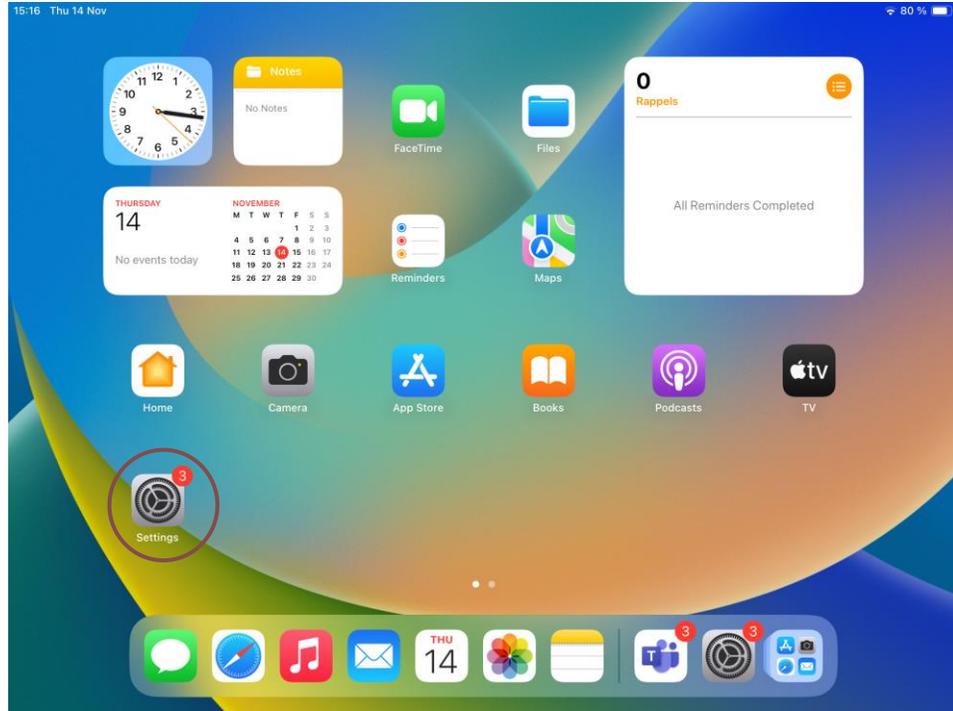


Start the **"Settings"** application on your tablet's home screen.

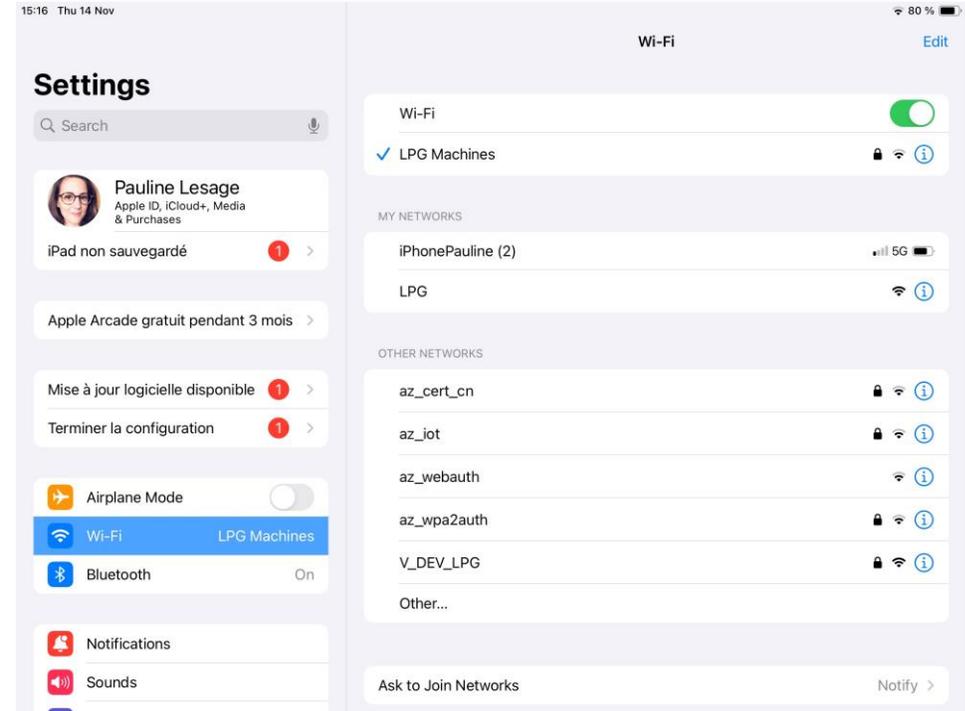


Select the **"WI-FI"** section, then choose your **WI-FI** network.

DOWNLOAD THE endermolink™ application

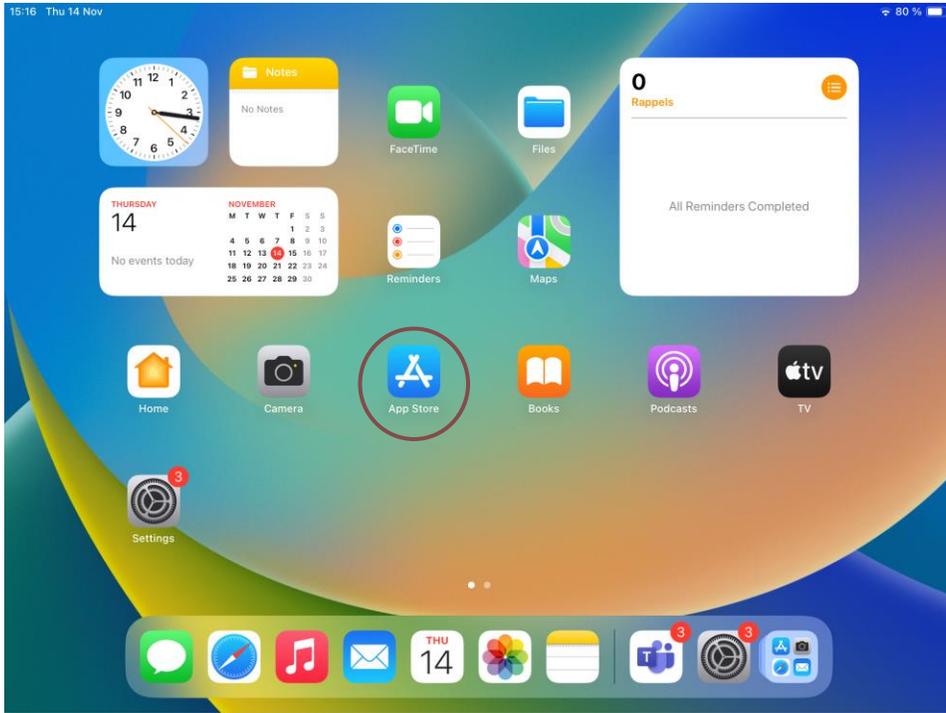

Start the **"App Store"** application on your tablet's home screen.

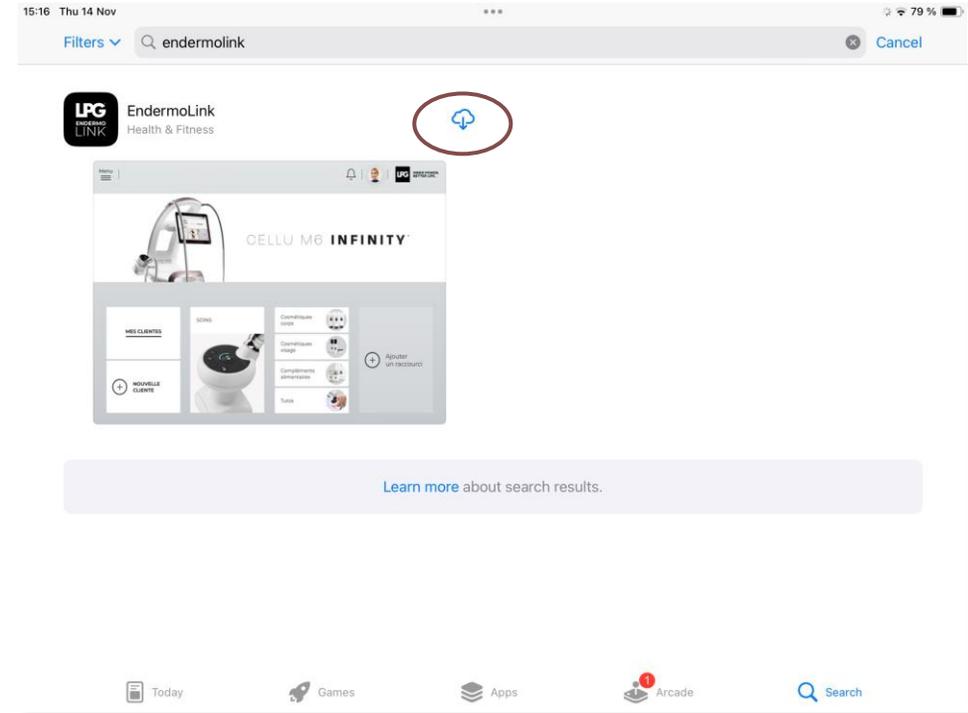

Search for the **Endermolink™** application in the search bar.
It will appear, **click on it.**


endermolink™ application

Cellu M6 Infinity®




**INNER POWER.
BETTER LIFE.**

DOWNLOAD THE endermolink™ application

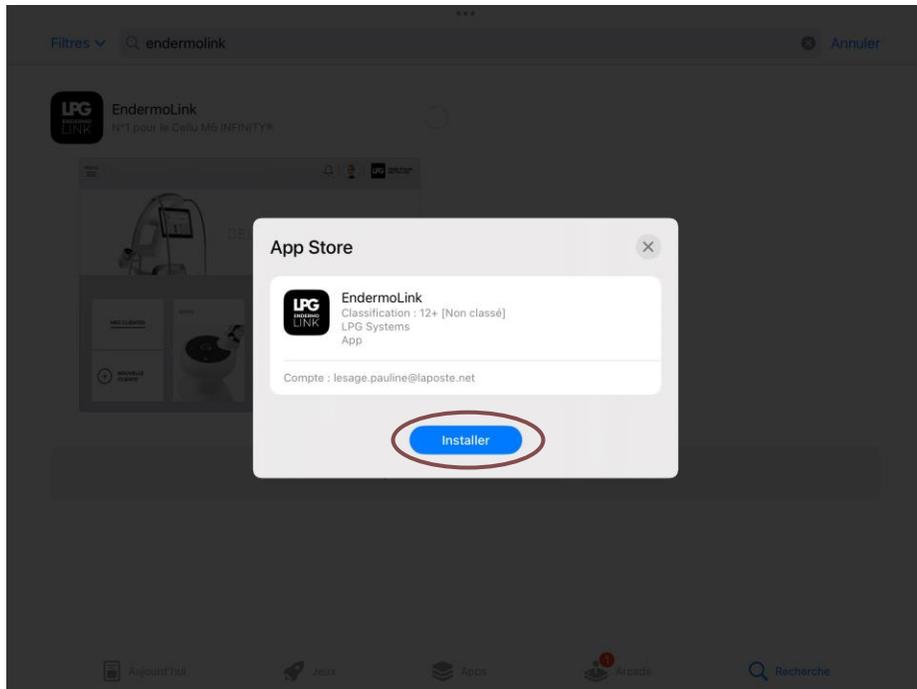

Then click on **Install**.

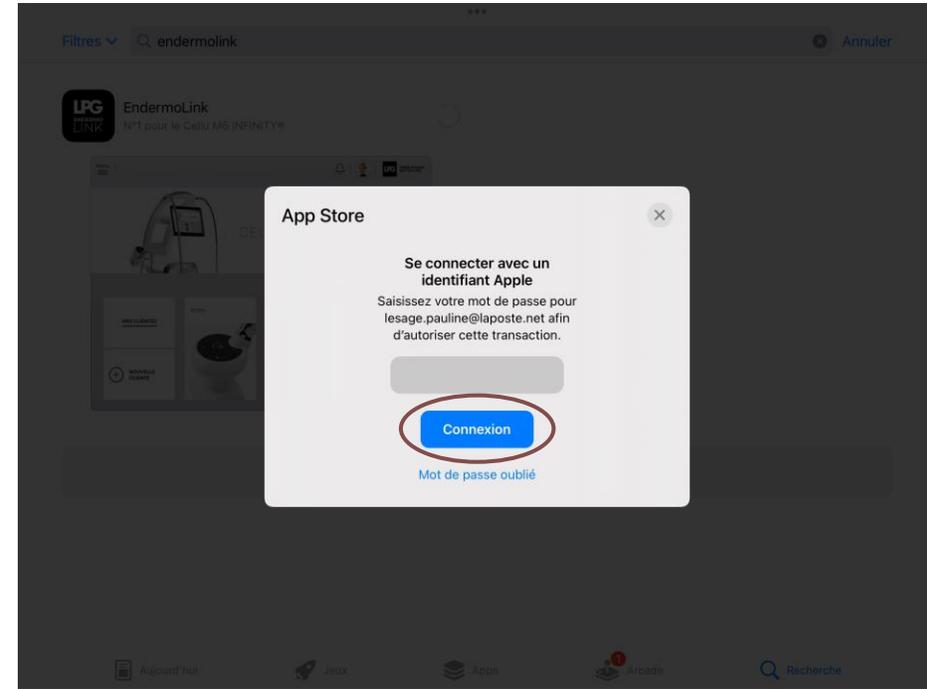

To download the application, sign in to **your Apple account**.
Enter your **password** and click on **Connection**.


endermolink™ application

Cellu M6 Infinity®




INNER POWER.
BETTER LIFE.

DOWNLOAD THE endermolink™ application

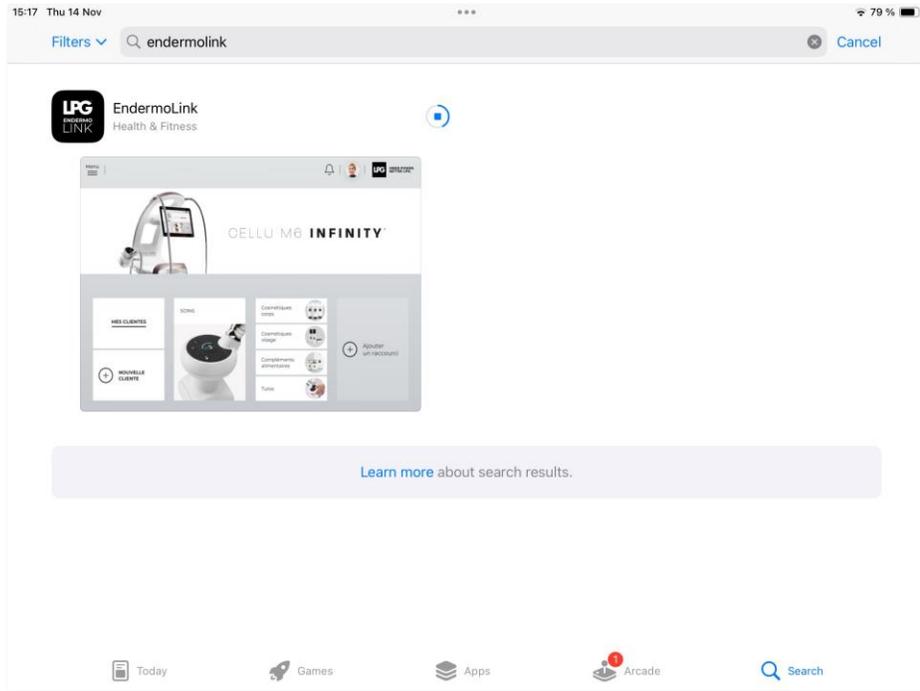

The **Endermolink™** app is now downloading.

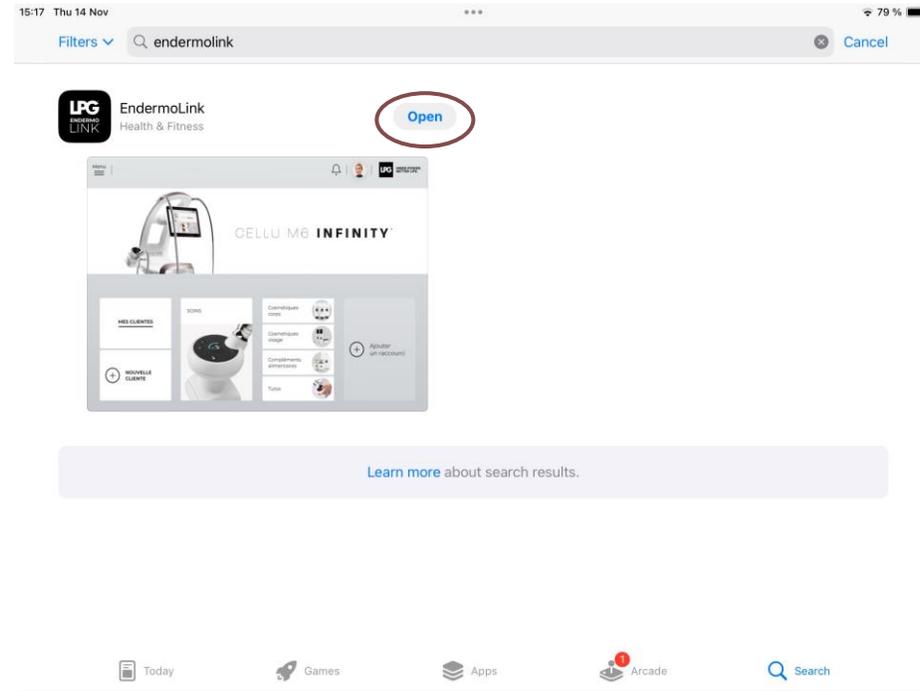

The **Endermolink™** application is downloaded, click **Open**.


endermolink™ application

Cellu M6 Infinity®




INNER POWER.
BETTER LIFE.

LAUNCH THE endermolink™ application

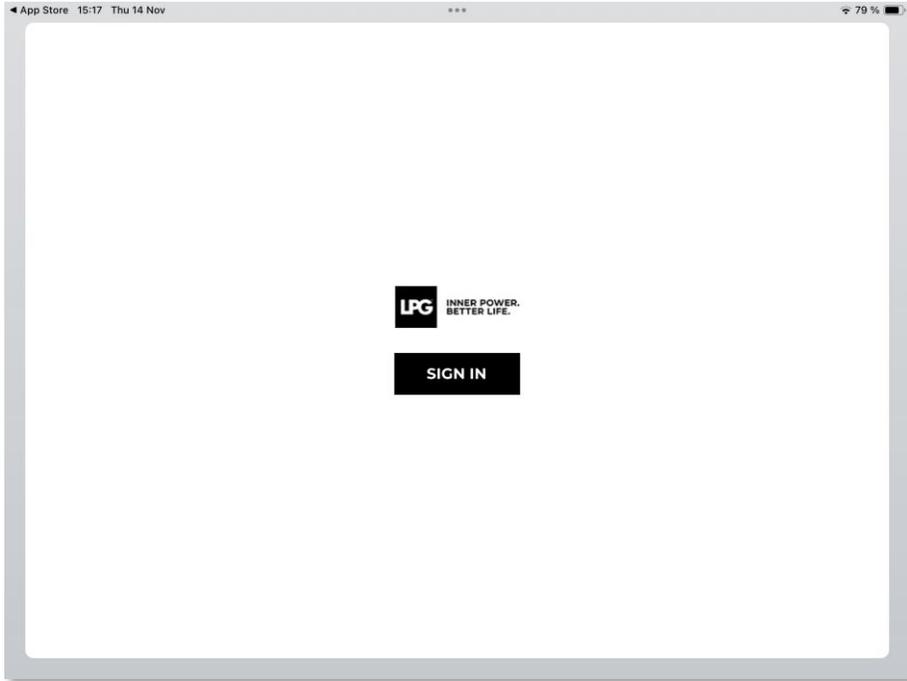

To open the **Endermolink™** application, click **SIGN IN**.

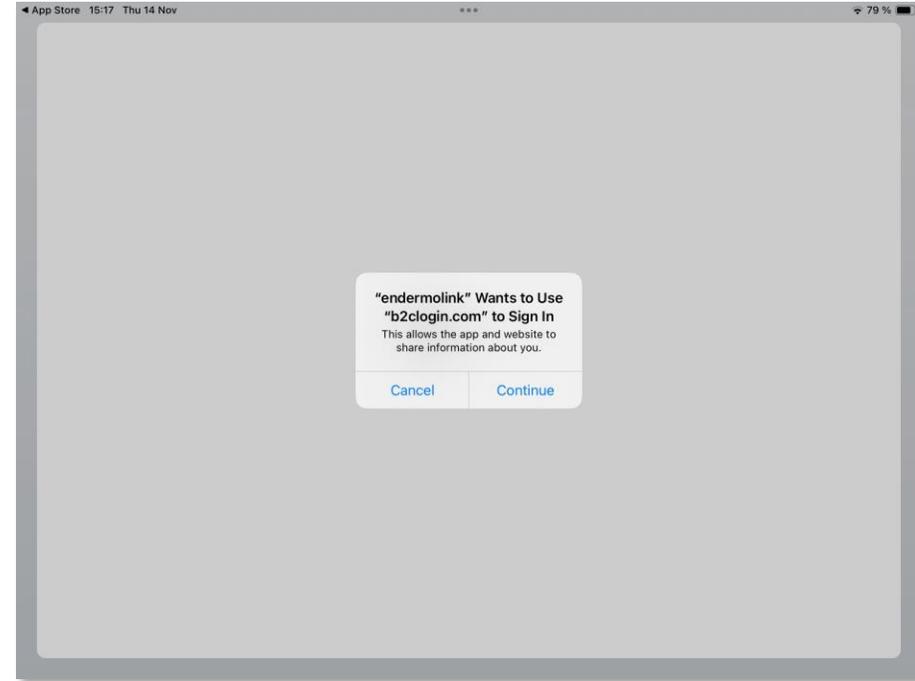

A **login pop-up** will appear, click **Continue**.



endermolink™ application

Cellu M6 Infinity®



INNER POWER.
BETTER LIFE.

LAUNCH THE endermolink™ application

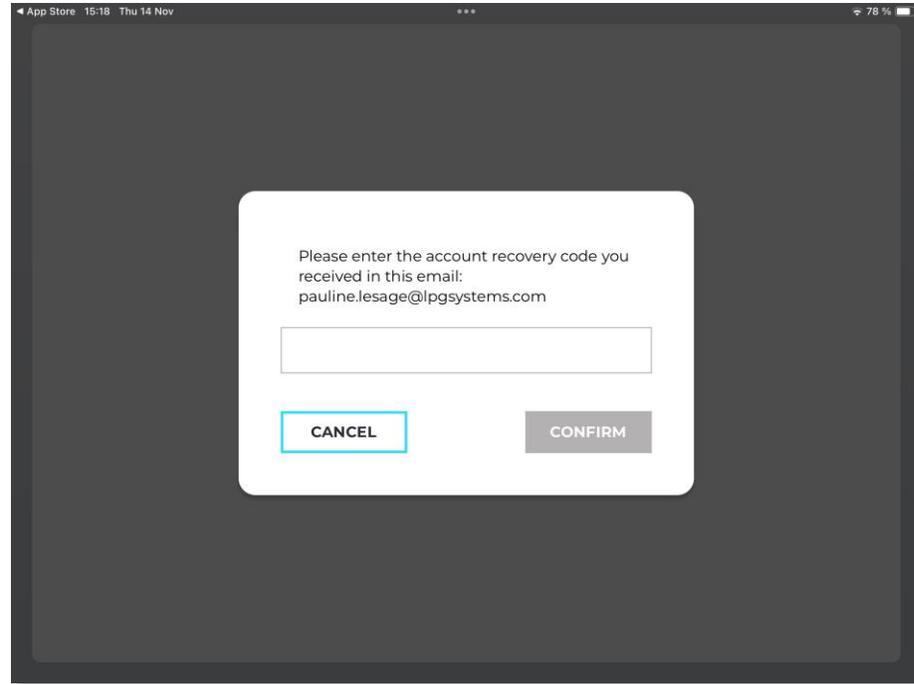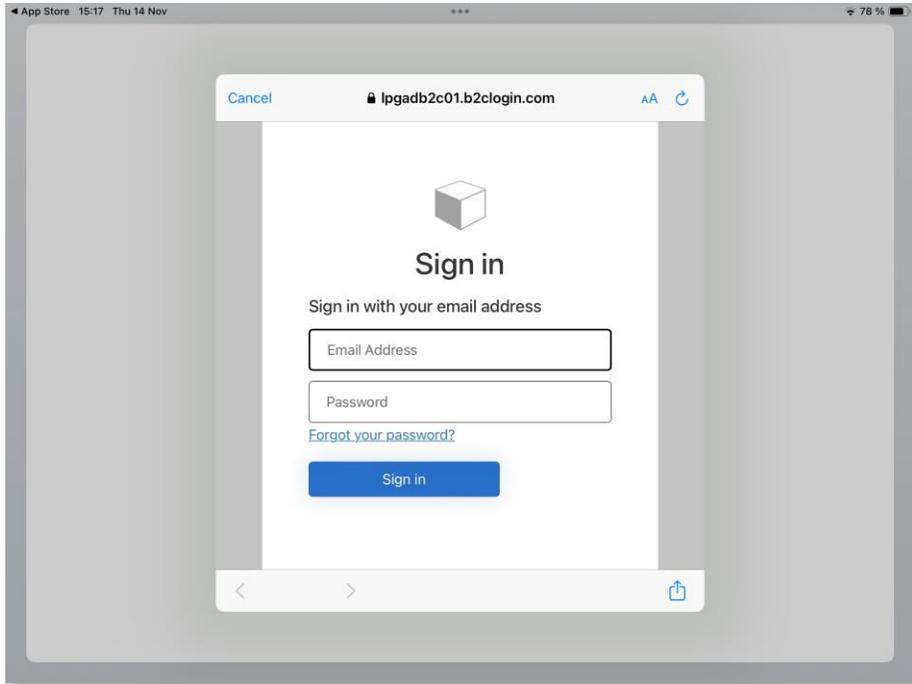

Enter your username (email address associated with your LPG® account) and password (existing if you are already an LPG® customer or chosen by you).

You will receive an **account creation code** by email. Fill it in and click **CONFIRM**.

endermolink™ application


Cellu M6 Infinity®

! If you have forgotten your password, click on "Forgot your password?" or on the following link: <https://portal.lpgsystems.com/password> to reset your password.



**INNER POWER.
BETTER LIFE.**

LAUNCH THE endermolink™ application

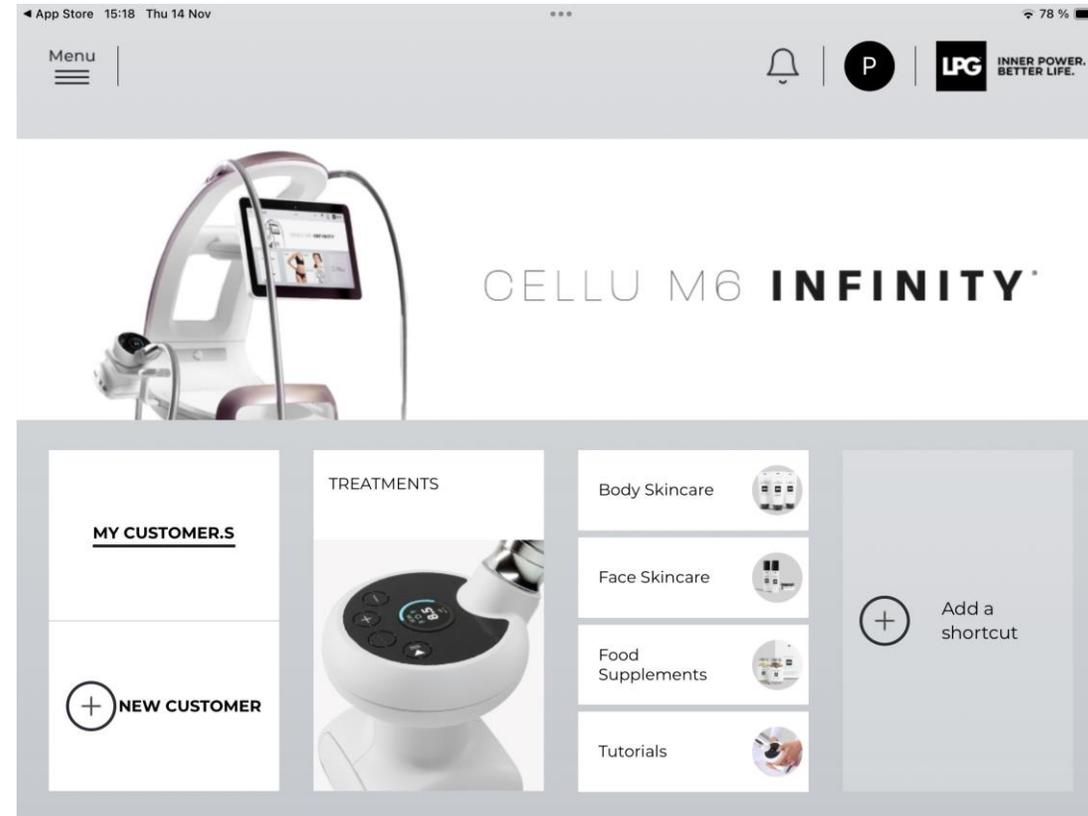

Welcome to the **Endermolink™ app!**
You can now use the app and enjoy **its many benefits.**

endermolink™ application

Cellu M6 Infinity®



INNER POWER.
BETTER LIFE.



**THANK YOU FOR YOUR
ATTENTION**

LPG[®]

**INNER POWER.
BETTER LIFE.**