CELLU M6 INFINITY®

GUIDE INTERFACE

INTERFACE GUIDE

ÉDITION 10/2024 | IU2402-EDA

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To better meet the needs and expectations of its customers, LPG Systems is constantly seeking to improve the design and quality of its products. For this reason, there may be slight differences between your appliance and the one described in this guide. All illustrations in this user guide are non-contractual.

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This document describes how to use the software on Cellu M6 Infinity[®]. It is not intended to provide details about implementation, but how to operate the software from a user's point of view.

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When the device starts up, the screen shown opposite appears.

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Press the desired language to select it. Then, press "OK" to continue.

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Please read and accept the terms of the EULA. Press OK to continue.

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Select the Wi-Fi network you want to connect. Press OK to continue.

Then, enter the Wi-Fi password and press OK to continue.

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Select the option above, then press OK to continue.

The activation file sent by LPG® to your email address must be downloaded onto a USB drive. Then insert it into the USB port on the back of your device's screen.

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Select your time zone. Press OK to continue.

Cht - 1 (Hundes, Listenne) Cht - 1 (Hundes, Bankel) Cht - 2 (Hesnel, Lis Cale Hasselw) Cht - 2 (Hesnel, Kal Cale Hasselw) Cht - 3 (Holowa, Nariab) Cht - 330 (Birkean) Cht - 330 (Birkean)

VÉRIFICATION COMPTE CLIENT CUSTOMER ACCOUNT VERIFICATION

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During the verification of your user account, the waiting screen appears.

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When you are connected, the screen shown opposite appears. Press OK to continue.

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The configuration of your Cellu M6 INFINITY® device is complete. You are now ready to discover the richness of its contents.

Scan the QR code with your tablet to download the endermolink $^{\rm TM}$ application.

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Select the treatment type you require from the menu on the home page by clicking on the corresponding icons.

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Select your customer's gender, skin type and treatment type, then click on **confirm**. This will filter the treatment types and display the corresponding protocols.

≣ <	CORPS FEMME	1006 1109	<u>چ</u> ¢	(f) us anase
		er um protocole	0	
	Celluite Lissant	50 minutes		⊳
	Resculpt & Firm	55 minutes		⊳

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Details of the protocol are then displayed on your screen (areas to be treated, treatment heads to be used. recommended LPG[®] skincare and food supplements, session sequence).

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Select customised treatment on the screen below.

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Select the treatment goals and the priority areas.

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Select the issues by area, then click on **confirm** to continue.

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For each area concerned, fill in the skin condition and prioritise your customer's issues.

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The customised treatment will then appear on your screen. Click on **Start session.**

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Follow the instructions that appear and click on 5 to start the treatment.

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During the treatment, the screen below appears, showing the various information and settings.

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UNE

LANCER

Press "Customers" on the homepage menu or "My customers" directly on the homepage.

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The customised session is only launched if you have created a customised programme on the endermolink™ app.

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at the top left of the screen.

Click on **Stop** via the menu button

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The screen shown opposite is displayed while the device shuts down.

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When an error is detected, a notification message appears on the screen.

Click on the icon and follow the instructions that appear.

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If the error persists, contact your authorised distributor or LPG[®] customer service.

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When the software needs to be updated, a notification appears on the screen. Click on the icon and follow the instructions.

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Click on **Install now** to implement the update.

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Wait while the update is installed.

APPLICATION PROFESSIONNELLE ENDERMOLINK™ ENDERMOLINK™ PROFESSIONAL APPLICATION

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When the update is installed, a message appears on your screen, followed by homepage.

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The application can be downloaded directly from the App Store for iOS, or Google Play Store for Android.

Open the application and log in with your credentials and password.

The connection between the machine interface and the endermolink M is made automatically.

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This application allows you to carry out customised assessments.

You will find customised programmes for your customers as well as tutorials and product information to develop your expertise.your know-how.

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