

Registered trademark

#### **OPERATOR'S MANUAL**

# CELLU M6° Alliance Lab

Please read the complete manual carefully before using your equipment

© LPG Systems 2024. LPG, CELLU M6 and Alliance are registered trademarks of LPG. Reproduction, even in part, is strictly prohibited.

GU 604 E Edition A 0 2024



Congratulations on the purchase of your CELLU  $M6^{\circ}$  Alliance. This model represents the fruits of numerous years of experience in the design and production of cutaneous tissue care systems. You will experience fully the technical perfection and reliability of LPG Systems, which is the leader in this sector.

With the aim to continuously provide satisfaction, your device is equipped with a specific software devised to ensure connection with the LPG dedicated server. The data collected through this software will allow LPG to give you better services in respect of support and maintenance.

This operator's manual contains the operating description, basic maintenance instructions to be performed periodically and the safety instructions.

Your device is intended for use in the care of connective tissue. It should be used only by a professional who has attended the manufacturer's training provided by LPG Systems or an approved provider, if you live outside of France.

If you have any doubts whatsoever concerning the operation or maintenance of your equipment, please do not hesitate to contact LPG via the Customer Service Department of your distributor.

+33 (0)4 75 78 69 00

#### **⊿** ATTENTION

In order to respond better to customer requirements and expectations, LPG Systems is continuously researching ways of improving the design and quality of its products. This will explain the few minor differences between your equipment and the item described in this guide.

#### PACKAGE CONTENTS

- → One CELLU M6<sup>®</sup> Alliance Lab device
- → One Alliance 80 head
- → One Alliance 50 head
- → One TR30 head
- → One Ergolift head
- → Two Ergolift chambers (Lift20 and Lift10)
- → One electrical power cord
- → One unpackaging instructions and quick installation guide
- → One marketing set

#### Liste of accessories provided with your device:

Depending on the version you have, (see serial number on the nameplate), some protocols are not activated and their accessories are not provided.

Accordingly, the paragraphs describing them do not concern this version (see table below). In any doubt about the operation of your unit or to evolve into a complete model, please contact the Customer Service of LPG Systems or your distributor.

	Pack body	Pack face	Pack spa
Alliance 80	√		<b>√</b>
Alliance 50			<b>√</b>
TR30			√
Ergolift		√	√
Ergolift chamber (Lift 10, Lift 20)		√	√
Unpackaging instructions and quick installation guide	√	√	√
Cord	√	<b>√</b>	<b>√</b>
Maketing set	√	√	<b>√</b>

## **SOMMAIRE**

## **DESCRIPTION OF THE DEVICE**

DESCRIPTION OF THE DEVICE
DESCRIPTION OF THE CONTROLS 8
PRECAUTIONS FOR USE
MAINTENANCE
TROUBLESHOOTING
TECHNICAL SPECIFICATIONS
CARE HEADS 28
ENDERMOWEAR™
GENERAL WARRANTY CONDITIONS41


#### INTENDED USE


The CELLU M6 $^\circ$  Alliance Lab device is intended for aesthetic uses to body and face care. It can be used to:

- 1. Smooth the body's contour
- 2. Redesign of silhouette
- 3. Firming the skin
- 4. Reduce wrinkles
- 5. Enhance the skin aspect
- 6. Face contouring

## THE CELLU M6® ALLIANCE LAB DEVICE

## **FILTER ACCES**

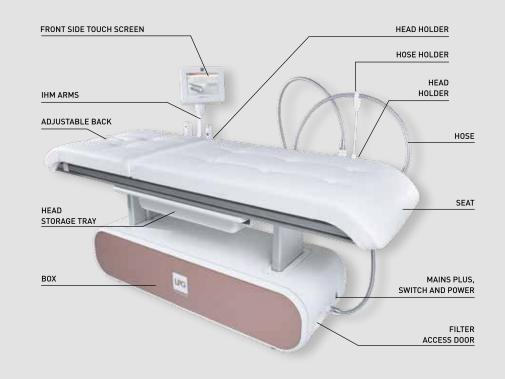

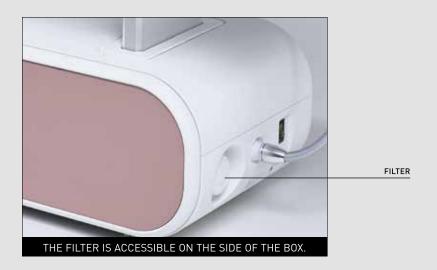


The unit can only operate if it is connected to the mains by its power cord and provided the ON switch has been actuated and the green voltage light is on. After switching on the unit, please wait a few seconds for the screen to display information.

DESCRIPTION


#### **CONTROL SCREEN**

## **ALLIANCE 50 HEAD AND TR30 HEAD**

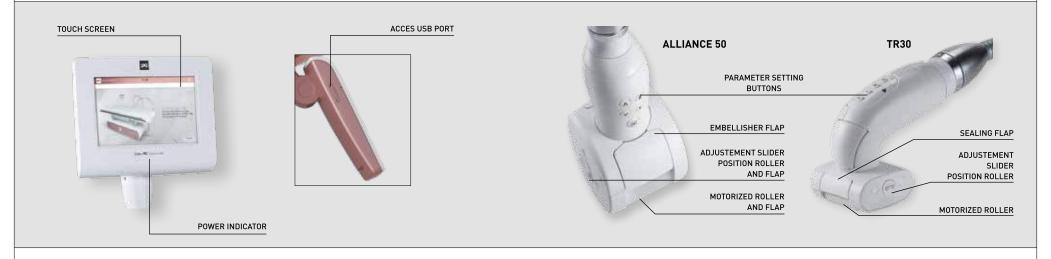

#### **ALLIANCE 80 HEAD**

## ADAPTER ERGOLIFT HEAD

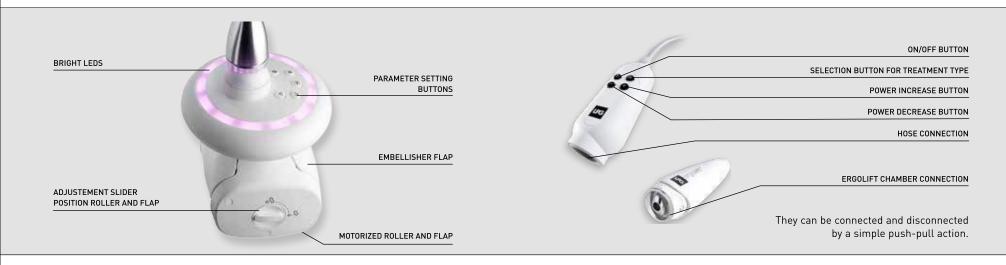

⊿ ATTENTION

For detailed instructions on using the touch interface, refer to the touch interface operating manual received during training and available from customer service.


THE CONTROLS

9

DESCRIPTION

## **HEIGHT ADJUSTMENT**

The table and back height are adjustable using the controls on the table (see photo). Simply press the appropriate buttons.

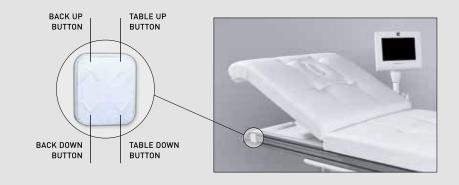

#### FRONT SIDE ADJUSTMENT

The front side can be adjusted using the articulated dial.


To adjust the front side, just move it to the desired position.

ARTICULATED DIAL



## **TABLETTE**

The CELLU M6 $^{\circ}$  Alliance Lab device is equipped with 2 shelfs to hold the different care heads.

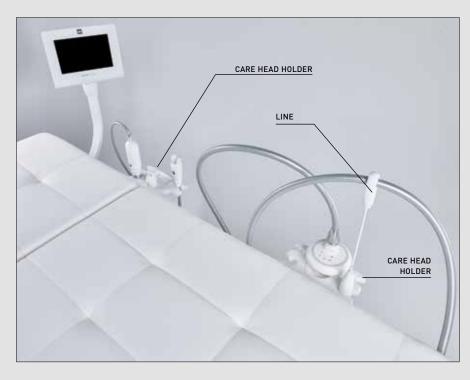

These shelfs can be moved along the length of the table by sliding it along the provided rail.

ΙU

#### IMPORTANT SAFETY INFORMATION

All safety precautions must be observed whilst using electrical equipment. Please read all safety notices and precautions prior to use of the equipment.

#### **DANGER** - TO MINIMISE THE RISK OF ELECTRICAL SHOCK:

Always disconnect the equipment from the electrical supply outlet after use and before cleaning and maintenance.

Check that the supply voltage of the unit indicated on the data plate complies with the mains voltage.

The unit must be connected by the power cord1 supplied to a grounded outlet in accordance with current electrical standards. Electrical adapters must not be used with this equipment. Ensure that the unit is connected to a system with a differential protection against DC and AC.

#### **WARNING**

#### • TO MINIMISE THE RISK OF BURNS, FIRE, ELECTRICAL SHOCK OR INJURY:

- The equipment must not be left unattended whilst connected to the electrical supply.
- Disconnect the unit from the electrical supply if it is not going to be used for a long period.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- Never use the unit for purposes other than those recommended by LPG Systems.
   Only use the care heads supplied with your unit or recommended by LPG.
- Never use the equipment if:

The electrical power cord or outlet is damaged.

The equipment does not function correctly.

The equipment is damaged or has fallen or been dropped.

The equipment has been exposed to excessive humidity.

- Do not move the unit by pulling the electrical power cord.
- Fully unwind the electrical power cord and keep it away from warm surfaces.
- Never use the equipment if the ventilation ports are obstructed. Ensure that the ventilation ports are kept clear of dust or other contaminants.
- Do not allow solid debris, liquid or other foreign bodies to fall or be sucked into the unit, as these could cause damage.
- Never use the equipment on a dusty, uneven floor, or in a moist atmosphere.
- Never use the equipment in the presence of aerosols or oxygen.
- Before disconnecting the unit from the electrical supply, set all controls to the 'off' position and unplug the unit. The disconnection of the unit is disconnecting the power outlet.
- It is prohibited to modify this equipment without prior authorisation from the manufacturer.
- It is prohibited to use components or spare parts non recommended by LPG.
- Return the device to LPG Systems Service Center or your distributors for examination and repair.

#### PRECAUTION FOR USE

#### **CONTRA-INDICATIONS**

#### ATTENTION: KEEP THESE INSTRUCTIONS.

Your device should be used on healthy skin. It is important to read and respect the following precautions and contraindications before using your device.

Never touch the patient and the device's unprotected cables or connectors simultaneously.

Never use the adapter as a care head or allow it to come into direct contact with the skin.

Only use care heads supplied with your unit or recommended by LPG.

Do not use the care heads directly on the skin. Wear the care suit provided by LPG Systems,  $ENDERMOWEAR^{TM}$ .

LPG Systems will not be liable for any inappropriate use of the equipment.

Improper use of the device can cause tissue damage or pain.

The operator must be particularly attentive to the sensations felt by the person undergoing care

The operator must ensure that the parameters (intensity, sequentiality, differential...) are always adapted to the tissue being careed.

Do not put more than 135 kg on the seat.

Do not put more than 10Kg on the head storage tray.

When the table is being used without supervision, it should be put in the lowest position to avoid the risk of falling.

After use, put the table in the lowest position to avoid the risk of falling.

Do not lean, rest, or sit on the head storage tray.

Do not use the USB connection during care
Do not operate the unit in unsuitable
environmental conditions (see technical

specifications).

It is recommended to use a care sheet on the seat.

The power plug is used as disconnect device. The disconnection of the unit is disconnecting the power plug.

Please position your device so that the mains unit is always accessible.

Do not touch the patient and the hose connectors simultaneously.

- Do not care open wounds, the eyes, intra-cavity areas, mucous membranes, the genitals or breasts.
- This device is not recommended for pregnant women. In the event of pregnancy, do not care the lumbar-abdominal region. Consult with a doctor with regard to this care.
- Do not care a patient with an infectious disease, a growing tumour, a phlebitis, a wound, or an infected area.
- Do not care a patient with skin cancer, a visible tumor, or other cancerous lesions. Consult with a doctor in cases where the patient has a case history of tumours or is in remission.
- Do not care inflamatory areas or scars without medical advice and LPG technical training for the affected areas.
- Do not care people with circulatory problems without first consulting their doctor and without training in LPG technology for the affected areas.

14

#### **CONTRA-INDICATIONS**

Do not care a patient with an unexplained and persistent pain without medical advice and without training in LPG technology for the affected areas.

Do not care a patient after an invasive medical care without medical advice or surgeon who carried out the care and without training in LPG technology for the affected areas.

To avoid bruising, exercise caution when determining a patient's level of sensitivity and avoid use on patients taking anticoagulant drugs.

Stop care immediately if the patient experiences pain and consult a doctor.

This device should not be used on all dermatoses, skin rashes, herpes, inflammatory or infectious acne or vitiligo.

Because of the risk of interference, it is important that the professional ensures the patient is not equipped with a personal medical device such as a pacemaker. If this is the case, the professional should obtain details about the device in question to ensure that any interference will not affect the correct use of the equipment.

As this list is not exhaustive, always seek out medical advice in the event of doubt.

For a more detailed list of indications and against indications, refer to training manuals.

Don't care a patient who takes anticaogulant drugs.

#### **ELECTROMAGNETIC COMPATIBILITY**

The Cellu M6 Alliance Lab device uses RF power only for its internal functions.

Therefore, its RF emissions are very low and are not likely to cause interference in a neighboring electronic device.

The Cellu M6 Alliance Lab device can be used in all establishments, including domestic premises and those directly connected to the public low-voltage power supply network supplying domestic buildings.

The Cellu M6 Alliance lab device must be installed on a floor made of wood, concrete or ceramic tile. If the floors are covered with synthetic materials, the relative humidity should be at least 30%.

The Cellu M6 Alliance Lab device must be connected to a power supply network whose quality is equivalent to that of a typical commercial or hospital environment.

If the Cellu M6 Alliance Lab device user requires continuous operation during power outages, it is recommended to supply power from an uninterruptible power supply or battery.

Portable and mobile RF communications devices must not be used too close to any part of the Cellu M6 Alliance Lab, including cables.

Interference may occur in the vicinity of equipment marked with the following symbol:



This unit contains programs to help the operator obtain the best anticipated results for each case undergoing care. Under no circumstances may these programs be construed as a guarantee of successful care, which varies, depending on the morphology, physiology and eating behavior of each patient.

#### **IDENTIFICATION RATING PLATE**

Your unit is identified by a serial number shown on the rating plate. The rating plate also shows the permitted supply voltage for the unit. If you need to contact LPG Systems because of a technical problem, please indicate the serial number of your. This serial number provides information on the year and month of manufacture of your unit. The letter indicates the year the device was manufactured. Z=2009, A=2010, B=2011, etc.

The two digits indicate the production month: 01=January; 02=February; 03=March; etc.

	LPG SYSTEMS FARRQUE EN FRANCE MADE N FRANCE フランス製 TECHNOPARC DE LA PLAINE 30 RUE DU DOCTEUR ABEL CS90035 26902 VALENCE CEDEX 09 FRANCE		ĆE	(i
SERIAL NUMBER	TYPE: CELLU M6 ALLIANCE LAB モデル	IP 20		夏
	N° DE SERIE SERIAL NUMBER LABG112061 シリアル番号			$ \bar{\Lambda} $
	100-240V 50Hz 830W 100-230V 60 Hz 830W			

#### **IDENTIFICATION RATING PLATE**

This icon indicates that the unit was sold after August 13, 2006. In conformity with the 2002/96/CE directive, it cannot be thrown away with standard household waste but must be disposed of by means of recycling. When your device reaches the end of its useful life, it must be brought to an appropriate recycling center or returned to your dealer. By doing so, you help the environment by contributing to the conservation of natural resources and the protection of human health.

This icon indicates that some specific warnings or precautions associated with this device are not

This icon means « refer to the user manual ».

A This icon indicates « dangerous voltage ».

VOLTAGE, FRENQUENCY AND POWER

#### **CLEANING THE UNIT**

It is recommended that you clean your unit as often as possible, not only for reasons of hygiene and aesthetics but also because cleaning the unit will help keep it in a good state of repair and extend its useful life.

#### Using a vacuum cleaner with a fine nozzle, clean the following sections:

- Interior of the head storage drawer.
- Interior of the head storage tray.
- Interior of the filter access door.

#### Using a moist sponge, clean the following sections:

- All the external hoods.
- Hoses.
- The electrical power cord.

# Using a cloth soaked with a small amount of alcohol-free domestic cleaning product, clean the following sections:

- Control screen and control panel.
- Interior of the head storage drawer.
- Interior of the head storage tray.
- Interior of the filter access door.

#### Using an antistatic cloth soaked or wipes, cleaning:

• The touch screen.

The maintenance of the care heads should be performed before the first use, see chapter «care heads».

#### REPLACING THE FILTER CARTRIDGES

Your device contains two filter cartridges and one filter foam.

These components guarantee the efficiency of your unit and prolongs its useful life.

Ensure that they are changed as soon as the command screen displays one of these messages **(Fig 1-2)**:

Icon indicating a filter change is required (Fig. 2).





FILTER TO CHANGE

Access the 'filter change' menu as follows :

Select the 'maintenance' menu by pressing the icon indicated (Fig. 3).

Select the 'filter' menu by pressing the icon indicated (Fig. 4).



\_\_\_\_\_\_

PRESS THIS ICON

The 'filter change' screen indicates which filter requires changing (Fig. 5).



FILTER TO CHANGE

PRESS THIS ICON



FILTER TO CHANGE

20

ATTENTION

Proscribe aggressive products such as acetone, trichlorethylene or alcohol at 90 °.

#### REPLACING THE FILTER CARTRIDGES

Your device contains one filter cartridge to guarantee the efficiency of your device and to porlong its life. Be sure to change it as soon as the screen displays a warning message.

Open the filter acces door. Unscrew the filter cartridge and replace it with a new one. Contact LPG Systems Customer Support to stock up on filter filter cartridges so that you always have spares.





## **CONNECTING / DISCONNECTING** THE MOTORIZED CARE HEADS

To connect the heads to the hose, follow the procedure below:


Position the locking ring in the locked position (fig. 1)

Position the end of the hose so that the hose key is lined-up with the key slot of the care head connection (fig. 2)

Push the hose into the care head connection until it cliks into place.





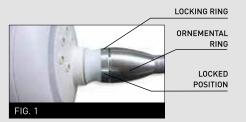



To disconnect the heads to the hose, follow the procedure below.

Position the locking ring in the unlocked position (fig. 3) Pull the locking ring towards the hose (fig. 4) Carefully remove the hose by pulling it by the white ring (fig. 5)







UNLOCKED POSITION

MAINTENANCE

#### INSTRUCTION FOR DISCONNECTING THE HOSE

INSTRUCTION TO REMOVE USB PROTECTIVE COVER

The hose can be disconnected from the box simply by following the procedure described above:

- Unlock the connection by turning the aluminum ring (fig. 1).
- Lifting the aluminum ring (fig. 2).
- And gently removing the hose by pulling on the white ring (fig. 3).









Remove the access cover to the USB port by using the appropiate tool.



#### REPLACING THE POWER CORD

If the power cord of your device is damaged, please contact LPG Systems Customer Service for a replacement.

LPG Systems Customer Service: +33(0)4 75 78 69 00

#### MAINTENANCE LOG SHEET

Replacement of filter cartridges: To be done when the warning message. Replacement of sealing flap: To be done when the flaps no longer treat the skin properly. They should be replaced after every 100 hours of use.

DATE	NBRE D'HEURES	OPÉRATIONS EFFECTUÉES

#### WHAT IF I'VE A PROBLEM?

# If your unit is not working properly, proceed with the following checks before calling Customer Services:

Is the unit properly connected to a mains plug?

Is the mains plug live?

Is the ON switch lit up?

Are the filter cartridges clean and correctly placed?

Are the hoses properly connected?

Are the care heads properly connected?

Is the connected care head connected corresponds to that shown on the screen?

Once these checks have been carried out and if the malfunction persists, please contact Customer Services of LPG Systems or the nearest authorized dealer, indicating the model of your unit and its serial number.

Service Client de LPG Systems :

+33 (0)4 75 78 69 00

#### **TECHNICAL SPECIFICATIONS**

	208x68x147 cm
•	
	by mechanical ventilation incorporated in the pump
Wifi:	2,4Ghz b/g/n
Lenght of Ergolift Hose:	
Operating environmental features:	
Ambient relative humidity:	+ 10 à + 30°C for normal use. 30 à 75% without condensation.
	a 1050 hPa (For use in a normally ventilated room)
Environmental characteristics of transpo	rt and storage:
Ambient relative humidity:	-20°C to +70°C
'	

Your device is equipped with patented care heads.

The CELLU M6® Alliance Lab is marked in compliance with the applicable texts and standards as part of the EMC 2004/108/EC and Low Voltage 2006/95/EC directives.

## INDEX

## **CARE HEADS**




Registered trademark

DESCRIPTION OF ALLIANCE 80 HEAD	30
DESCRIPTION OF ALLIANCE 50 HEAD	32
DESCRIPTION OF TR30 HEAD	34
DESCRIPTION OF ERGOLIFT HEAD	35
MAINTENANCE	36

## **DESCRIPTION OF THE ALLIANCE 80 HEAD**

#### **DESCRIPTION OF THE ALLIANCE 80 HEAD**

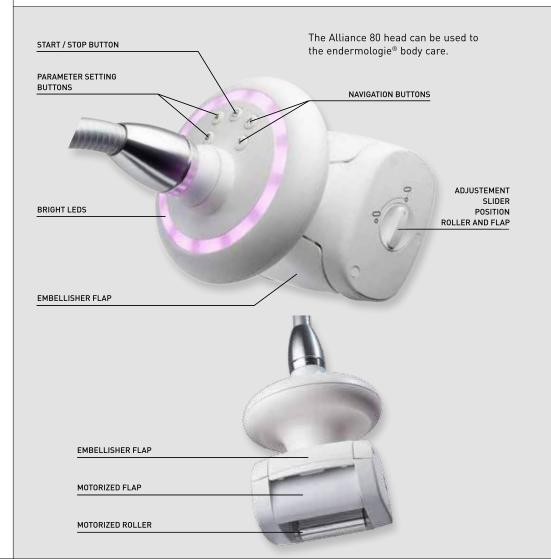

#### ADJUSTMENT OF ROLLER AND FLAP POSITION

Alliance head 80 has an adjustable stop with 2 positions for adjusting the distance between the motorized roller and of the damper.

When the cursor is in the horizontal position, the maximum mobility of the rollers is ensured.

When the cursor is in the vertical position, the minimum mobility of the rollers is ensured.



To change the adjustment slider turns it into the holding down to the desired position, as shown in the photo below:






HEADS

CARE

## **DESCRIPTION OF ALLIANCE 50 HEAD**

## **DESCRIPTION OF ALLIANCE 50 HEAD**

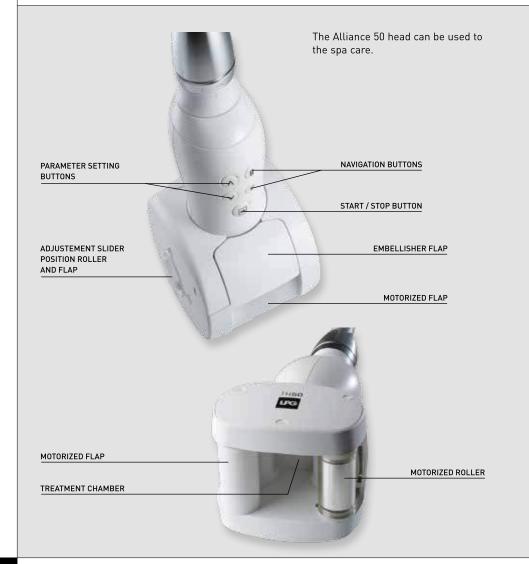


#### ADJUSTMENT OF ROLLER AND FLAP POSITION

Alliance head 50 has an adjustable stop with 2 positions for adjusting the distance between the motorized roller and of the damper.

When the cursor is in the horizontal position, the maximum mobility of the rollers is ensured.

When the cursor is in the vertical position, the minimum mobility of the rollers is ensured.

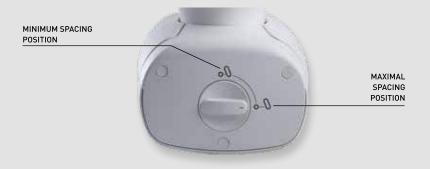


To change the adjustment slider turns it into the holding down to the desired position, as shown in the photo below:






## **DESCRIPTION OF TR30 HEAD**

## DESCRIPTION OF THE ERGOLIFT HEAD

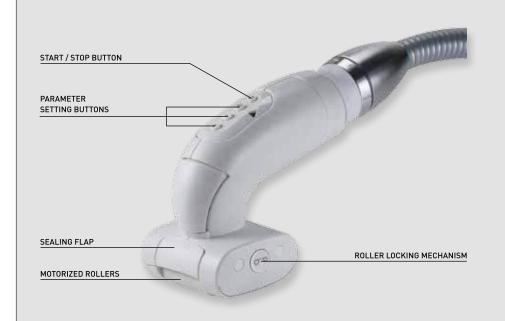

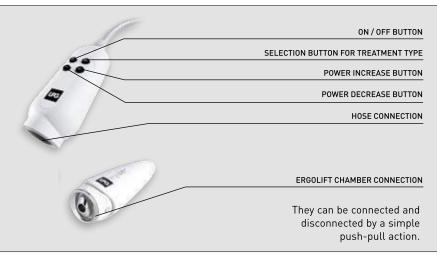

#### **DESCRIPTION OF ERGOLIFT CHAMBERS**

The Egolift chamber Lift 20 can be used to the care of large areas with fine tissues, and sensible areas. The Ergolift chamber Lift 10 can be used to the care to tight areas, eyes and lip contour, hands and figers.



Only the LIFT 20 and LIFT 10 can be connected to the Ergolift head. Thay can be connected and disconnected by a simple push-pull action.



**Lift 10** Care chamber with removable flap





#### **MAINTENANCE**

For reasons of hygiene, the maintenance of the care heads should be performed after each use, using antiseptic wipes impregnated with a bactericidal and fungicidal solution. Special attention must be given to the maintenance of the parts that are in contact with the client.

#### THE ALLIANCE 80 HEAD

- 1. Remove the sealing flaps (2 up flaps and 1 down flap) as showed in the pictures below **[fig 1 to 4]**.
- 2. Thoroughly rub for at least one minute with the wipes as describe here below.:
  - a) Flaps and their housing (fig 5 to 7).
  - b) The casing on both sides of the rollers (turn the head over, rotate the rollers manually to clean the entire surface) (fig. 8).
  - c) The motorized flap (Don't mobilize the motorized flap) (fig 9 & 10).
  - d) Le sabot
- 3. Reattach the sealing flaps
- 4. Maintain the storage drawer using LPG wipes, then place the head in it.



#### **MAINTENANCE**

#### THE ALLIANCE 50 HEAD

- 1. Remove the sealing flaps (2 up flaps and 1 down flap) as showed in the pictures below (fig 1 to 4).
- 2. Thoroughly rub for at least one minute with the wipes as describe here below.:
  - a) Flaps and their housing (fig 5 to 7).
  - b) The casing on both sides of the rollers (turn the head over, rotate the rollers manually to clean the entire surface) (fig. 8).
  - c) The motorized flap (Don't mobilize the motorized flap) (fig 9 & 10).
  - d) Le sabot
- 3. Reattach the sealing flaps
- 4. Maintain the storage drawer using LPG wipes, then place the head in it.



3.6

#### **MAINTENANCE**

#### THE TR30 HEAD

- 1. Remove the sealing flaps as showed in the pictures below (fig. 1).
- 2. Thoroughly rub for at least one minute with the wipes as describe here below: a) The flaps and their housing (fig 2 & 3).
  - b) The casing on both sides of the rollers (turn the head over, rotate the rollers manually to clean the entire surface) (fig. 4).
  - c) Le sabot
- 3. Reattach the sealing flaps
- 4. Maintain the storage drawer using LPG wipes, then place the head in it.







Fig 3



Fig 1

#### **MAINTENANCE**

#### **ERGOLIFT HEAD AND ERGOLIFT CHAMBERS**

For reasons of hygiene, the maintenace of the care heads should be performed after each use, using antiseptic wipes impregnated with a bactericidal and fungicidal solution. Special attention must be given to the cleanliness of the parts that are in contact with the client.

- 1. Disconnect the chamber from the Ergolift care head. (fig. 1).
- 2. Remove the flap thanks to the dedicated tool. (fig. 2).
- 3. Thoroughly rub the Ergolift chamber, the flap and the tool for at least one minute with the wipes as describe here below. (fig. 3).
- 4. Put the flap back in the Ergolift chamber by following the same steps in the reverse order (fig. 4).









HEADS

CARE

#### ENDERMOWEAR™

#### **GENERAL WARRANTY CONDITIONS**

LPG Endermowear<sup>™</sup> is available in several sizes for men and women and has been specially designed for body treatments. It is intended for personal use, guarantees hygiene, and its opaque areas cover the patient's intimate parts during the treatment. Endermowear<sup>™</sup>'s unique material guarantees excellent adhesion to the skin which facilitates movement of the care head.

The products are delivered in a bag that the customer can personalize by putting their name on the label. It becomes the customer's property and can be used for several sessions. For esthetic reasons and hygiene, it should be washed after each use. Please refer to the washing instructions indicated on the bag label.

You have recently acquired an appliance distributed by LPG Systems or an LPG Systems approved distributor. It is the purchaser/ user's responsibility to find out from the local authorities the conditions and professional qualifications required before using the appliance.

The purchase of this equipment implies the legal acceptance by the purchaser/professional user of these general warranty conditions. If the appliance was sold to you by an approved LPG Systems distributor, the purchaser/user should refer to the supplier's warranty conditions. These may in no way increase the undertakings made by LPG in these present warranty conditions.

The warranty can only be implemented and is only valid if the warranty slip has been duly filled out and returned to LPG Systems within two weeks of delivery, irrespective of the country. Warranty slips that are only partially completed will be rejected. The appliance is guaranteed against manufacturing flaws and defects in the raw materials.

The warranty extends for the shorter of the following two periods: one (1) year OR two thousand (2000) hours of use from the invoice date. During this period, LPG Systems undertakes to exchange or repair free of charge, as quickly as possible, any part that LPG Systems acknowledges as defective,

however LPG Systems does not undertake to replace the entire appliance.

Traveling and living expenses for our technicians and transportation costs of the appliance or parts to and from the aftersales service workshop are not covered by this warranty. Replacements and repairs performed within this warranty, with or without immobilization of the equipment, shall not have the effect of extending the warranty period.

Replaced parts become the property of LPG Systems or the approved distributor. No compensation shall be paid for loss of use. Subject to other conditions hereafter, warranty shall apply if the purchaser/professional user has allowed LPG to proceed to necessary repair works.

#### **GENERAL WARRANTY CONDITIONS**

#### Warranty exclusion:

Damage occurring during transportation. Transportation of this equipment and/or spare parts is at the recipient's own risk. Before taking delivery, it is the recipient's responsibility to verify the state of the goods and to make a claim against the transport company in the manner usual in the delivery country.

Non-observance of the installation and operating instructions, failure to carry out maintenance and/or negligence in maintaining the appliance and/or its filter cartridges, connection to a faulty electricity supply or a non-earthed electricity supply or a power supply whose voltage is different to the one indicated on the appliance.

If an appliance is sold before the end of the warranty period, the warranty is transferred to the purchaser for the remaining warranty period, on the condition that:

- I The original invoice is provided.
- **II** That the initial vendor is informed of the sale.

Modification, mounting of accessories or dismantling of the equipment.

Any operation and/or intervention not specified in the LPG Systems Operating Instructions and performed on the equipment by the purchaser/user and/or any party not approved by LPG Systems.

Use of consumables, spare parts, inappropriate components or parts not supplied by LPG Systems.

Blockage of the appliance through aspiration of a foreign body.

Normal wear and tear of any of the equipment's parts resulting from normal usage.

Damages or default resulting of accidental events (falls, impacts, ...)
Damages or default resulting of natural disasters (lightening, water damages, ...)
Fire, negligence or abuses.

#### **GENERAL WARRANTY CONDITIONS**

The warranty is limited to the replacement of the components of the device which comply with conditions described above. Under no circumstances shall LPG be liable for any loss or damage as a result or in connection with the device and/or its use, including any financial loss, loss of margin, loss of use, etc. This clause shall apply under any and all legal basis. Whenever the above restriction may not be applicable or enforceable, LPG liability shall be limited to the price for the device and/or the service.

9

WARRANTY CONDITIONS

ENERAL

42

#### WARRANTY ACTIVATION

## OR FILL IN THE FORM AND RETURN TO LPG SYSTEMS

9

GENERAL WARRANTY CONDITIONS

You can activate your warranty online by connecting to our warranty webpage:

http://warranty.lpgsystems.com

•
Name*:
Address*:
Country*:
Tel*:
Email:
Type of business:
Profession:
Type of unit: CELLU M6® ALLIANCE LAB
Serial number*:
Date:

The personal data collected in this form are sent to LPG Systems, which is responsible for processing it in order to manage customers relations, including after-sales service and warranties. Information with an asterisk must be provided in order for this form to be processed. According to modified French law 78-17 from January 6, 1978 relating to computers, digital files, and freedoms, also known as the "Informatique et Libertés" law, you have a right of access, rectify, and remove personal data about you by contacting LPG Systems in writing at the following address: LPG SYSTEMS - 30, rue Dr Abel - Technoparc de la Plaine - 26000 VALENCE (FRANCE).



#### **HEADQUARTERS LPG SYSTEMS S.A.S**

TECHNOPARC DE LA PLAINE

30, RUE DU DR. ABEL - CS 900 35 - 26902 VALENCE CEDEX 09 - FRANCE TEL.: +33 (0)4 75 78 69 00 - FAX: +33 (0)4 75 42 80 85

#### INTERNATIONAL/MARKETING

ECOLUCIOLES - BAT A

955 RTE DES LUCIOLES - BP 243 - 06905 SOPHIA-ANTIPOLIS - FRANCE TEL.: +33 (0)4 92 38 39 00 - FAX: +33 (0)4 92 96 09 65

