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TOUCHSCREEN INTERFACE - OPERATOR'S MANUAL

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© Copyright 2017. LPG, CELLU M6 and Alliance are registered trademarks of LPG.
Reproduction, even in part, is strictly prohibited.

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This document describes how to use the software on the CELLU M6 Alliance and CELLU M6 Alliance Lab. It is not intended to provide details about implementation, but how to operate the software from a user's point of view.

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When the device starts up, the below screen appears, push on « start ».

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Press the desired language to select it. Then, press "Next" to continue.

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Please read and accept the terms of the CLUF. Press «Next» to continue

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Select the activation mode by pressing the appropriated touch :

Select this option, if you are connected to internet

Select this option, if you are in offline mode

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Select the WiFi network you want to connect. Follow the indications related to the selected network (security key).

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You must download the activation file sent by mail on a usb key. Then, insert it into your device.

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Enter your username and password sent by LPG to your mailbox.

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During the verification of your user account, the below screen appears.

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The configuration of your device is finished. Please, press on the touch « Discover » to continue.

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You can choose to work from customized sessions, or in guest sessions. For this press the appropriated button.

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Select the protocol category to press the appropriated touch.

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Select the genre of your patient/customer and the body area (body or face). This allows to filter the types of care and view the protocols.

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Select the care category by pressing on the appropriated touch.

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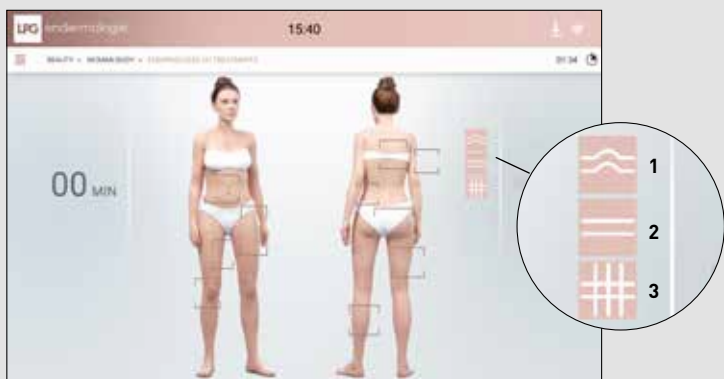
Select the areas you want to target by clicking on it.

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Select the type of skin corresponding to the selected area.

1. Tissu souple / soft
2. Tissu dense / dense
3. Tissu fibreux / fibrous

You can adjust the time corresponding to the area after its selection.

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You can add or remove areas by selecting them again.

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Your personalized care Endermologie ID is ready. Press the arrow to start the session.

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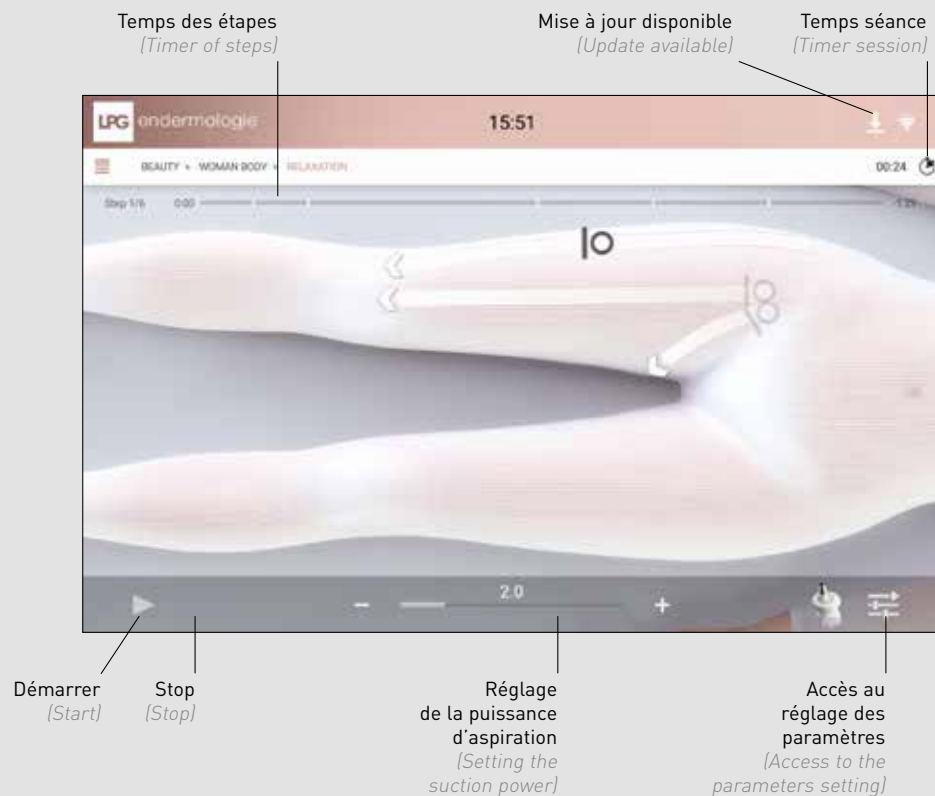
During treatment, the following screen appears, it shows the various information and settings related to treatment.

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The status of all subunits is monitored once the application is started. There are several possible default or error levels:

- «Critical» level (danger for the user)
- «Warning» level (no danger for the user)

These two error levels are indicated in the toolbar as follows:

BUTTON	ACTION	DESCRIPTION
	Error display	Critical error level
	Error display	Warning error level

These two error levels behave differently.

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Once a Critical level error appears, an error popup appears on the screen:

Critical Critical error popup

This popup lists all of the detected errors (regardless of whether it is a Critical or Warning error level). All subunits are set to «Safe» mode

- The pumps and suction are stopped
- Treatment head rollers are stopped
- The fan operates as minimum speed

Passing through this message closes the popup, but the toolbar icon remains, even if the error no longer exists. All Critical level errors are kept. Similarly, the pumps cannot be started (except in maintenance mode) after a Critical level error has appeared. A machine startup resets the system and clears all errors.

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Once a Warning level error appears, the Warning icon appears in the toolbar. No popups are displayed, and subunits are not placed in «Safe» mode. The error can only be viewed by clicking the Warning icon in the toolbar.

View of warning
level error

When a Warning level error has been corrected, it is removed from the list of defects.

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TECHNOPARC DE LA PLAINE

30, RUE DU DR. ABEL - CS 90035 - 26902 VALENCE CEDEX 09 - FRANCE

TEL.: +33 (0)4 75 78 69 00 - FAX: +33 (0)4 75 42 80 85

INTERNATIONAL/MARKETING

2753, ROUTE DES DOLINES - BP 243

06905 SOPHIA-ANTIPOLIS CEDEX - FRANCE

TEL.: +33 (0)4 92 38 39 00 - FAX: +33 (0)4 92 96 09 65

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SIÈGE SOCIAL LPG SYSTEMS S.A.