INNER POWER. BETTER LIFE.

SOMMAIRE

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
The screenshot shows the LPG mobile application interface for creating a client profile. The top status bar displays the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "15:58", and a Wi-Fi signal icon. Below the status bar, a navigation bar shows a hamburger menu icon and the breadcrumb "ACCUEIL > CRÉATION DE COMPTE > INFORMATIONS PERSONNELLES". The main content area is a form with the following fields: a gender selection with radio buttons for "Femme" (selected) and "Homme"; "Nom *" and "Prénom *" text input fields; a "Date de naissance *" text input field; "Adresse", "Ville", and "Code postal" text input fields; "Téléphone" and "Email" text input fields; "Taille (en cm)" and "Poids (en kg)" text input fields; a "suivi médical" text input field; and an "interventions chirurgicales" text input field. A large grey button at the bottom of the form is labeled "CRÉER CETTE FICHE CLIENT". A back arrow is visible on the left side of the form area. A placeholder for a profile picture is shown on the right side of the form.

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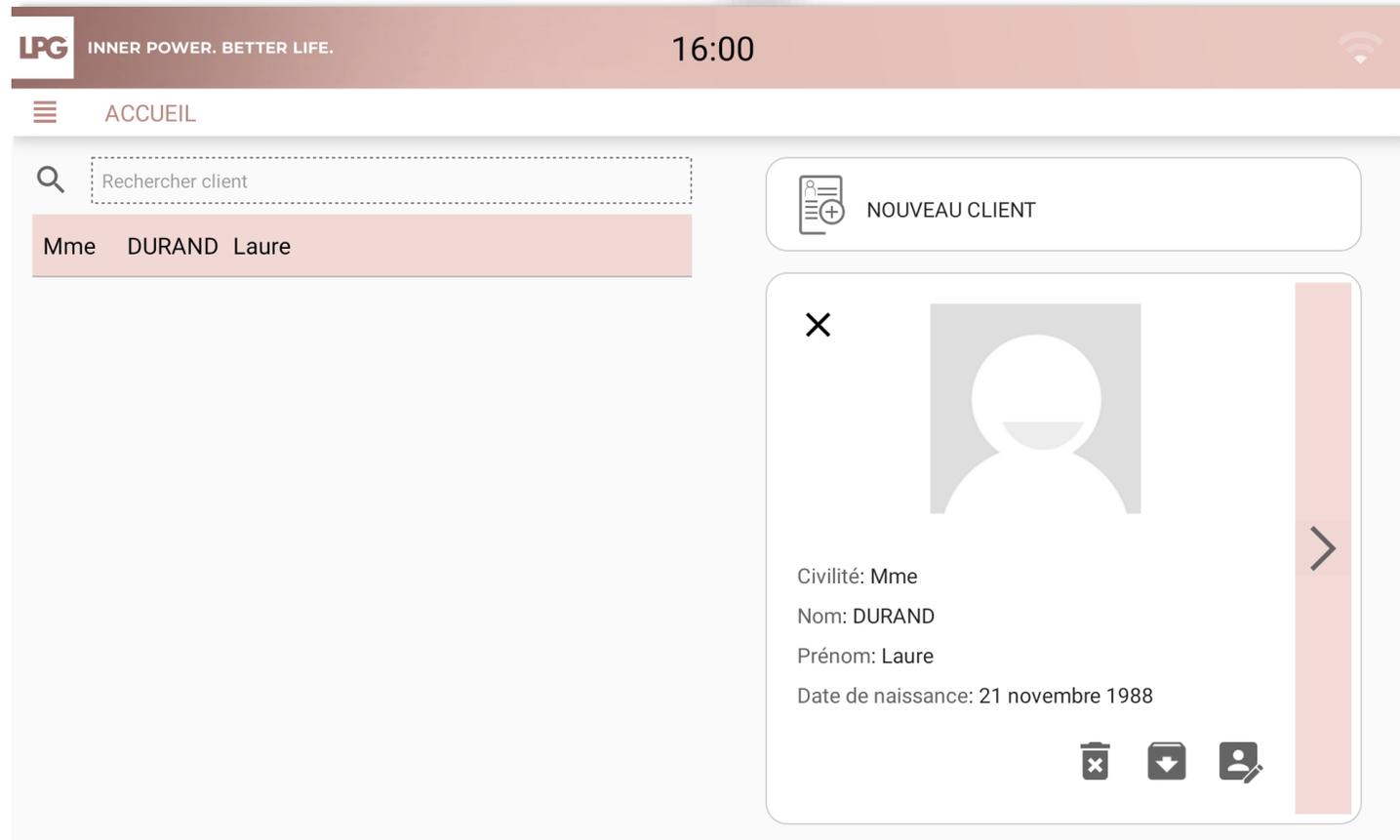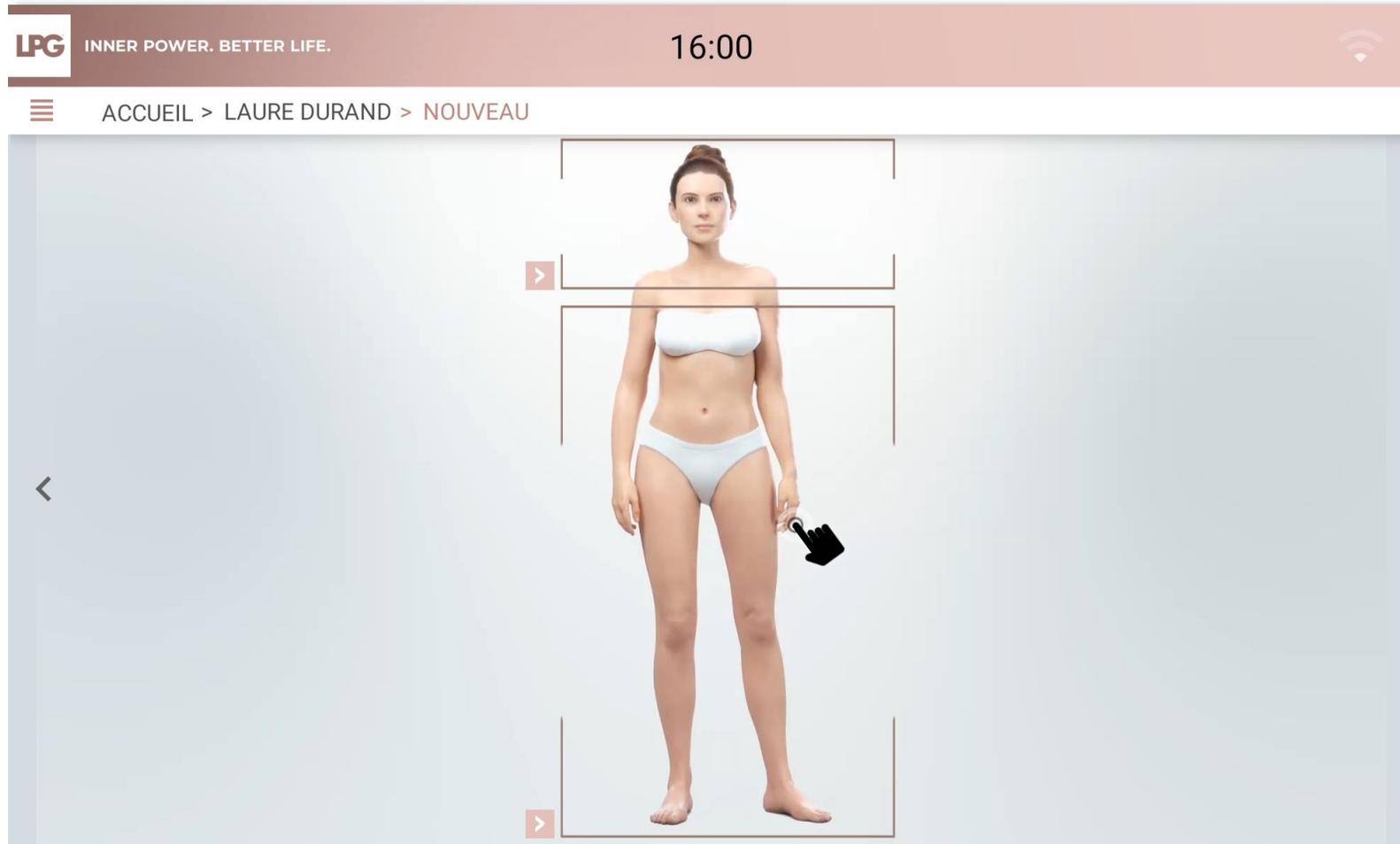
The screenshot shows a mobile application interface. At the top, there is a header with the LPG logo and the tagline 'INNER POWER. BETTER LIFE.' on the left, the time '16:00' in the center, and a Wi-Fi signal icon on the right. Below the header is a navigation bar with a hamburger menu icon on the left and the text 'ACCUEIL > LAURE DURAND > NOUVEAU'. The main content area is a slide titled 'LE TRIO GAGNANT'. The slide text reads: 'Les résultats dépendront de sa capacité à combiner les soins endermologie® à une alimentation équilibrée et à une activité physique régulière.' Below this, there are two sections: 'UNE ALIMENTATION ÉQUILIBRÉE' and 'UNE ACTIVITÉ PHYSIQUE RÉGULIÈRE', each with a list of bullet points. The slide is flanked by left and right navigation arrows.

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
The screenshot shows a mobile application interface for a body evaluation form. At the top, there is a header bar with the LPG logo and the tagline "INNER POWER. BETTER LIFE." on the left, the time "16:01" in the center, and a Wi-Fi signal icon on the right. Below the header, a navigation bar shows a hamburger menu icon followed by the text "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is titled "ÉVÉNEMENTS FÉMININS" and contains three sections: "GROSSESSE" with radio buttons for "aucune", "1", "2", "3", and "+ de 3"; "CONTRACEPTION" with radio buttons for "non" and "oui"; and "MENOPAUSE" with radio buttons for "non" and "oui". Below these sections is a "COMMENTAIRES" section. Navigation arrows are visible on the left and right sides of the form area.

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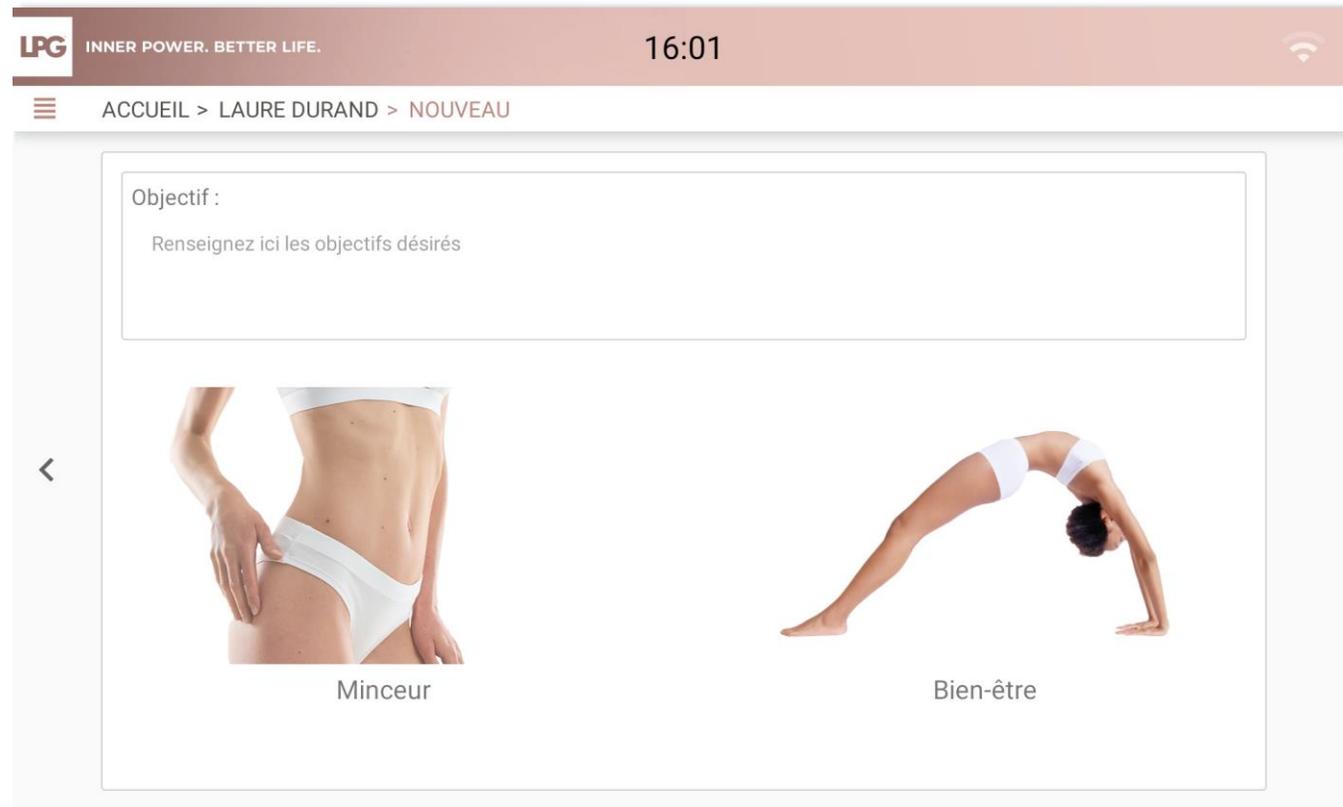The screenshot shows the LPG mobile application interface. At the top, there is a header with the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "16:01", and a Wi-Fi icon. Below the header, a navigation bar shows "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area features a text input field labeled "Objectif :" with the placeholder text "Renseignez ici les objectifs désirés". Below this field, there are two selectable options: "Minceur" (represented by an image of a woman's waist) and "Bien-être" (represented by an image of a woman performing a backbend yoga pose).

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
The screenshot shows a mobile application interface with a dark red header. On the left is the LPG logo and the text "INNER POWER. BETTER LIFE.". On the right is the time "16:03" and a Wi-Fi icon. Below the header is a navigation bar with a hamburger menu icon and the text "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is a large white box with a thin border. At the top left of this box is the label "Objectif :". Below it is a large, empty text input field with the placeholder text "Renseignez ici les objectifs désirés". On the left and right sides of the bottom of this box are navigation arrows: a left-pointing chevron on the left and a right-pointing chevron on the right.

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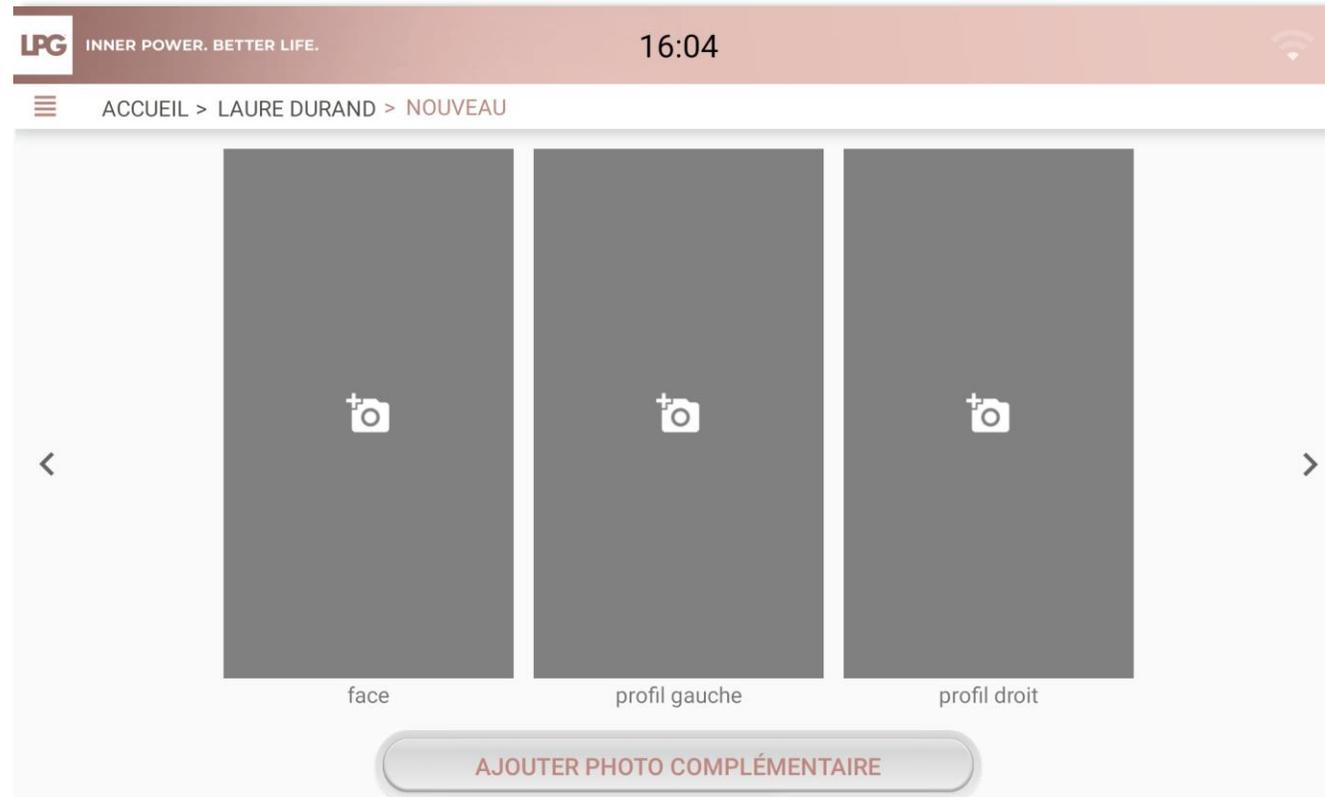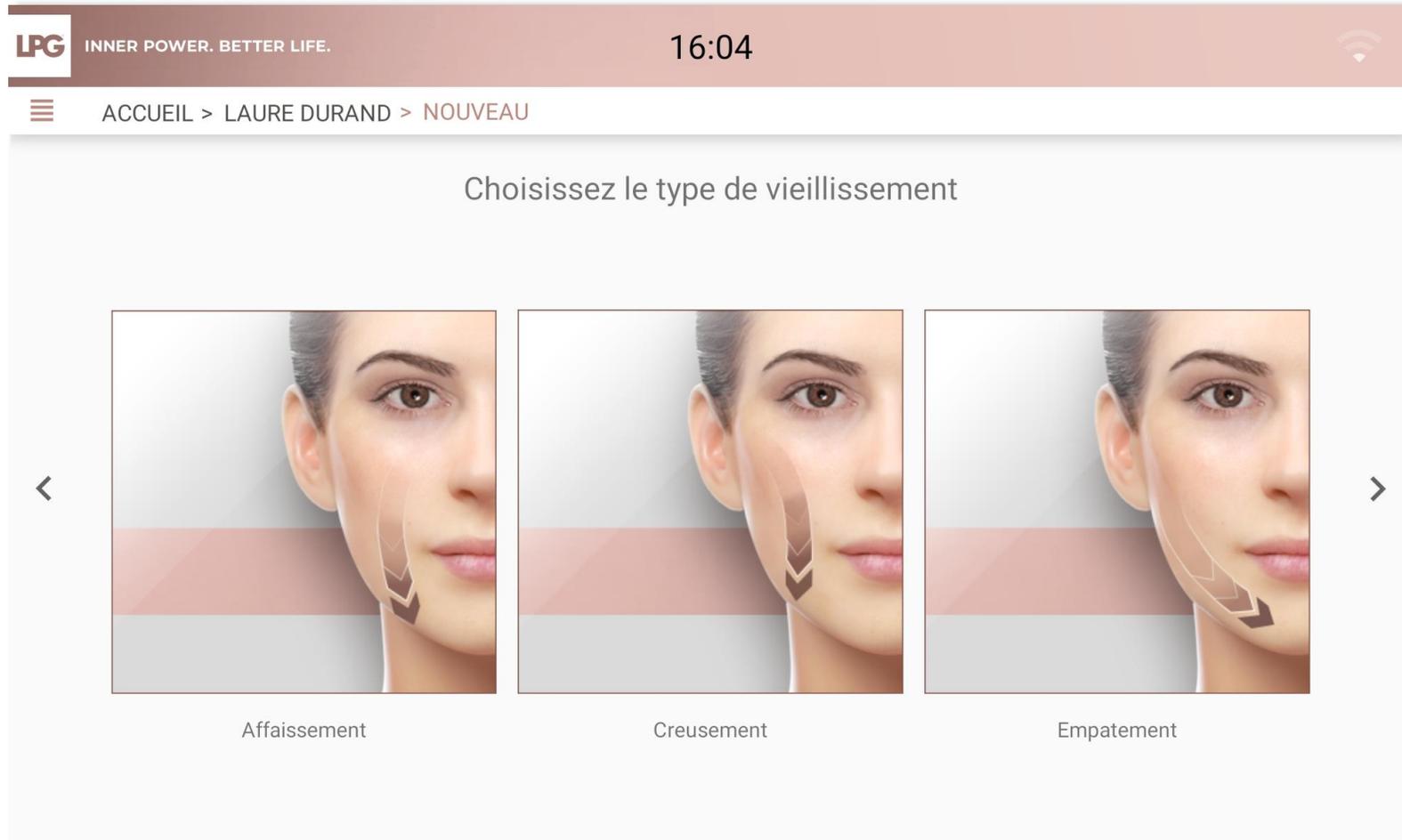
The screenshot shows a mobile application interface for skin aging evaluation. At the top, there is a header with the LPG logo and the tagline "INNER POWER. BETTER LIFE." on the left, the time "16:04" in the center, and a Wi-Fi icon on the right. Below the header is a navigation bar with a menu icon and the text "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is titled "Choisissez le type de vieillissement" and features three side-by-side images of a woman's face, each illustrating a different type of skin aging: "Affaissement" (sagging), "Creusement" (wrinkling), and "Empatement" (swelling). Each image has a red horizontal bar at the bottom and a dark arrow pointing downwards on the right side of the face. Navigation arrows are visible on the left and right sides of the image row.

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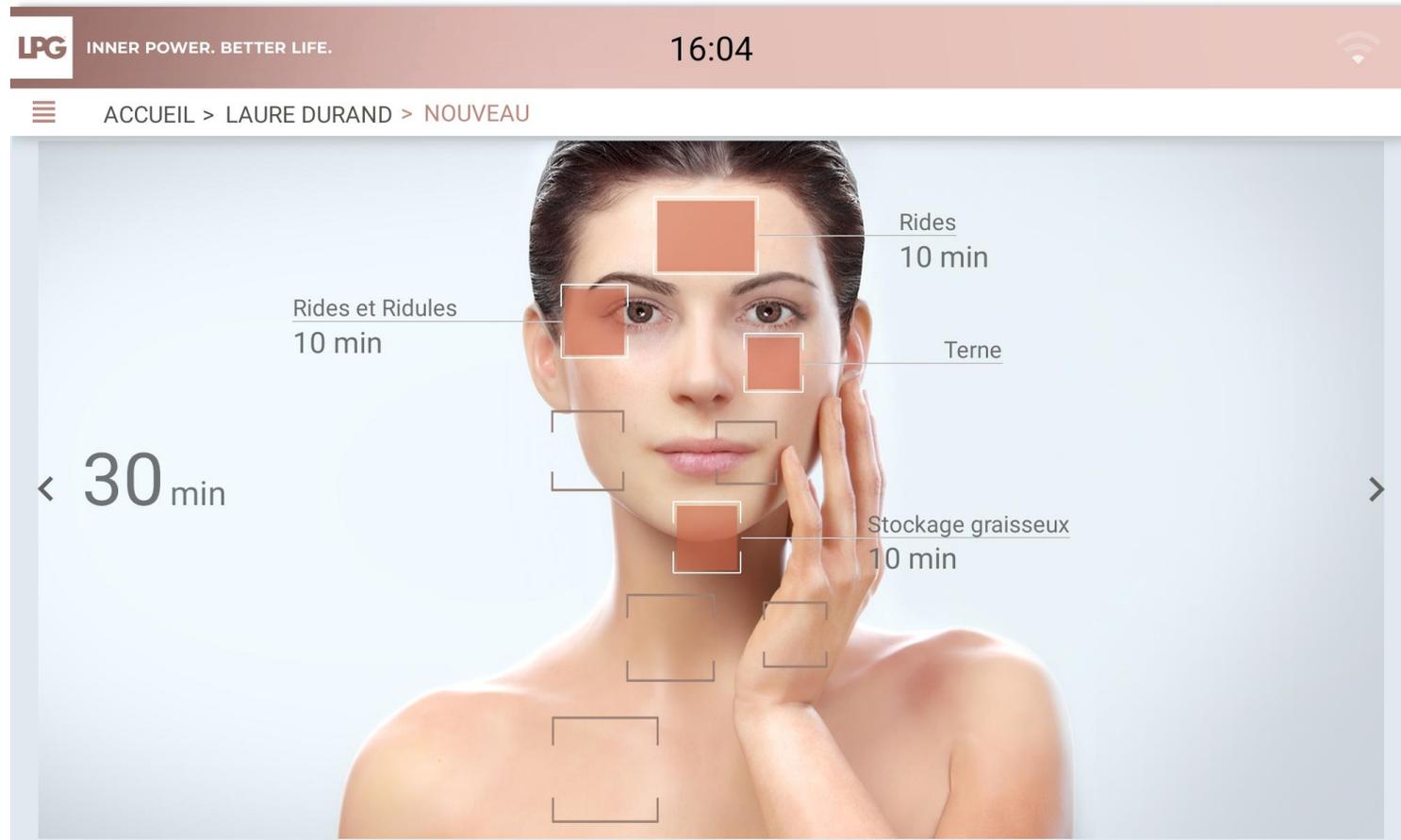
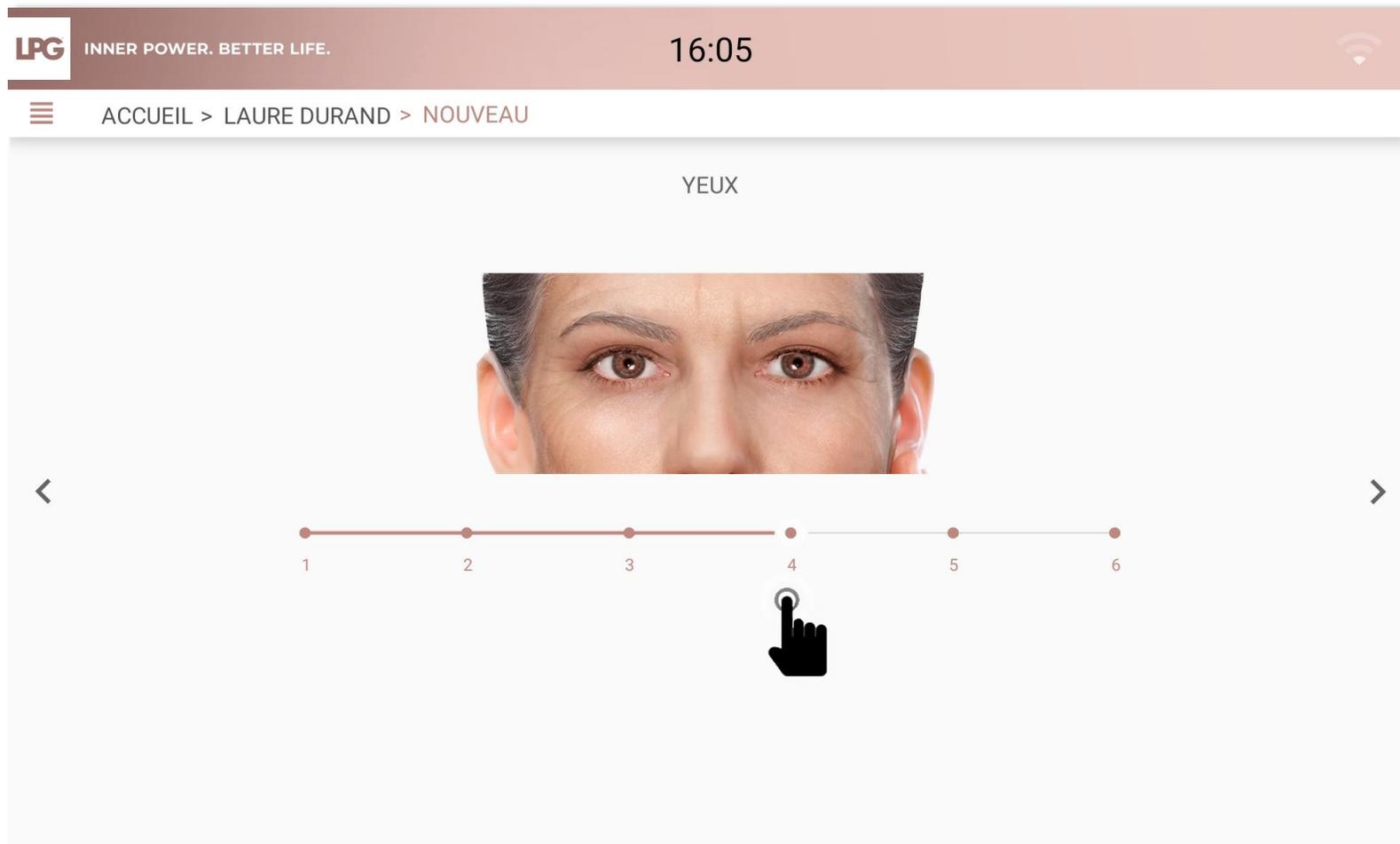
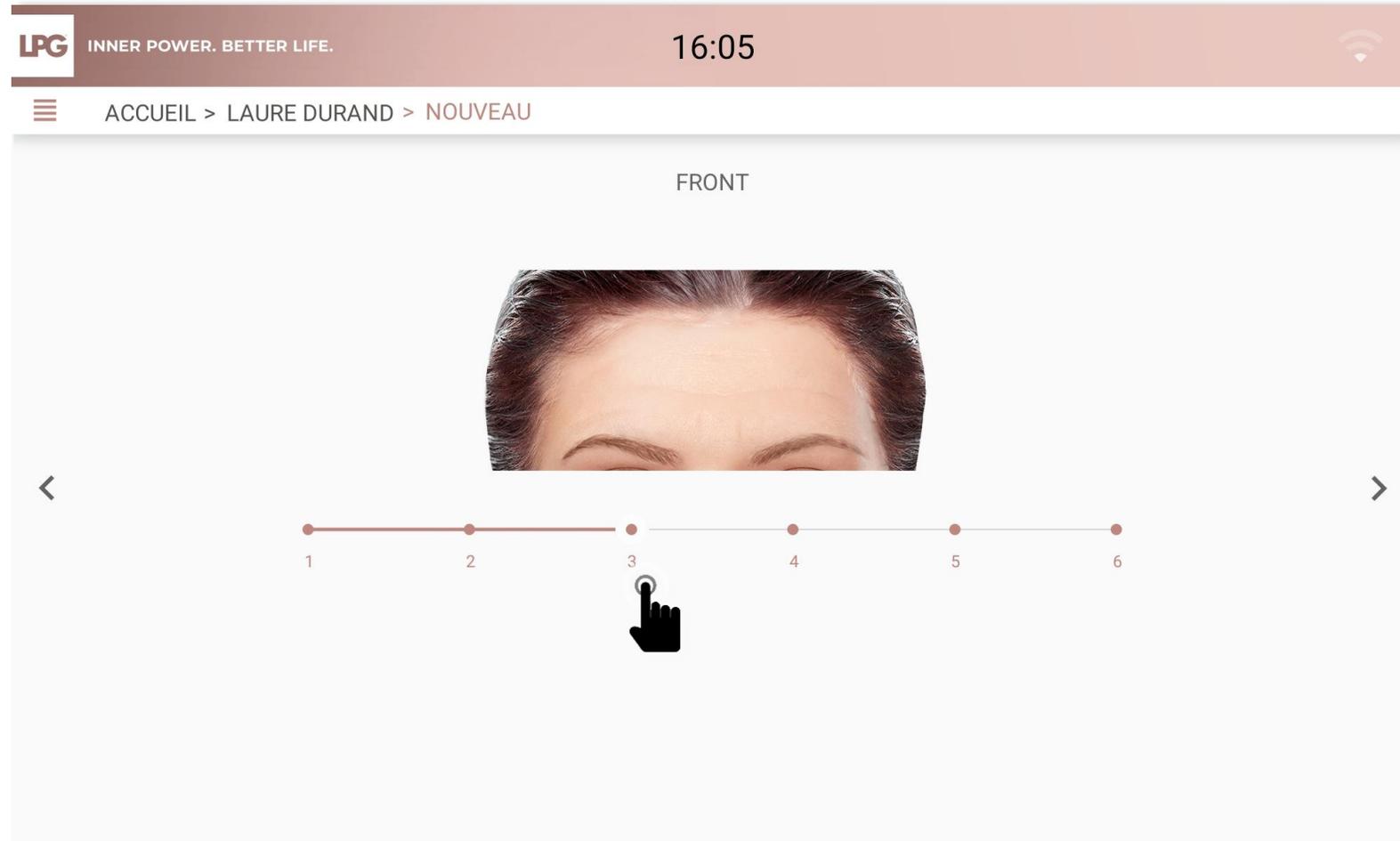
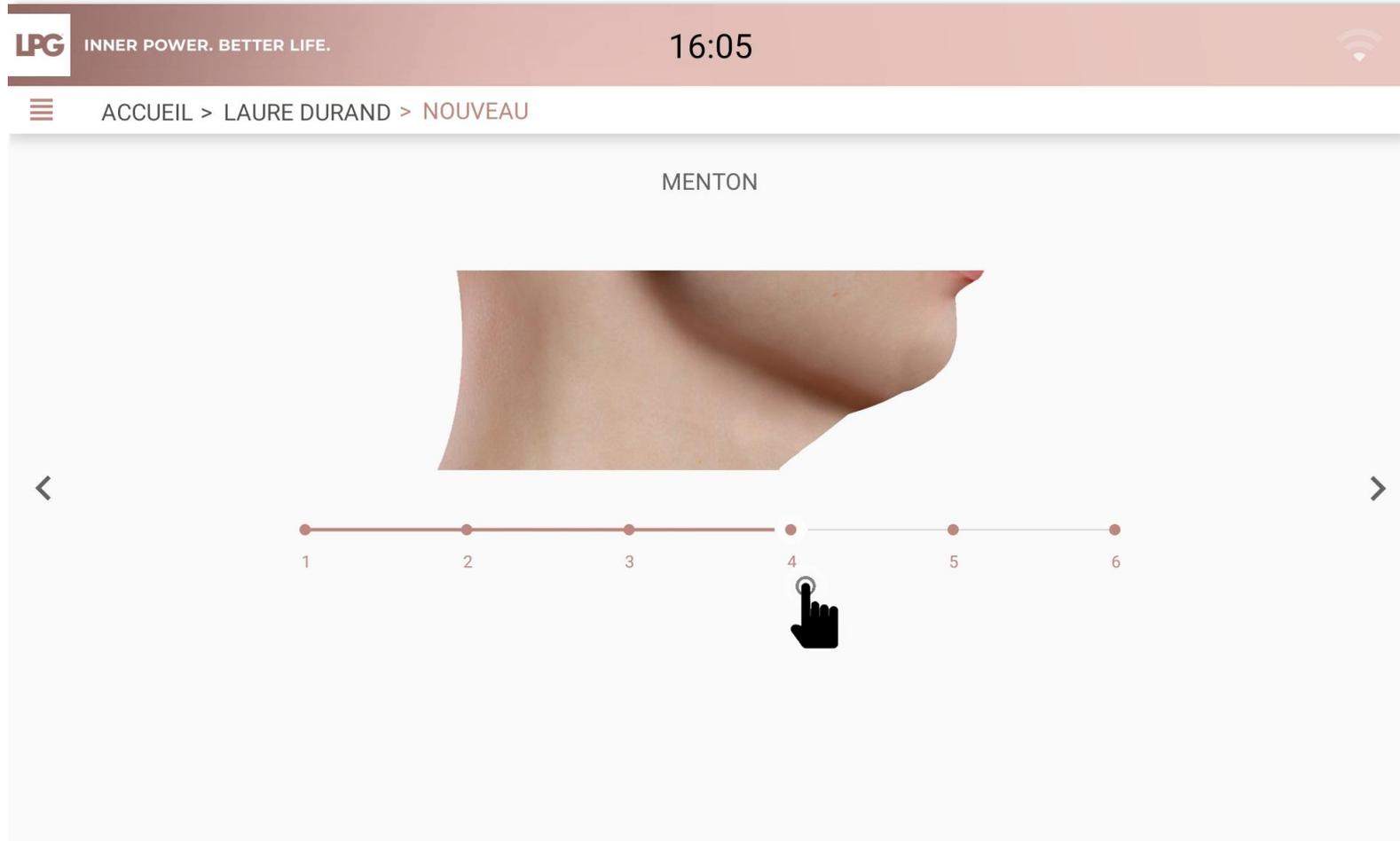
The screenshot shows the LPG mobile application interface. At the top, the LPG logo and tagline 'INNER POWER. BETTER LIFE.' are displayed, along with the time '16:07'. Below the navigation bar, the breadcrumb trail reads 'ACCUEIL > LAURE DURAND > NOUVEAU'. The main content area is split into two columns: 'MON PROGRAMME' and 'MA ROUTINE BEAUTÉ'. The 'MON PROGRAMME' section lists three treatment types with their durations and session counts:

Soin(s) zone	Regard Anti-rides	Zone Frontale	Double-menton
10 min	10 min	10 min	10 min

Number of sessions: 6 (with minus and plus buttons)

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Number of sessions: 8 (with minus and plus buttons)

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The 'MA ROUTINE BEAUTÉ' section lists several skincare products, each with an information icon (i):

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A 'VALIDER' button is located at the bottom center of the screen, with a hand icon pointing to it.

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The screenshot shows a mobile application interface for LPG skincare. At the top, there is a header with the LPG logo and the slogan "INNER POWER. BETTER LIFE." on the left, the time "16:07" in the center, and a Wi-Fi signal icon on the right. Below the header, a navigation bar shows "ACCUEIL > LAURE DURAND" with a menu icon on the left and a vertical ellipsis on the right. The main content is divided into two sections: "VISAGE" and "CORPS", both dated "10 JUIN 2024".

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The "VISAGE" section includes two photos of a woman's face (frontal and profile), a calendar icon, a camera icon, and a collection of LPG skincare products. Below the photos are icons for sharing, editing, and deleting.

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The "CORPS" section includes two photos of a woman's body (front and back), a calendar icon, a camera icon, and a collection of LPG body care products. Below the photos are icons for sharing, editing, and deleting.

At the bottom of the application, there is a large grey button with a white plus sign (+).

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The screenshot displays the LPG mobile application interface for a client named Laure Durand. At the top, the LPG logo and tagline 'INNER POWER. BETTER LIFE.' are visible, along with the time 16:07. Below the header, the navigation bar shows 'ACCUEIL > LAURE DURAND'. The main content area is divided into two sections: 'VISAGE' and 'CORPS', both dated '10 JUIN 2024'. The 'VISAGE' section lists treatments: 'Regard Anti-rides - Zone Frontale - Double-menton', 'Éclat', and 'Rénovateur Anti-âge'. It includes two photos of the client's face (front and profile), a calendar icon, a camera icon, and a product image of skincare items. The 'CORPS' section lists treatments: 'Ventre - Fesses - Intérieur cuisses' and 'Detox'. It includes two photos of the client's body (front and side), a calendar icon, a camera icon, and a product image of body care items. At the bottom of the screen, there is a grey bar with a white plus sign icon, indicating the option to create a new program.

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The screenshot shows the LPG mobile application interface. At the top, the LPG logo and slogan 'INNER POWER. BETTER LIFE.' are visible, along with the time '16:07'. Below this is a navigation bar with 'ACCUEIL > LAURE DURAND > MES SÉANCES'. The main content area displays a session titled 'REGARD ANTI-RIDES - ZONE FRONTALE - DOUBLE-MENTON' with a progress indicator '0/6 30 min' and an add icon. The session is broken down into four rows, each representing a step:

Step	Activity	Duration	Actions
1/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min	Calendar, Comments, Trash
2/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min	Calendar, Comments, Trash
3/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min	Calendar, Comments, Trash
4/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min	Calendar, Comments, Trash

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The screenshot shows a mobile application interface with a dark red header. On the left is the LPG logo and the text "INNER POWER. BETTER LIFE.". In the center is the time "16:09". On the right is a Wi-Fi signal icon. Below the header is a navigation bar with a hamburger menu icon on the left and the text "ACCUEIL > LAURE DURAND > BILAN PHOTOS". The main content area has a light gray background and contains two input fields. The first is a text box labeled "Objectif :". The second is a larger text area labeled "Ajouter votre commentaire ici" with left and right arrow navigation icons. The bottom of the screen shows a dark red bar with the LPG logo and the text "INNER POWER. BETTER LIFE.".

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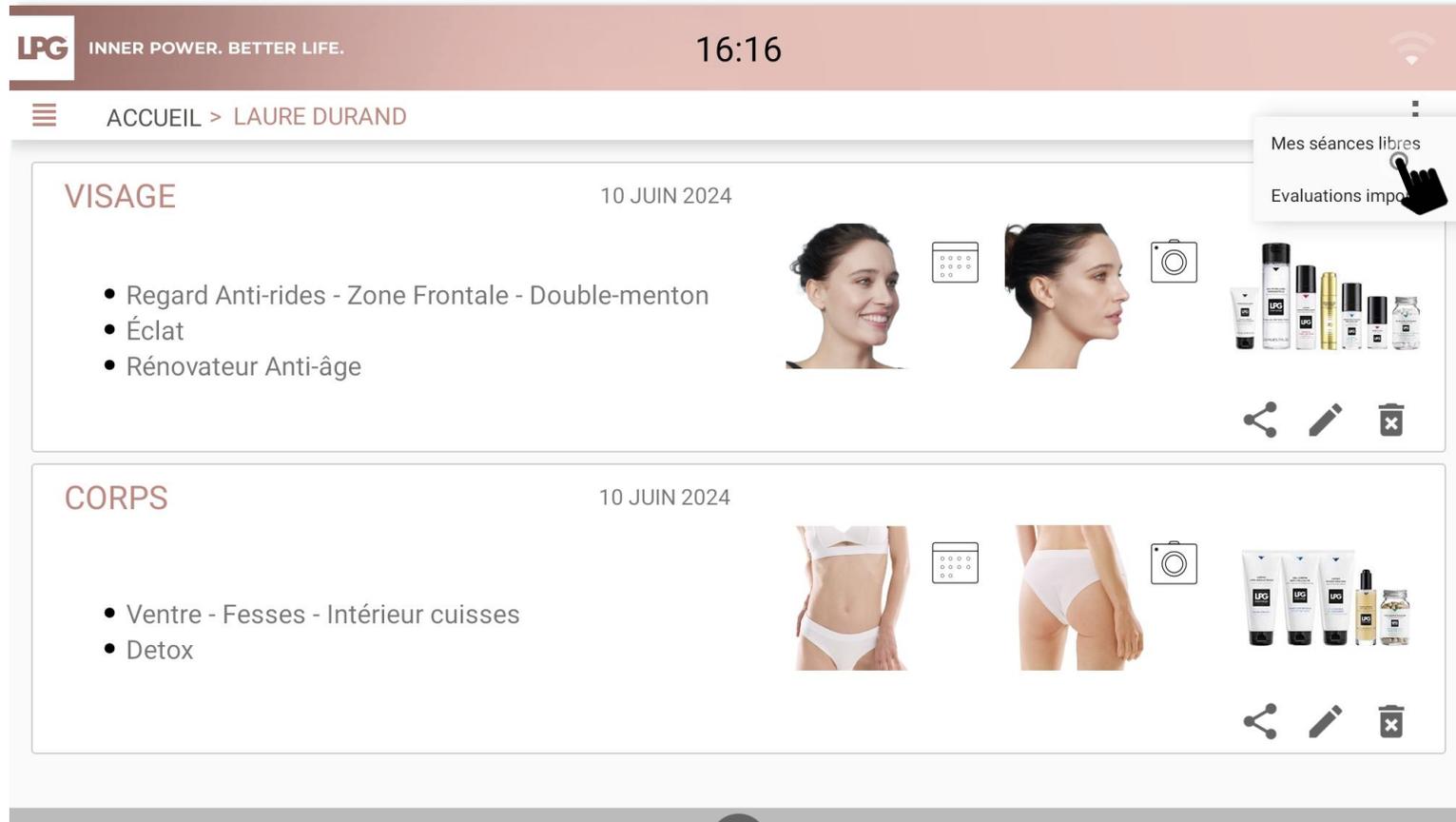
The screenshot displays the LPG mobile application interface. At the top, the LPG logo and tagline "INNER POWER. BETTER LIFE." are visible on the left, and the time "16:16" is centered. Below the header, a navigation bar shows "ACCUEIL > LAURE DURAND". A dropdown menu is open, highlighting "Mes séances libres" with a hand cursor icon. The main content area is divided into two sections:

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Each session card includes a front and side profile photo of the client, a calendar icon, a camera icon, and a collection of LPG skincare products. At the bottom of each card are icons for sharing, editing, and deleting.

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