



INNER POWER. BETTER LIFE.

SUMMARY

○ Start the application	p. 3
○ Create a client account	p. 4
○ Body evaluations and treatment programs	p. 9
○ Face evaluations and treatment programs	p. 19
○ Photo assessment	p. 36
○ Compare photo assessments	p. 41
○ Send treatment program to your client	p. 44
○ Send treatment program to your CELLU M6 ALLIANCE®	p. 47
○ Free sessions	p. 52
○ Backup	p. 54
○ Updates	p. 57
○ Data import (LPG SCAN tablet to the new one)	P. 61
○ Data export	P. 63



INNER POWER.
BETTER LIFE.

START THE APPLICATION

Select the **endermoscan™** icon on your tablet's home screen to launch the application.






CREATE A CLIENT
ACCOUNT



INNER POWER.
BETTER LIFE.

CREATE CLIENT ACCOUNT

Once the application is open, select NEW CLIENT on the home screen.

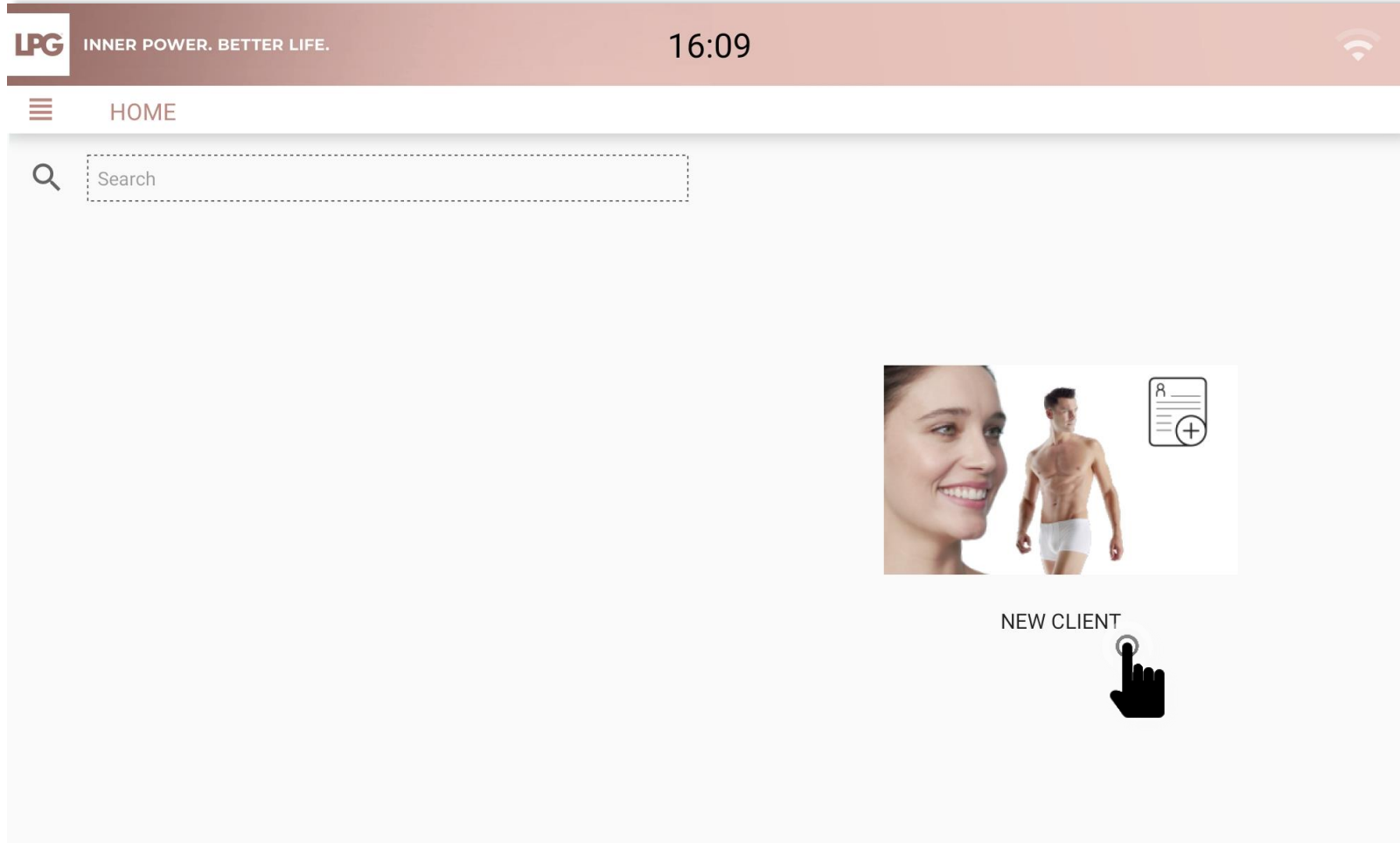

CREATE CLIENT ACCOUNT

After discussing all contraindications with your client, please tick the «Contraindications for endermologie® treatments were discussed during your evaluation» box to start filling in the client's profile.

LPGINNER POWER. BETTER LIFE.

16:09

HOME > CREATE AN ACCOUNT > CONTRAINDICATIONS

INFORMATION ON ENDERMOLOGIE® TREATMENTS

Strict general contraindications:

- Progressing cancer (except with written medical recommendation)
- Anticoagulant treatment
- Blood disease
- Disease with inflammatory eruption
- Infection or skin rash

Body endermologie® contraindications:

- Pregnancy (except for lower limbs)
- Phlebitis
- Organ transplant
- Kidney failure

Face endermologie® contraindications:

- Vitiligo
- Infectious, inflammatory acne
- Herpes

If in doubt, ask for a written medical opinion.

☐ Contraindications for endermologie® treatments were discussed during your evaluation.

Localized Contraindications:

- Rosacea
- Varicosity and varicose veins
- Vein surgery (for 2 months)
- Areas that underwent plastic surgery (for 6 months)
- Skin lesions and wounds
- Keloid, hypertrophic, or recent scars
- Inguinal or abdominal hernias
- Lipoma
- Raised moles
- Filler injections (for 2 weeks)
- Botox® injections (for 2 weeks)
- Piercing
- Angioma
- Subcutaneous implants
- Gold and serrated thread lifts
- Acute osteo-articular or muscle injury

CREATE CLIENT ACCOUNT

- Fill in client information.
- Click on CREATE THIS CLIENT PROFILE.
- THE BMI is calculated automatically after filling in the height and the weight.

LPG INNER POWER. BETTER LIFE. 16:10

HOME > CREATE AN ACCOUNT > PERSONAL INFORMATION

☒ Woman ☐ Man

Last Name * First Name *

Date of Birth *

Address City Zip Code

Phone Email



Height (in cm) Weight (in kg)


medical follow-up

surgical procedures

CREATE THIS CLIENT PROFILE

CREATE CLIENT ACCOUNT

- To view or to edit a client profile, select your client's name and click on 
- To search for a client in the existing database, type the client's name in the SEARCH bar.
- To archive a client profile, click on 

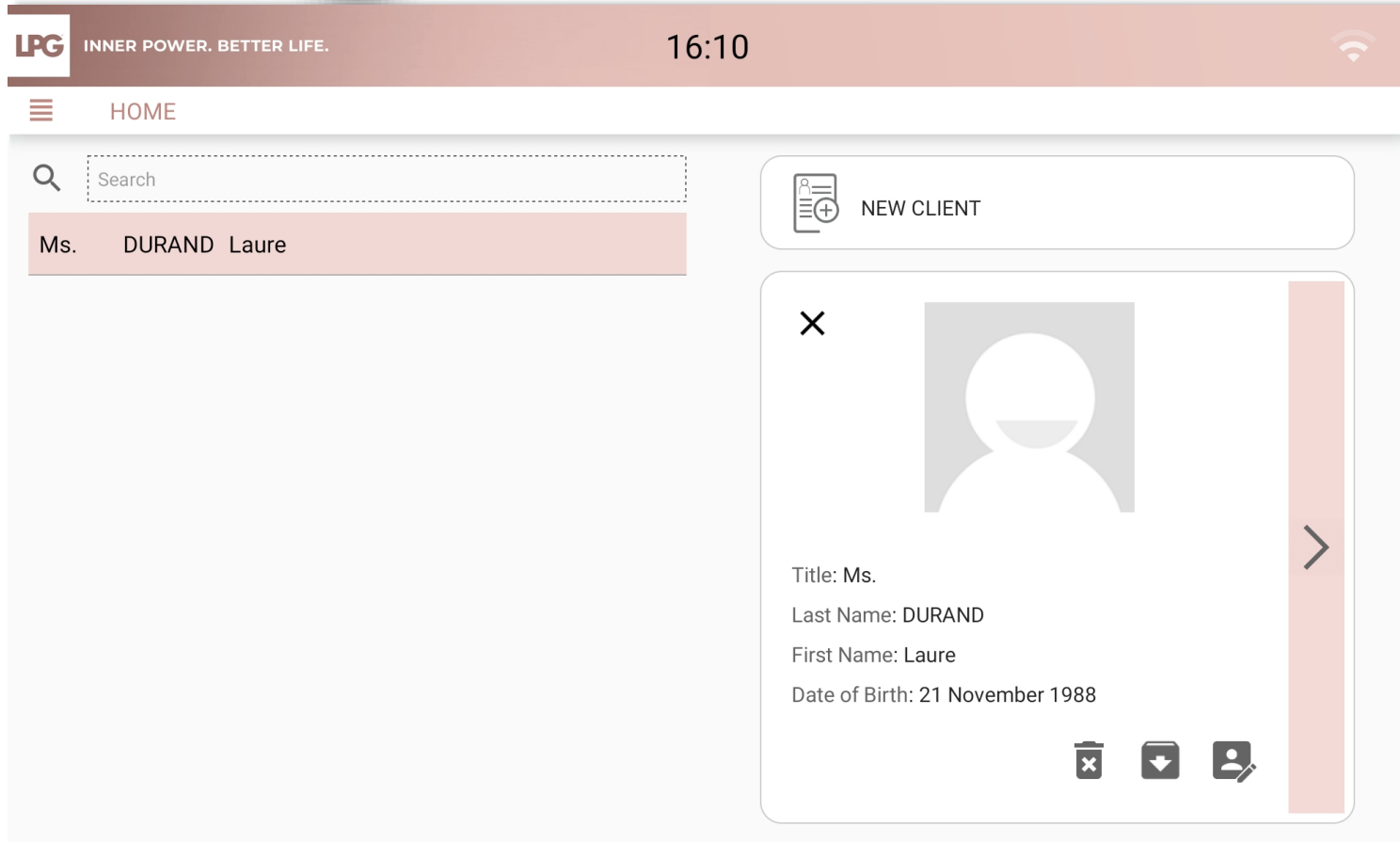

The screenshot displays the LPG client management interface. At the top, the header includes the LPG logo, the tagline "INNER POWER. BETTER LIFE.", the time "16:10", and a Wi-Fi icon. Below the header, a navigation bar shows a hamburger menu icon and the word "HOME". The main content area is divided into two sections. On the left, there is a search bar with a magnifying glass icon and the placeholder text "Search". Below the search bar, a list of clients is shown, with the first entry highlighted in a light red box: "Ms. DURAND Laure". On the right, there is a "NEW CLIENT" button with a plus icon. Below this, a detailed view of a client profile is shown. The profile includes a placeholder for a profile picture, a close button (X), and the following information: "Title: Ms.", "Last Name: DURAND", "First Name: Laure", and "Date of Birth: 21 November 1988". At the bottom of the profile view, there are three icons: a trash can (delete), a download arrow (archive), and an edit person icon.




BODY EVALUATION & TREATMENT PROGRAMS



INNER POWER.
BETTER LIFE.

BODY EVALUATION

Once the client account has been created, choose to carry out a **body evaluation** by selecting the concerned area.

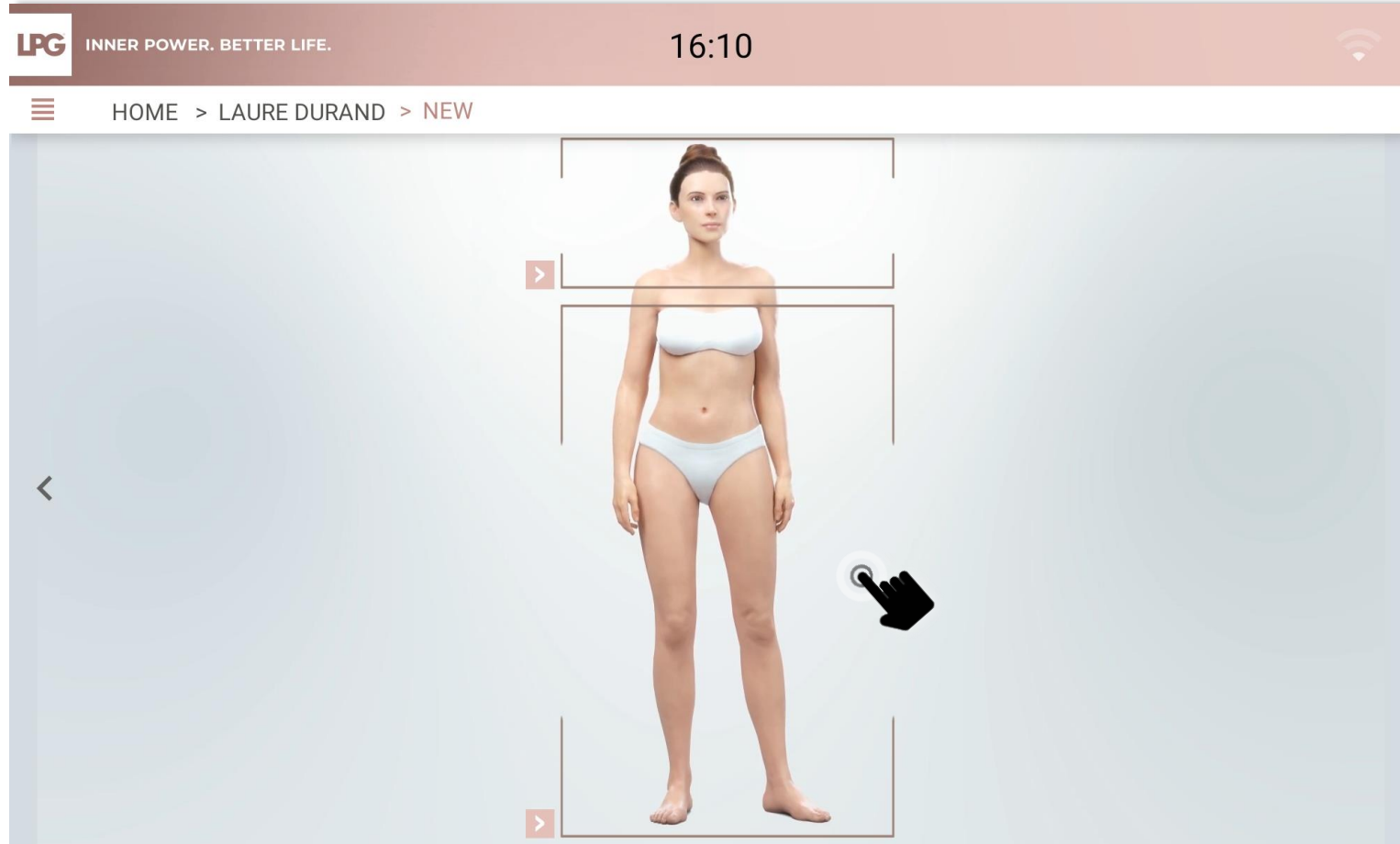

INNER POWER.
BETTER LIFE.

BODY EVALUATION

It is important to inform your client of the different endermologie® treatment programs success levers.

The screenshot shows a mobile application interface. At the top, there is a header bar with the LPG logo and the tagline 'INNER POWER. BETTER LIFE.' on the left, the time '16:11' in the center, and a Wi-Fi icon on the right. Below the header is a navigation bar with a hamburger menu icon, the text 'HOME > LAURE DURAND > NEW', and a right arrow. The main content area is titled 'THE WINNING TRIO'. It contains a paragraph stating that results depend on the client's ability to combine endermologie® treatments with a balanced diet and regular exercise. Below this, there are two sections: 'A BALANCED DIET' and 'REGULAR EXERCISE', each with a list of recommendations. The interface also features left and right navigation arrows on the sides of the content area.

LPG INNER POWER. BETTER LIFE. 16:11

HOME > LAURE DURAND > NEW

THE WINNING TRIO

Results will depend on her/his ability to combine endermologie® treatments with a balanced diet and regular exercise.

A BALANCED DIET

- 5 fruits and vegetables per day; proteins, carbohydrates, and various foods at each meal; limit sweet, salty, and fatty foods.
- It is recommended that you drink regularly to keep your body hydrated (ideally between meals).

REGULAR EXERCISE

- The equivalent of walking at least 30 minutes per day is recommended.
- For fat-elimination results, it is recommended that you exercise after your body endermologie® session.

BODY EVALUATION

It is important to inform your client on the different endermologie® treatment programs success levers.
(i.e.: Results may vary depending on to the hormonal stability)

LPG

INNER POWER. BETTER LIFE.

16:11

HOME > LAURE DURAND > NEW

FEMALE EVENTS

PREGNANCY

☒ none ☐ 1 ☐ 2 ☐ 3 ☐ 3+

CONTRACEPTION

☒ no ☐ yes

MENOPAUSE

☒ no ☐ yes

COMMENTS

<

>

BODY EVALUATION

- Fill in client's desired objectives.
- Select the type of treatment desired according to your client's objectives :
 - Slimming (*endermologie® ID treatments*)
 - Well-being treatments (*these do not require evaluation*)

LPG

INNER POWER. BETTER LIFE.


16:11

HOME > LAURE DURAND > NEW


Objective:

Fill in desired objectives here

<




Slimming



Well-being

BODY EVALUATION


- Take 4 photos (front, right profile, left profile, back). Maintain you tablet in vertical (portrait) format. The photo must be taken when level sensors are green. Report to the integrated markers of the photo assistant to position your client properly. Once the picture is taken, click on **REGISTER** or **CANCEL** if you are not satisfied.
- To take other pictures, click on **ADD ADDITIONAL PICTURE** and choose a title.

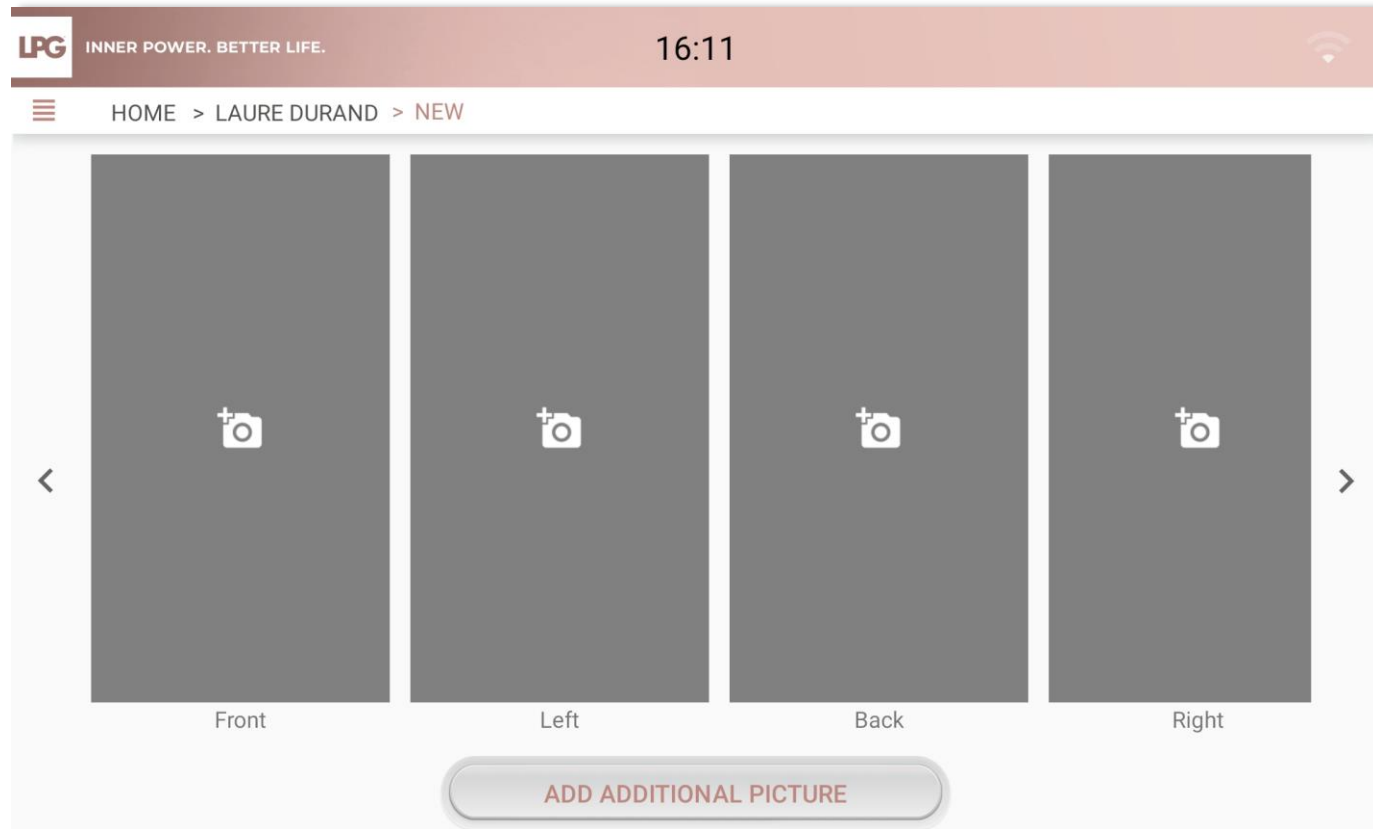


INNER POWER.
BETTER LIFE.


BODY EVALUATION

Fill it in if your client is concerned with this issue to determine the endermologie® cosmetics best adapted to their needs.

 INNER POWER. BETTER LIFE.

16:11



 HOME > LAURE DURAND > NEW

Water retention questionnaire

1 - Where is your cellulite?

☐ diffuse

☐ in one area

2 - Can your weight vary 1 kg to 3 kg within a 48-hour period?

☐ yes

☐ no

3 - How do your legs feel?

☐ Only heavy and painful at the end of the day

☐ Already swollen in the morning and worsening at the end of the day

☐ Only swollen during my premenstrual period

☐ No problem with my legs

4 - Describe your ankles.

☐ They are swollen after prolonged sitting (for example, a flight)

☐ They are always swollen with edema around the ankle bones

☐ Nothing in particular regarding my ankles



5 - Do you have any right now?

☐ Swollen hands and feet?

☐ Tree-trunk legs?

☐ Noticed a varying increase in volume from one week to the next?

☐ A cortisone-based medical treatment?





INNER POWER.
BETTER LIFE.

BODY EVALUATION

- Select the client's targeted areas.
- Select the type of tissue (supple, dense, fibrous) corresponding to each selected area.
- Adjust treatment time corresponding to each area.



INNER POWER.
BETTER LIFE.

BODY EVALUATION

Recommended endermologie® protocols

Endermologie® ID treatment(s) previously selected are automatically checked.
According to your client's objectives, you can add a well-being treatment.

LPG

INNER POWER. BETTER LIFE.

16:12

HOME > LAURE DURAND > NEW

Recommended endermologie® protocols

endermologie® ID treatment(s)

- ☒ Stomach
- ☒ Buttocks
- ☒ Saddlebags

Specific treatment(s)

- ☐ Young mum

Well-being treatment(s)

- ☐ Relaxation
- ☐ Fluid Mobilization
- ☐ Endermopuncture
- ☐ Vitality - Stress - Sleep





INNER POWER.
BETTER LIFE.

BODY EVALUATION

Final evaluation and protocol, cosmetics & endermonutri recommendation

- Adjust the number of recommended sessions.
- To confirm the evaluation and to create the customized program, select **SUBMIT**.
- Your client's program is then saved.

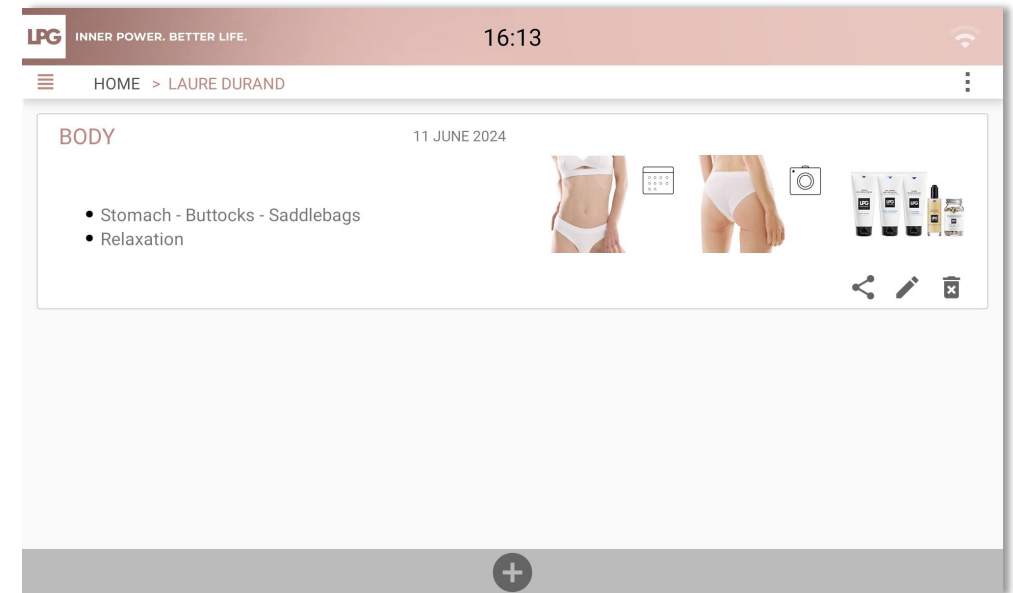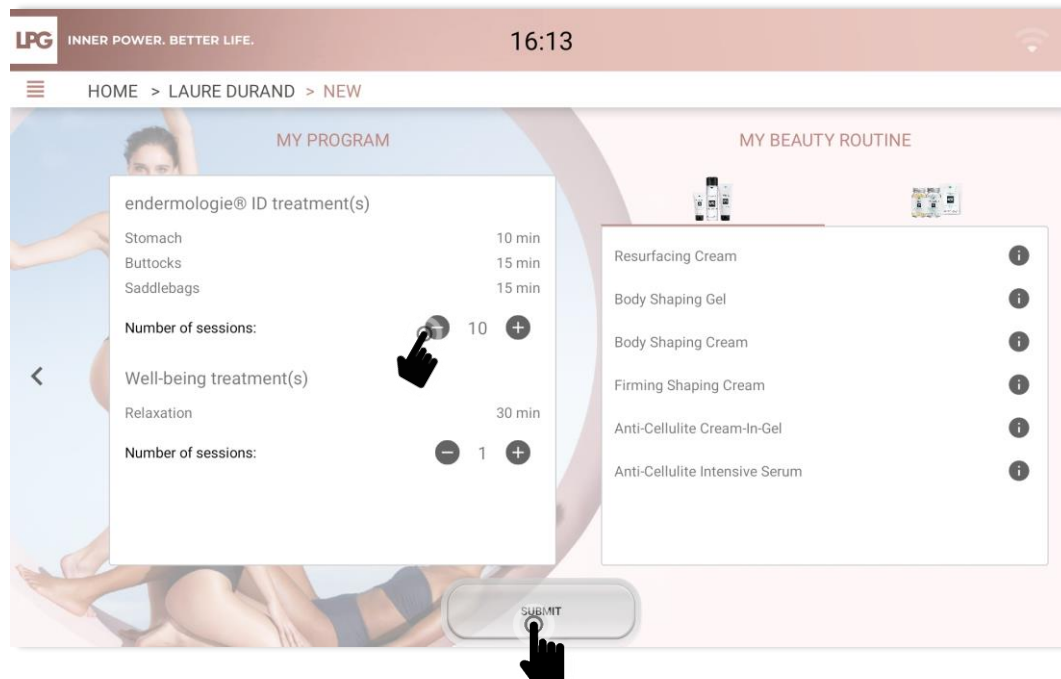




FACE EVALUATION & TREATMENT PROGRAMS



INNER POWER.
BETTER LIFE.


FACE EVALUATION

To start a new evaluation, click on

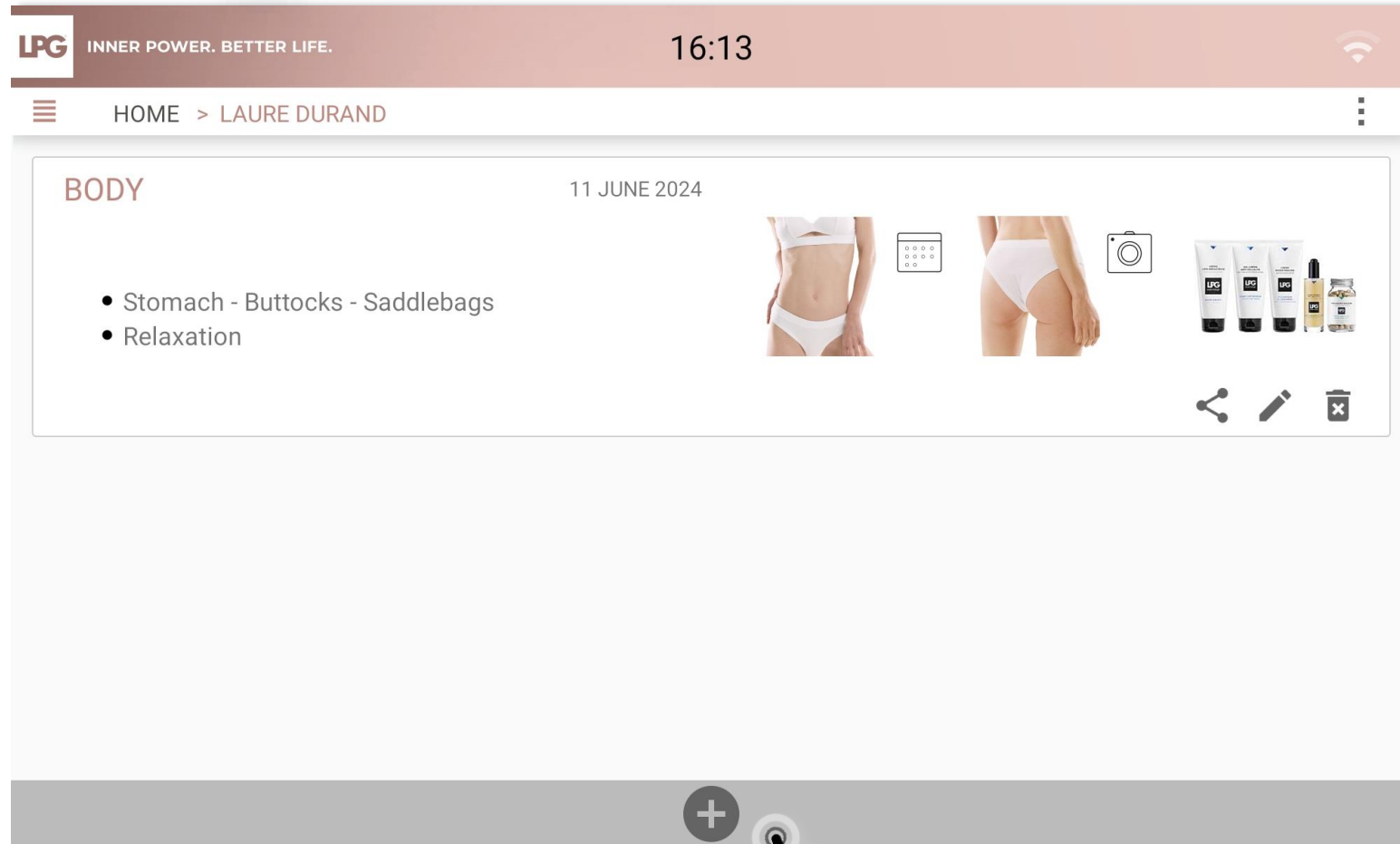

FACE EVALUATION

To start a face evaluation, click on the face area.

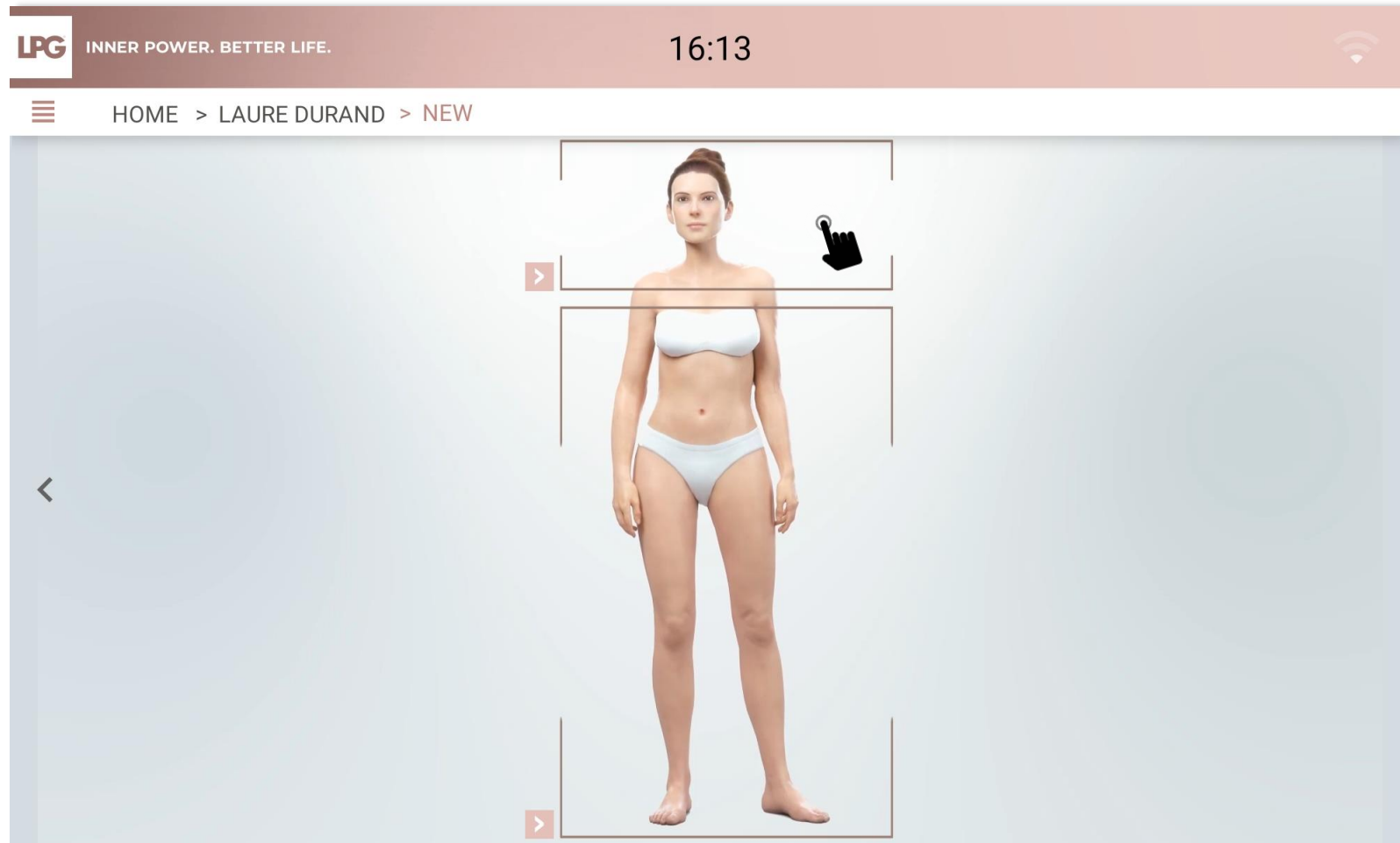

INNER POWER.
BETTER LIFE.

FACE EVALUATION

Fill in client's desired objectives.


The screenshot shows a mobile application interface. At the top, there is a dark red header bar. On the left of this bar is the 'LPG' logo in white, followed by the text 'INNER POWER. BETTER LIFE.' in white. In the center of the header bar is the time '16:13' in white. On the right is a white Wi-Fi icon. Below the header bar is a white navigation bar. On the left is a hamburger menu icon. To its right is the breadcrumb trail 'HOME > LAURE DURAND > NEW' in dark red. The main content area is a large white rectangle with a thin gray border. Inside this rectangle, at the top left, is the text 'Objective:'. Below it is a larger white area with the placeholder text 'Fill in desired objectives here' in a light gray font. On the left side of the main content area, there is a gray vertical bar with a white left-pointing chevron '<'. On the right side, there is a gray vertical bar with a white right-pointing chevron '>'.



INNER POWER.
BETTER LIFE.


FACE EVALUATION

- Take 3 photos (right profile, face, left profile). Maintain you tablet in vertical (portrait) format. The photo must be taken when level sensors are green. Report to the integrated markers of the photo assistant to position your client properly. Once the picture is taken, click on **REGISTER** or **CANCEL** if you are not satisfied.
- To take other pictures, click on **ADD ADDITIONAL PICTURE** and choose a title.

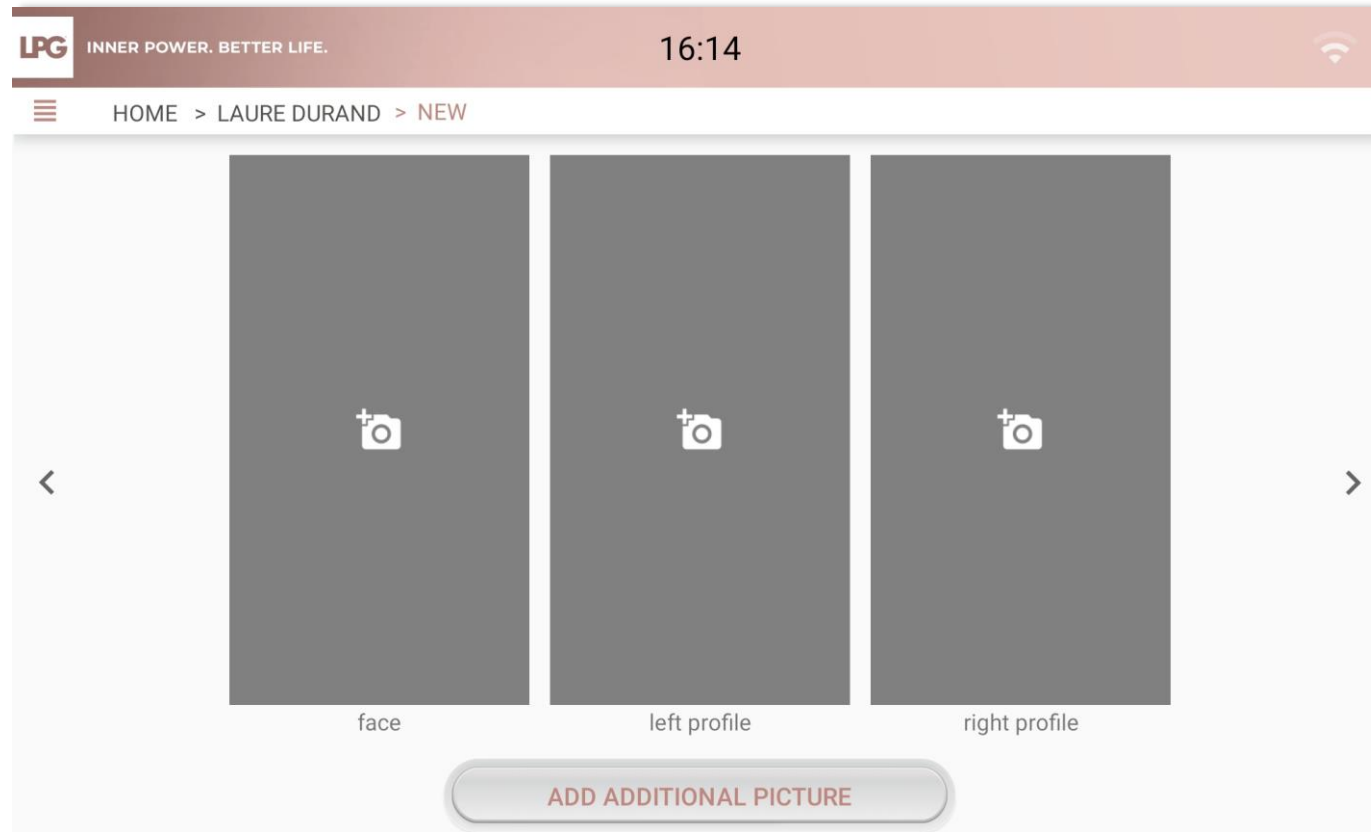

FACE EVALUATION

Select the corresponding skin aging type.

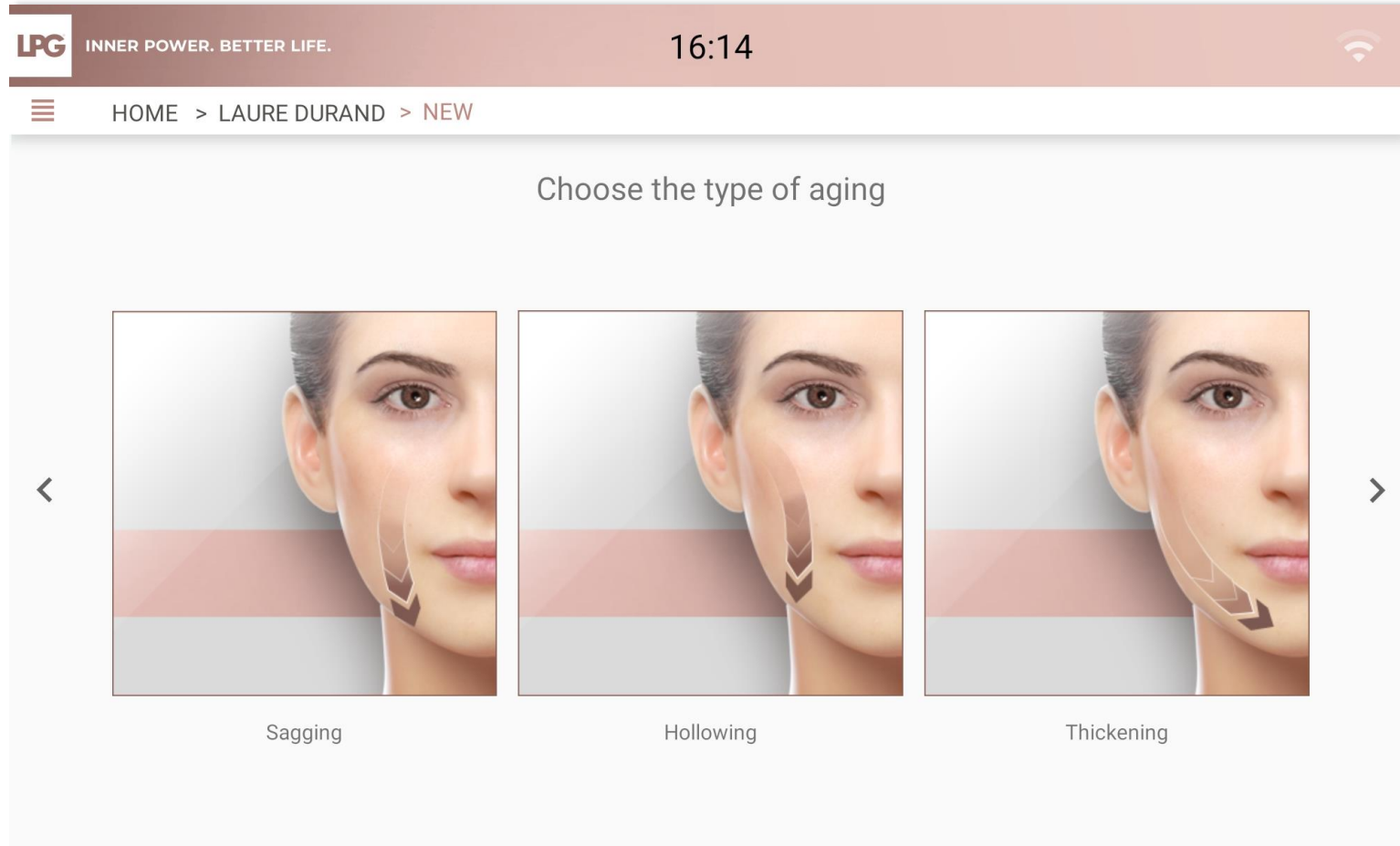

FACE EVALUATION

Select up to 4 priority areas.


For each selected area, chose the main objective.



INNER POWER.
BETTER LIFE.

FACE EVALUATION


Move the cursor to select the corresponding level.

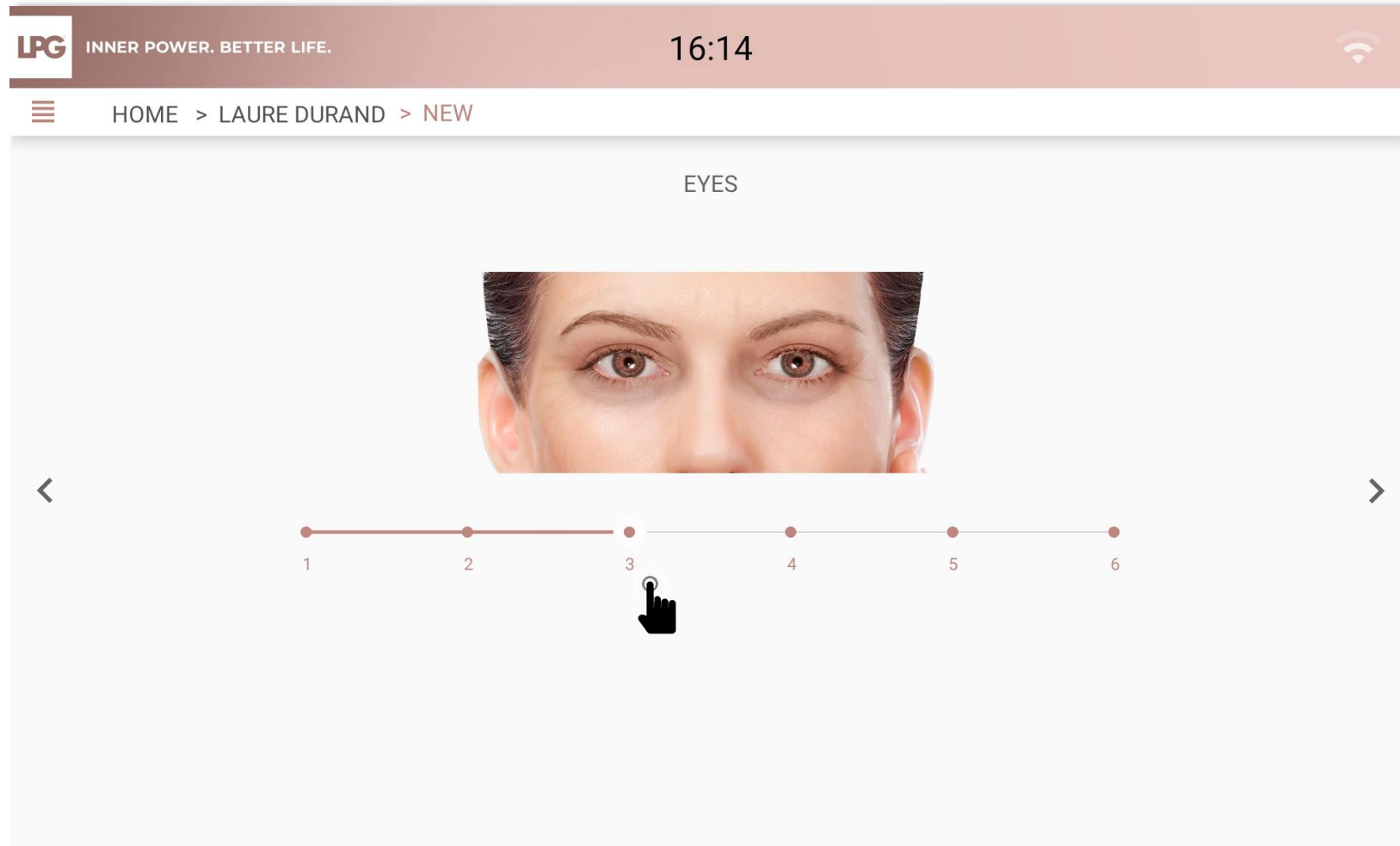

INNER POWER.
BETTER LIFE.

FACE EVALUATION


Move the cursor to select the corresponding level.

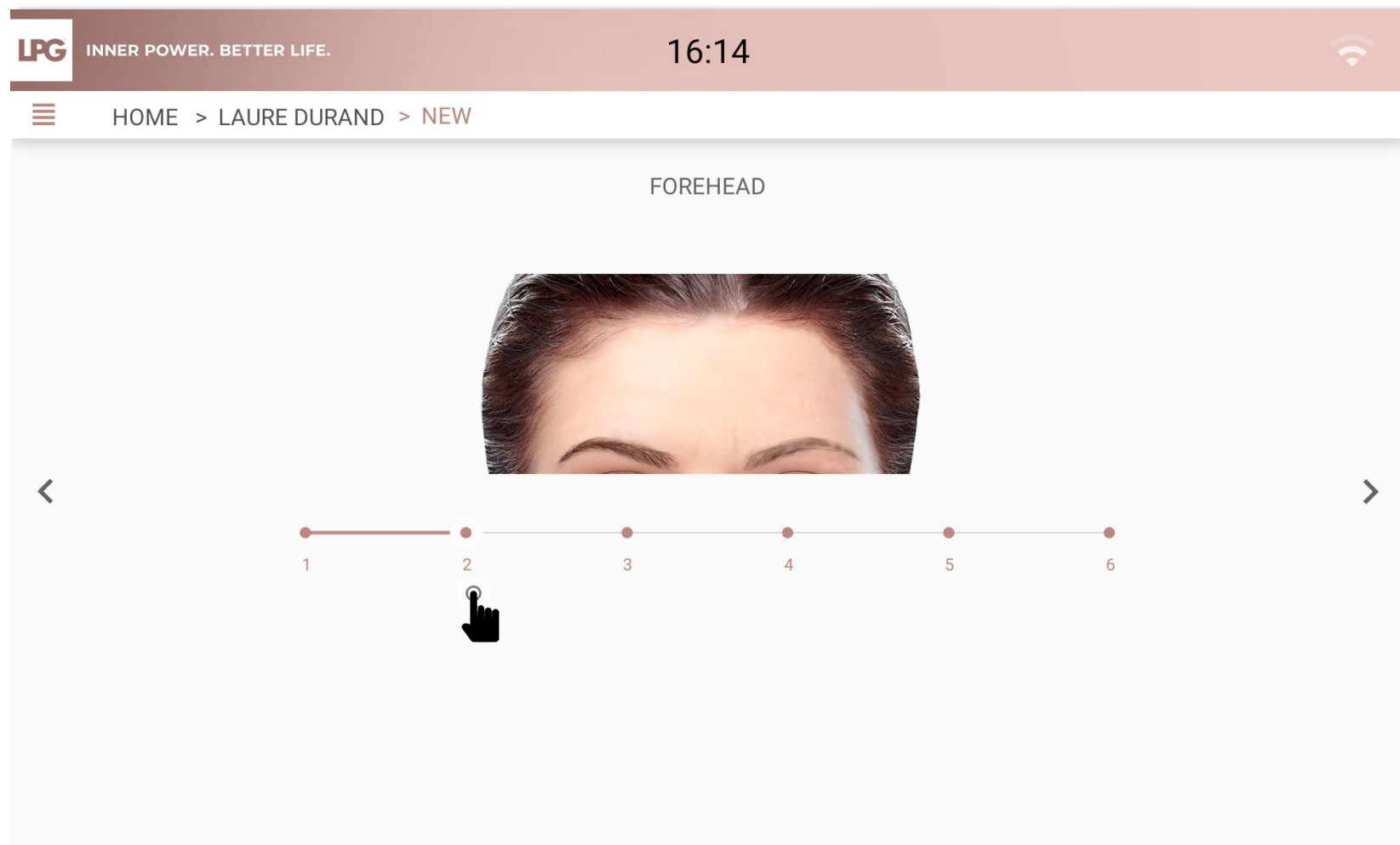

INNER POWER.
BETTER LIFE.


FACE EVALUATION

Move the cursor to select the corresponding level.

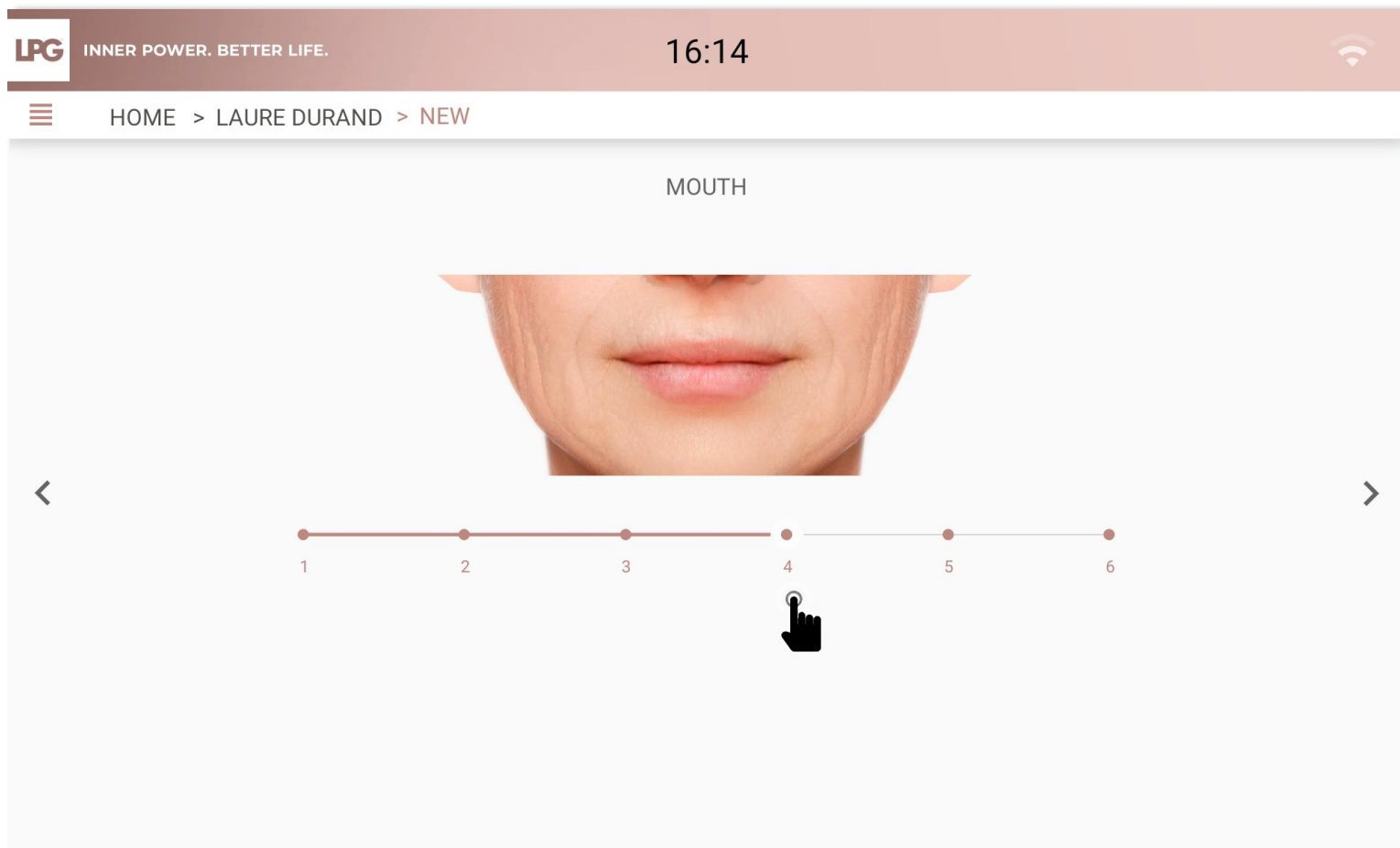

FACE EVALUATION

Move the cursor to select the corresponding level.

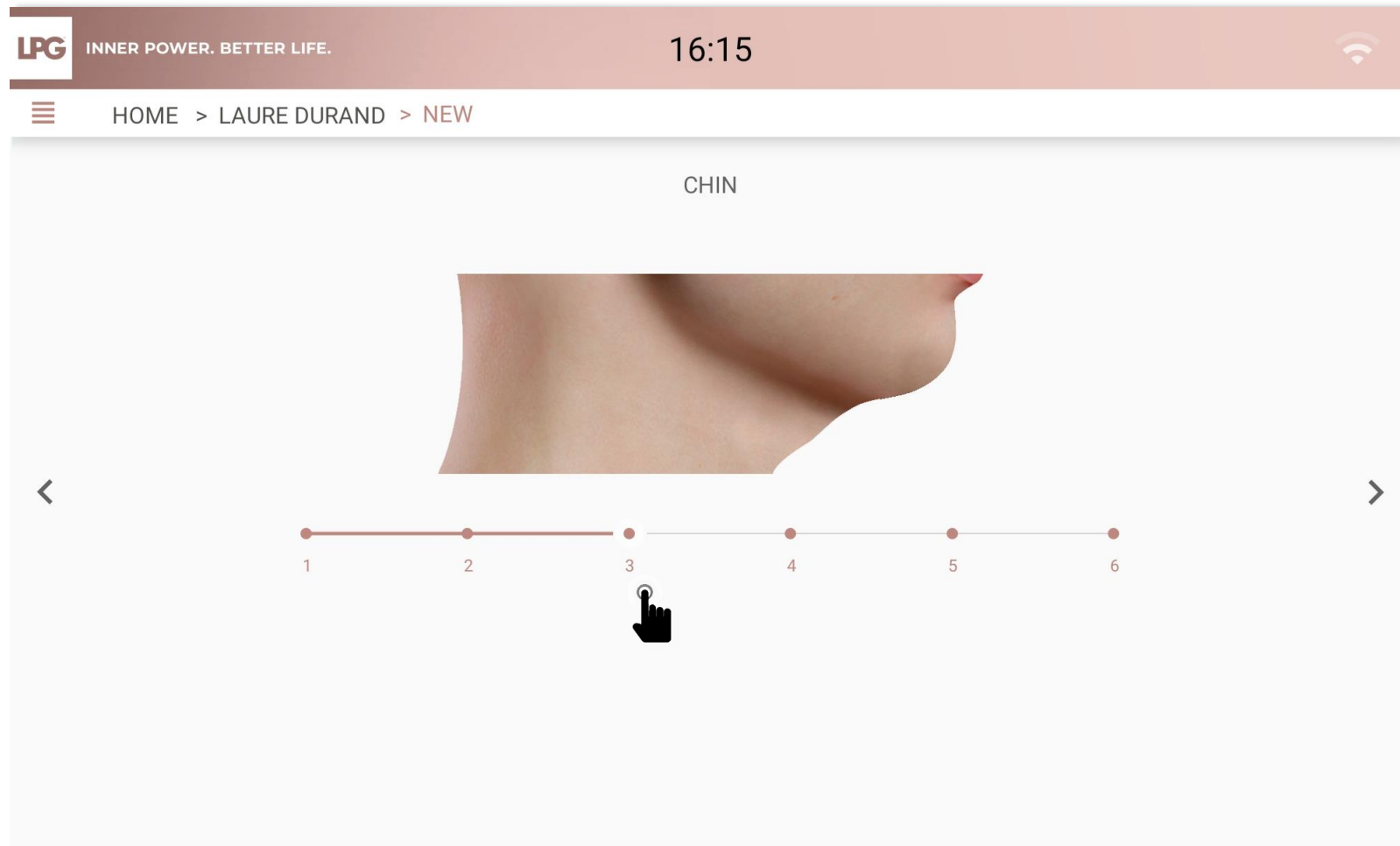

INNER POWER.
BETTER LIFE.

FACE EVALUATION

Select most adapted protocol(s) according to your client's objectives.

LPG

INNER POWER. BETTER LIFE.

16:15

HOME > LAURE DURAND > NEW

Recommended endermologie® protocols

Targeted treatment(s)

- ☒ Eye Wrinkles
- ☒ Forehead
- ☐ Mouth Area
- ☐ Double Chin

Classic treatment(s)

- ☐ Age-defying Skin Toning
- ☐ Age-defying Replumping
- ☒ Glow
- ☐ Age-defying Resculpting



Signature treatment(s)

- ☐
- ☐
- ☐
- ☐
- ☐

FACE EVALUATION

Final evaluation and protocol, cosmetics and endermonutri recommendation

- Adjust the number of recommended sessions.
- To confirm the evaluation and program creation, select **SUBMIT**.
- Your client's program is then saved.

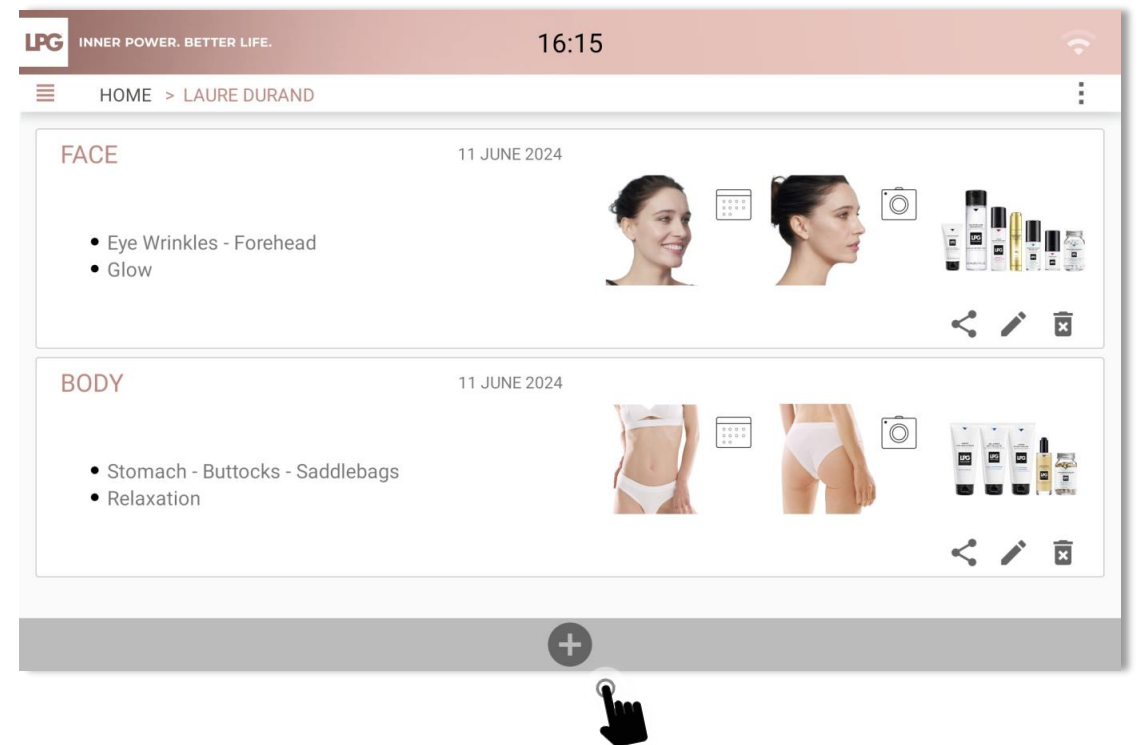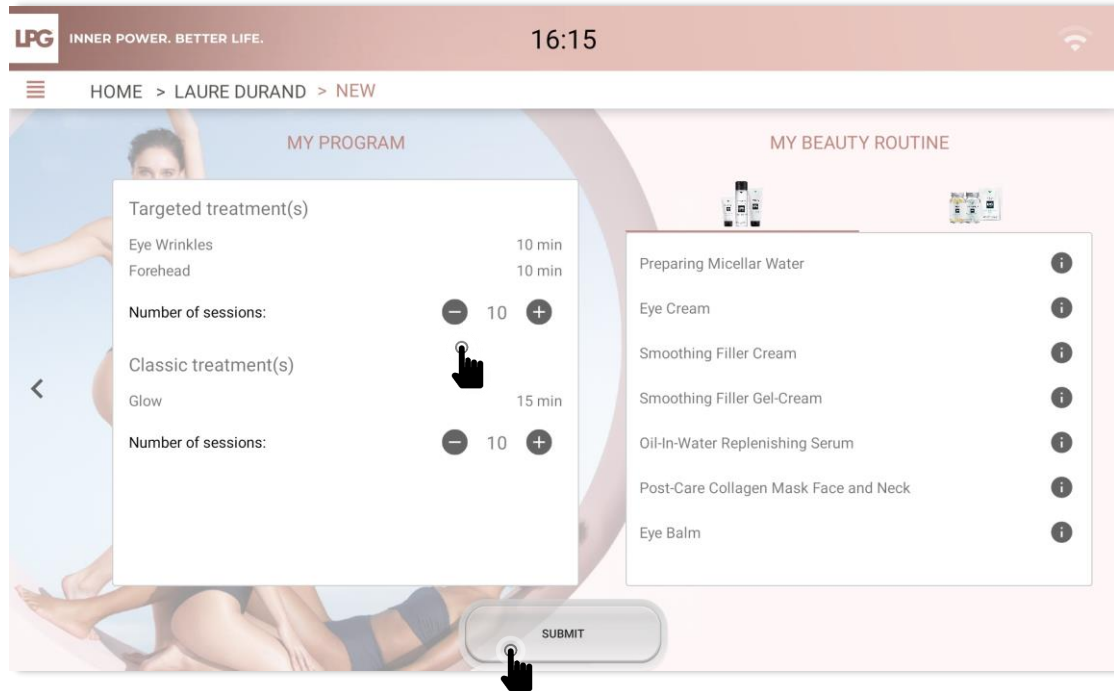

VIEWING PROGRAMS

SESSIONS

To view the completed endermologie® sessions (protocol description, date). This will enable a precise treatment program follow-up.

PHOTOS ASSESSMENT

To make, to consult and to compare photo assessments.

COSMETICS

To view cosmetics associated with your client's programs.

ENDERMONUTRI

To view food supplements associated with your client's programs.

DATE OF THE
FIRST
EVALUATION




TO CREATE NEW FACIAL OR BODY TREATMENT PROGRAM



INNER POWER.
BETTER LIFE.

VIEWING SESSIONS

 : To confirm a session in case of connection loss with your CELLU M6 ALLIANCE®






 : To comment a session (settings used...)

 : To delete a session

 : To add a session



The screenshot shows the LPG app interface. At the top, there's a header with the LPG logo, the tagline 'INNER POWER. BETTER LIFE.', the time '16:16', and a Wi-Fi icon. Below the header is a navigation bar with a menu icon, 'HOME', '> LAURE DURAND', and 'MY SESSIONS'. The main content area shows a session titled 'EYE WRINKLES - FOREHEAD' with a sub-header '1/10 20 min' and a plus icon. Below this is a list of sessions, each with a number (1/10 to 6/10), a description 'Eye Wrinkles Forehead', a duration '10 min', and a date 'Performed 11 June 2024'. Each session row has three icons: a calendar with a checkmark, a speech bubble, and a trash can.

EYE WRINKLES - FOREHEAD				1/10 20 min	+
1/10	Eye Wrinkles Forehead	10 min	10 min	Performed 11 June 2024	  
2/10	Eye Wrinkles Forehead	10 min	10 min		  
3/10	Eye Wrinkles Forehead	10 min	10 min		  
4/10	Eye Wrinkles Forehead	10 min	10 min		  
5/10	Eye Wrinkles Forehead	10 min	10 min		  
6/10	Eye Wrinkles	10 min			

IN EXPERT MODE

- Possibility to save the settings used during the session for next sessions.
- Possibility to save the time spent on each step.

VIEWING COSMETICS

You can view cosmetics associated with each treatment program.

LPG

INNER POWER. BETTER LIFE.

16:16

HOME > LAURE DURAND > MY BEAUTY ROUTINE



EYE WRINKLES - FOREHEAD



PREPARING MICELLAR WATER



EYE CREAM



SMOOTHING FILLER CREAM



SMOOTHING FILLER GEL-CREAM

GLOW

PREPARING MICELLAR WATER



- Removes impurities and makeup Moisturizes and soothes


Soak a cotton pad and cleanse face, eyes and neck, morning and evening.
Dry the skin gently without rinsing.

LPG


INNER POWER.
BETTER LIFE.

VIEWING ENDERMONUTRI


You can view food supplements associated with each treatment program.


 INNER POWER. BETTER LIFE.


16:10





HOME > LAURE DURAND > MY BEAUTY ROUTINE




 VITALITY BOOSTER


 HYALURONIC ACID

 COLLAGEN

 OMEGA

 WATERGLOW

VITALITY BOOSTER



- Stimulates vitality and physical & mental performance
- Reduces fatigue
- Strengthens the immune system

2 capsules per day preferably in the morning and before 3 pm to prevent any difficulty in falling asleep.
For optimal results, it is recommended to follow this program for at least 2 consecutive months. It may be repeated at the start of any endermologie® program and/or change in season.



INNER POWER.
BETTER LIFE.





PHOTO ASSESSMENT



INNER POWER.
BETTER LIFE.

PHOTO ASSESSMENT

After clicking on « Photo assessment » icon, click on  to make a new photo assessments.
All photos taken during the first evaluation are saved here.

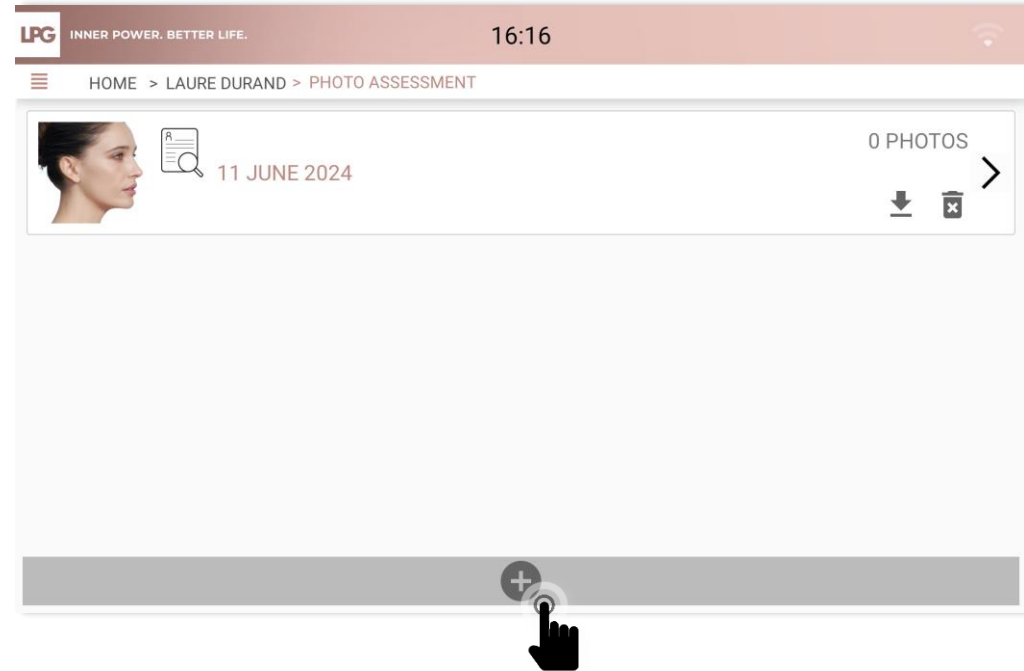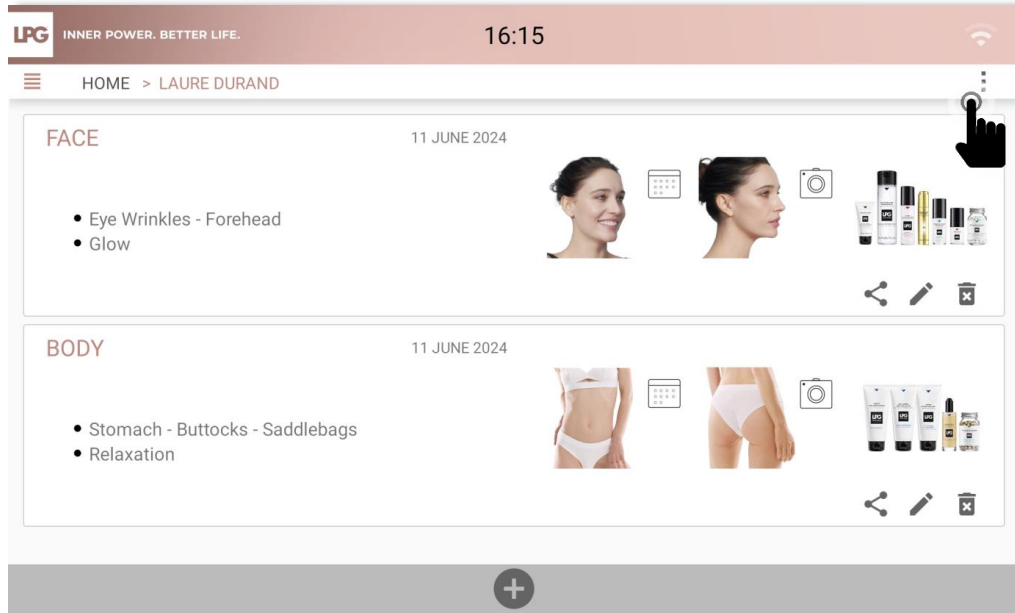





-  : To delete photos assessment
-  : To save photos assessment on your tablet


PHOTO ASSESSMENT

The first evaluation's objectives are mentioned (reminder).
You can add comments (number of sessions, results..)

 INNER POWER. BETTER LIFE.



16:17



 HOME > LAURE DURAND > PHOTO ASSESSMENT

Objective:

Add your comments here






INNER POWER.
BETTER LIFE.

PHOTO ASSESSMENT


- Take 3 photos (right profile, face, left profile). Maintain you tablet in vertical (portrait) format. The photo must be taken when level sensors are green. Report to the integrated markers of the photo assistant to position your client properly. Once the picture is taken, click on **REGISTER** or **CANCEL** if you are not satisfied.
- To take other pictures, click on **ADD ADDITIONAL PICTURE** and choose a title.

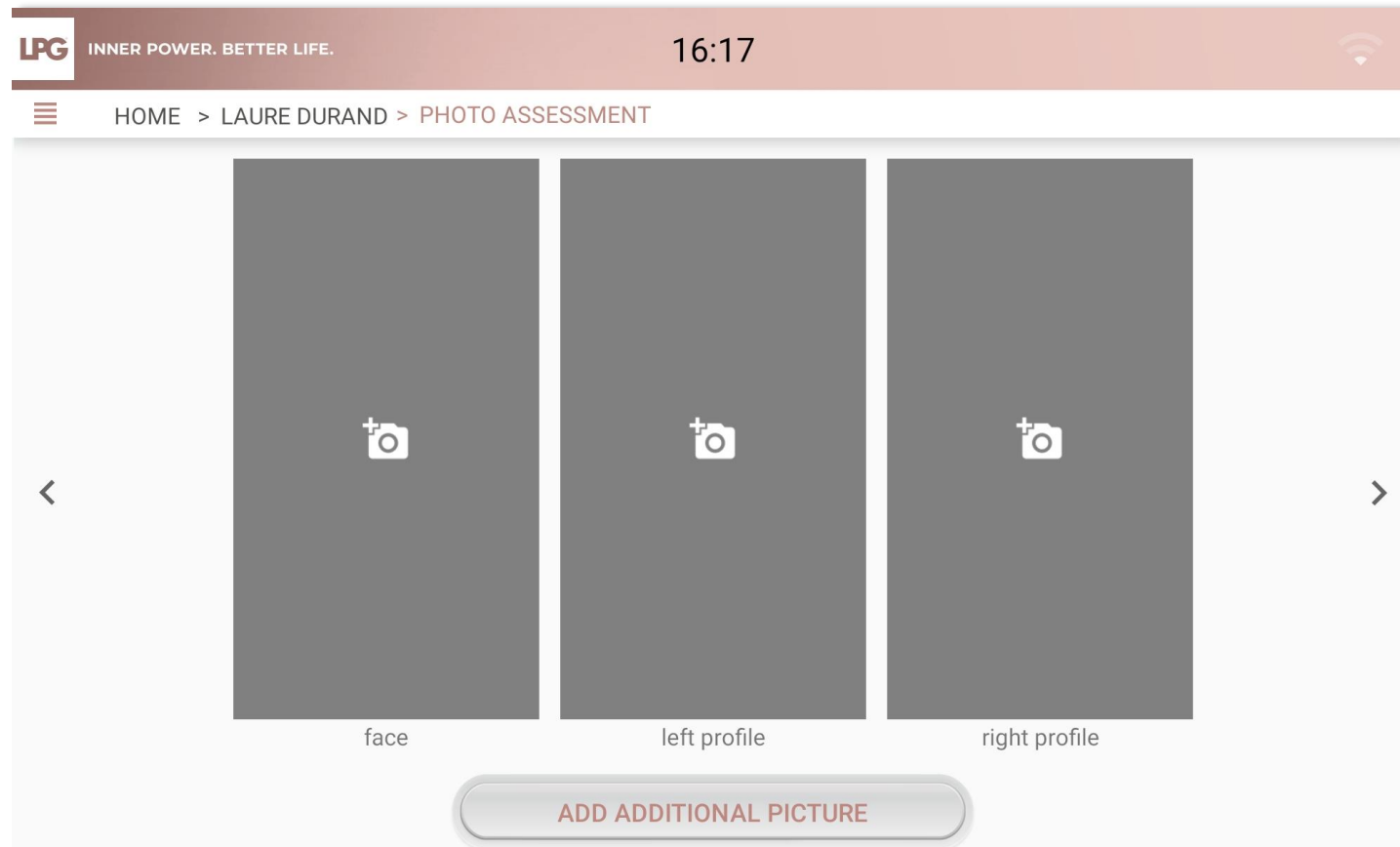

INNER POWER.
BETTER LIFE.

PHOTO ASSESSMENT

Move the cursor to select the corresponding level.

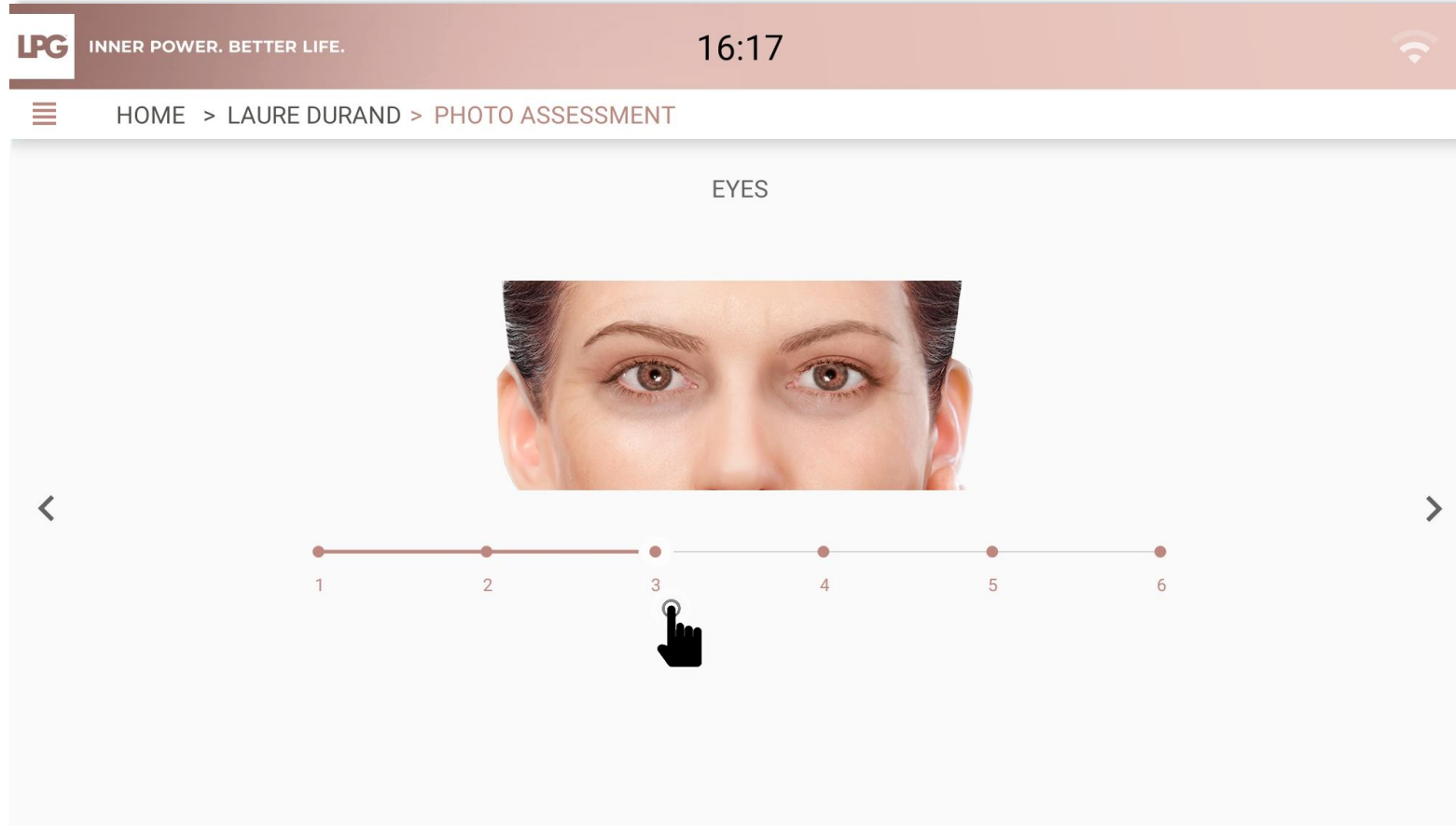




COMPARE PHOTO ASSESSMENTS



INNER POWER.
BETTER LIFE.


COMPARE PHOTO ASSESSMENTS

Click on 

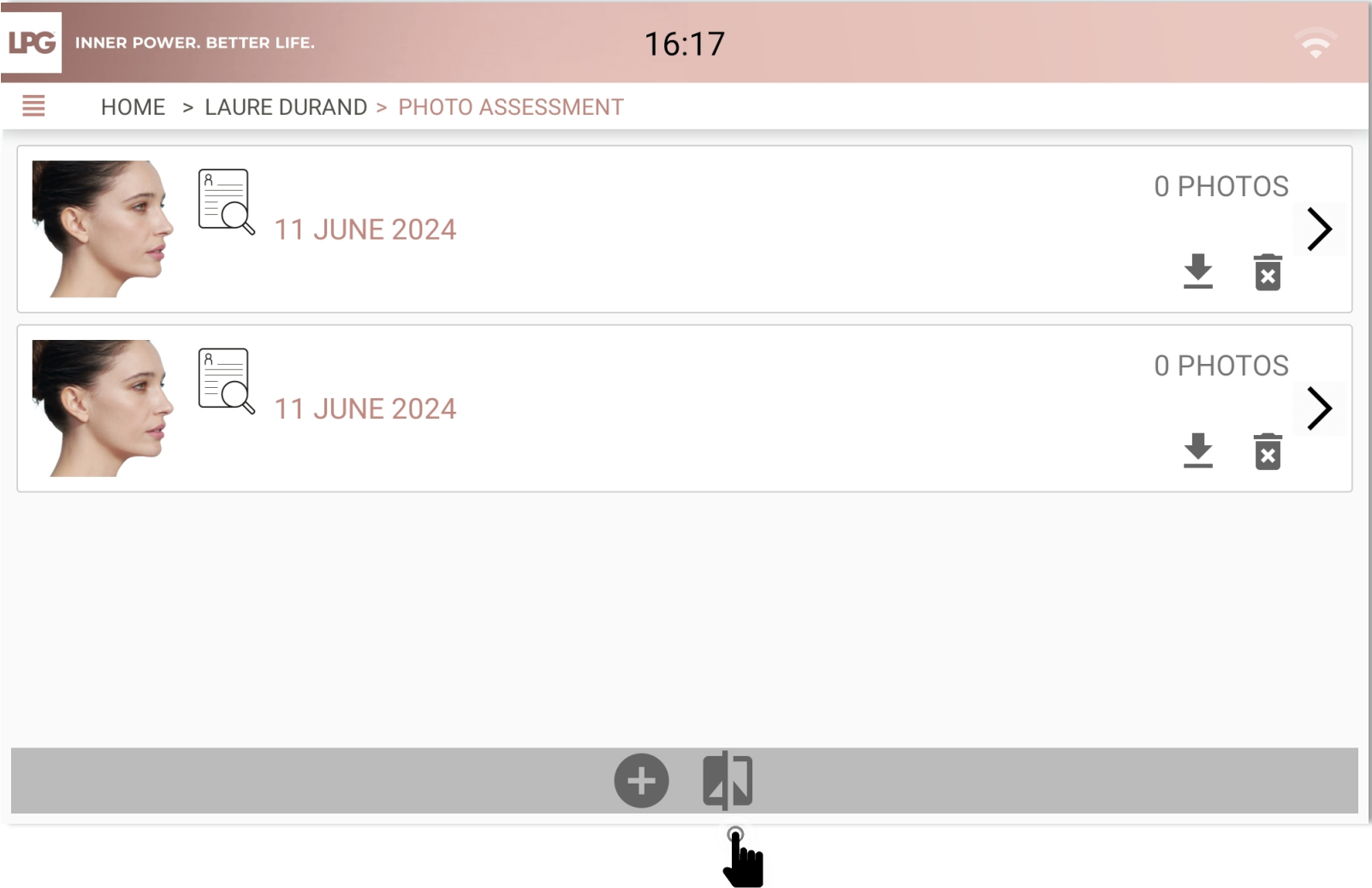

COMPARE PHOTO ASSESSMENTS

- Using the arrows, select the photo assessments
- Next, select the picture : face, back, left profile, right, profile
- To save the comparison, click on ↓
- Choose a title and save it in a folder on your tablet.

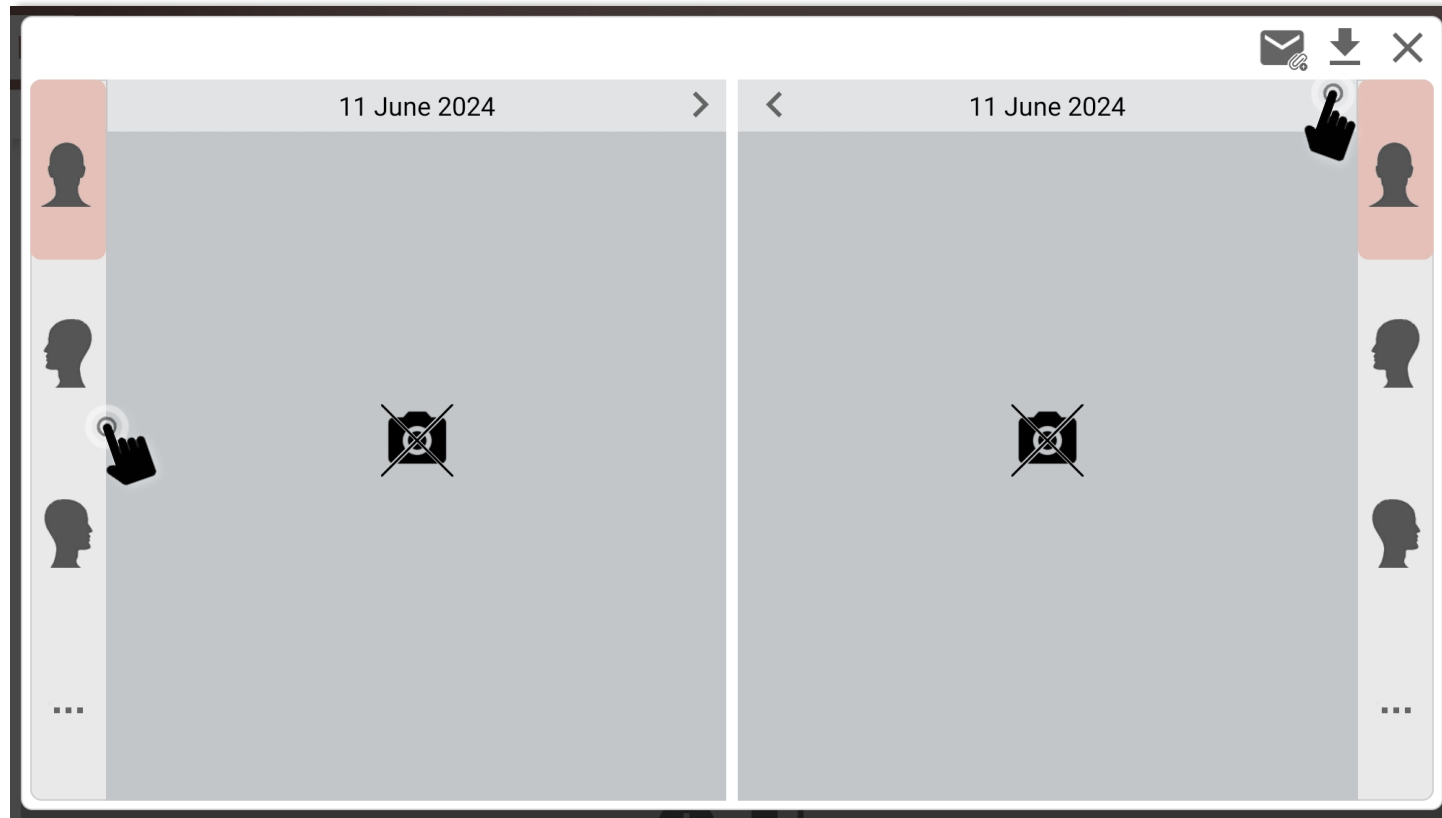




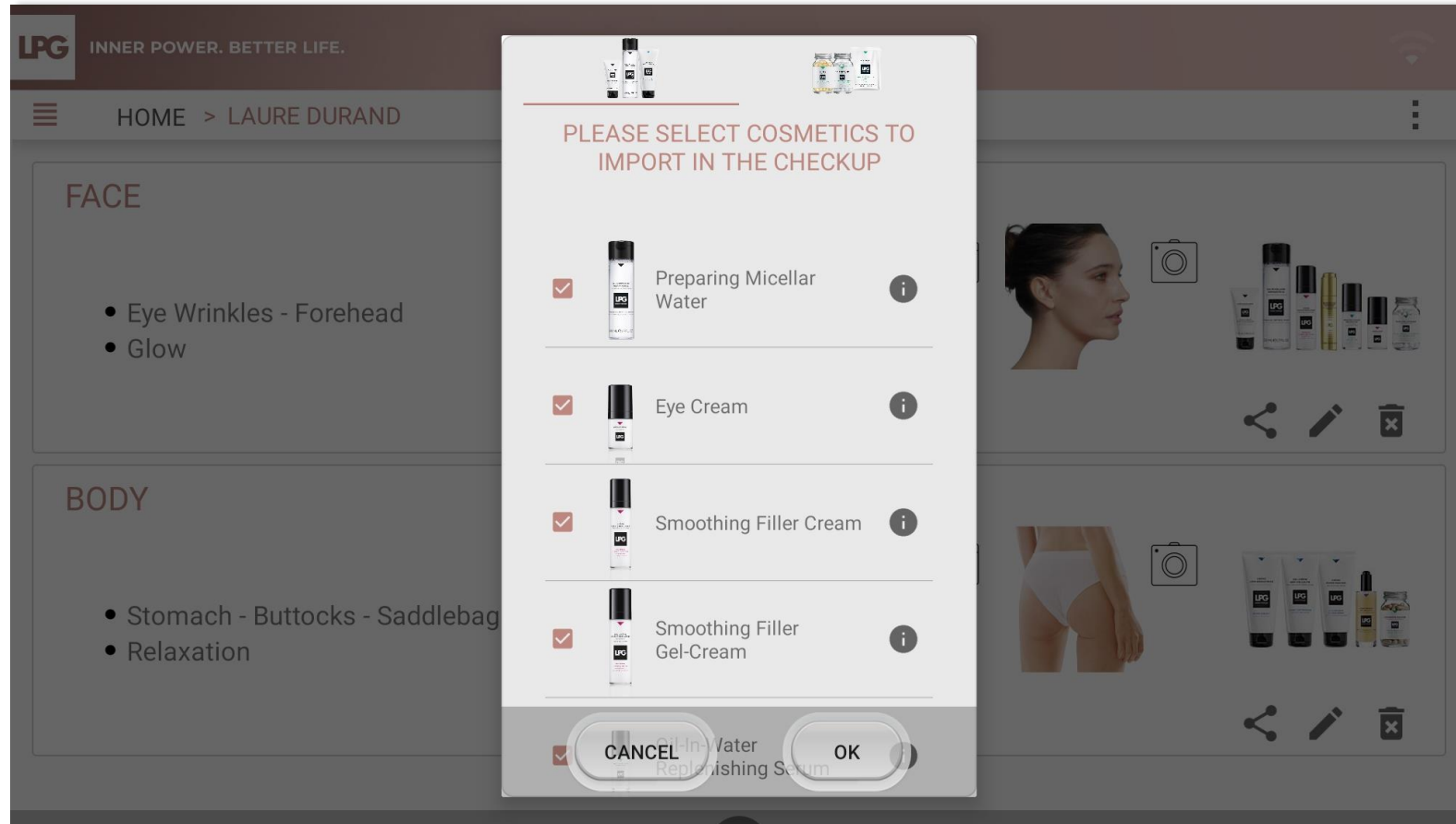
SEND TREATMENT
PROGRAM
TO YOUR CLIENT



INNER POWER.
BETTER LIFE.

SEND TREATMENT PROGRAM


- Choose the concerned program and click on 
- Select cosmetics and food supplements to import in the document. Next, click on OK.



INNER POWER.
BETTER LIFE.

SEND TREATMENT PROGRAM

After setting up your email account, customize the message which will be sent.
The program (PDF document) is automatically enclosed with your email. Next, click on SEND.

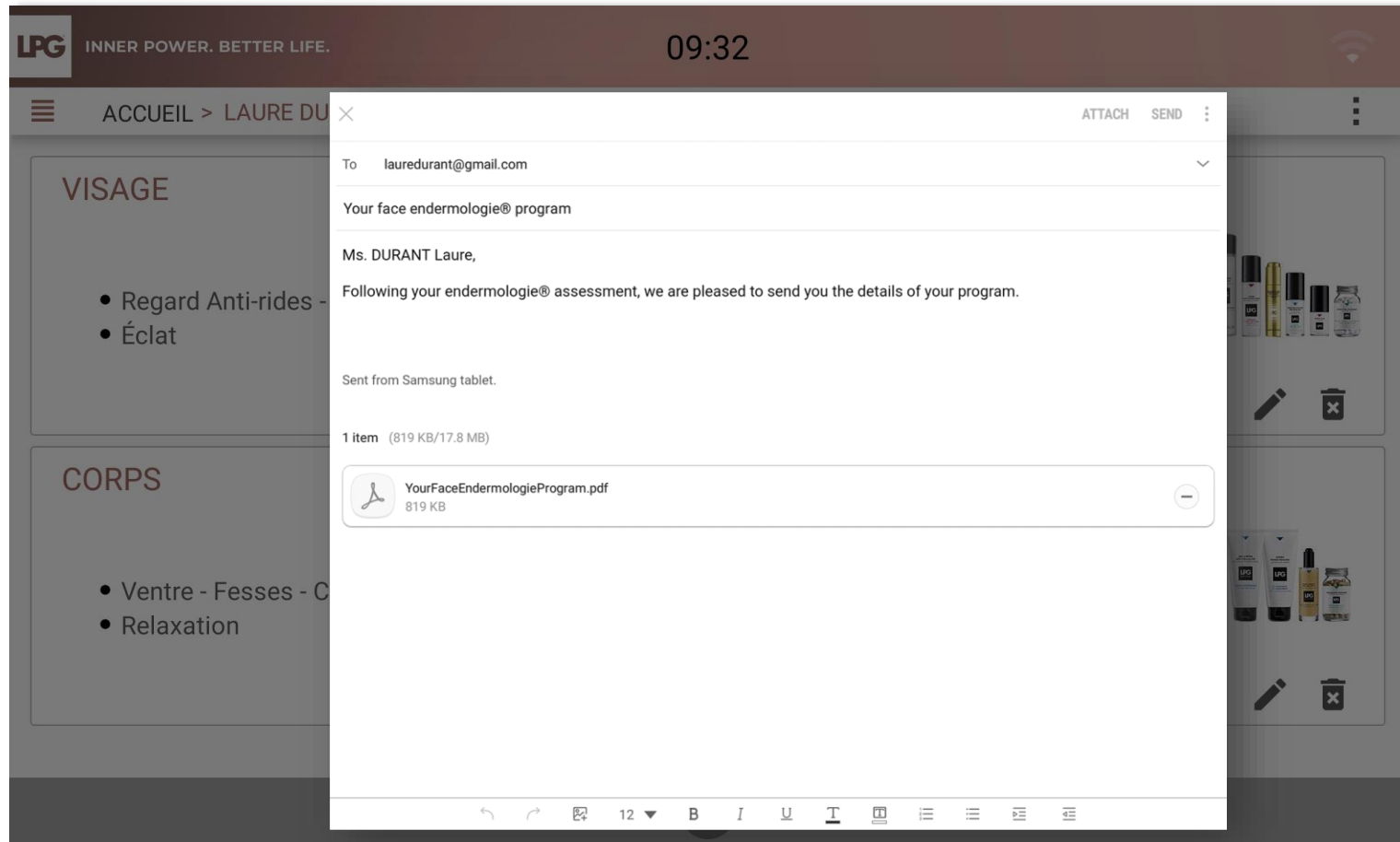

INNER POWER.
BETTER LIFE.




SEND TREATMENT
PROGRAMS
TO YOUR CELLU M6
ALLIANCE®




INNER POWER.
BETTER LIFE.

TREATMENT PROGRAM LAUNCH

After connecting your tablet to your CELLU M6® device (please refer to the Tablet Quickstart guide if necessary), select **CUSTOMIZED SESSION**.


If your device is connected to your tablet, this icon  appears on your tablet's screen.

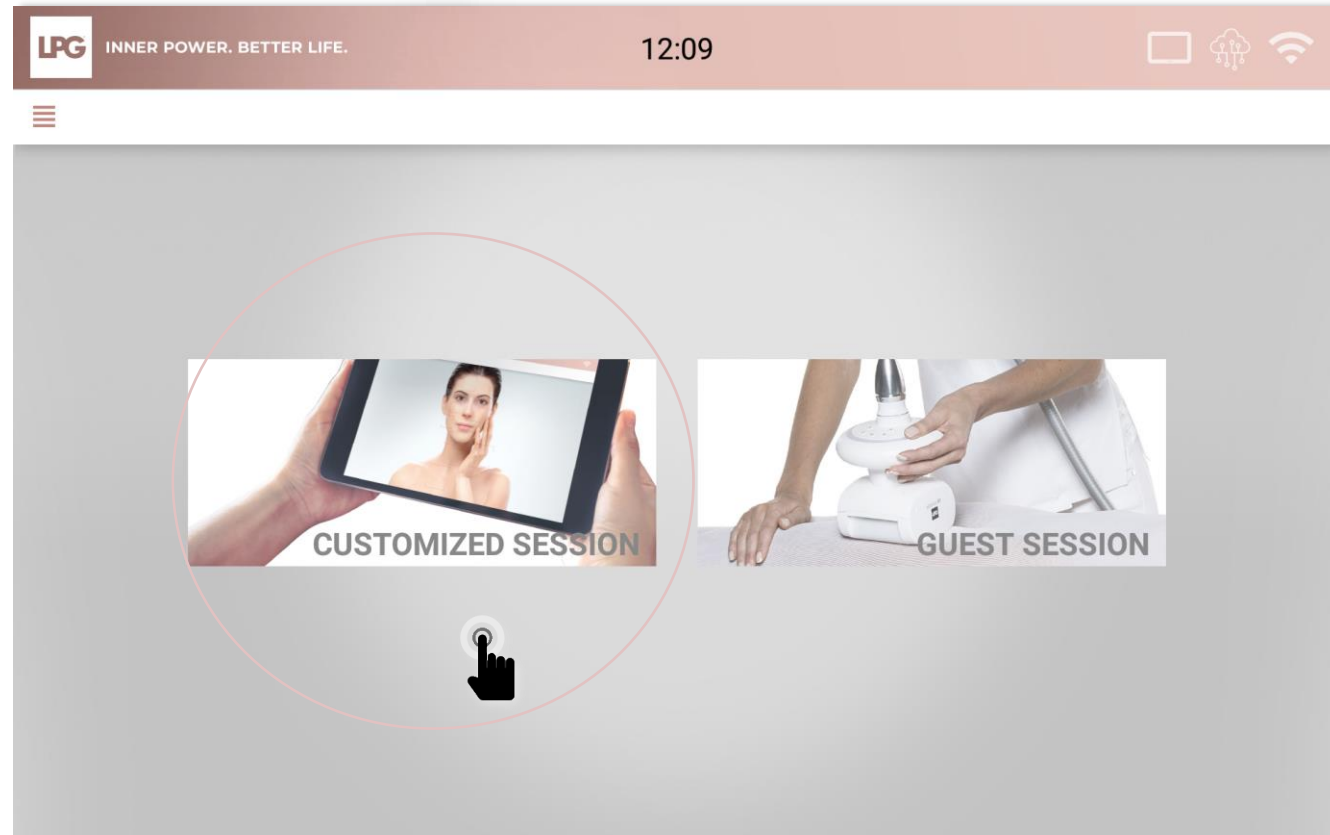

INNER POWER.
BETTER LIFE.

TREATMENT PROGRAM LAUNCH

Select the concerned client in your database.

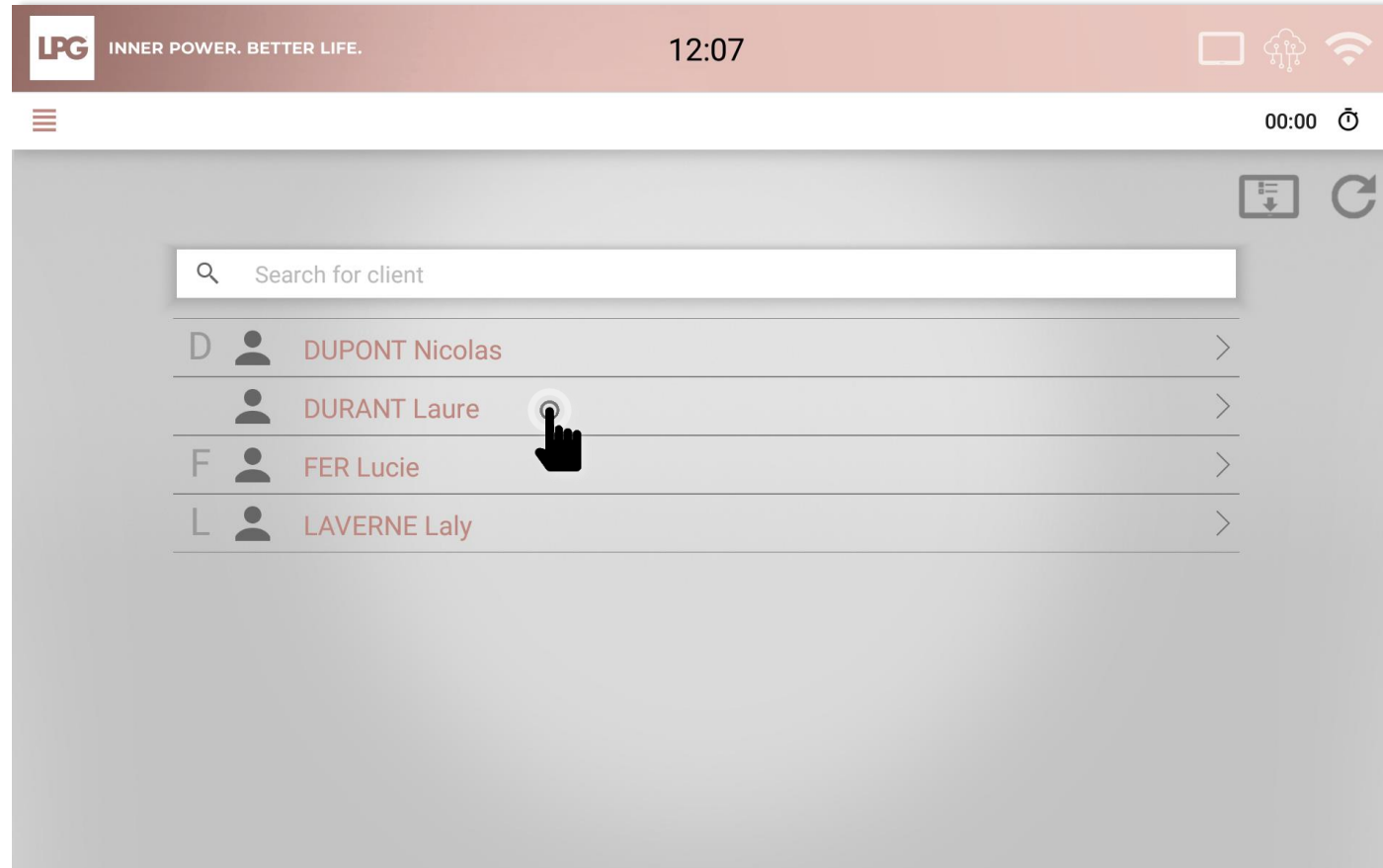


Client programs can be downloaded when the tablet is connected. They will be accessible at any time. The results will be sent as soon as the tablet reappears.



**INNER POWER.
BETTER LIFE.**

TREATMENT PROGRAM LAUNCH

Select the desired program.

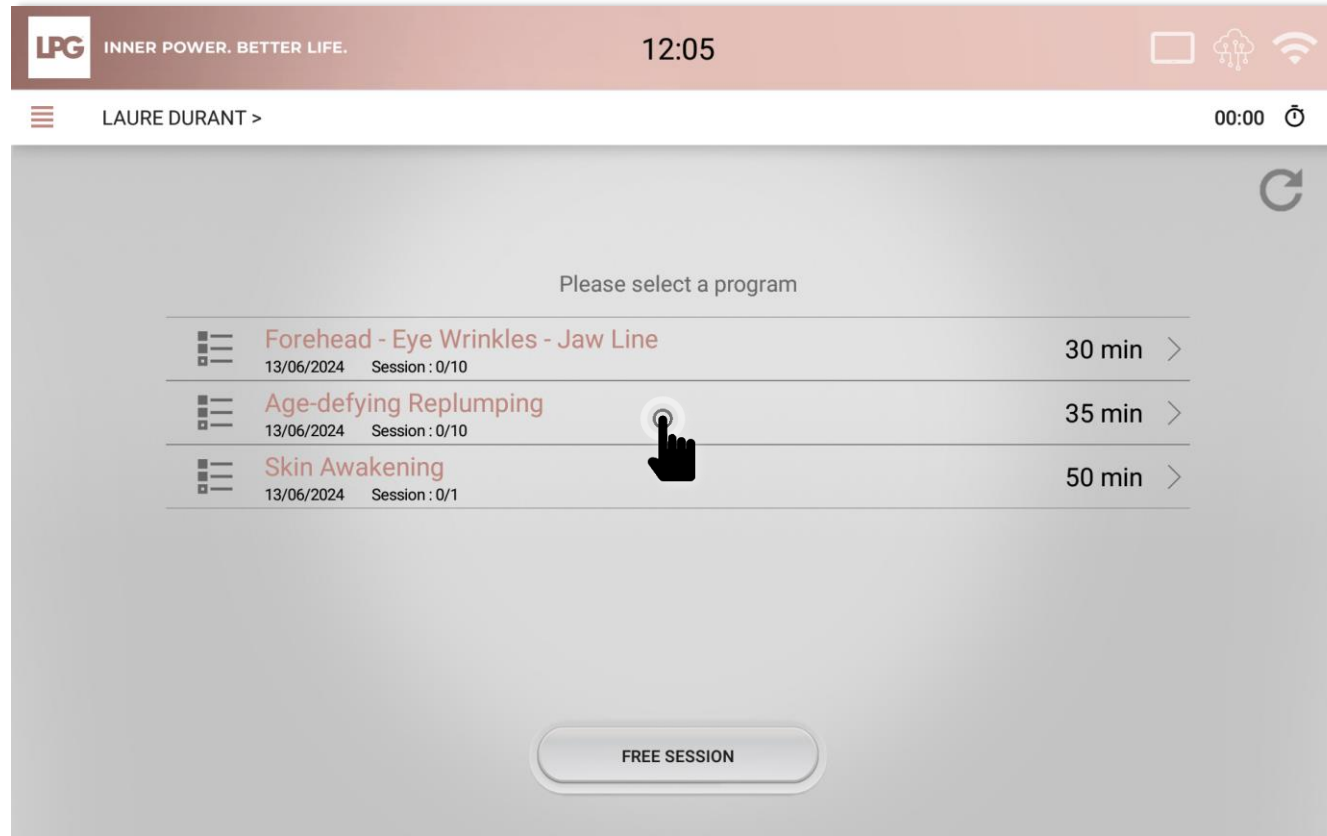


In case of connection loss, possibility to update manually the sessions on the machine validated on the tablet by clicking on



**INNER POWER.
BETTER LIFE.**

TREATMENT PROGRAM LAUNCH

The session can start!

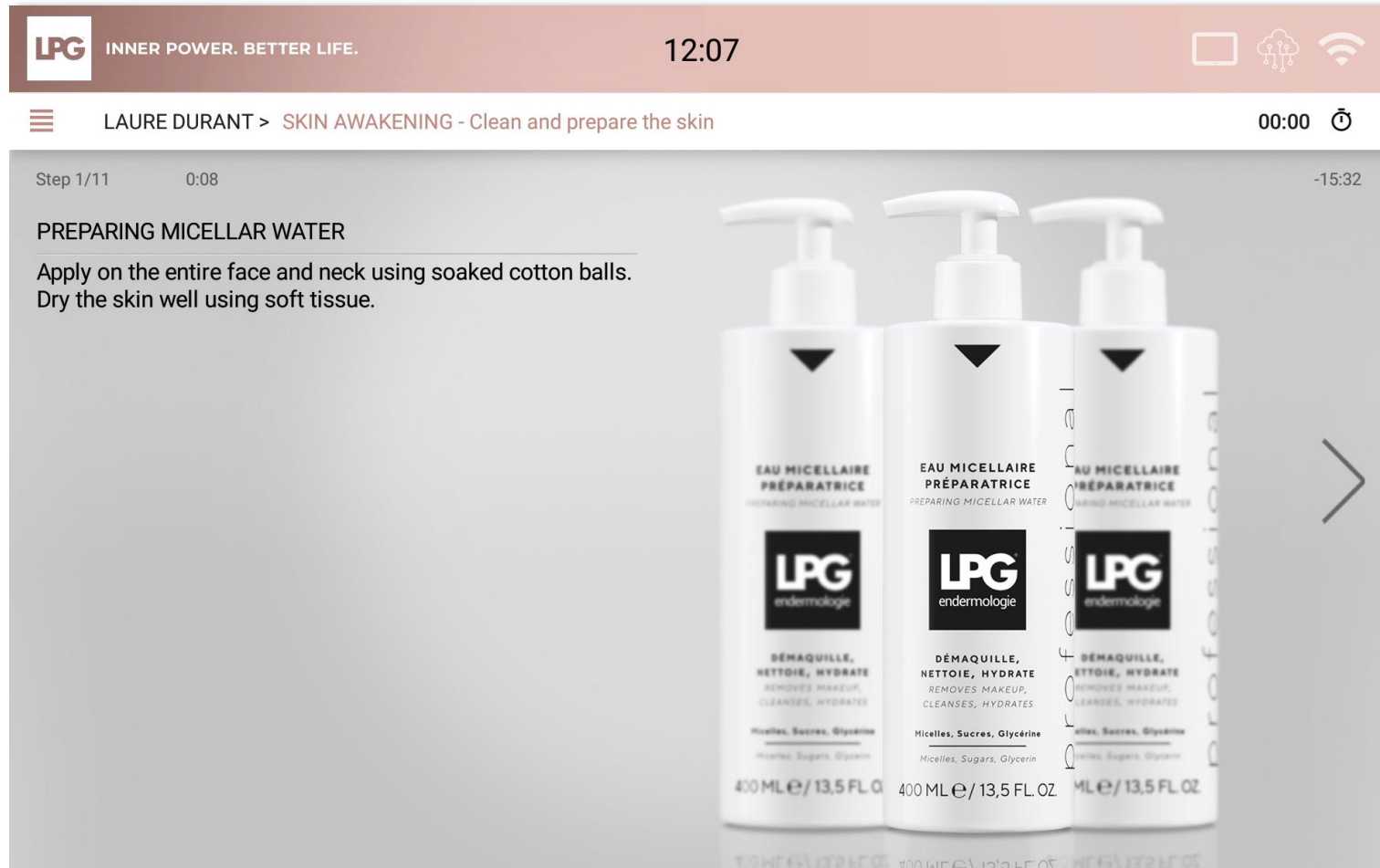




FREE SESSIONS



INNER POWER.
BETTER LIFE.

VIEWING FREE SESSIONS

To access your client's free sessions, click on 
Next, click on **MY FREE SESSIONS**.

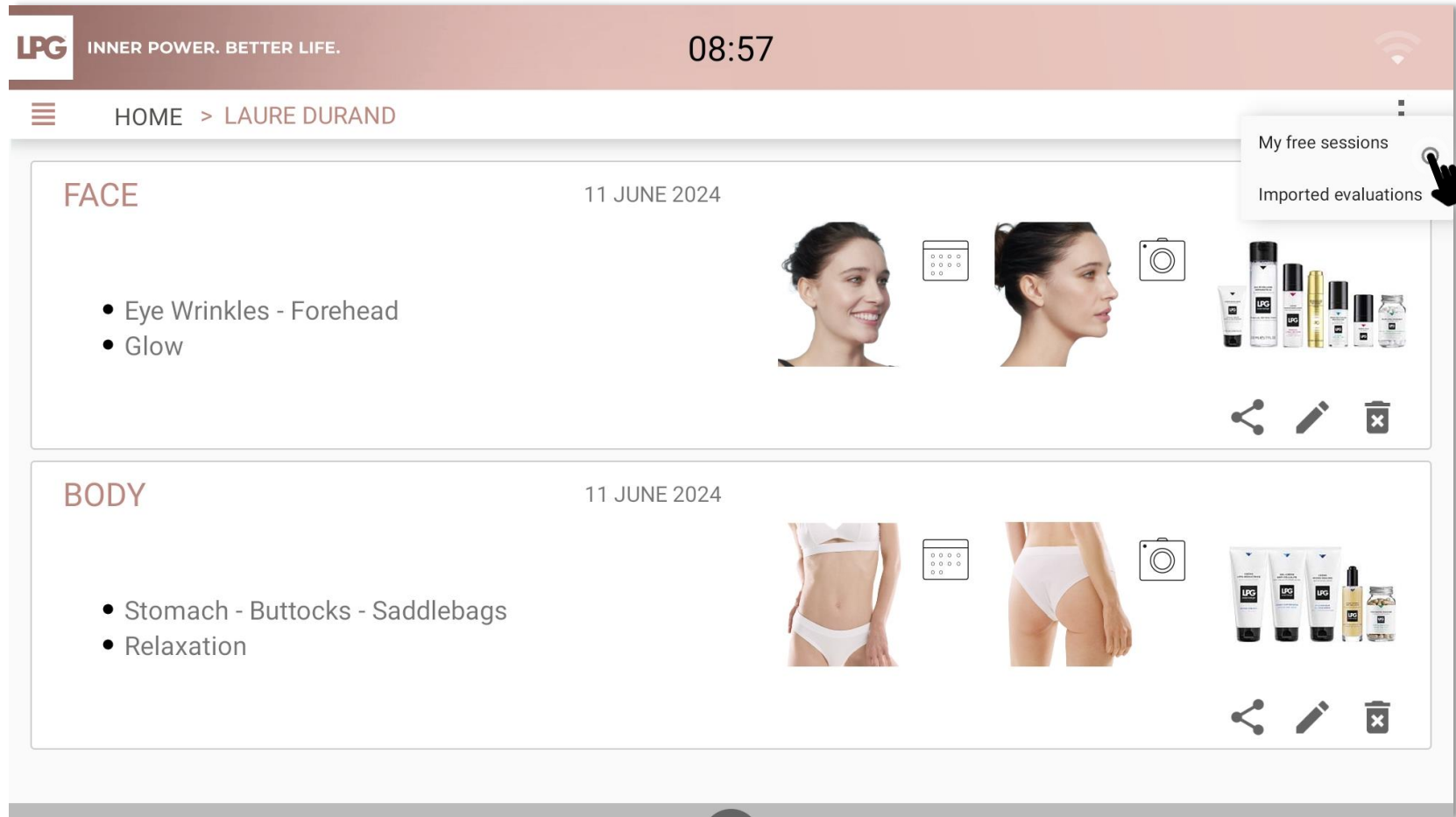

INNER POWER.
BETTER LIFE.



BACKUP



INNER POWER.
BETTER LIFE.


BACKUP



Should you skip this essential step and should any problem occur (operating error, technical failure, theft...), you will lose your data

For more security, save all your data on an external support of your choice: micro SD card available in supermarkets or in specialized stores, personal computer, external hard drive...
By default, your database will be stored on your tablet.

Click on the menu button on the upper left side of the screen and select **SETTINGS**. Click on **DATABASE**. Click on « **BACKUP DATABASE** » and choose your micro SD card as backup location. The number of possible backups is conditioned by your micro SD card's memory size.

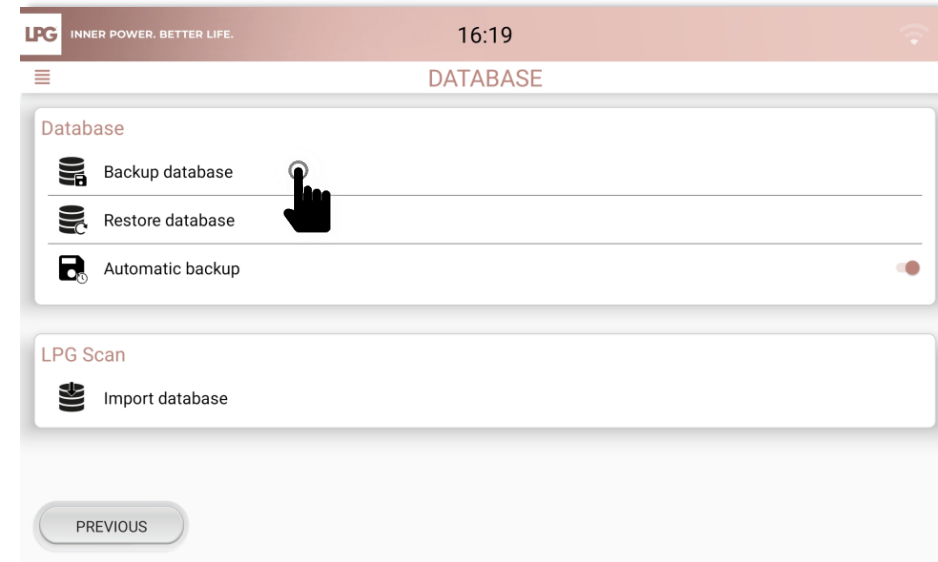

AUTOMATIC BACKUP

To plan an automatic backup, click on **AUTOMATIC BACKUP**.



Click on the cursor to activate the automatic backup



Select the periodicity, the location.

Enter your password for security protection. Please keep it safe because without it, it will be impossible to restore your backups. For privacy reasons, LPG SYSTEMS does not have access to your password and cannot ask you for it. LPG SYSTEMS can not be held responsible for any loss of your data.

Click on **OK**.

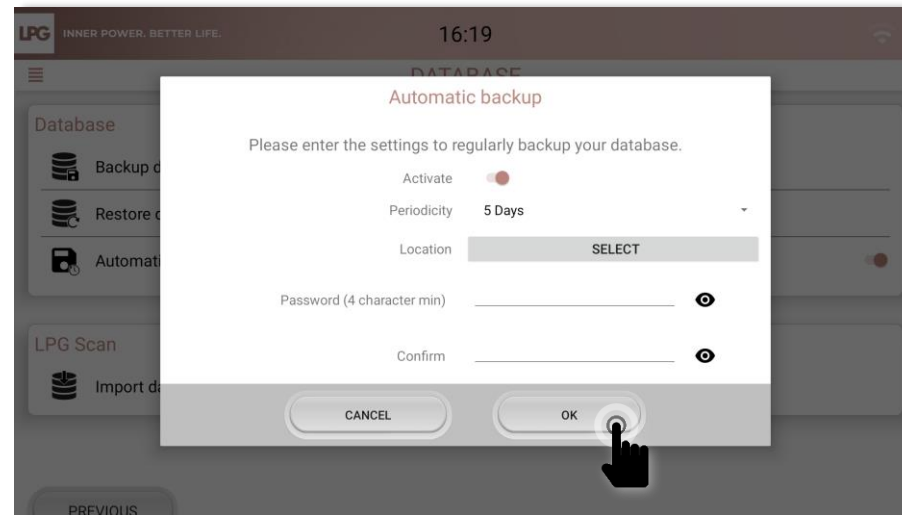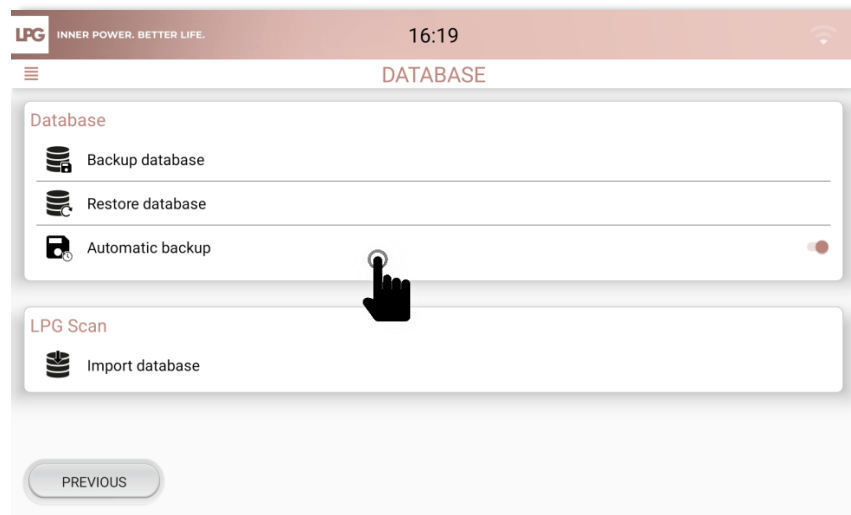

INNER POWER.
BETTER LIFE.




UPDATES



INNER POWER.
BETTER LIFE.



UPDATES

When an update is available, the following screen appears when you start the machine.
The "i" allows to get information about the update.
Click on **OK** to start the update.

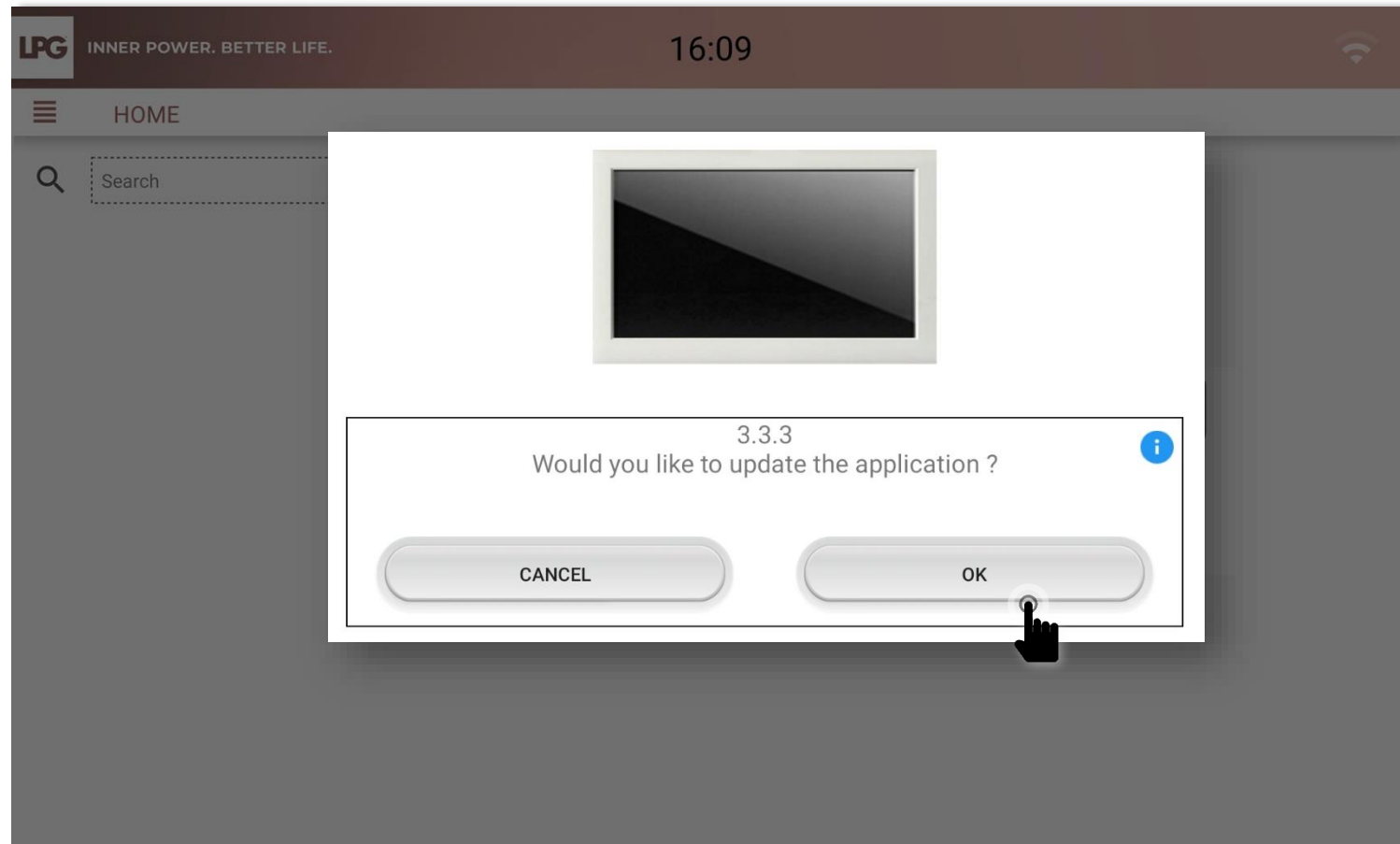

UPDATES

Please follow the instructions on the screen and continue by clicking on **OK**.

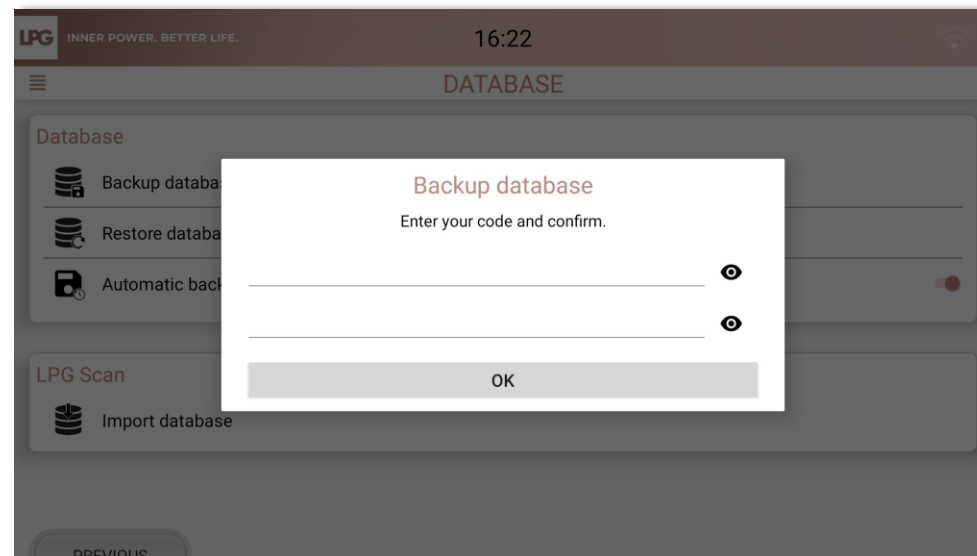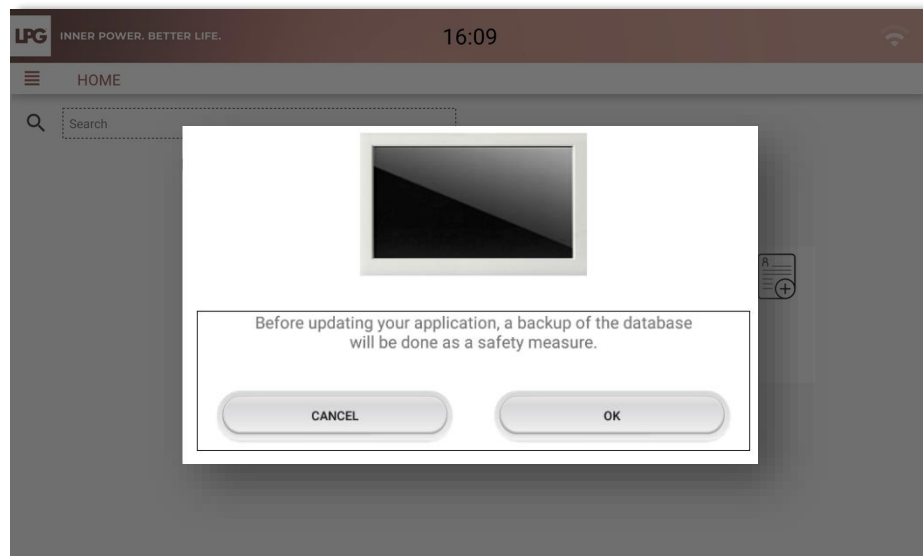

Enter your password for security protection. Please keep it safe because without it, it will be impossible to restore your backups. For privacy reasons, LPG SYSTEMS does not have access to your password and cannot ask you for it.


LPG SYSTEMS can not be held responsible for any loss of your data.



INNER POWER.
BETTER LIFE.

UPDATES

At the end of the backup, your tablet can display this message. Click on **Settings**.
Allow the installation of unknown sources with the cursor.

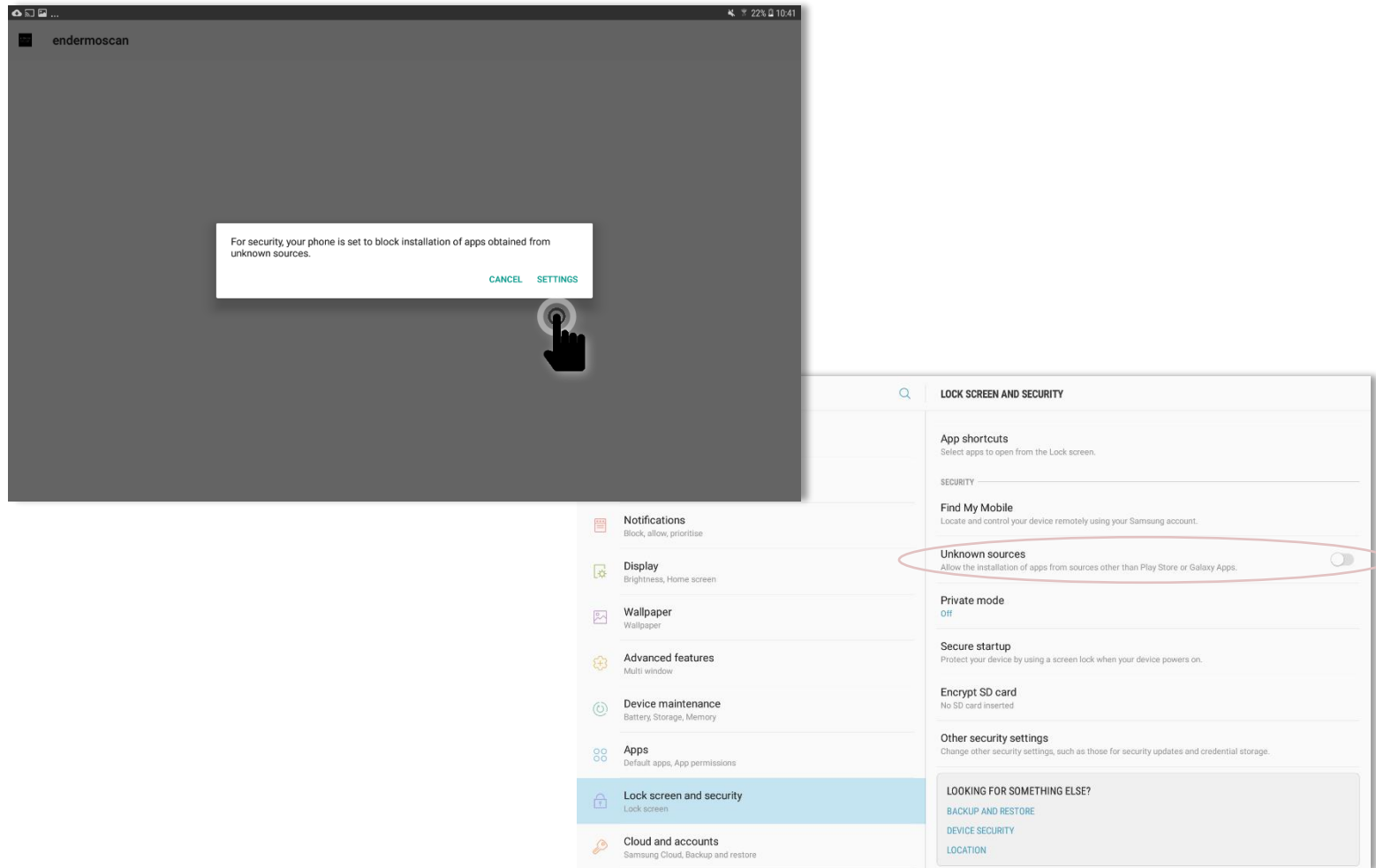



DATA IMPORT (LPG® SCAN tablet to the new one)





INNER POWER.
BETTER LIFE.

LPG® SCAN DATABASE

Before importing, please save your database.

To import your LPG SCAN database (client profile, photos), please insert a micro SD card into the intended location on the tablet. Then click on the menu button on the upper left side of the screen and select **SETTINGS**.

In **DATABASE**, click on **IMPORT DATABASE** and choose a backup location.

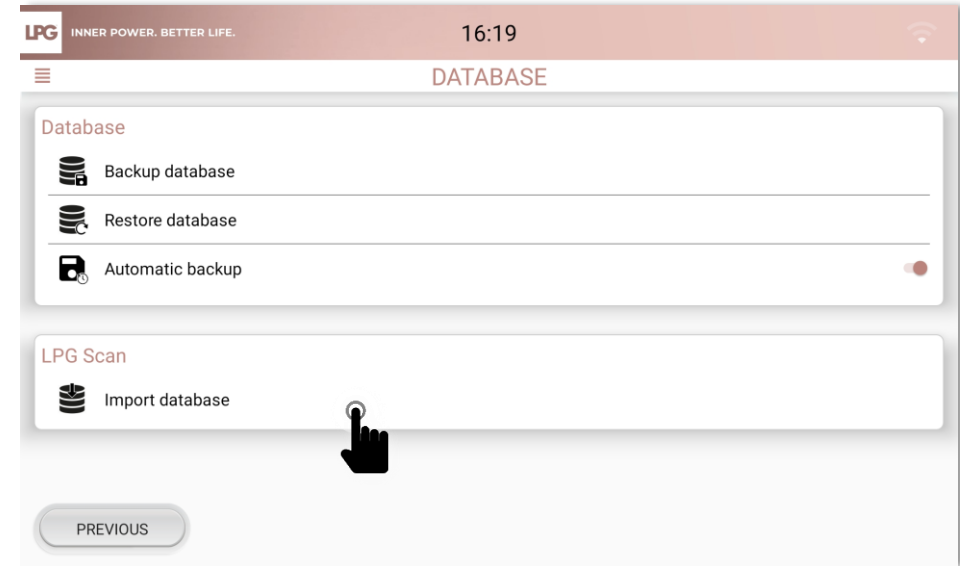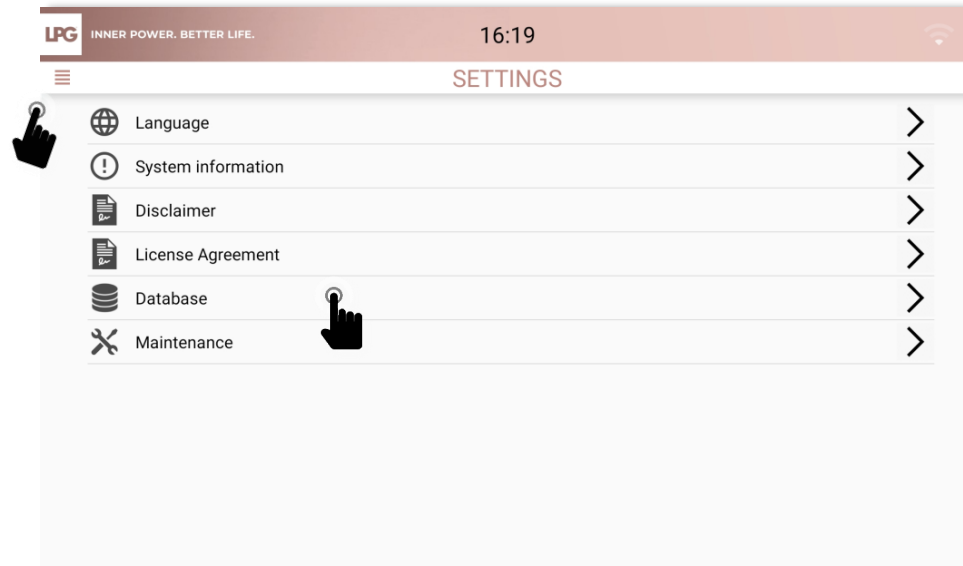




DATA EXPORT





INNER POWER.
BETTER LIFE.

DATA EXPORT

To access your tablet's settings, click on 

« **Disclaimer** » makes you aware of confidentiality and integrity of data if you decide to export our customer data (please check *the export of customer data*).

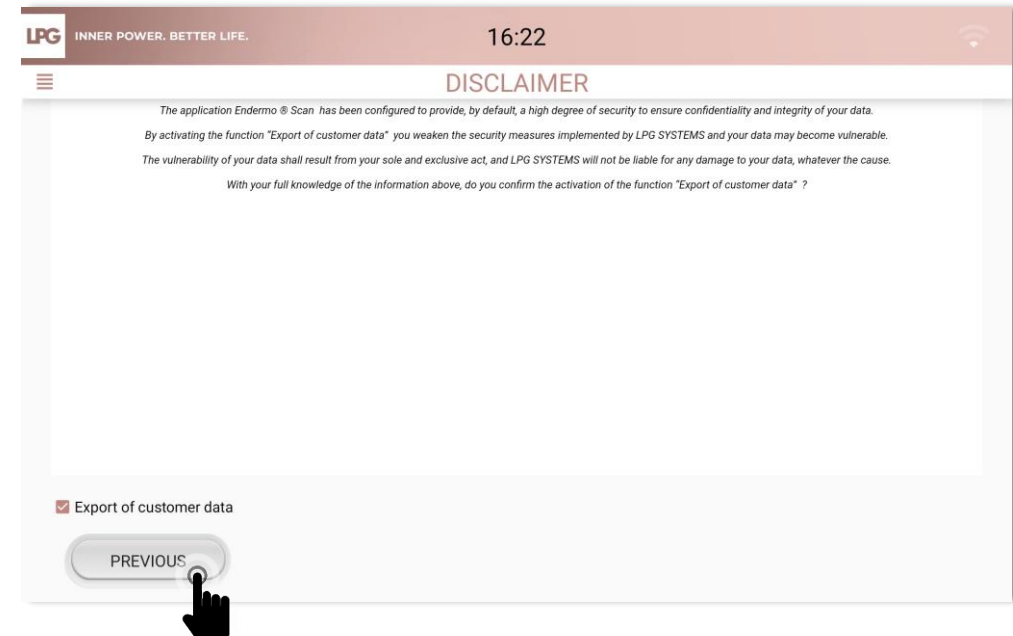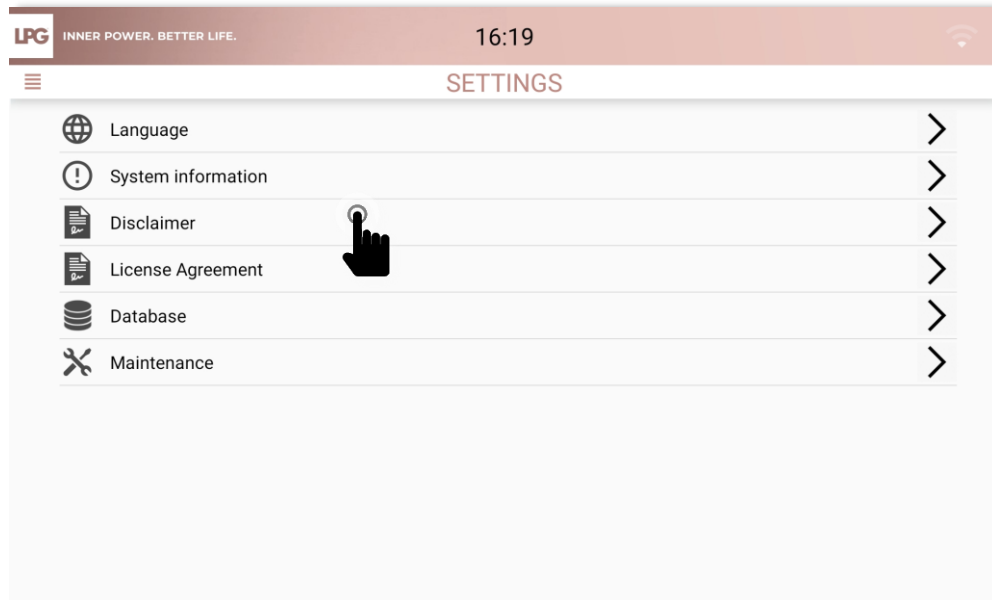

LPG® is not responsible for the use of data outside of its application.