INNER POWER. BETTER LIFE.

SOMMAIRE

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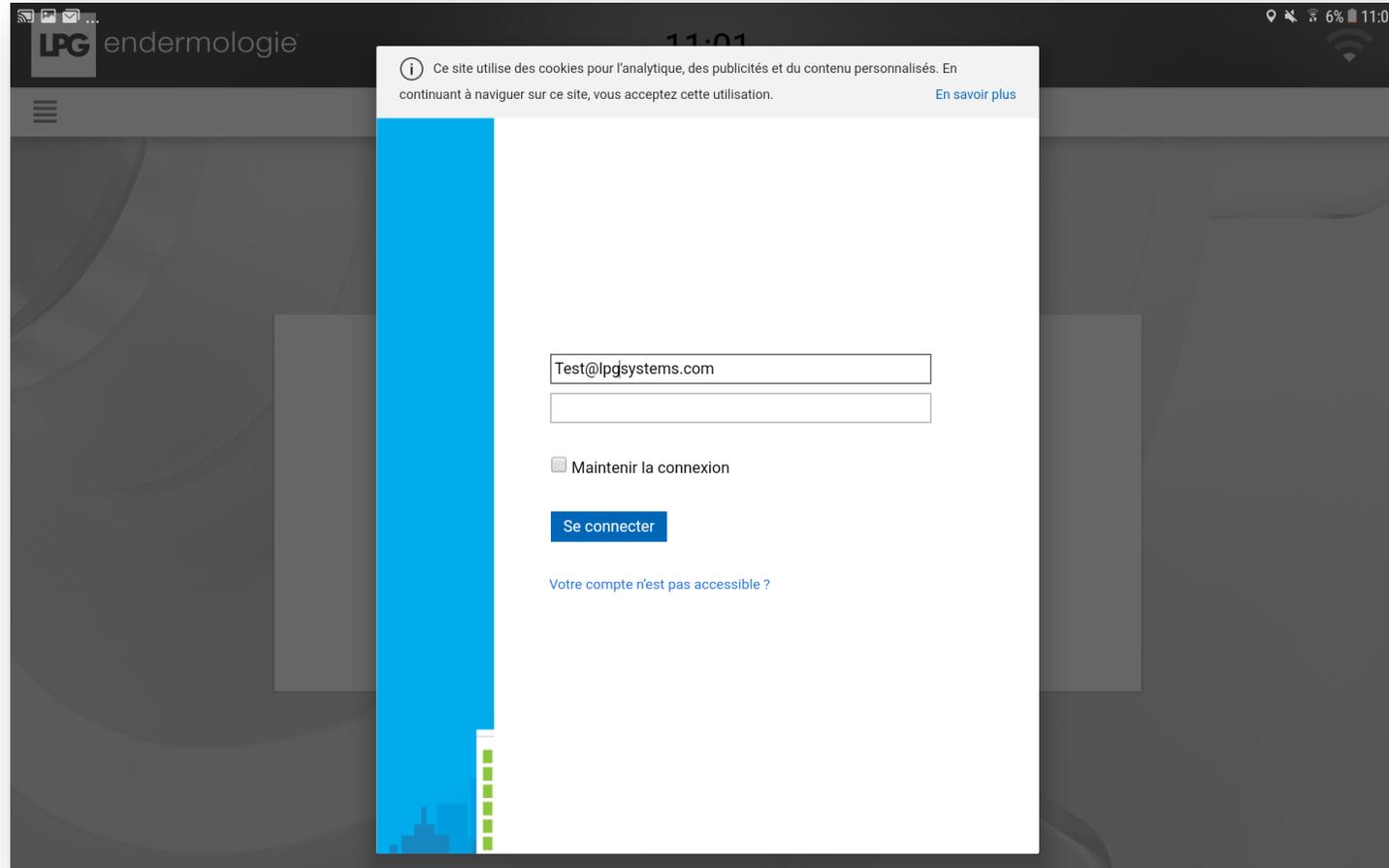
The screenshot displays the mobile application interface for LPG endermologie. At the top, there is a dark header with the LPG logo and the text 'endermologie'. Below the header, a white login form is centered. The form includes an email input field containing 'Test@lpgsystems.com', a password input field, a checkbox labeled 'Maintenir la connexion', and a blue 'Se connecter' button. A blue vertical bar is positioned to the left of the form. Above the form, a grey cookie consent banner reads: 'Ce site utilise des cookies pour l'analytique, des publicités et du contenu personnalisés. En continuant à naviguer sur ce site, vous acceptez cette utilisation.' with a link 'En savoir plus'. Below the form, a blue link reads 'Votre compte n'est pas accessible ?'. The background of the app is dark grey with a subtle pattern.

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
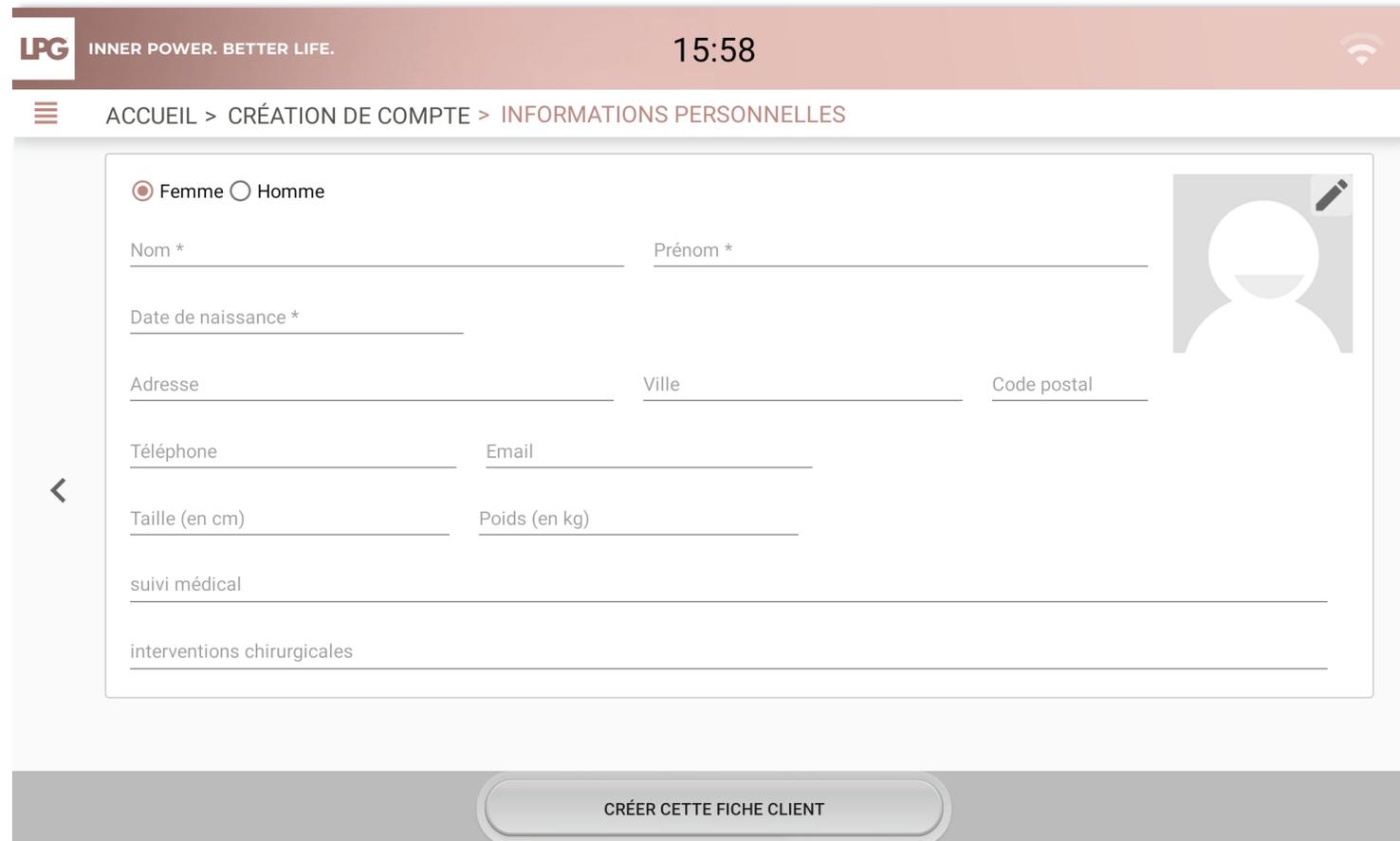
The screenshot shows the LPG mobile application interface for creating a client profile. The top navigation bar is dark red with the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "15:58", and a Wi-Fi icon. Below the navigation bar, a breadcrumb trail reads "ACCUEIL > CRÉATION DE COMPTE > INFORMATIONS PERSONNELLES". The main content area is a white form with a light gray border. At the top left of the form, there are radio buttons for "Femme" (selected) and "Homme". To the right is a placeholder for a profile picture with a pencil icon. The form contains several input fields: "Nom *" and "Prénom *" (required), "Date de naissance *", "Adresse", "Ville", "Code postal", "Téléphone", "Email", "Taille (en cm)", "Poids (en kg)", "suivi médical", and "interventions chirurgicales". A back arrow is visible on the left side of the form. At the bottom of the form, there is a large, rounded button labeled "CRÉER CETTE FICHE CLIENT".

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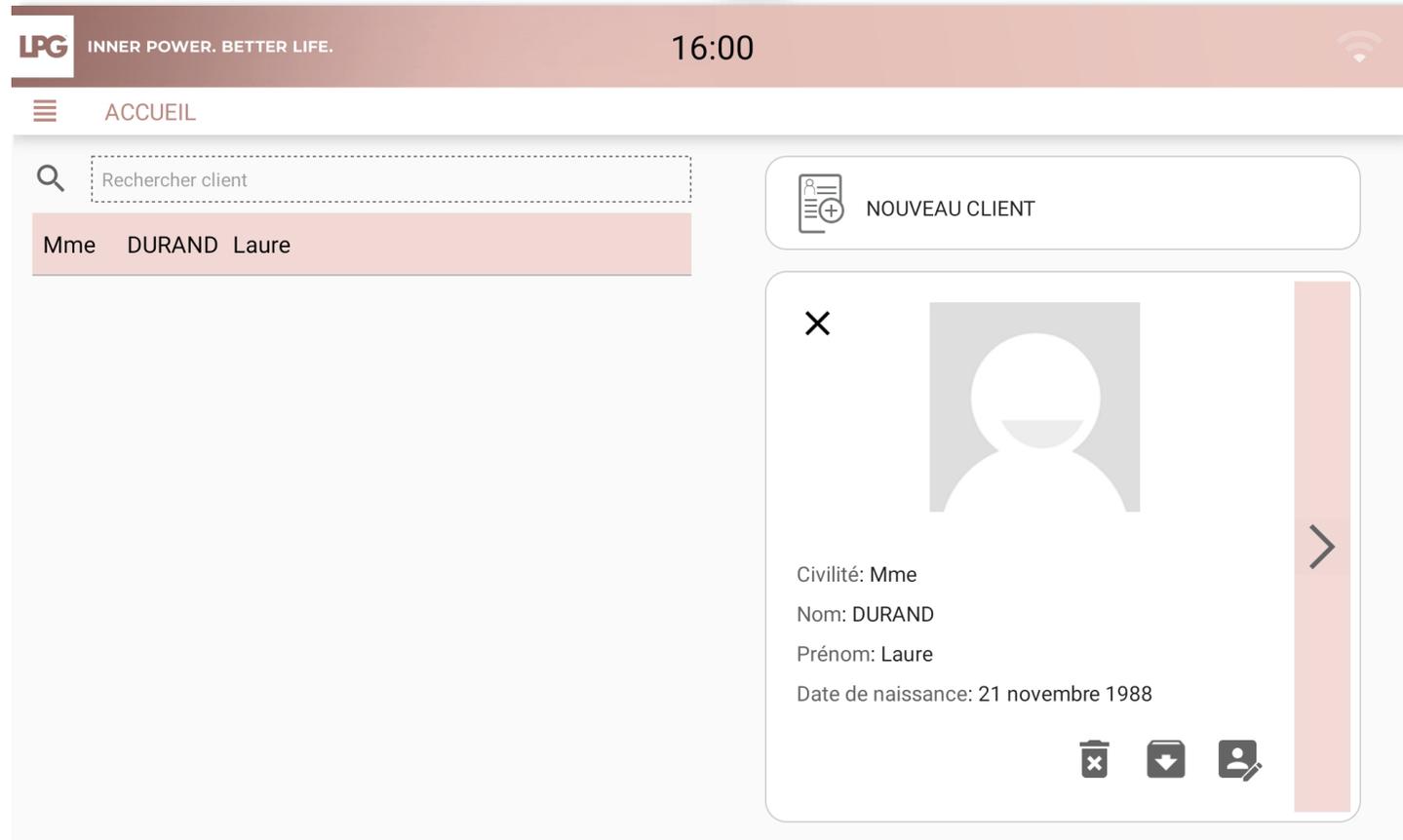
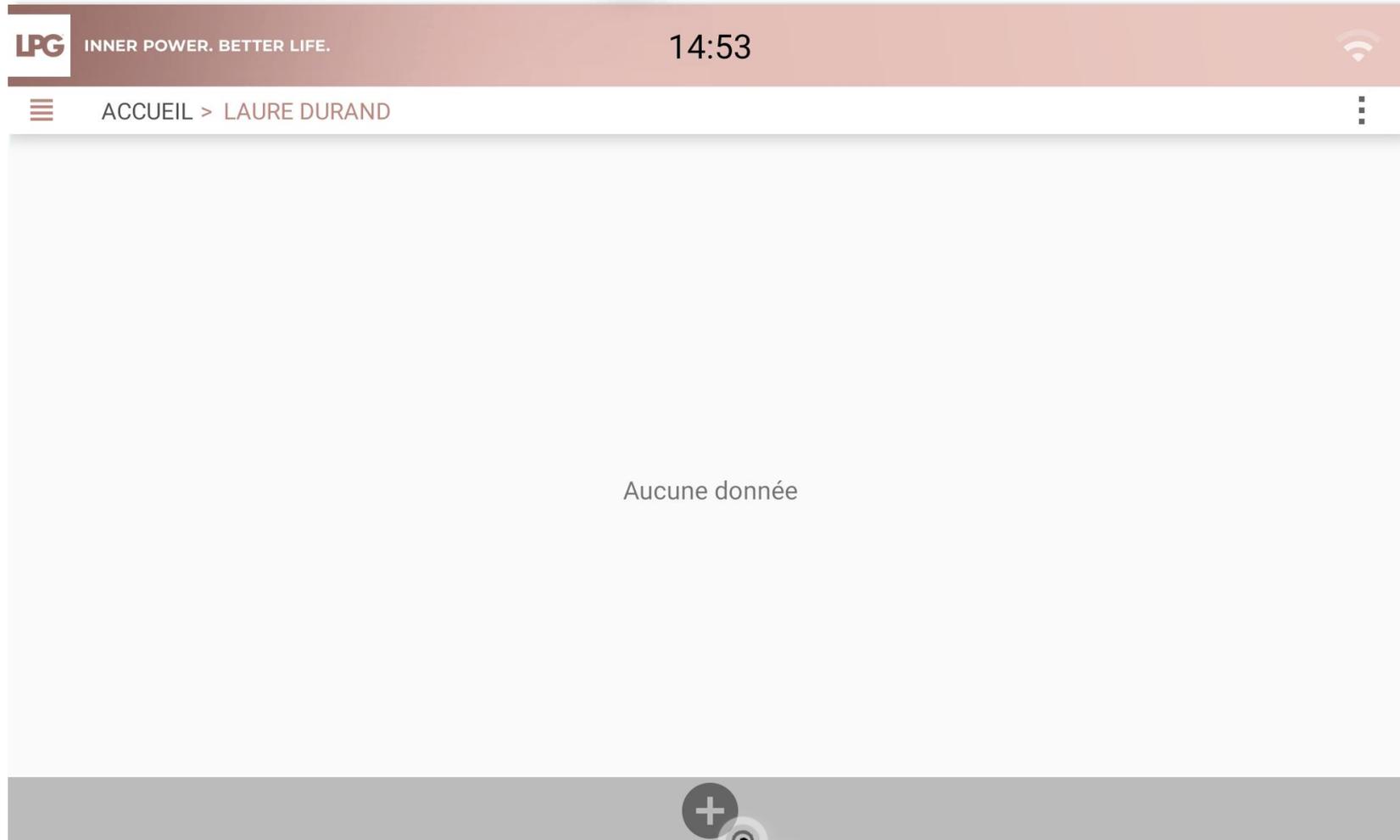
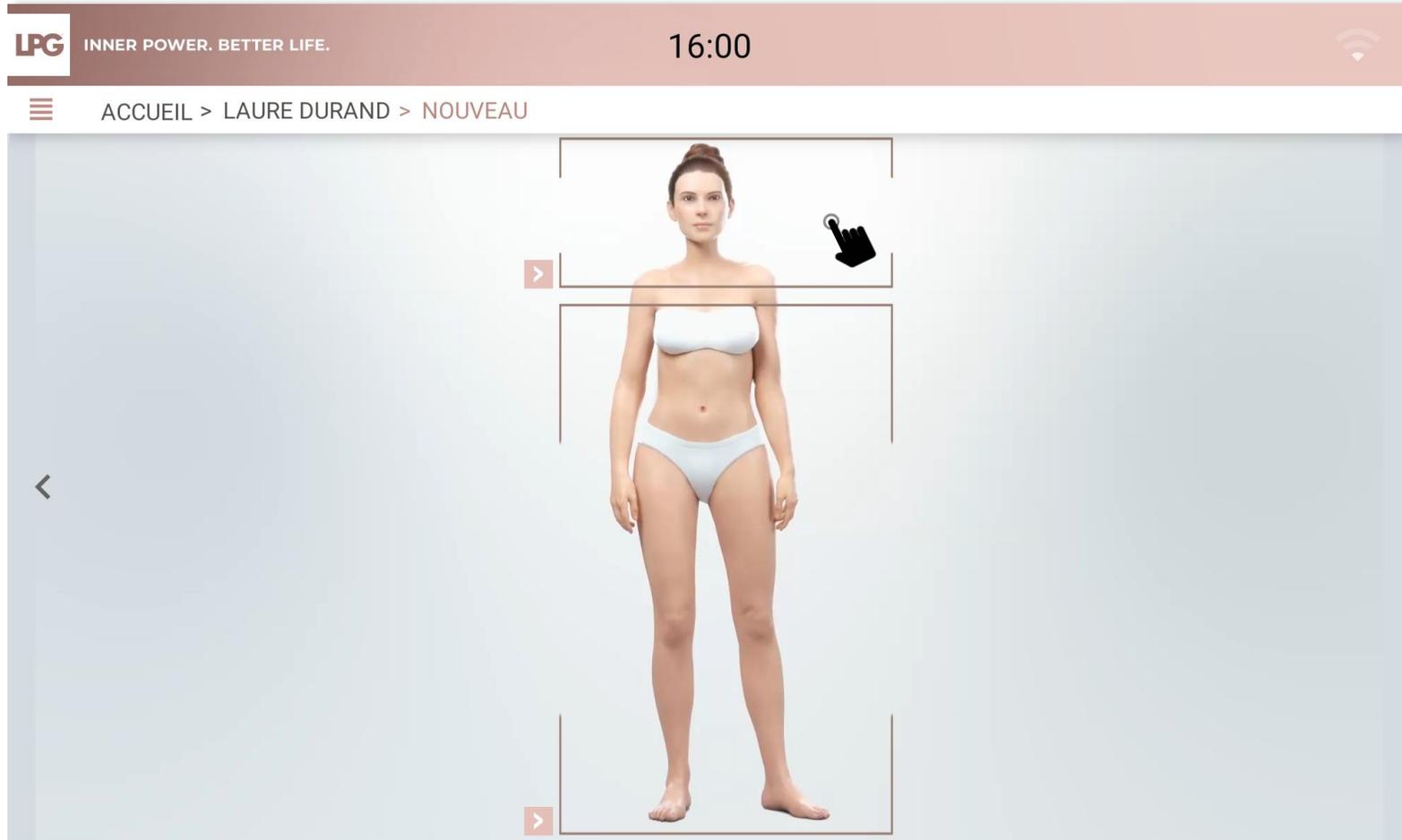
The screenshot shows a mobile application interface with a dark red header. On the left, the LPG logo is followed by the tagline "INNER POWER. BETTER LIFE.". In the center, the time "16:03" is displayed. On the right, there is a Wi-Fi signal icon. Below the header, a navigation bar contains a hamburger menu icon and the breadcrumb "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is a large white box with a thin border. At the top left of this box is the label "Objectif :". Below it is a large, empty text input field with the placeholder text "Renseignez ici les objectifs désirés". On the left and right sides of the main content area, there are grey vertical bars containing left and right arrow navigation icons, respectively.

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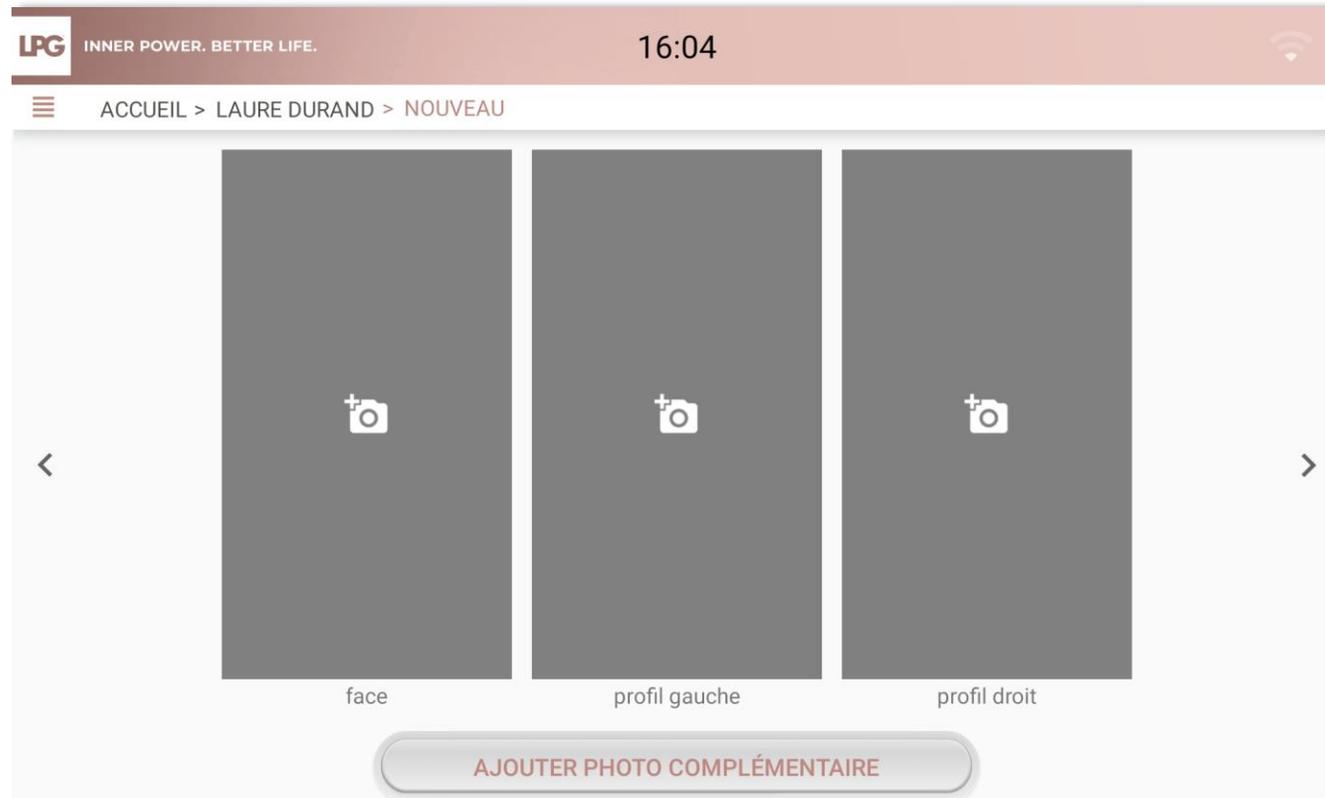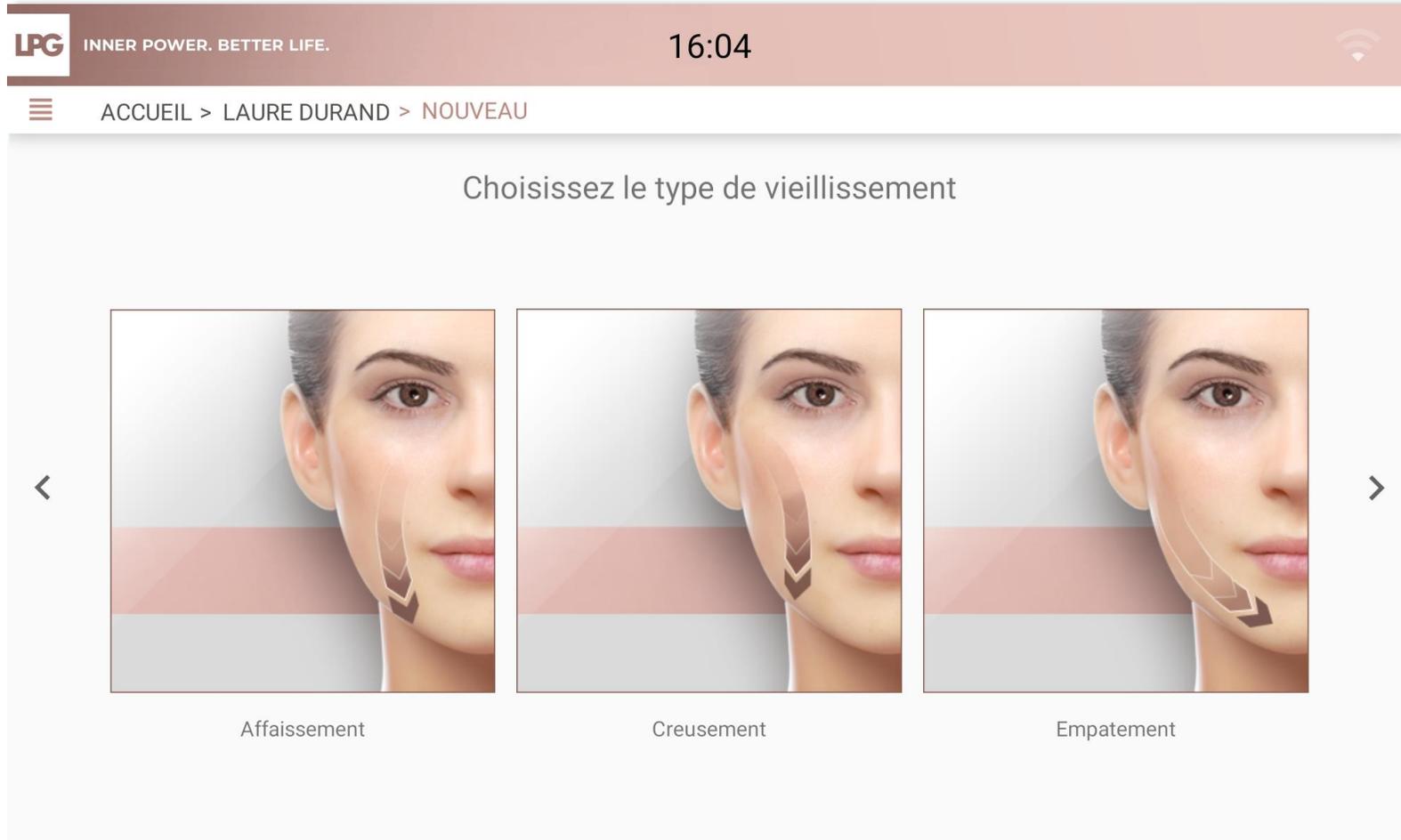
The screenshot shows a mobile application interface for skin aging evaluation. At the top, there is a header bar with the LPG logo and the tagline "INNER POWER. BETTER LIFE." on the left, the time "16:04" in the center, and a Wi-Fi icon on the right. Below the header is a navigation bar with a menu icon and the text "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is titled "Choisissez le type de vieillissement" and features three side-by-side images of a woman's face. Each image shows a different type of skin aging: "Affaissement" (sagging), "Creusement" (wrinkles), and "Empatement" (swelling). Below each image is a corresponding label. The interface includes left and right navigation arrows.

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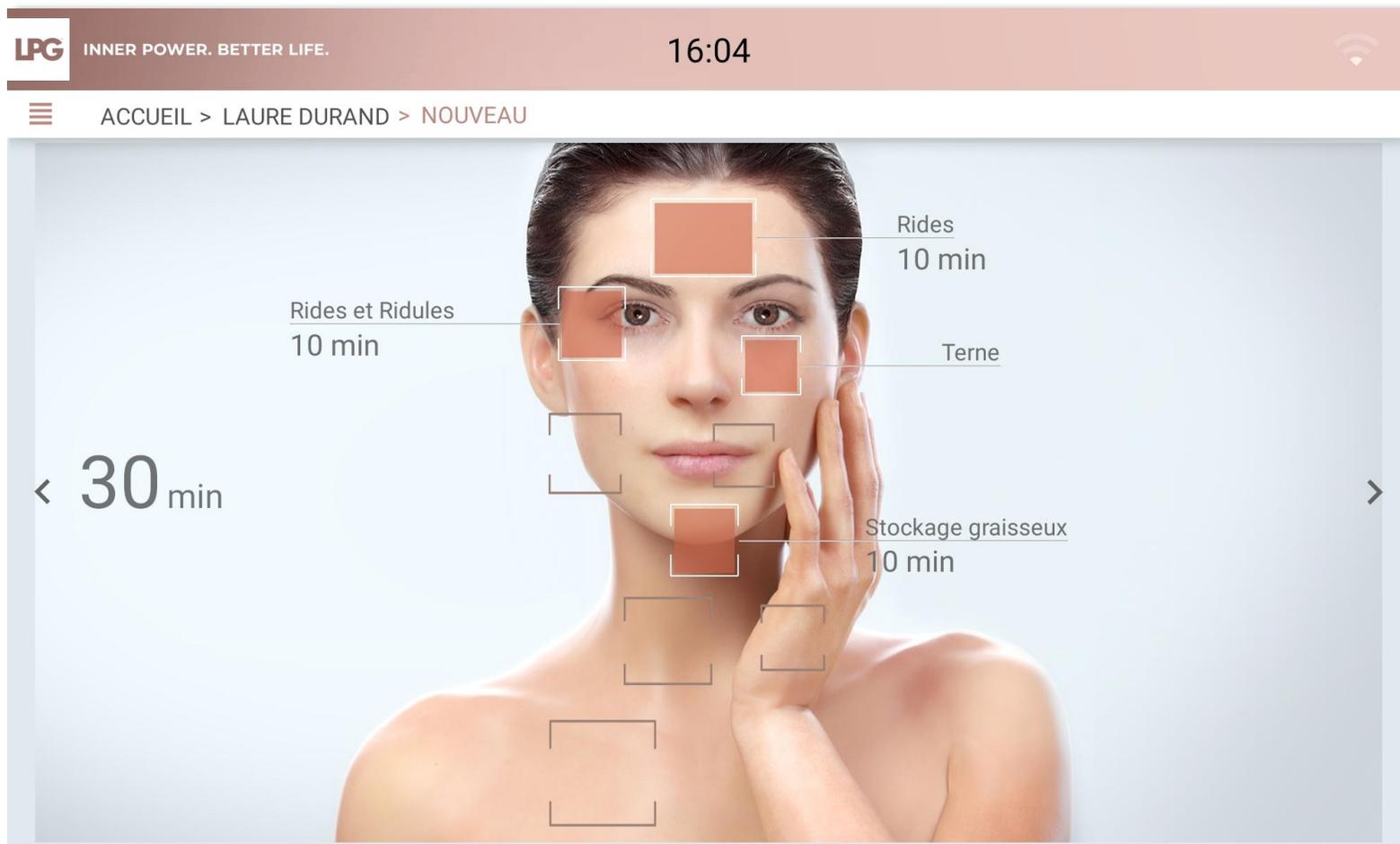
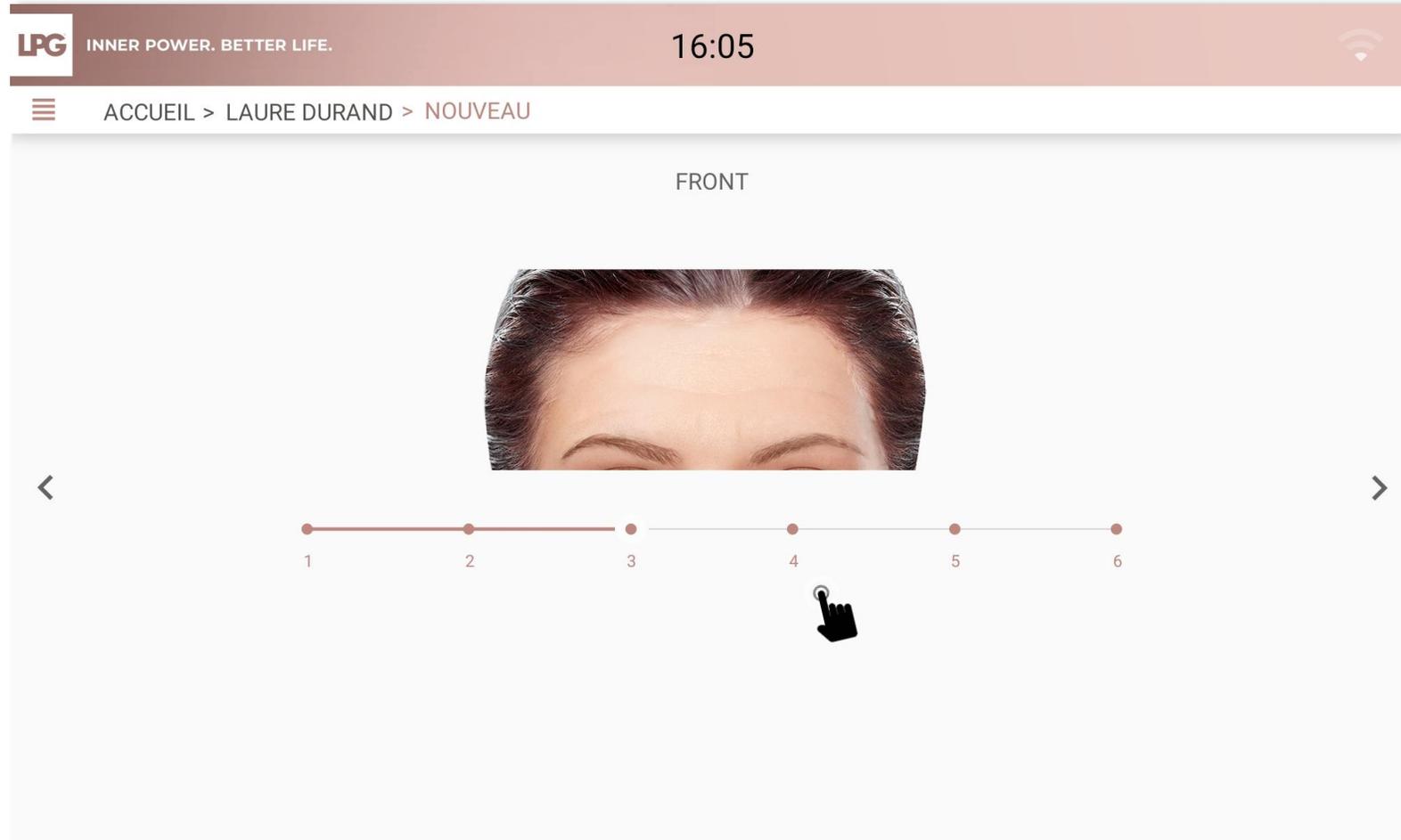
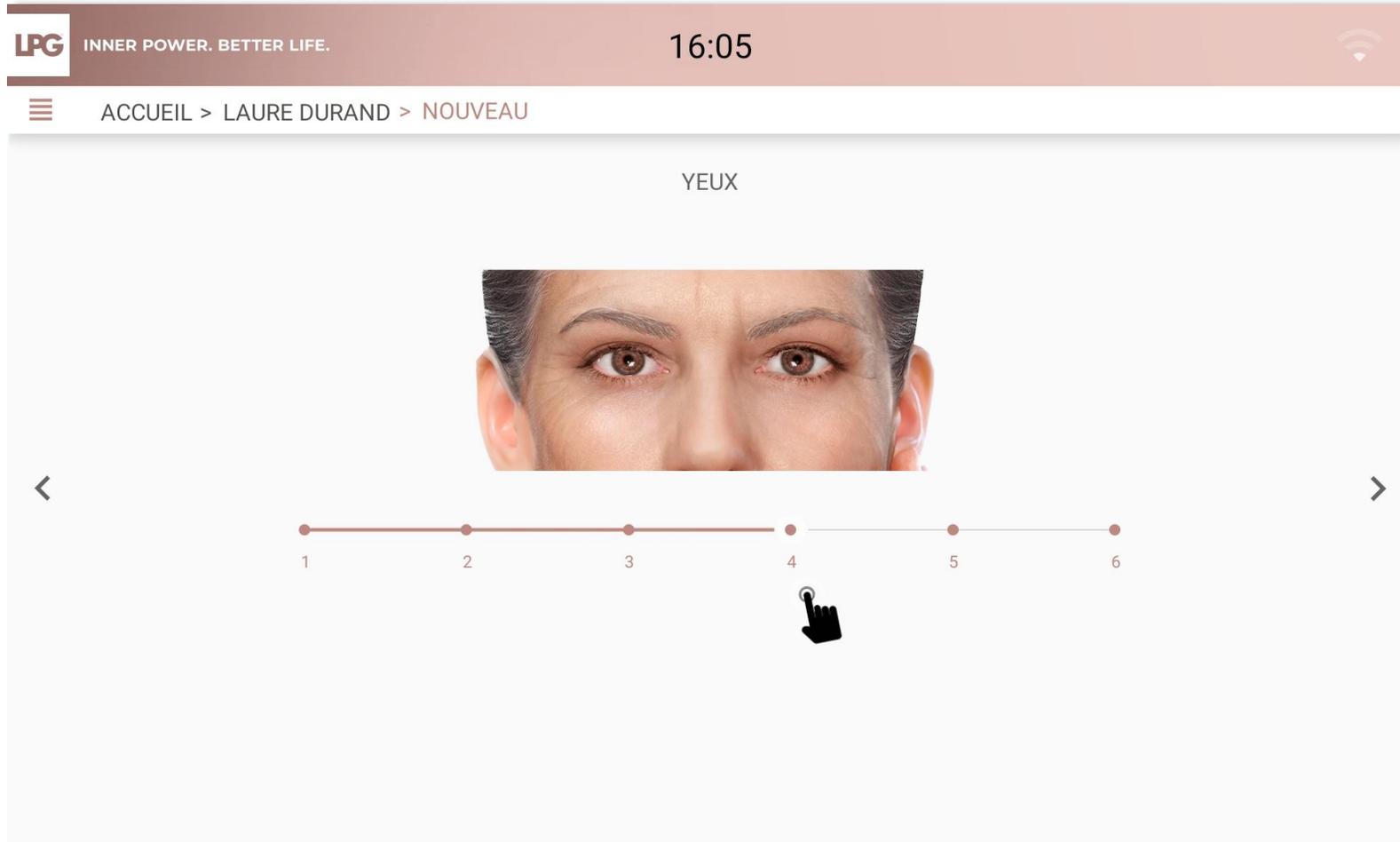
The screenshot shows a mobile application interface for eye evaluation. At the top, there is a status bar with the LPG logo and the slogan "INNER POWER. BETTER LIFE." on the left, the time "16:05" in the center, and a Wi-Fi icon on the right. Below the status bar is a navigation bar with a hamburger menu icon on the left and the text "ACCUEIL > LAURE DURAND > NOUVEAU" in the center. The main content area is titled "YEUX" and features a close-up image of a woman's eyes. Below the image is a horizontal slider with six numbered markers (1 to 6). A hand cursor is positioned over the marker "4", indicating the selected grade. Navigation arrows are visible on the left and right sides of the slider.

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The screenshot displays the LPG mobile application interface. At the top, the LPG logo and the slogan "INNER POWER. BETTER LIFE." are visible on the left, and the time "16:07" is on the right. Below the header, a navigation bar shows "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is divided into two sections: "MON PROGRAMME" on the left and "MA ROUTINE BEAUTÉ" on the right. The "MON PROGRAMME" section contains three treatment options, each with a list of specific treatments and a "Nombre de séances" field with minus and plus buttons. The "MA ROUTINE BEAUTÉ" section lists various skincare products, each with an information icon. A large "VALIDER" button is positioned at the bottom center, with a hand icon pointing to it.

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The screenshot shows a mobile application interface for a facial evaluation. At the top, there is a header bar with the LPG logo and the tagline "INNER POWER. BETTER LIFE." on the left, the time "15:04" in the center, and a Wi-Fi signal icon on the right. Below the header, a navigation bar shows a menu icon, the text "ACCUEIL > LAURE DURAND", and a vertical ellipsis menu icon. The main content area is titled "VISAGE" and dated "12 JUIN 2024". It features a list of evaluation points: "• Regard Anti-rides - Zone Frontale - Double-menton", "• Éclat", and "• Rénovateur Anti-âge". To the right of the text are two profile photos of a woman, a calendar icon, a camera icon, and a row of five LPG skincare products. At the bottom right of the main content area are icons for sharing, editing, and deleting. A large plus sign icon is centered at the bottom of the screen.

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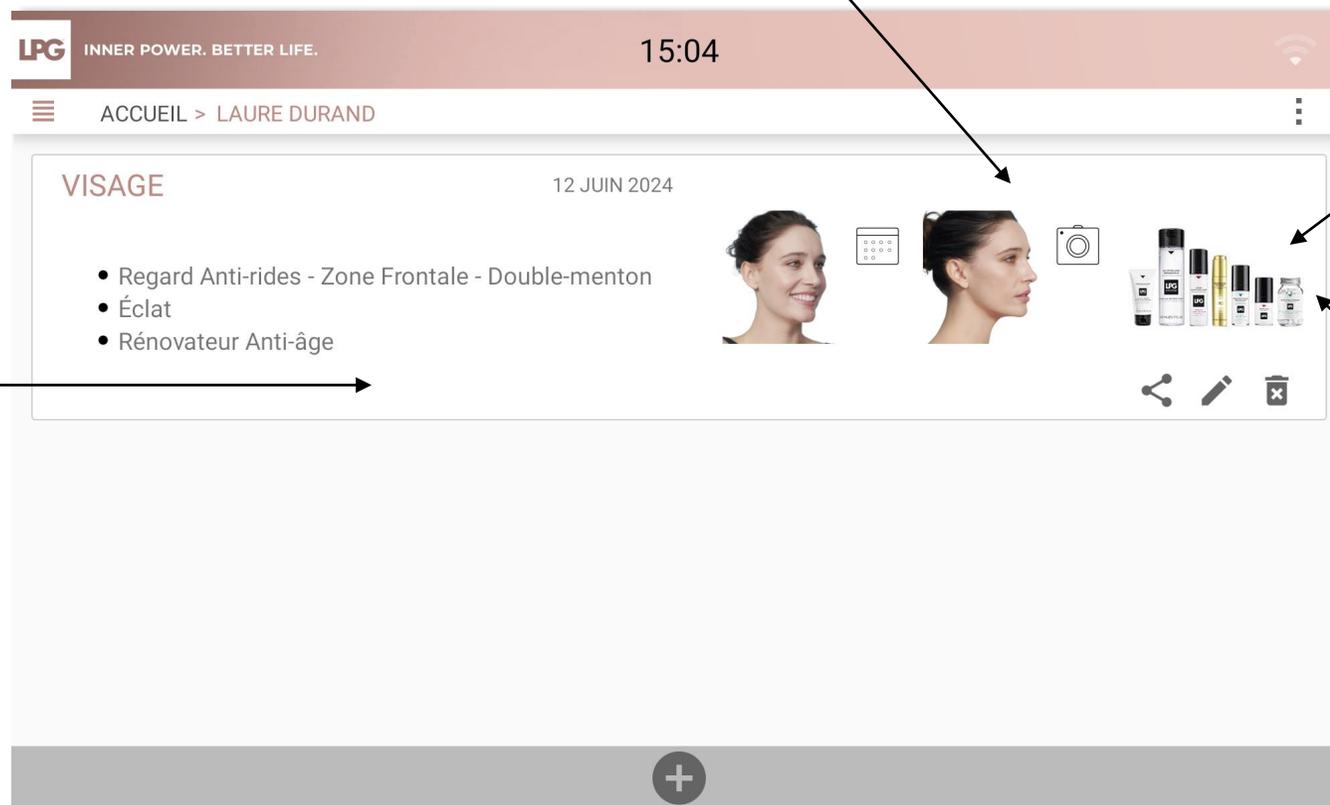
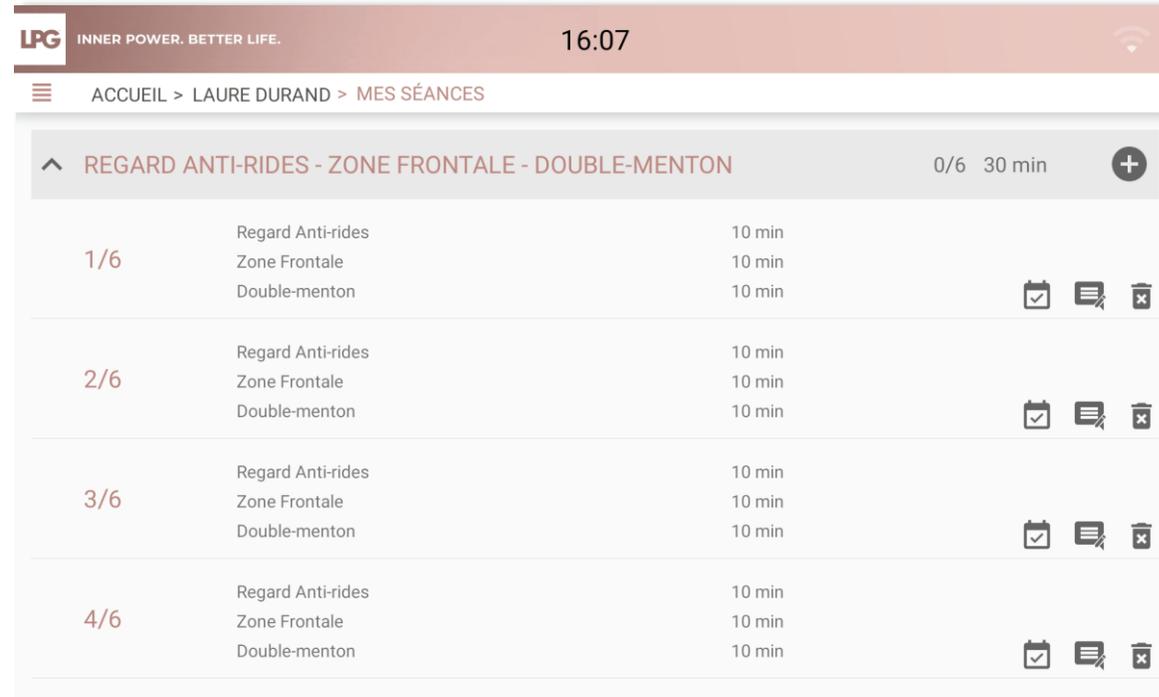
The screenshot shows a mobile application interface for a user named 'LAURE DURAND'. The top status bar displays the time '16:07' and a signal strength indicator. Below the status bar, the navigation path is 'ACCUEIL > LAURE DURAND > MES SÉANCES'. The main content area shows a session titled 'REGARD ANTI-RIDES - ZONE FRONTALE - DOUBLE-MENTON' with a progress indicator '0/6' and a duration of '30 min'. The session is divided into four rows, each representing a sub-session. Each row contains the sub-session number (1/6, 2/6, 3/6, 4/6), the sub-session name, and the duration (10 min). To the right of each row are three icons: a calendar icon, a comments icon, and a trash icon.

Progress	Sub-session Name	Duration	Calendar	Comments	Trash
1/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min			
2/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min			
3/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min			
4/6	Regard Anti-rides Zone Frontale Double-menton	10 min 10 min 10 min			

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The screenshot displays the LPG mobile application interface. At the top, the header includes the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "16:08", and a Wi-Fi signal icon. Below the header, a breadcrumb trail reads "ACCUEIL > LAURE DURAND > MA ROUTINE BEAUTÉ". The main content area is split into two columns. The left column features a category "REGARD ANTI-RIDES - ZONE FRONTALE - DOUBLE-MENTON" with an upward arrow icon. Underneath, five product items are listed, each with a small image and text: "EAU MICELLAIRE PRÉPARATRICE" (highlighted in a light orange box), "CRÈME YEUX", "CRÈME LISSANTE REPULPANTE", "GEL-CRÈME LISSANT REPULPANT", and "SOIN FERMETÉ AFFINANT". The right column is dedicated to the selected product, "EAU MICELLAIRE PRÉPARATRICE", featuring a larger product image, a list of benefits: "• Démaquille et nettoie les impuretés" and "• Hydrate et apaise", and usage instructions: "Imbibez un coton et nettoyez le visage, les yeux et le cou, matin et soir. Séchez la peau délicatement sans rincer."

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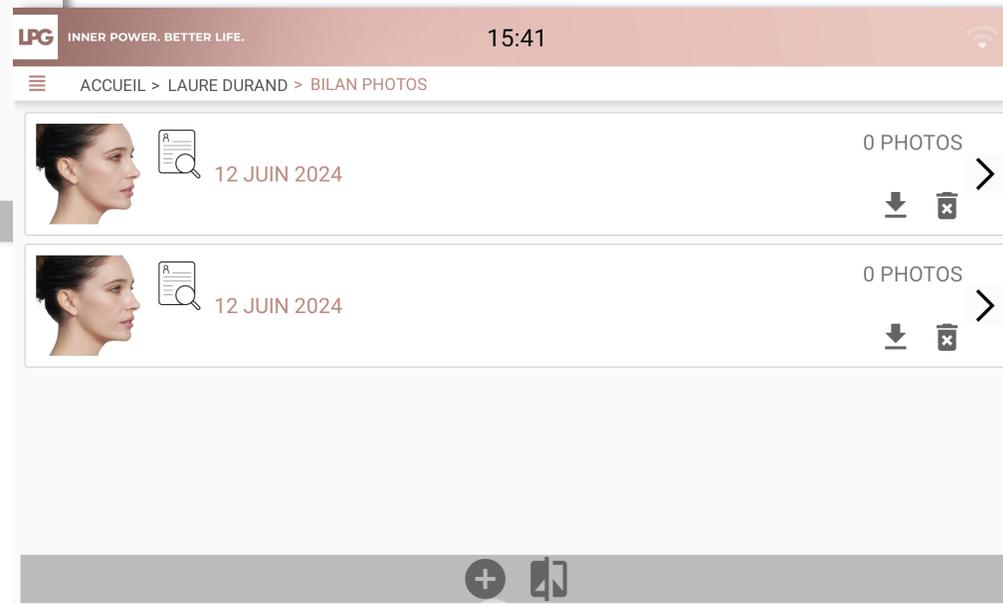
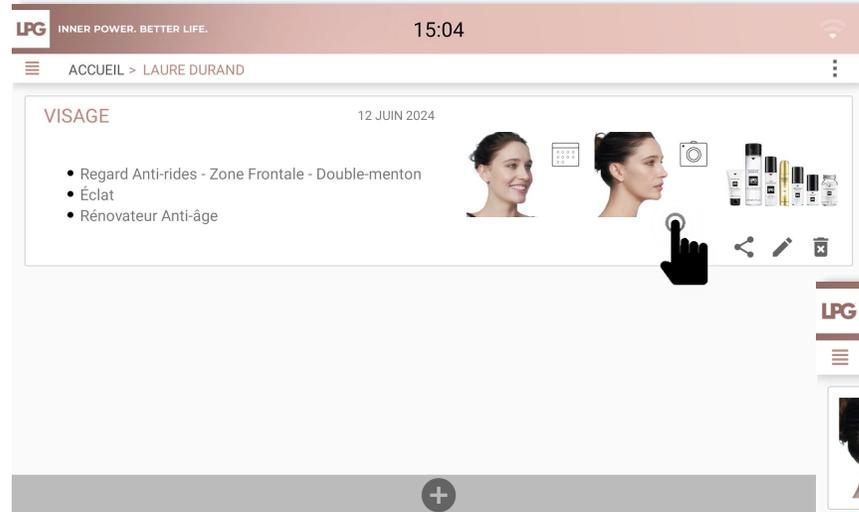
The screenshot shows a mobile application interface with a dark red header. The header contains the LPG logo and the slogan 'INNER POWER. BETTER LIFE.' on the left, the time '16:09' in the center, and a Wi-Fi signal icon on the right. Below the header is a navigation bar with a hamburger menu icon on the left and the breadcrumb text 'ACCUEIL > LAURE DURAND > BILAN PHOTOS'. The main content area is a light gray rectangle containing two text input fields. The first field is labeled 'Objectif :'. The second field is larger and contains the placeholder text 'Ajouter votre commentaire ici'. On the left and right sides of this second field, there are small black arrow icons pointing left and right respectively.

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