



INNER POWER. BETTER LIFE.

SUMMARY


○ Start the application	p. 3
○ Create a client account	p. 4
○ Face evaluations and treatment programs	p. 9
○ Photo assessment	p. 25
○ Compare photo assessments	p. 31
○ Send treatment program to your client	p. 34
○ Send treatment program to your MOBILIFT M6® CONNECT	p. 37
○ Free sessions	p. 42
○ Backup	p. 44
○ Updates	p. 47
○ Data import (LPG SCAN tablet to the new one)	p. 51
○ Data export	P. 53



INNER POWER.
BETTER LIFE.

START THE APPLICATION

Select the **endermoscan** icon on your tablet's home screen to launch the application.

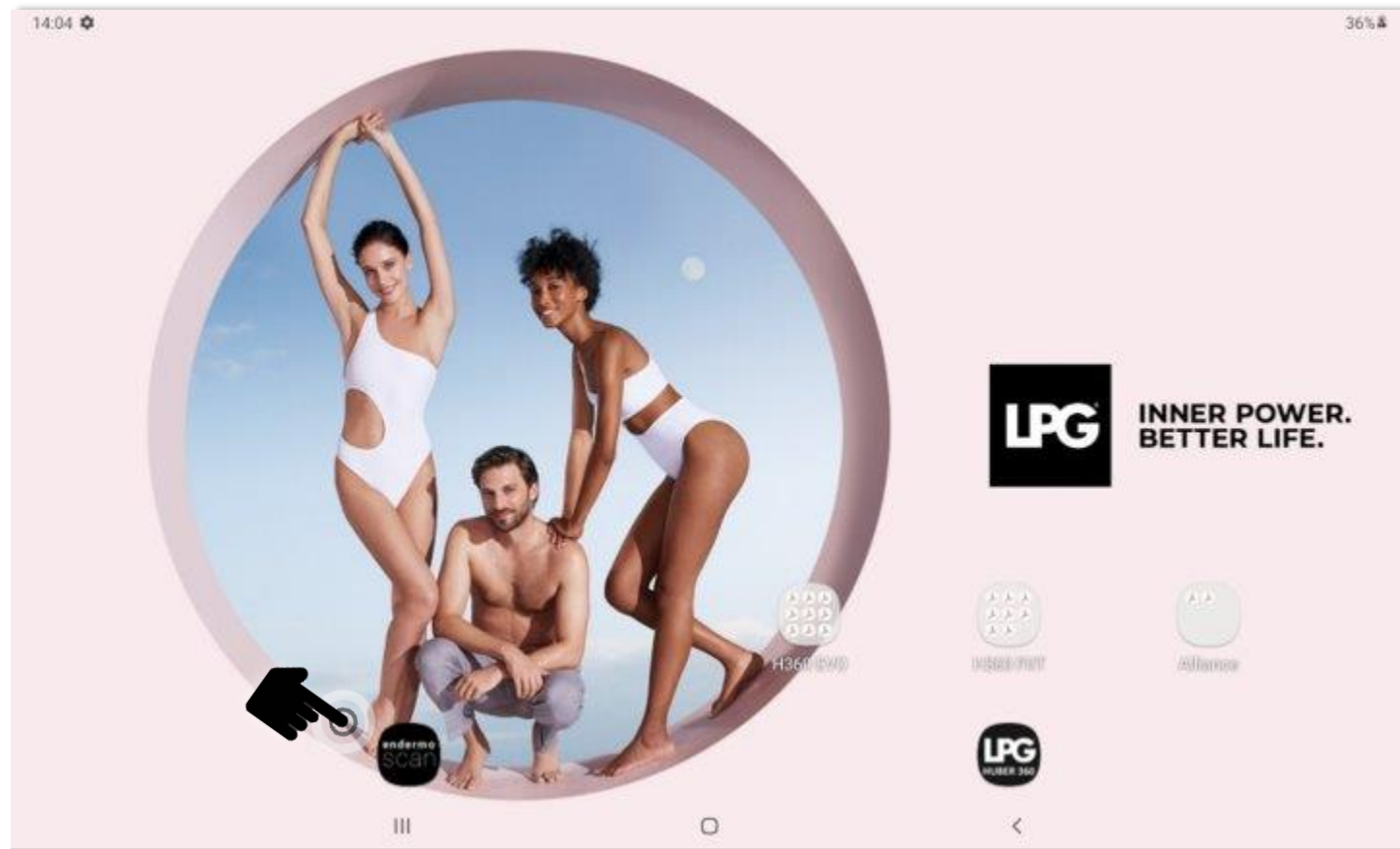




CREATE A CLIENT ACCOUNT



INNER POWER.
BETTER LIFE.

CREATE CLIENT ACCOUNT

Once the application is open, select **NEW CLIENT** on the home screen.

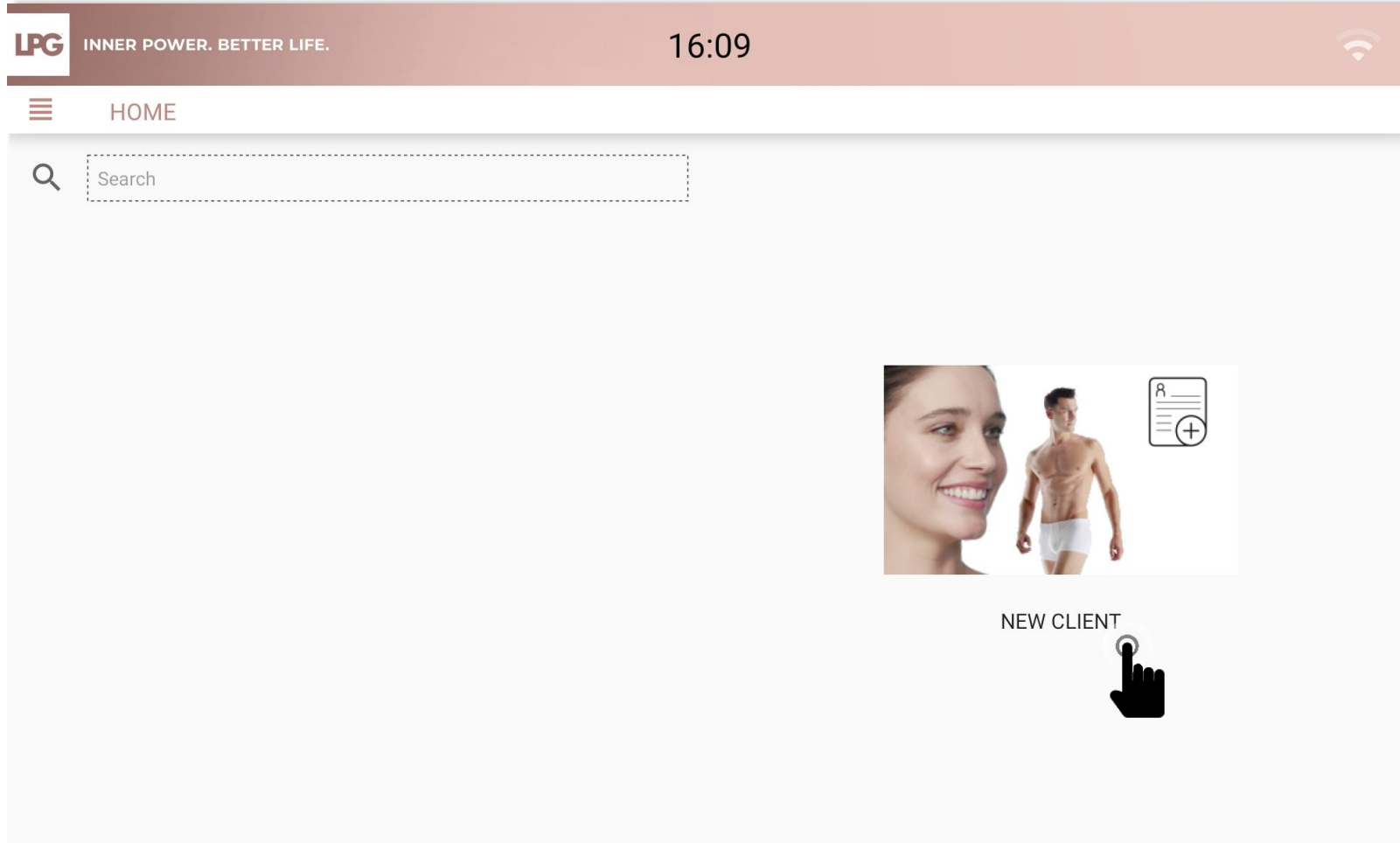

INNER POWER.
BETTER LIFE.

CREATE CLIENT ACCOUNT

After discussing all contraindications with your client, please tick the «**Contraindications for endermologie® treatments were discussed during your evaluation**» box to start filling in the client's profile.

LPG

INNER POWER. BETTER LIFE.

16:09

HOME > CREATE AN ACCOUNT > CONTRAINDICATIONS

INFORMATION ON ENDERMOLOGIE® TREATMENTS

Strict general contraindications:

- Progressing cancer (except with written medical recommendation)
- Anticoagulant treatment
- Blood disease
- Disease with inflammatory eruption
- Infection or skin rash

Body endermologie® contraindications:

- Pregnancy (except for lower limbs)
- Phlebitis
- Organ transplant
- Kidney failure

Face endermologie® contraindications:

- Vitiligo
- Infectious, inflammatory acne
- Herpes

If in doubt, ask for a written medical opinion.

☐ Contraindications for endermologie® treatments were discussed during your evaluation.

Localized Contraindications:

- Rosacea
- Varicosity and varicose veins
- Vein surgery (for 2 months)
- Areas that underwent plastic surgery (for 6 months)
- Skin lesions and wounds
- Keloid, hypertrophic, or recent scars
- Inguinal or abdominal hernias
- Lipoma
- Raised moles
- Filler injections (for 2 weeks)
- Botox® injections (for 2 weeks)
- Piercing
- Angioma
- Subcutaneous implants
- Gold and serrated thread lifts
- Acute osteo-articular or muscle injury

CREATE CLIENT ACCOUNT

Fill in client information.

Click on **CREATE THIS CLIENT PROFILE**.

THE BMI is calculated automatically after filling in the height and the weight.

LPG INNER POWER. BETTER LIFE. 16:10

HOME > CREATE AN ACCOUNT > PERSONAL INFORMATION

☒ Woman ☐ Man

Last Name * First Name *

Date of Birth *

Address City Zip Code

Phone Email

Height (in cm) Weight (in kg)

medical follow-up



surgical procedures


CREATE THIS CLIENT PROFILE



INNER POWER.
BETTER LIFE.

CREATE CLIENT ACCOUNT

- To view or to edit a client profile, select your client's name and click on 
- To search for a client in the existing database, type the client's name in the **SEARCH bar**.
- To archive a client profile, click on 

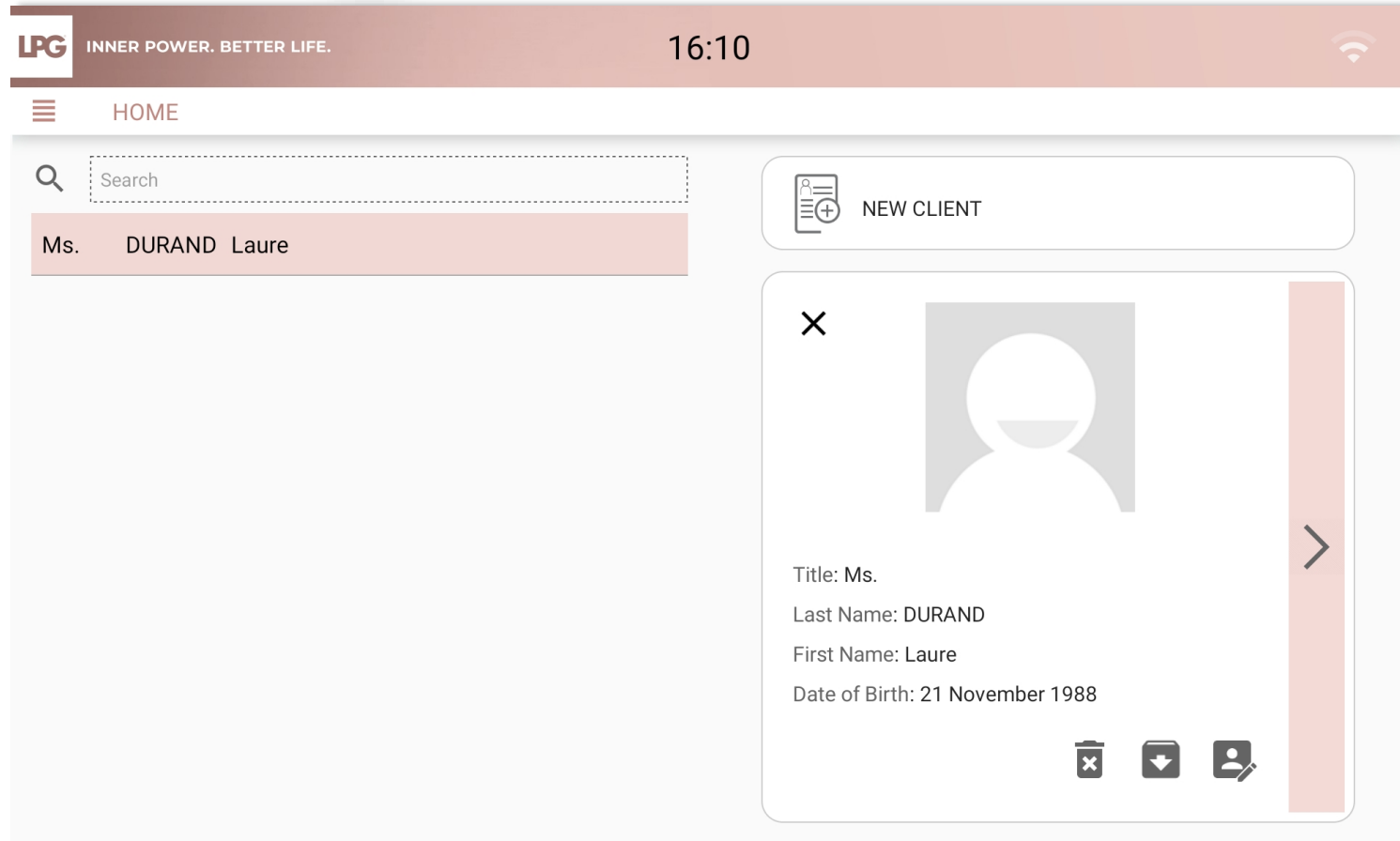




FACE EVALUATION & TREATMENT PROGRAMS



INNER POWER.
BETTER LIFE.

FACE EVALUATION

To start a face evaluation, click on the face area.

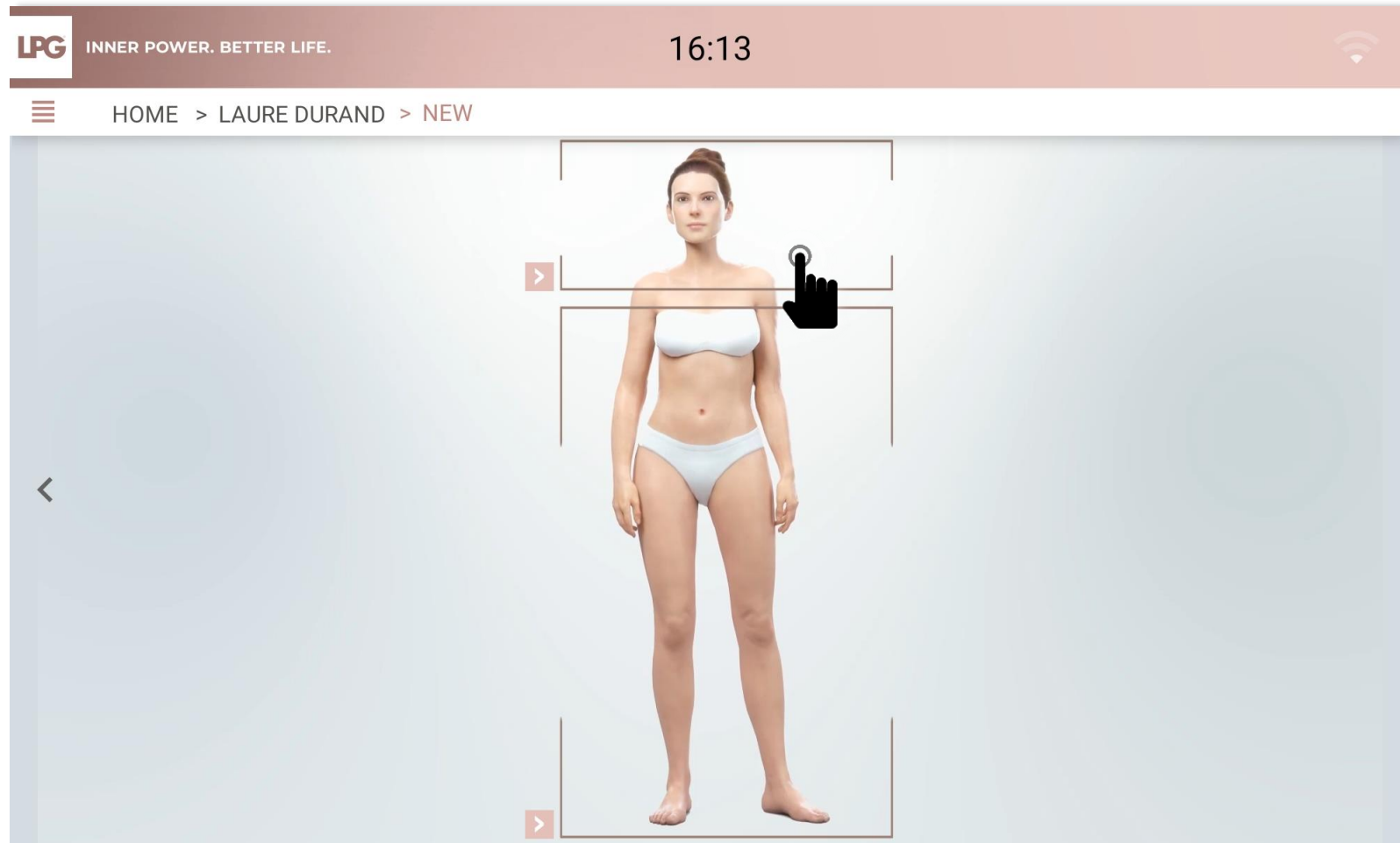

INNER POWER.
BETTER LIFE.

FACE EVALUATION

Fill in client's desired objectives.


The screenshot shows a mobile application interface. At the top, a dark red header bar contains the 'LPG' logo on the left, the text 'INNER POWER. BETTER LIFE.' in the center, the time '16:13' on the right, and a Wi-Fi signal icon on the far right. Below the header, a white navigation bar displays a hamburger menu icon on the left and the breadcrumb 'HOME > LAURE DURAND > NEW' in the center. The main content area is a large white rectangle with a thin border. Inside this rectangle, the word 'Objective:' is at the top left, followed by a large text input field containing the placeholder text 'Fill in desired objectives here'. On the left and right sides of the main content area, there are vertical grey bars, each featuring a black arrow pointing outwards (< and > respectively).



INNER POWER.
BETTER LIFE.


FACE EVALUATION

- Take 3 photos (right profile, face, left profile). Maintain you tablet in vertical (portrait) format. The photo must be taken when level sensors are green. Report to the integrated markers of the photo assistant to position your client properly. Once the picture is taken, click on **REGISTER** or **CANCEL** if you are not satisfied.
- To take other pictures, click on **ADD ADDITIONAL PICTURE** and choose a title.

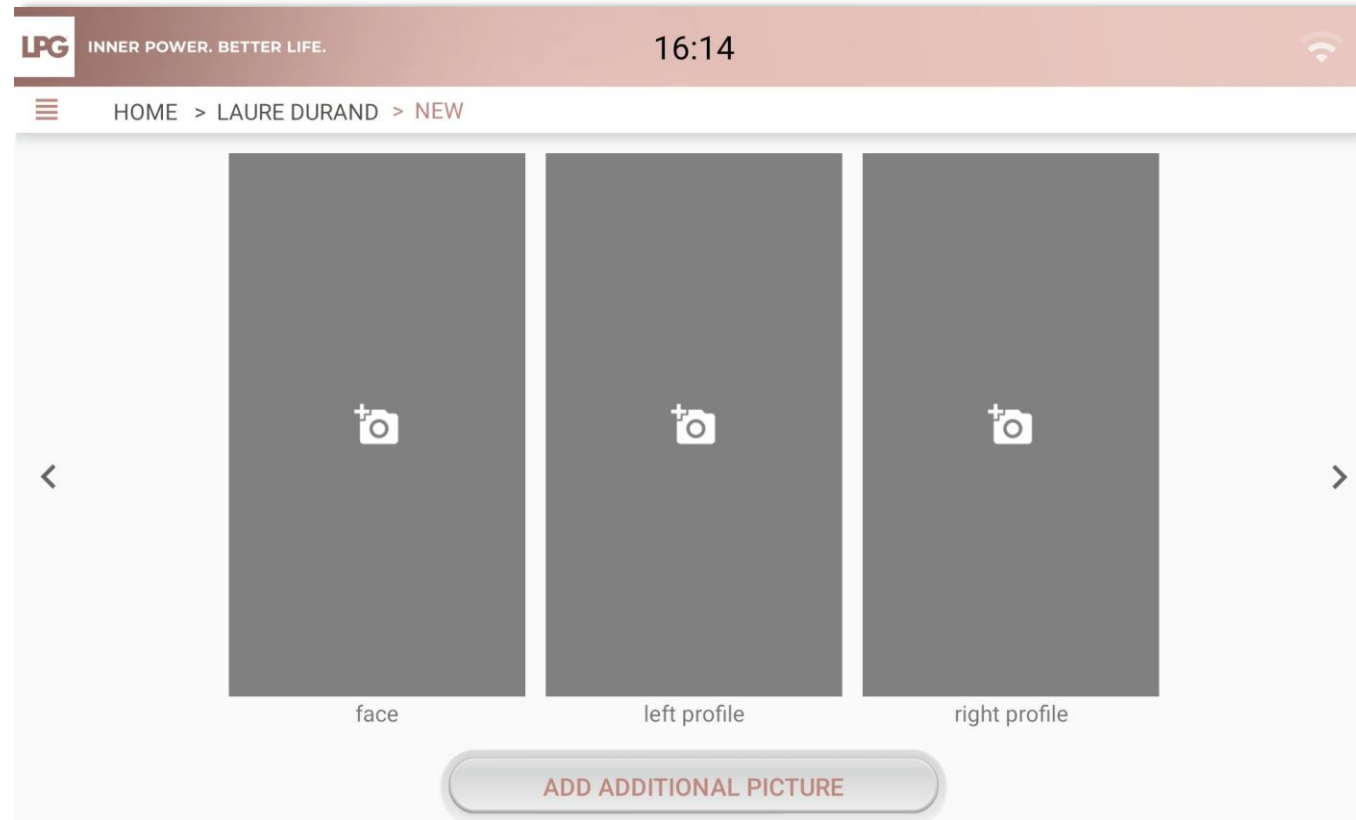

FACE EVALUATION

Select the corresponding skin aging type.

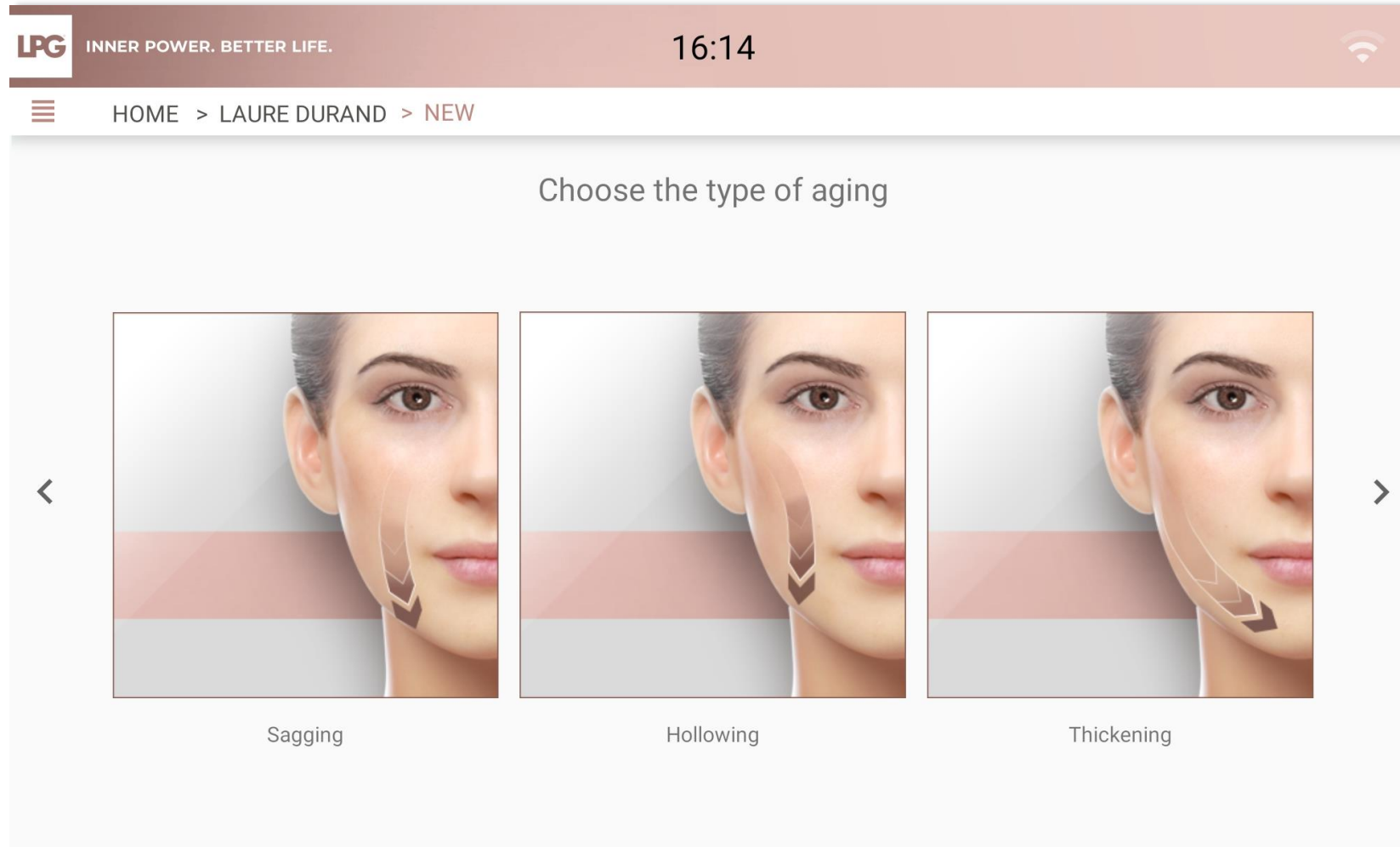

INNER POWER.
BETTER LIFE.


FACE EVALUATION

Select up to 4 priority areas.
For each selected area, chose the main objective.



FACE EVALUATION


Move the cursor to select the corresponding level.

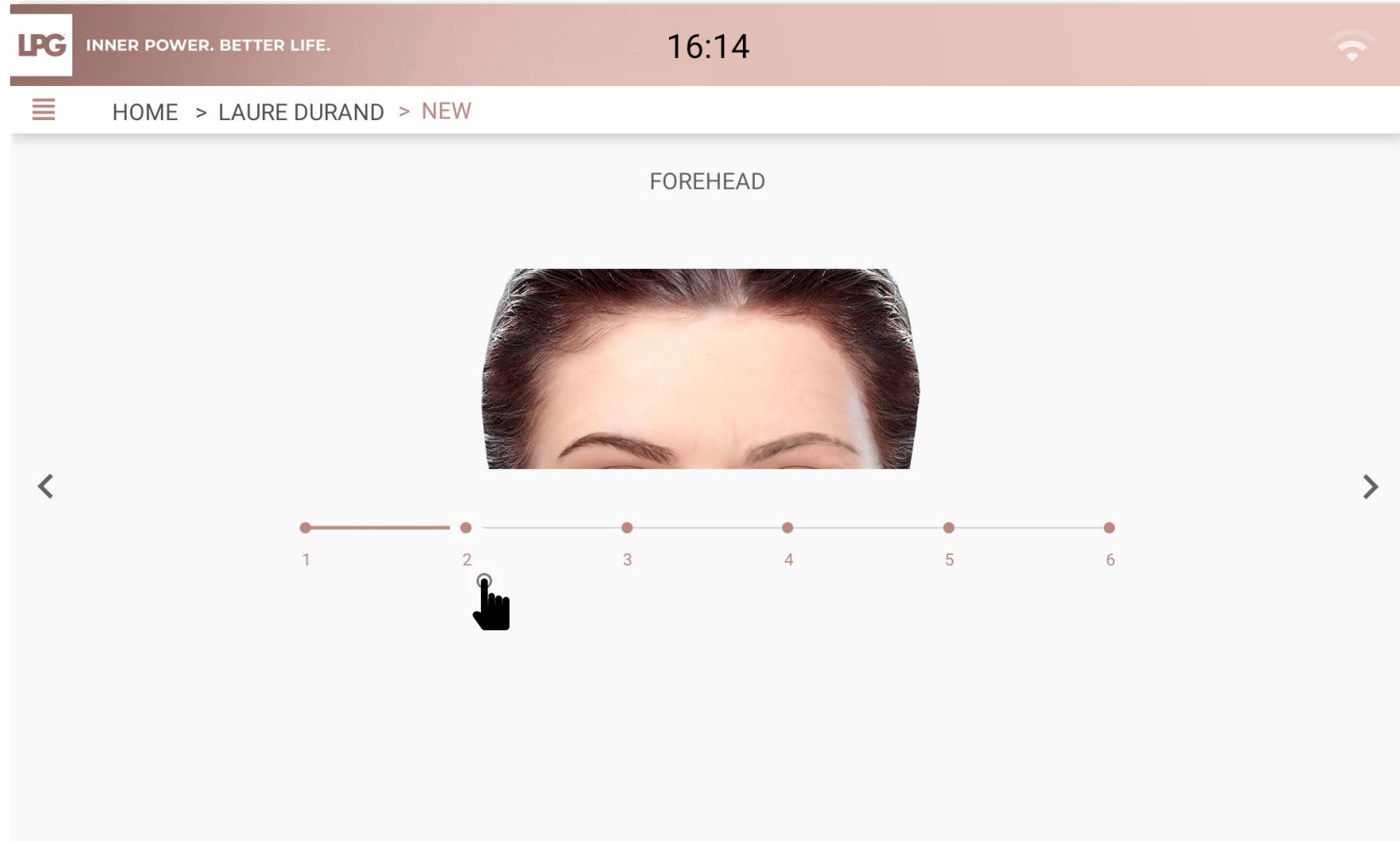

INNER POWER.
BETTER LIFE.

FACE EVALUATION


Move the cursor to select the corresponding level.

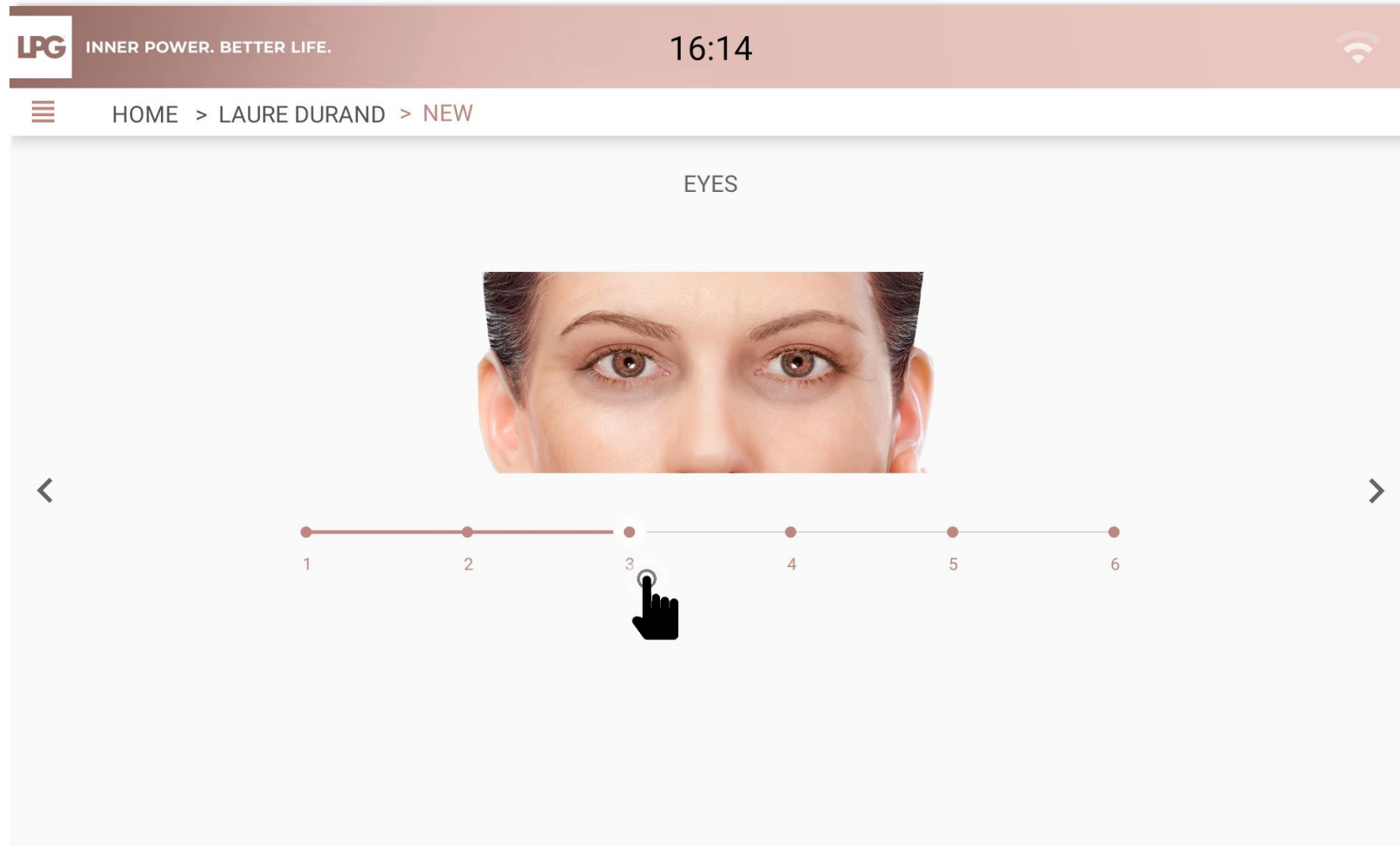

INNER POWER.
BETTER LIFE.

FACE EVALUATION


Move the cursor to select the corresponding level.

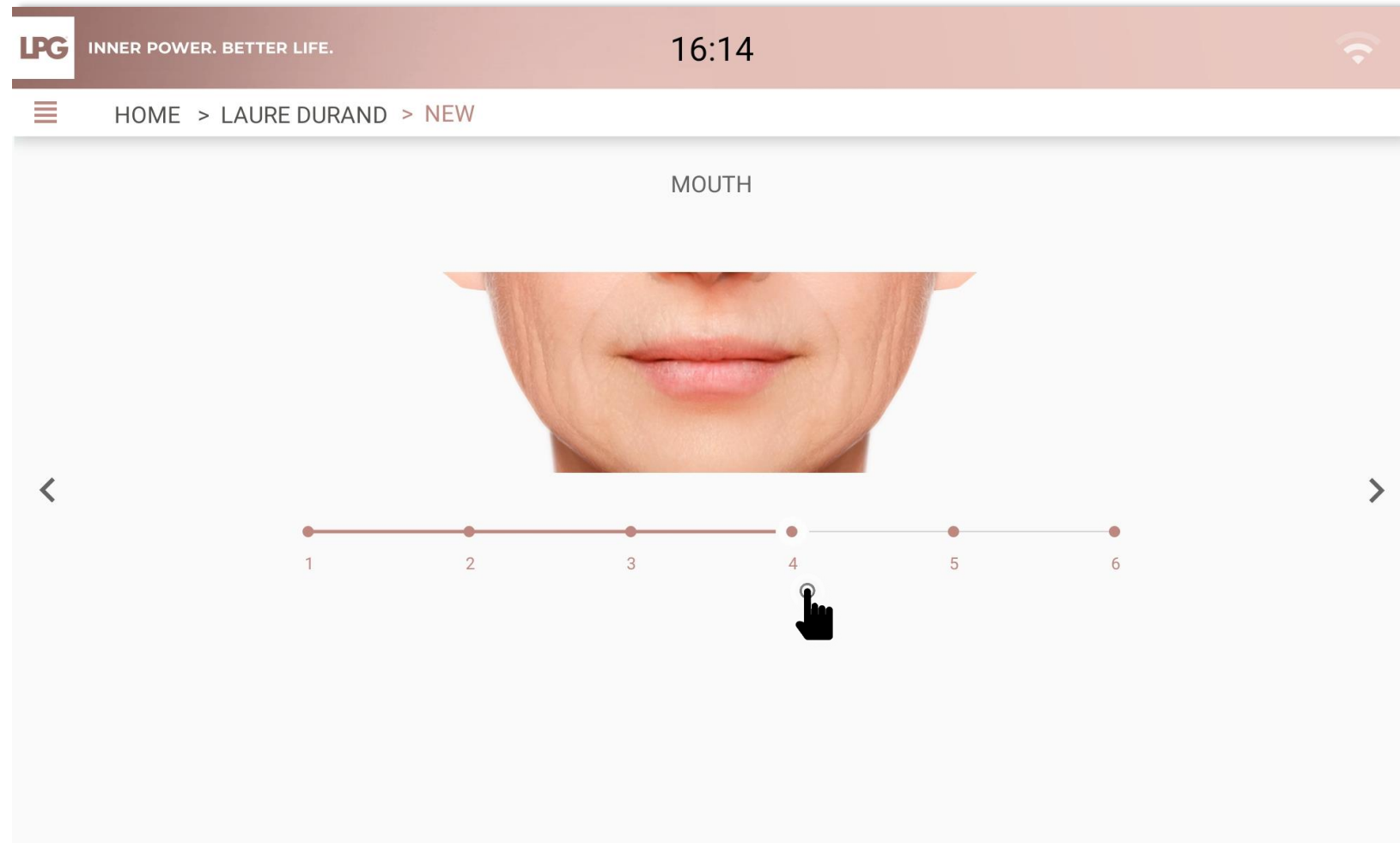

INNER POWER.
BETTER LIFE.

FACE EVALUATION

Move the cursor to select the corresponding level.

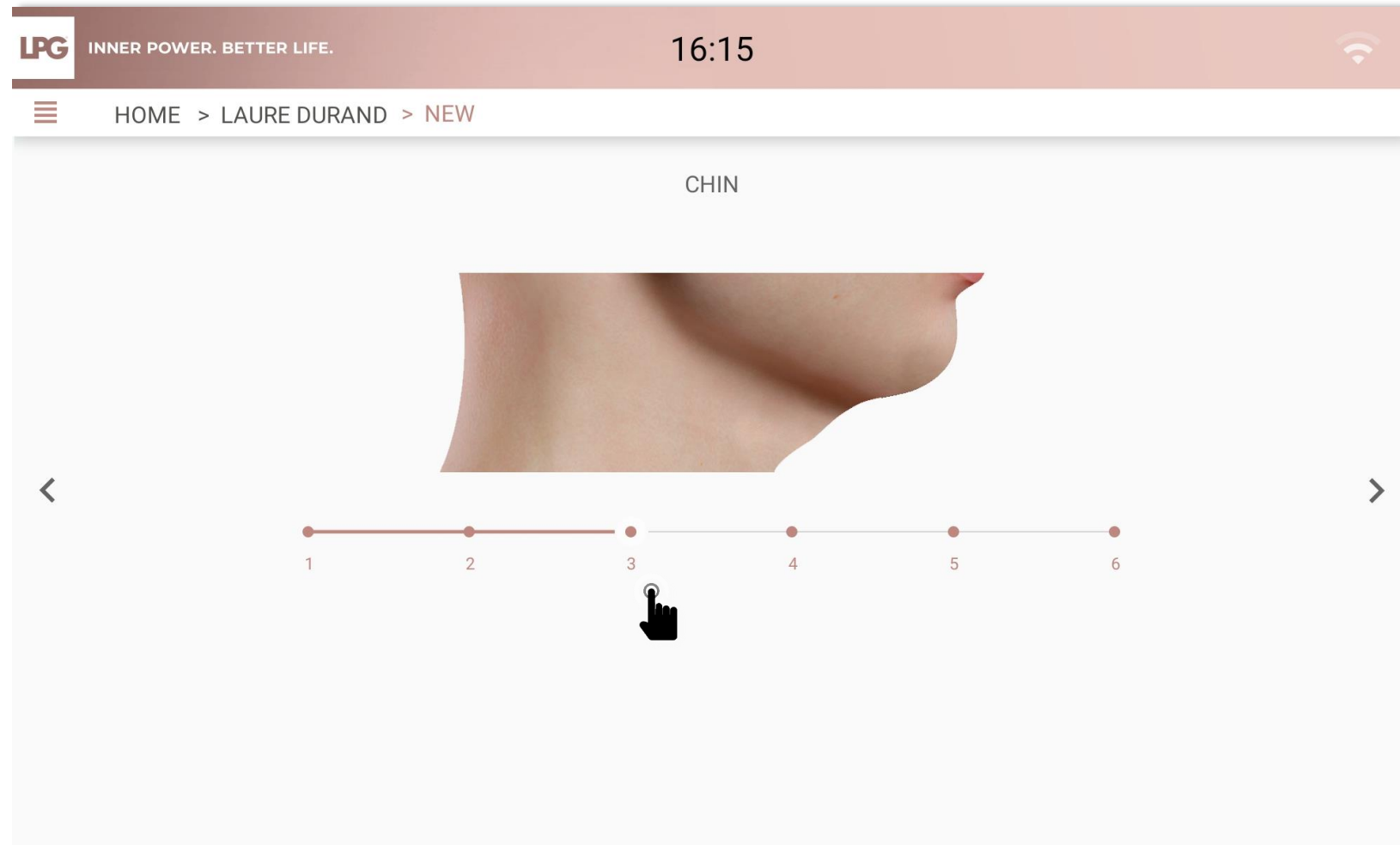

INNER POWER.
BETTER LIFE.

FACE EVALUATION

Select most adapted protocol(s) according to your client's objectives.

LPG

INNER POWER. BETTER LIFE.

16:15

HOME > LAURE DURAND > NEW

Recommended endermologie® protocols

Targeted treatment(s)

- ☒ Eye Wrinkles
- ☒ Forehead
- ☐ Mouth Area
- ☐ Double Chin

Classic treatment(s)

- ☐ Age-defying Skin Toning
- ☐ Age-defying Replumping
- ☒ Glow
- ☐ Age-defying Resculpting



Signature treatment(s)

- ☐
- ☐
- ☐
- ☐
- ☐

FACE EVALUATION

Final evaluation and protocol & cosmetics recommendation

- Adjust the number of recommended sessions.
- To confirm the evaluation and program creation, select **SUBMIT**.
- Your client's program is then saved.

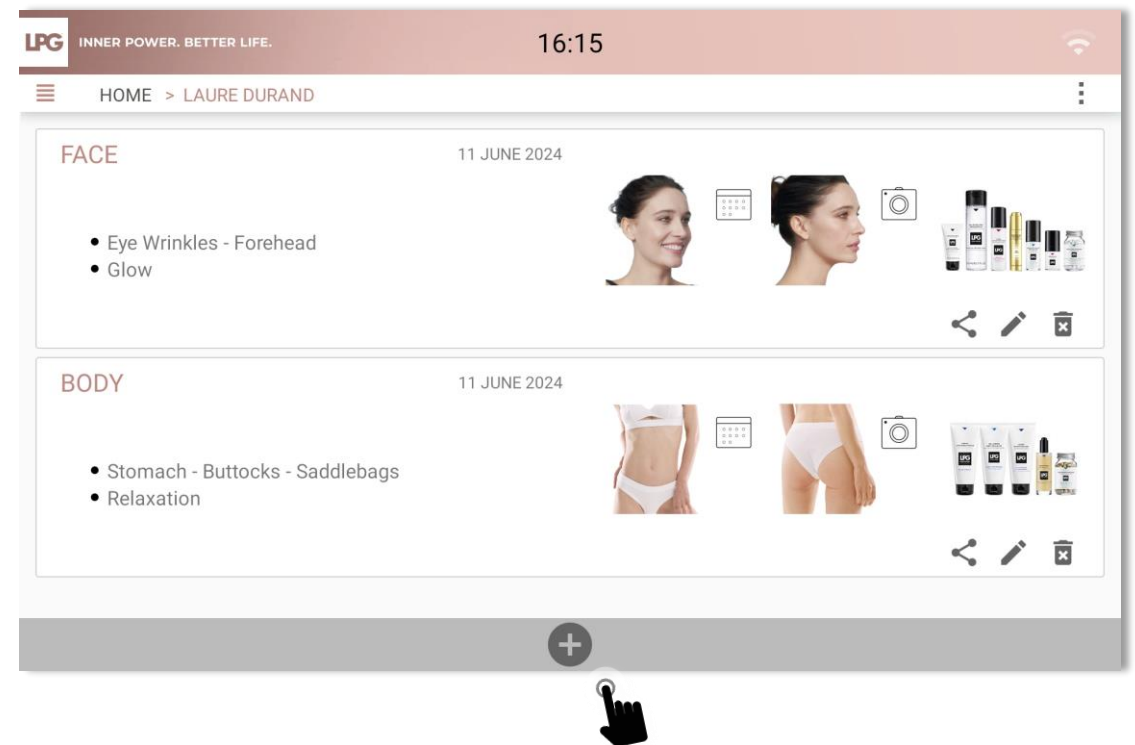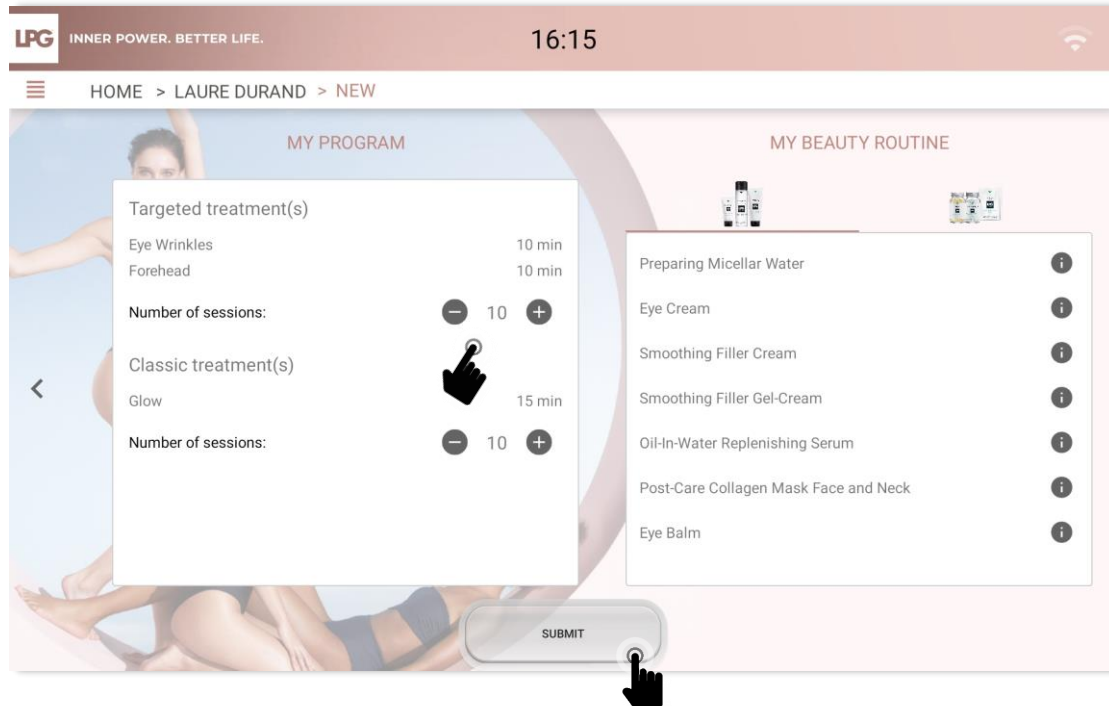

VIEWING PROGRAMS

SESSIONS

To view the completed endermologie® sessions (protocol description, date). This will enable a precise treatment program follow-up.

PHOTOS ASSESSMENT


To make, to consult and to compare photo assessments.

COSMETICS

To view cosmetics associated with your client's programs.

ENDERMONUTRI

To view food supplements associated with your client's programs.

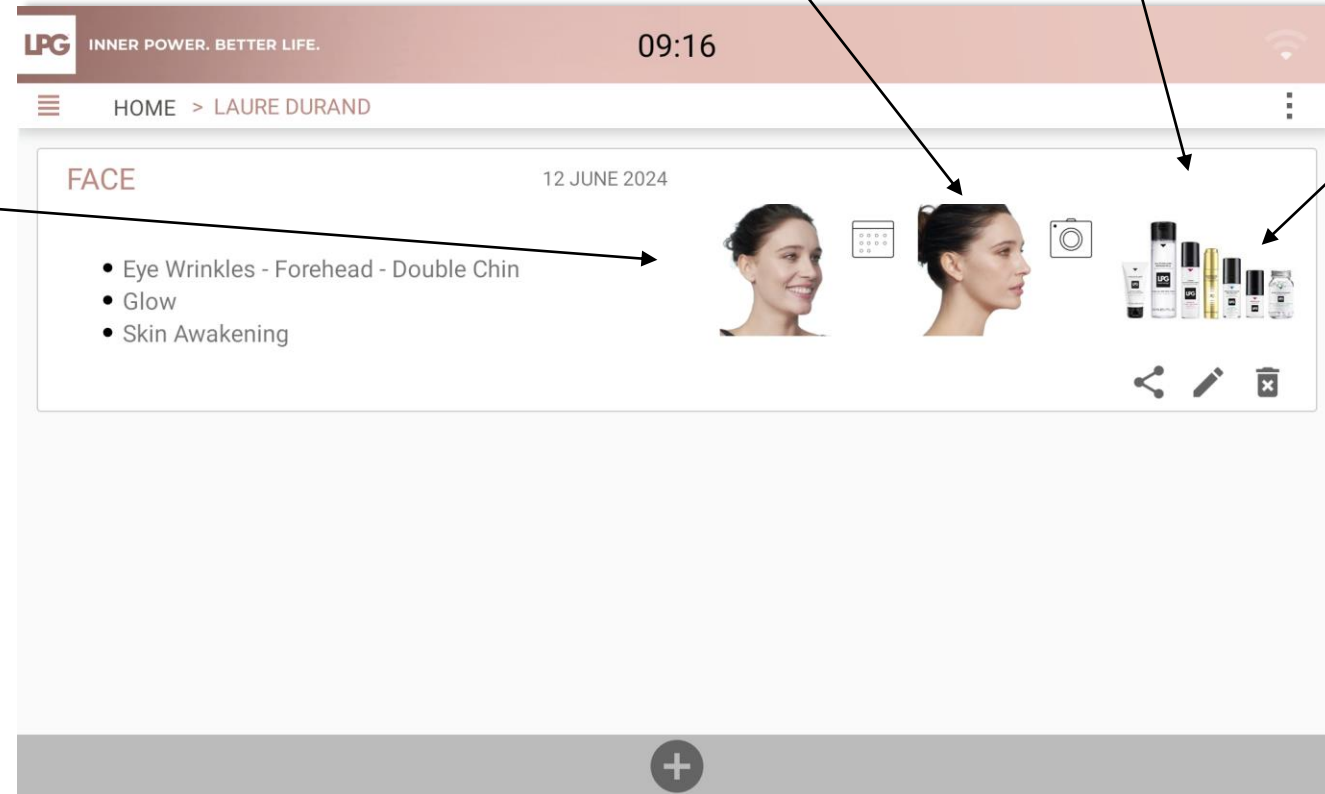

TO CREATE NEW FACIAL TREATMENT PROGRAM



INNER POWER.
BETTER LIFE.

VIEWING SESSIONS



: To confirm a session in case of connection loss with your MOBILIFT M6® CONNECT



: To comment a session (settings used...)



: To delete a session



: To add a session

LPGINNER POWER. BETTER LIFE.

09:18

HOME > LAURE DURAND > MY SESSIONS

^

EYE WRINKLES - FOREHEAD - DOUBLE CHIN

0/10 30 min

+

1/10

Eye Wrinkles

Forehead

Double Chin

10 min

10 min

10 min

2/10

Eye Wrinkles

Forehead

Double Chin

10 min

10 min

10 min

3/10

Eye Wrinkles

Forehead

Double Chin

10 min

10 min

10 min

4/10

Eye Wrinkles

Forehead

Double Chin

10 min

10 min


10 min




INNER POWER.
BETTER LIFE.

VIEWING COSMETICS



You can view cosmetics associated with each treatment program.

 INNER POWER. BETTER LIFE.


16:16





HOME > LAURE DURAND > MY BEAUTY ROUTINE




^ EYE WRINKLES - FOREHEAD

 PREPARING MICELLAR WATER


 EYE CREAM

 SMOOTHING FILLER CREAM

 SMOOTHING FILLER GEL-CREAM

^ GLOW

PREPARING MICELLAR WATER




- Removes impurities and makeup Moisturizes and soothes


Soak a cotton pad and cleanse face, eyes and neck, morning and evening.
Dry the skin gently without rinsing.

VIEWING ENDERMONUTRI


You can view food supplements associated with each treatment program.


 INNER POWER. BETTER LIFE.


16:29





☰ HOME > LAURE DURAND > MY BEAUTY ROUTINE



^ EYE WRINKLES - DOUBLE CHIN - NECK

 VITALITY BOOSTER


 HYALURONIC ACID

 COLLAGEN

 OMEGA

 WATERGLOW

VITALITY BOOSTER



- Stimulates vitality and physical & mental performance
- Reduces fatigue
- Strengthens the immune system

2 capsules per day preferably in the morning and before 3 pm to prevent any difficulty in falling asleep.
For optimal results, it is recommended to follow this program for at least 2 consecutive months. It may be repeated at the start of any endermologie® program and/or change in season.




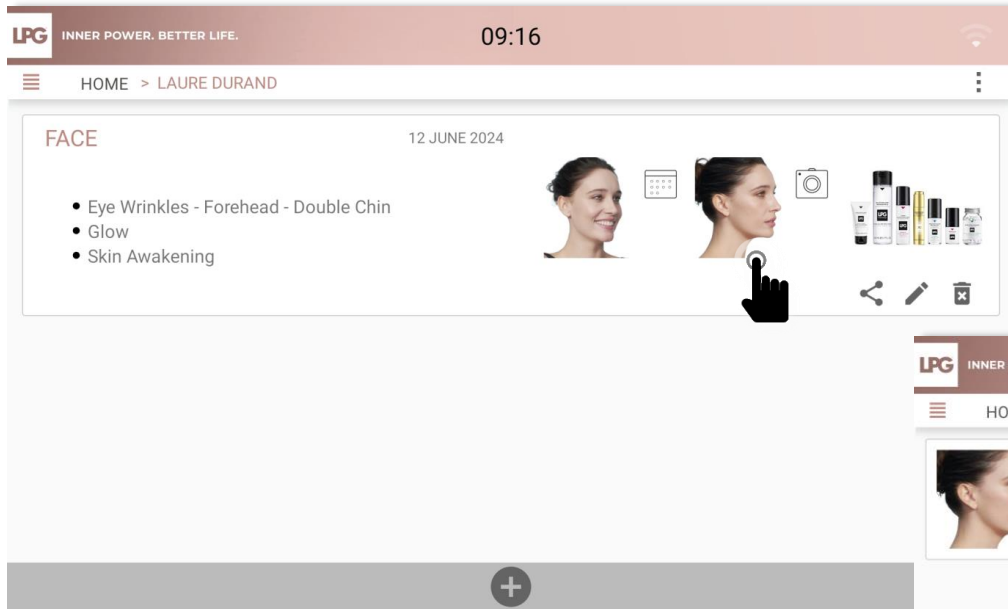
PHOTO ASSESSMENT



INNER POWER.
BETTER LIFE.

PHOTO ASSESSMENT


After clicking on « Photo assessment » icon, click on  to make a new photo assessment. All photos taken during the first evaluation are saved here.



:To delete photos assessment




:To save photos assessment on your tablet

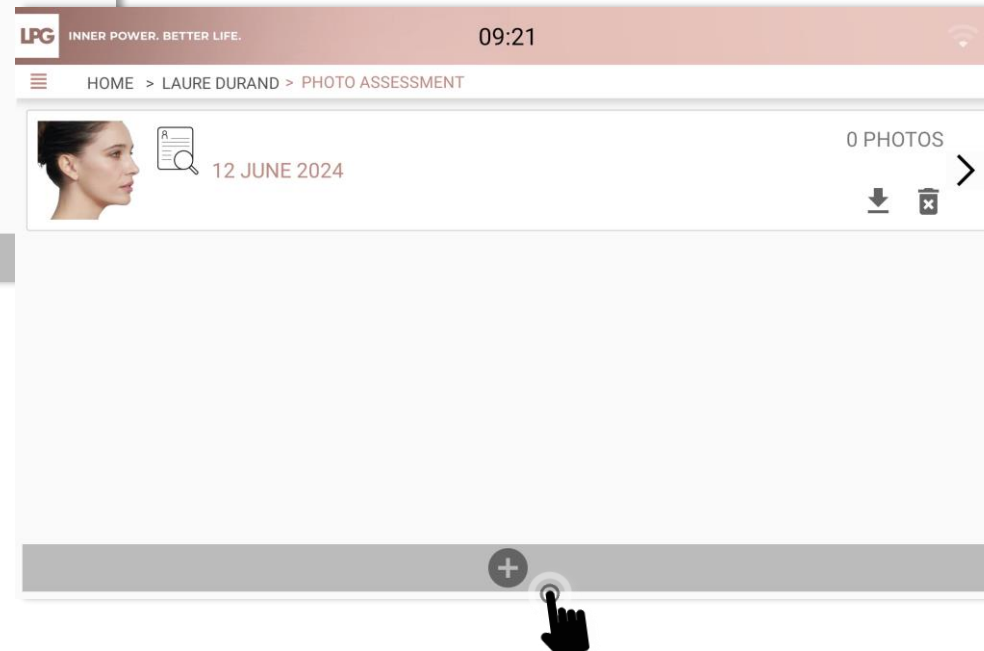

INNER POWER.
BETTER LIFE.

PHOTO ASSESSMENT


The first evaluation's objectives are mentioned (reminder).
You can add comments (number of sessions, results..)

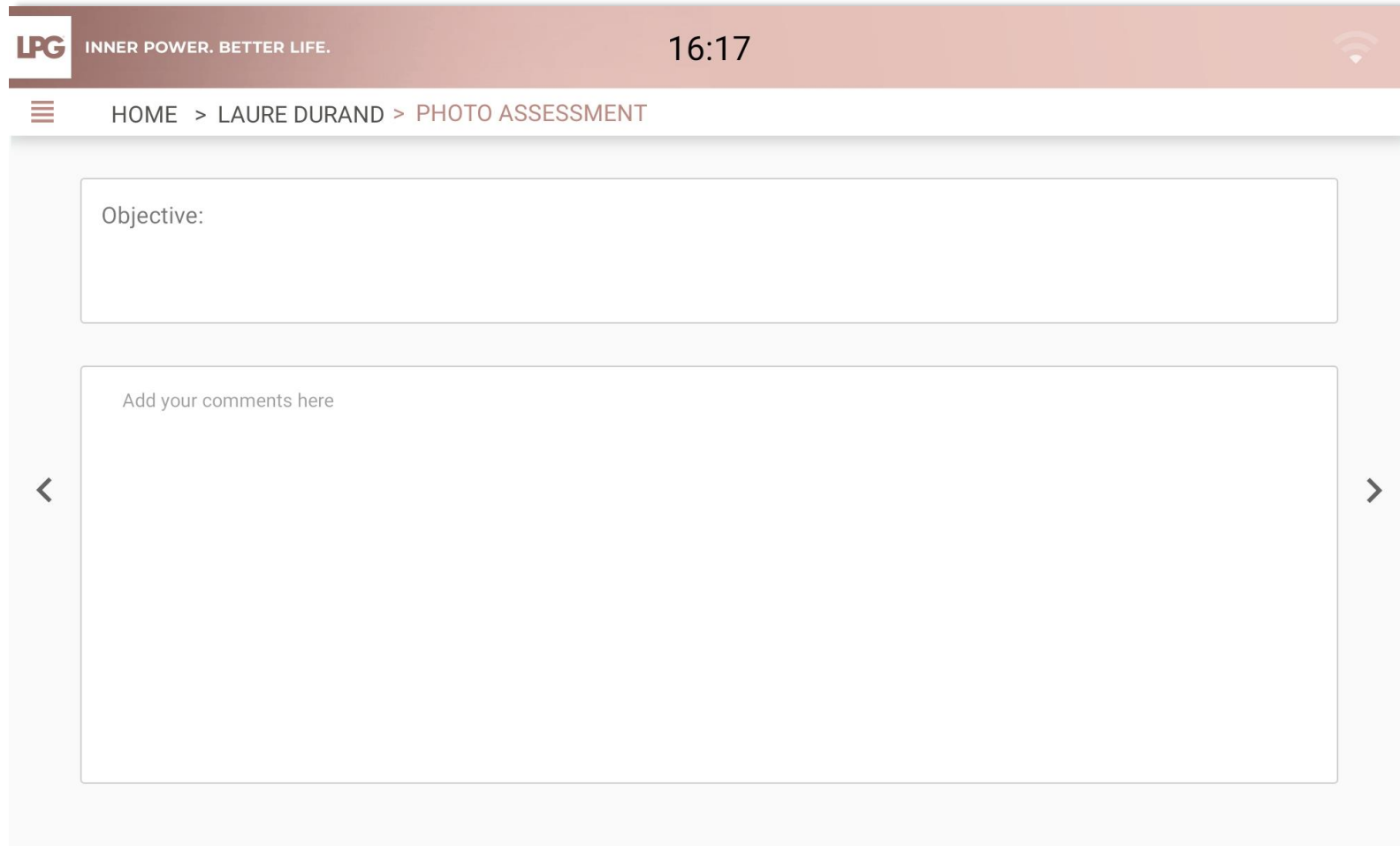

The screenshot shows a mobile application interface for 'LPG INNER POWER. BETTER LIFE.' The top status bar is red with the time '16:17' and a Wi-Fi icon. Below the status bar is a navigation bar with a hamburger menu icon, the text 'HOME > LAURE DURAND > PHOTO ASSESSMENT', and a red background. The main content area is white and contains two text input fields. The first field is labeled 'Objective:' and is empty. The second field is labeled 'Add your comments here' and is also empty. To the left of the second field is a left arrow icon, and to the right is a right arrow icon. The bottom of the screen shows a red bar with the LPG logo and the text 'INNER POWER. BETTER LIFE.'

PHOTO ASSESSMENT


- Take 3 photos (right profile, face, left profile). Maintain you tablet in vertical (portrait) format. The photo must be taken when level sensors are green. Report to the integrated markers of the photo assistant to position your client properly. Once the picture is taken, click on **REGISTER** or **CANCEL** if you are not satisfied.
- To take other pictures, click on **ADD ADDITIONAL PICTURE** and choose a title.

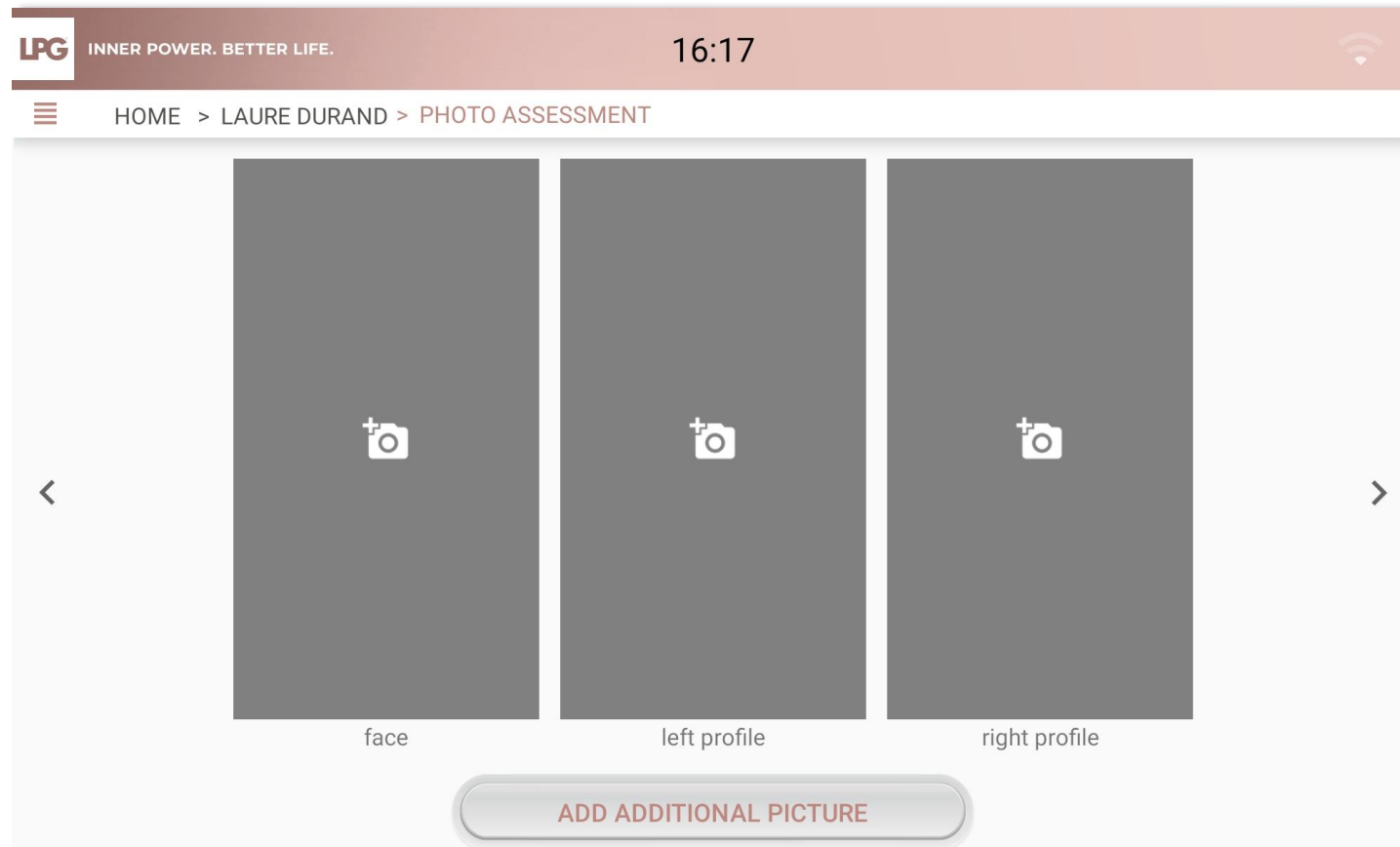

INNER POWER.
BETTER LIFE.

PHOTO ASSESSMENT


Move the cursor to select the corresponding level.

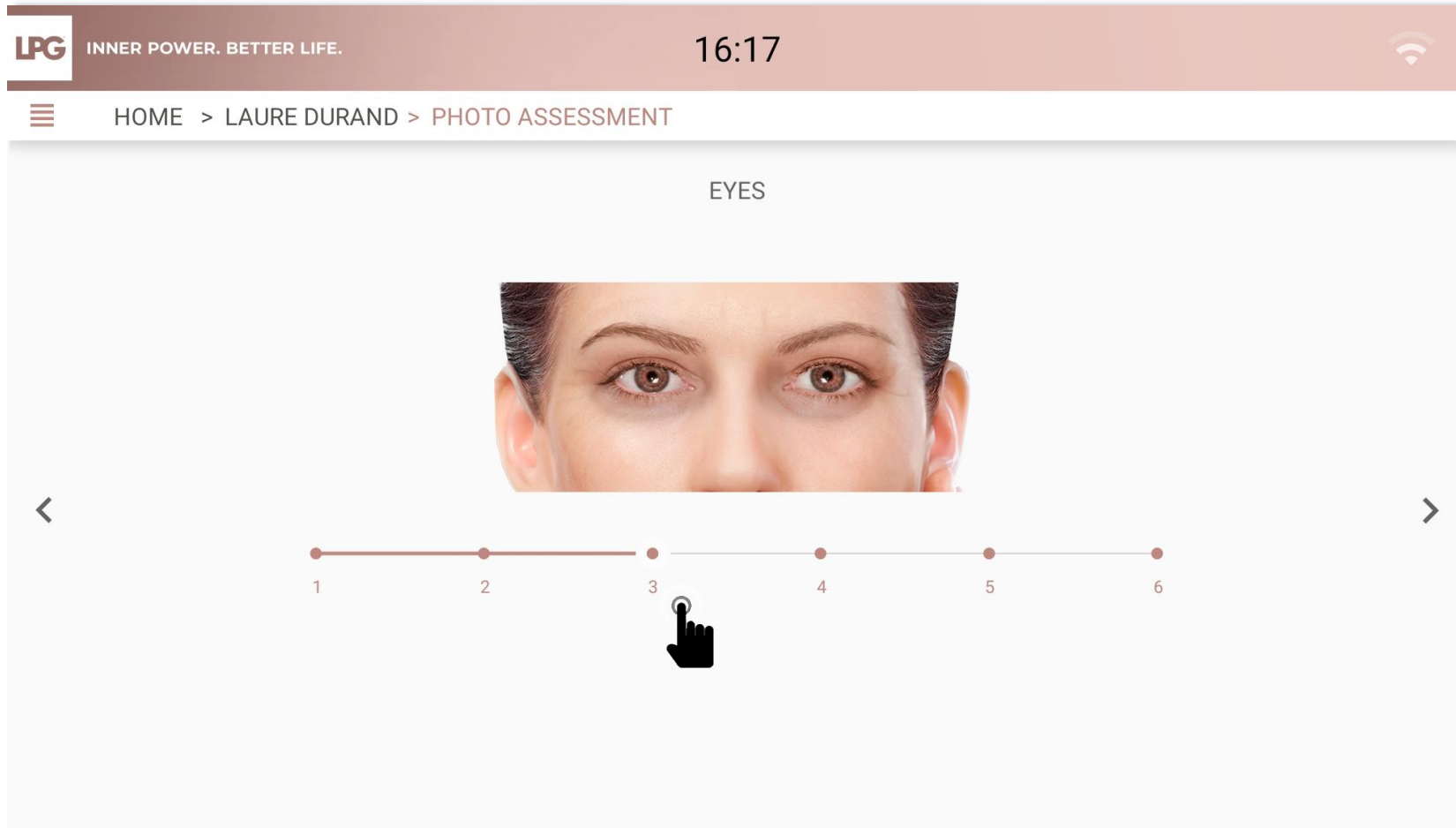

INNER POWER.
BETTER LIFE.

FACE EVALUATION

Move the cursor to select the corresponding level.

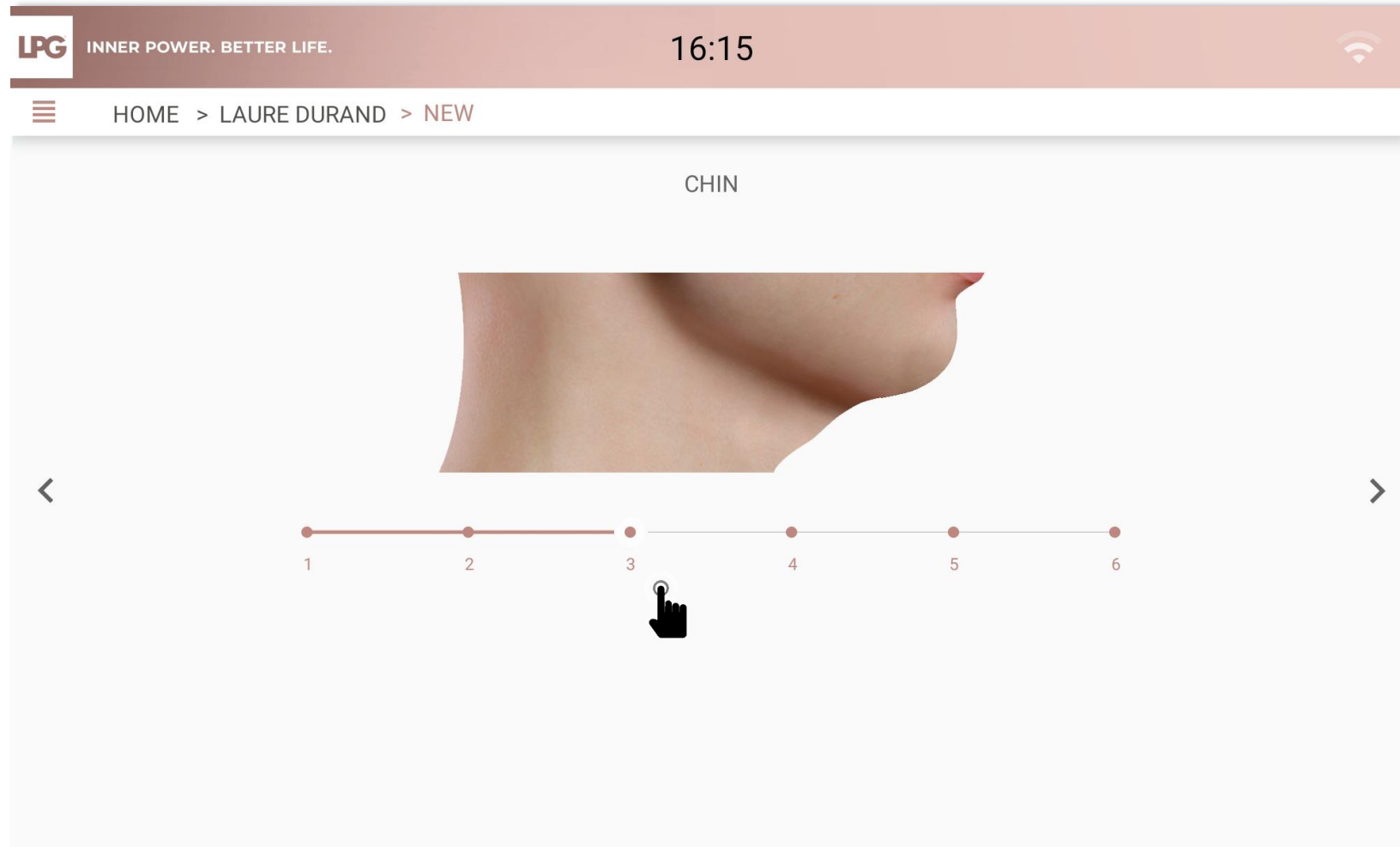

INNER POWER.
BETTER LIFE.




COMPARE PHOTO ASSESSMENTS




INNER POWER.
BETTER LIFE.

COMPARE PHOTO ASSESSMENTS

Click on 

 INNER POWER. BETTER LIFE.

16:17




HOME > LAURE DURAND > PHOTO ASSESSMENT

 11 JUNE 2024

0 PHOTOS



 11 JUNE 2024


0 PHOTOS








COMPARE PHOTO ASSESSMENTS

- Using the arrows, select the photo assessments
- Next, select the picture : face, left profile, right profile
- To save the comparison, click on 
- Choose a title and save it in a folder on your tablet.

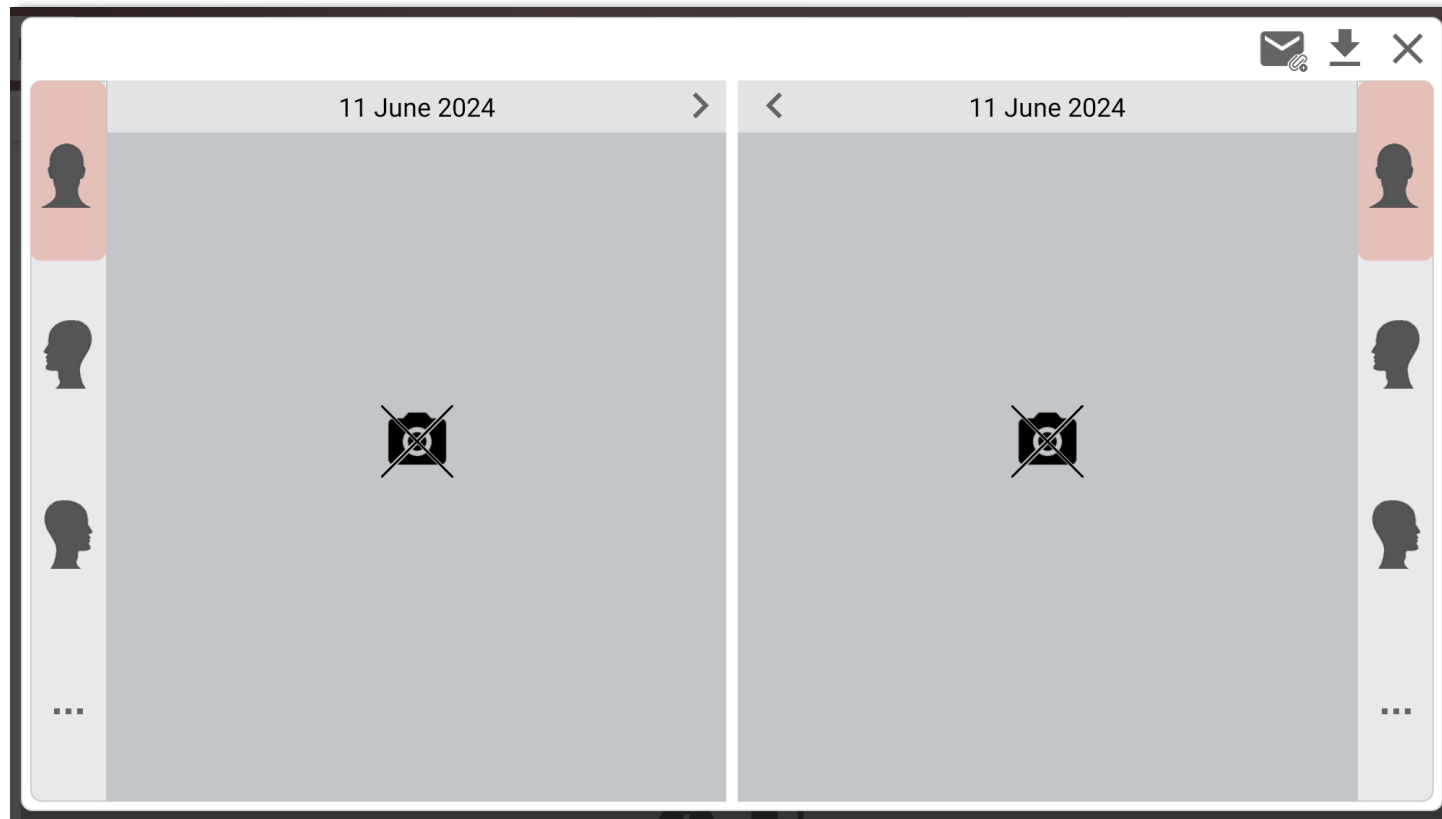




SEND TREATMENT PROGRAM TO YOUR CLIENT



INNER POWER.
BETTER LIFE.


SEND TREATMENT PROGRAMS

- Choose the concerned program and click on 
- Select cosmetics and food supplements to import in the document. Next, click on OK.

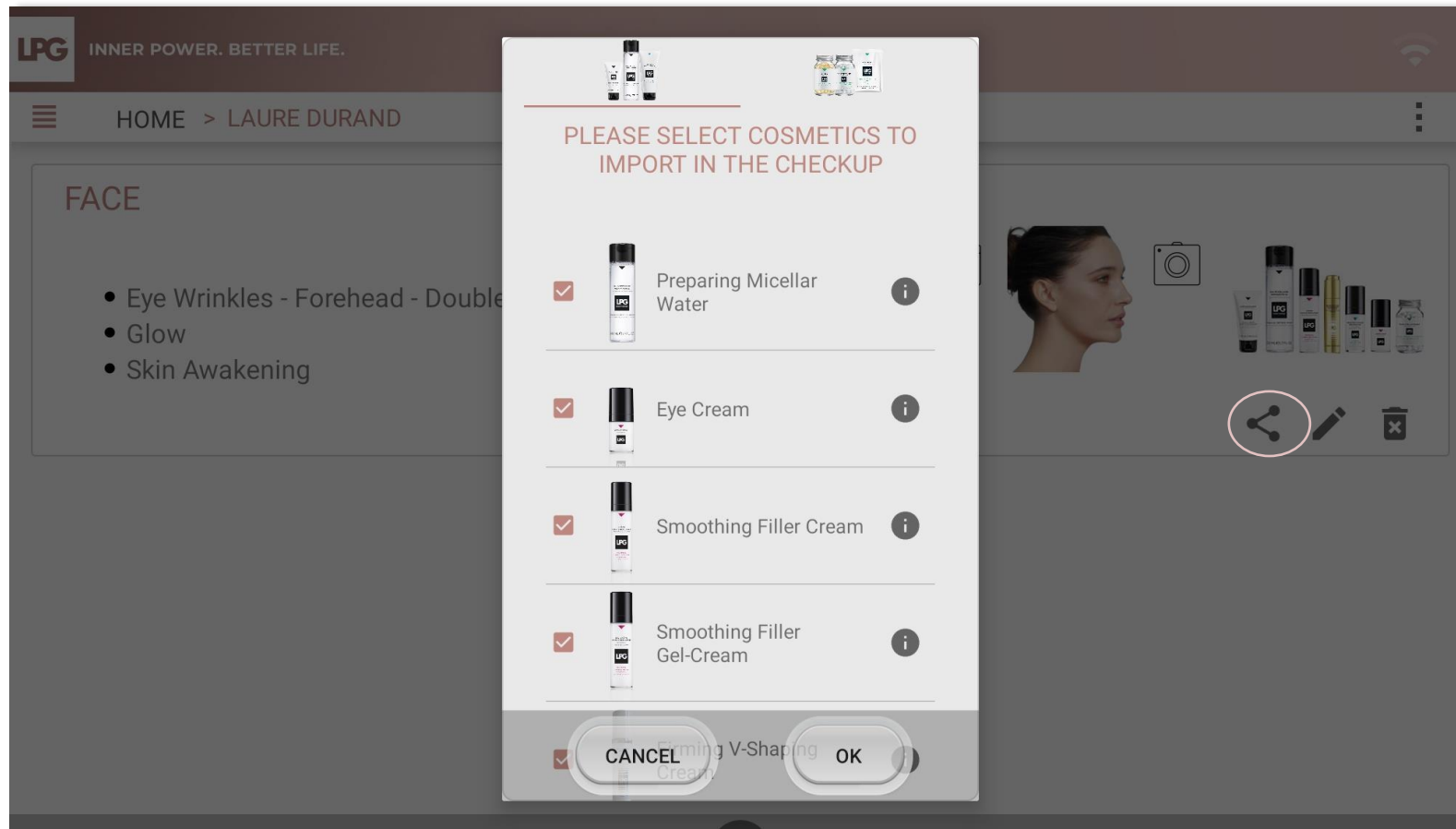


SEND TREATMENT PROGRAMS

- Choose the concerned program and click on 
- Select cosmetics and food supplements to import in the document. Next, click on OK.

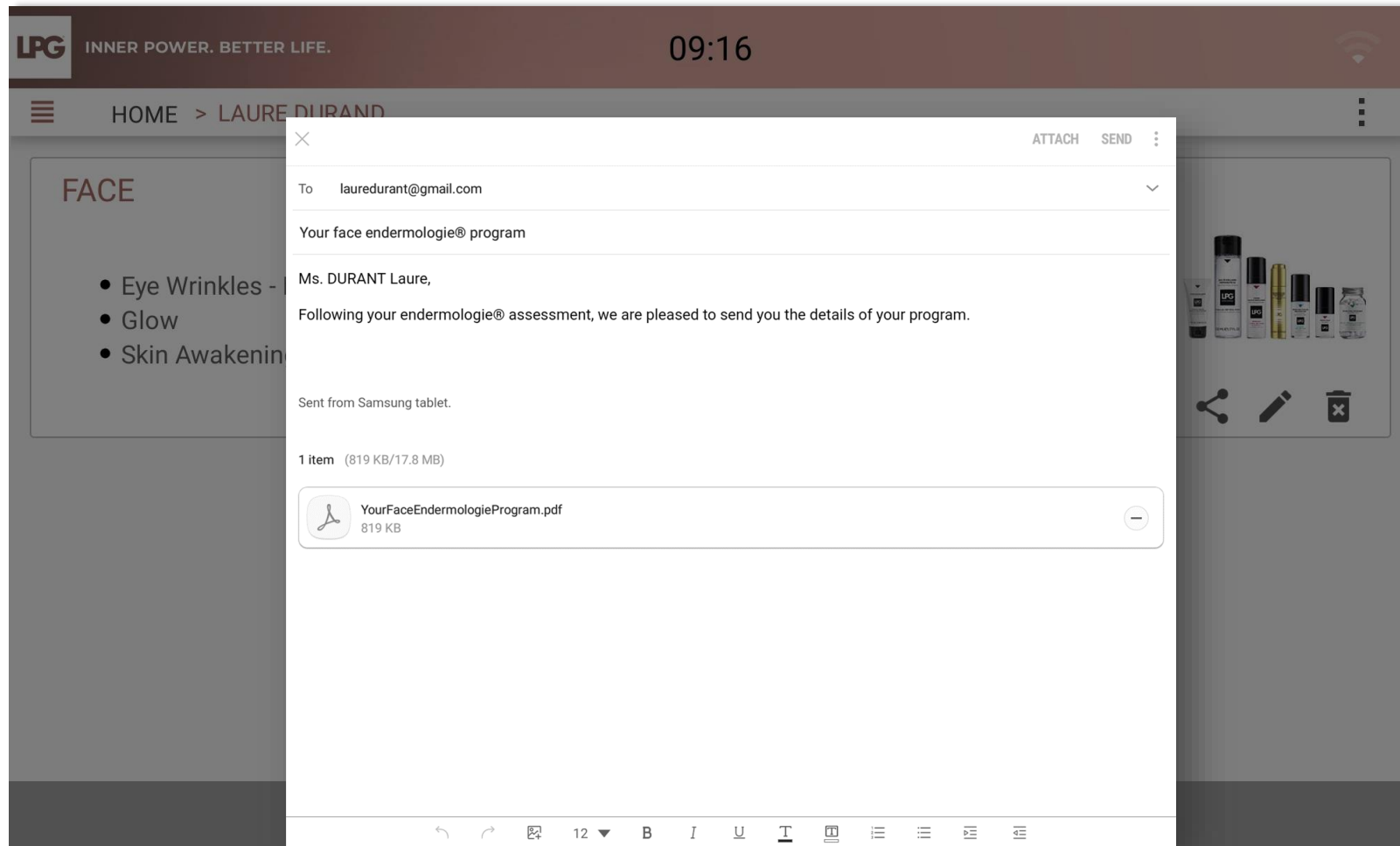

INNER POWER.
BETTER LIFE.


SEND TREATMENT PROGRAMS TO YOUR MOBILIFT M6® CONNECT

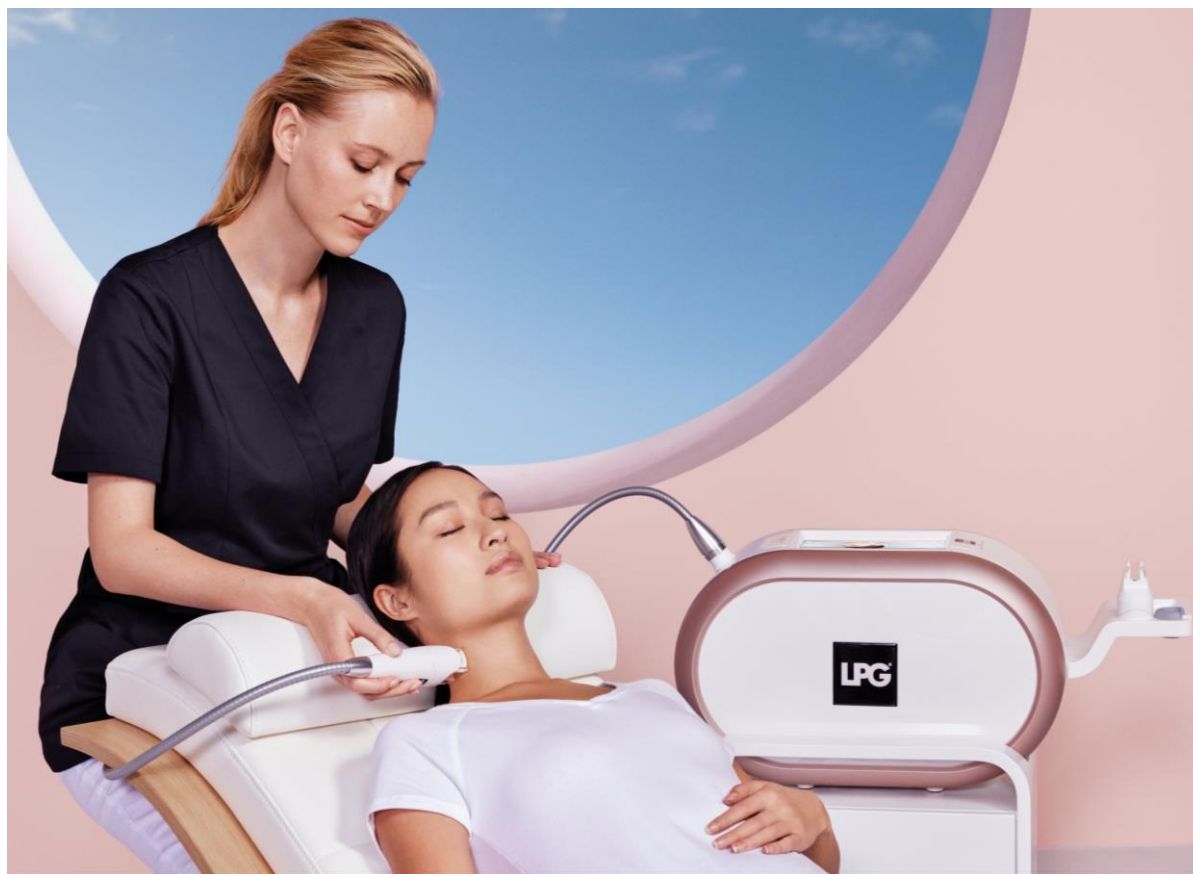


INNER POWER.
BETTER LIFE.

TREATMENT PROGRAM LAUNCH

After connecting your tablet to your MOBILIFT M6® CONNECT device (please refer to the Tablet Quickstart guide if necessary), select **CUSTOMIZED SESSION**.


If your device is connected to your tablet, this icon  appears on your tablet's screen.

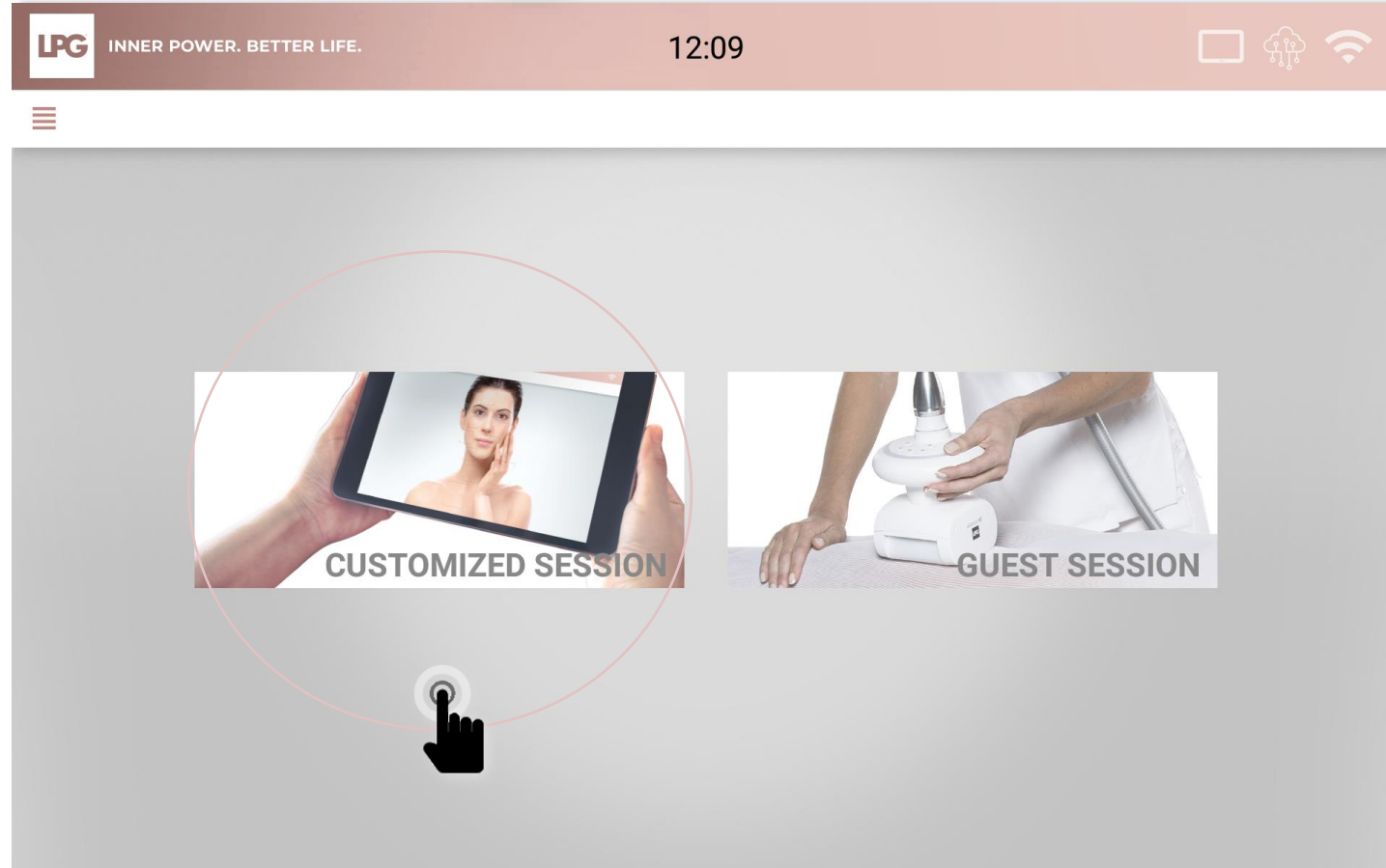

INNER POWER.
BETTER LIFE.

TREATMENT PROGRAM LAUNCH

Select the concerned client in your database.

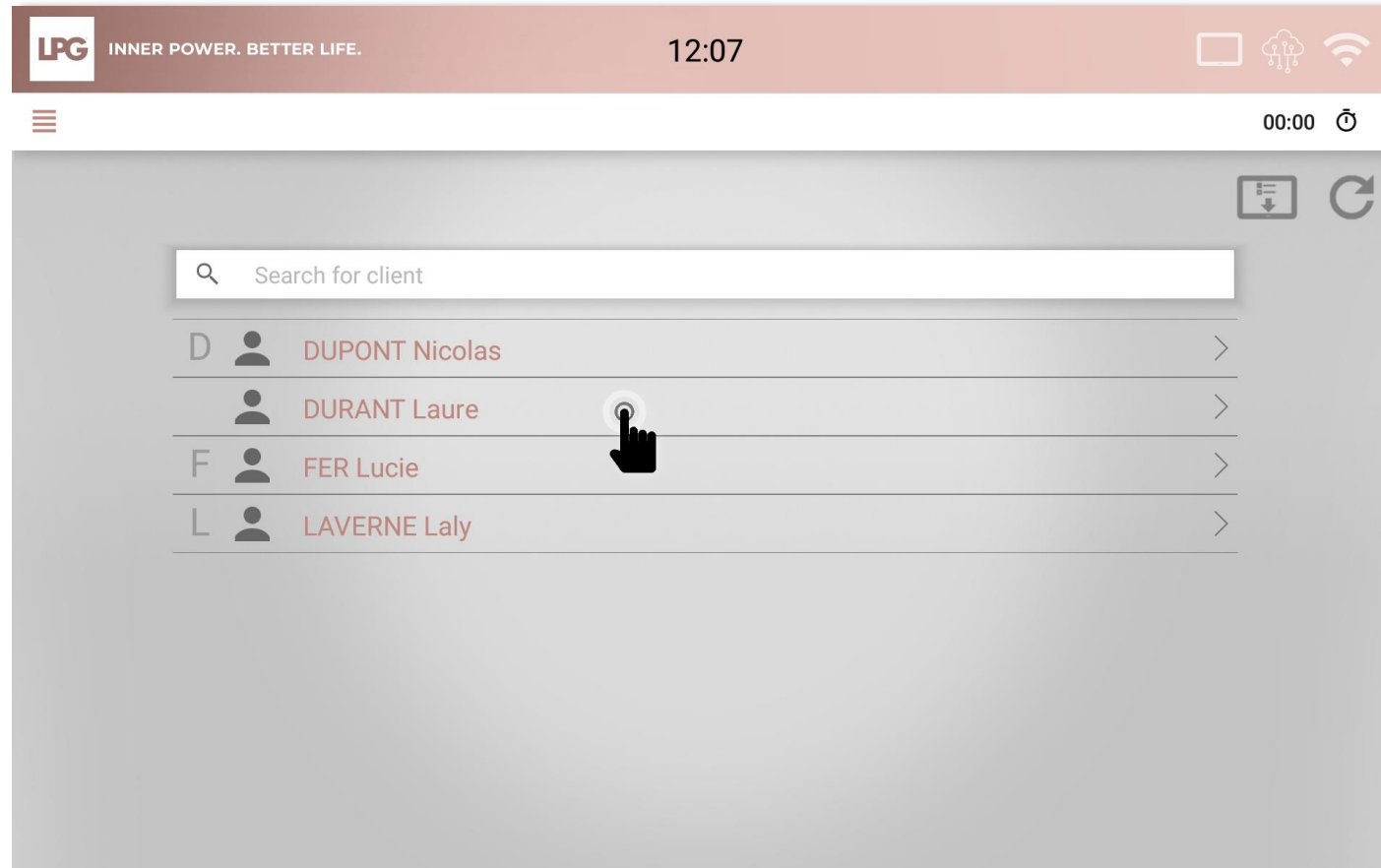


Client programs can be downloaded when the tablet is connected. They will be accessible at any time. The results will be sent as soon as the tablet reappears.



**INNER POWER.
BETTER LIFE.**

TREATMENT PROGRAM LAUNCH

Select the desired program.

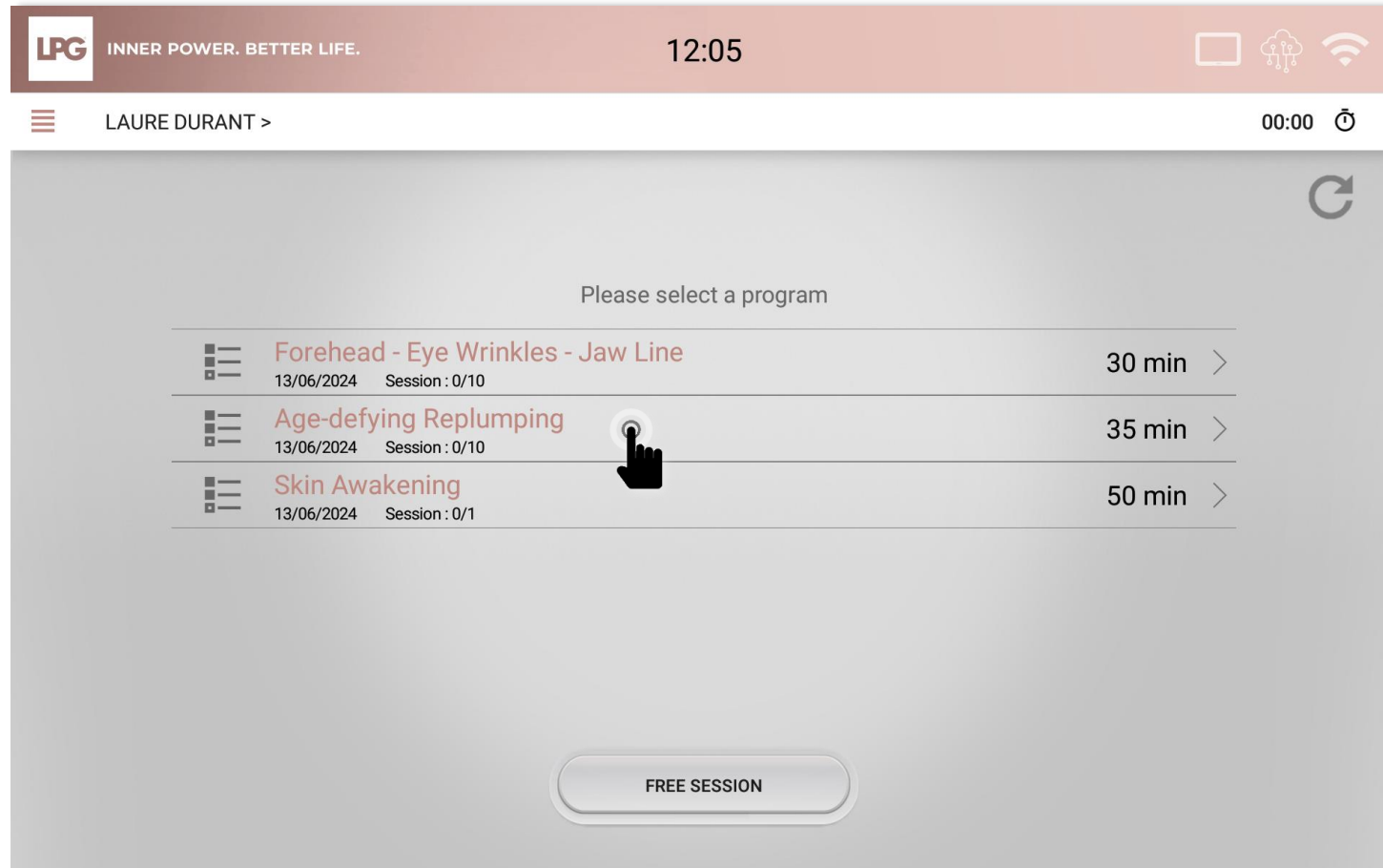

In case of connection loss, possibility to update manually the sessions on the machine validated on the tablet by clicking on.



INNER POWER.
BETTER LIFE.

TREATMENT PROGRAM LAUNCH

The session can start!

LPG

INNER POWER. BETTER LIFE.

12:07

LAURE DURANT > SKIN AWAKENING - Clean and prepare the skin

00:00

Step 1/110:08-15:32

PREPARING MICELLAR WATER

Apply on the entire face and neck using soaked cotton balls.
Dry the skin well using soft tissue.

EAU MICELLAIRE
PRÉPARATRICE
PREPARING MICELLAR WATER

LPG
endermologie

DÉMAQUILLE,
NETTOIE, HYDRATE
REMOVES MAKEUP,
CLEANSES, HYDRATES

Micelles, Sucres, Glycérine
Micelles, Sugars, Glycerin

400 MLe/13,5 FL.OZ

EAU MICELLAIRE
PRÉPARATRICE
PREPARING MICELLAR WATER

LPG
endermologie

DÉMAQUILLE,
NETTOIE, HYDRATE
REMOVES MAKEUP,
CLEANSES, HYDRATES

Micelles, Sucres, Glycérine
Micelles, Sugars, Glycerin

400 MLe/13,5 FL.OZ

EAU MICELLAIRE
PRÉPARATRICE
PREPARING MICELLAR WATER

LPG
endermologie

DÉMAQUILLE,
NETTOIE, HYDRATE
REMOVES MAKEUP,
CLEANSES, HYDRATES

Micelles, Sucres, Glycérine
Micelles, Sugars, Glycerin

400 MLe/13,5 FL.OZ

LPG

INNER POWER.
BETTER LIFE.




FREE SESSIONS



INNER POWER.
BETTER LIFE.

VIEWING FREE SESSIONS

To access your client's free sessions, click on 
Next, click on **MY FREE SESSIONS**.

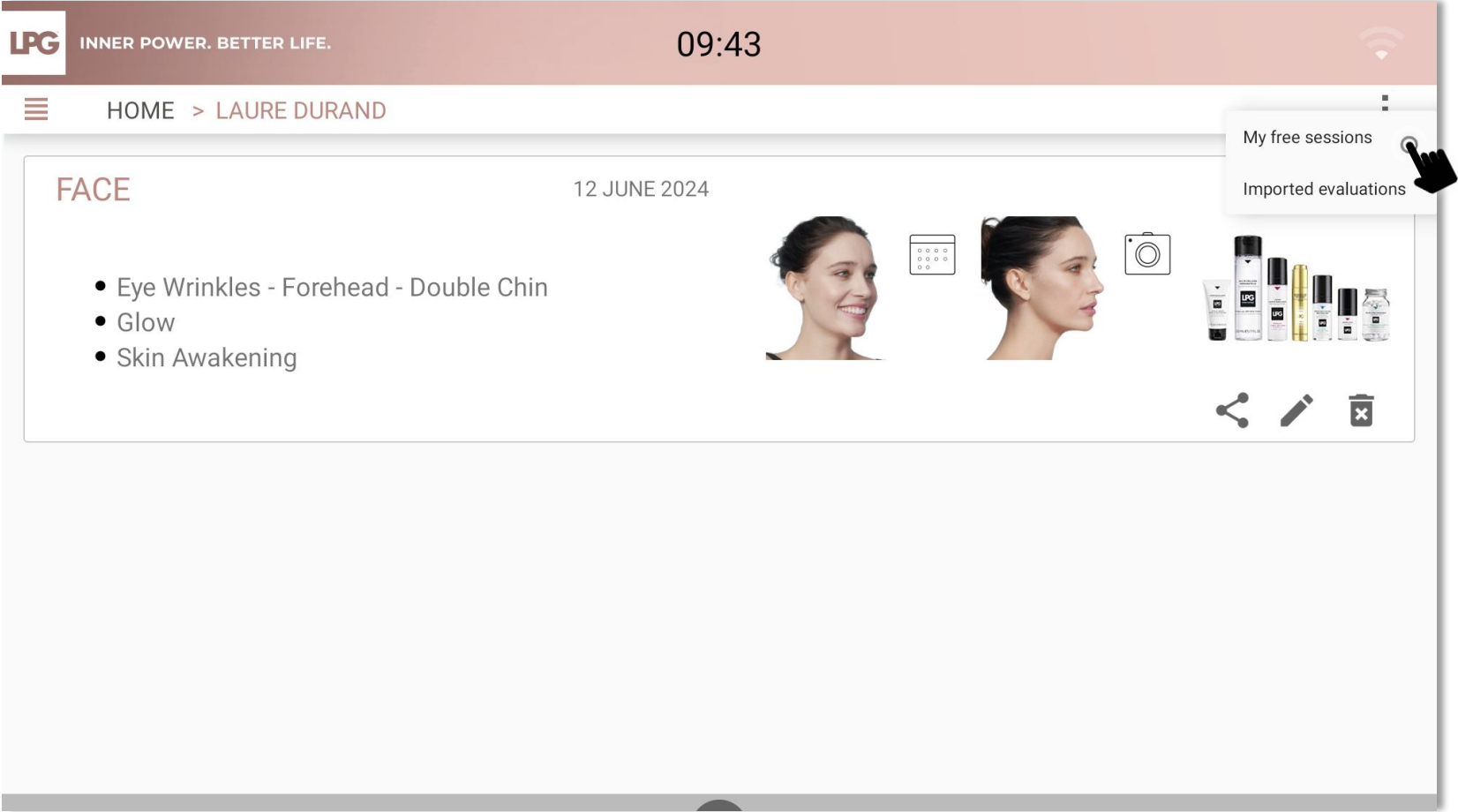



BACK UP



INNER POWER.
BETTER LIFE.



BACKUP



Should you skip this essential step and should any problem occur (operating error, technical failure, theft...), you will lose your data

For more security, save all your data on an external support of your choice: micro SD card available in supermarkets or in specialized stores, personal computer, external hard drive...
By default, your database will be stored on your tablet.

Click on the menu button on the upper left side of the screen and select **SETTINGS**. Click on **DATABASE**. Click on « **BACKUP DATABASE** » and choose your micro SD card as backup location. The number of possible backups is conditioned by your micro SD card's memory size.

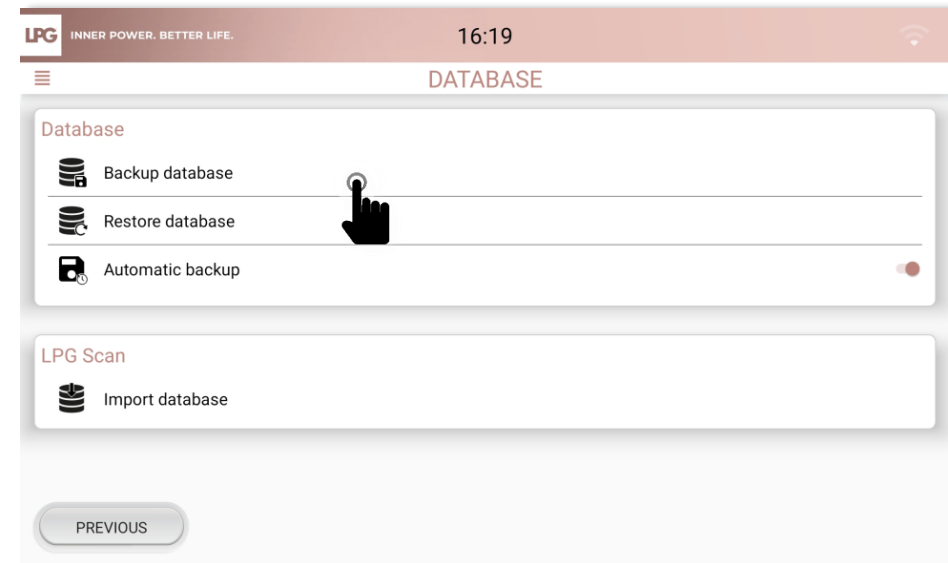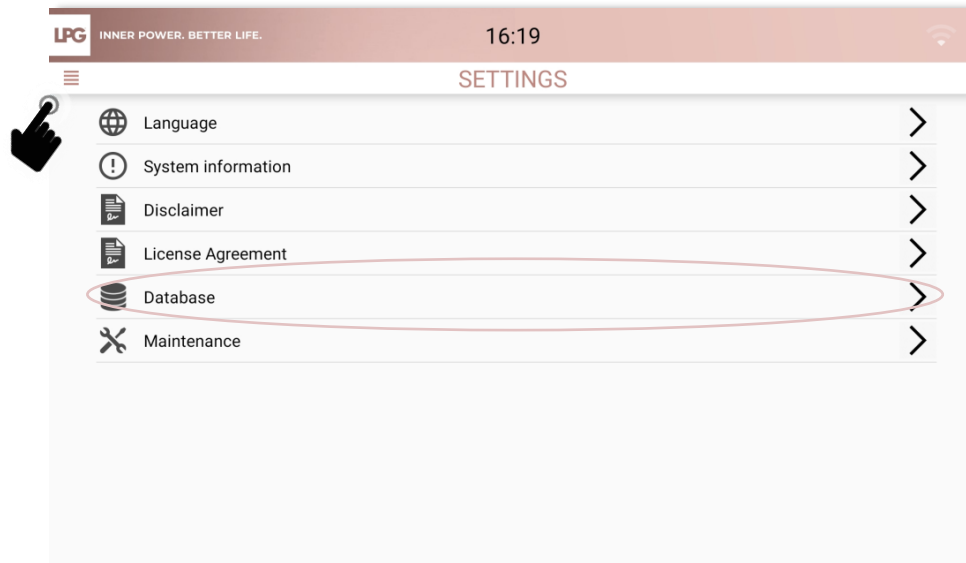

**INNER POWER.
BETTER LIFE.**

AUTOMATIC BACKUP

To plan an automatic backup, click on **AUTOMATIC BACKUP**.



Click on the cursor to activate the automatic backup



Select the periodicity, the location.

Enter your password for security protection. Please keep it safe because without it, it will be impossible to restore your backups. For privacy reasons, LPG SYSTEMS does not have access to your password and cannot ask you for it. LPG SYSTEMS can not be held responsible for any loss of your data.

Click on **OK**.

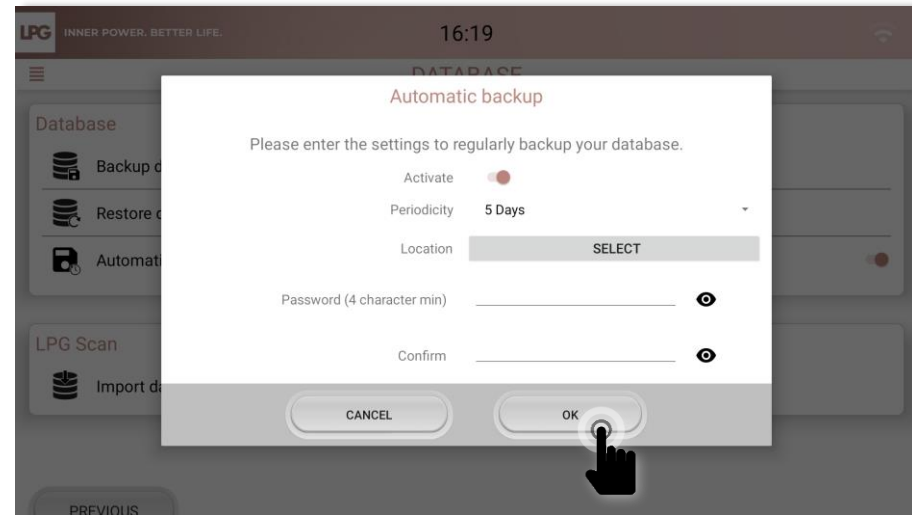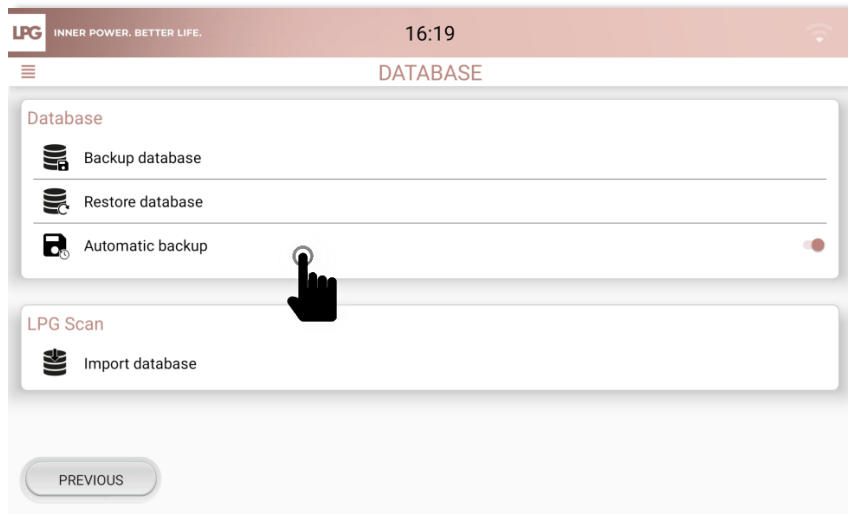

INNER POWER.
BETTER LIFE.




UPDATES



INNER POWER.
BETTER LIFE.


UPDATES

When an update is available, the following screen appears when you start the application. The "i" allows to get information about the update. Click on **OK** to start the update.

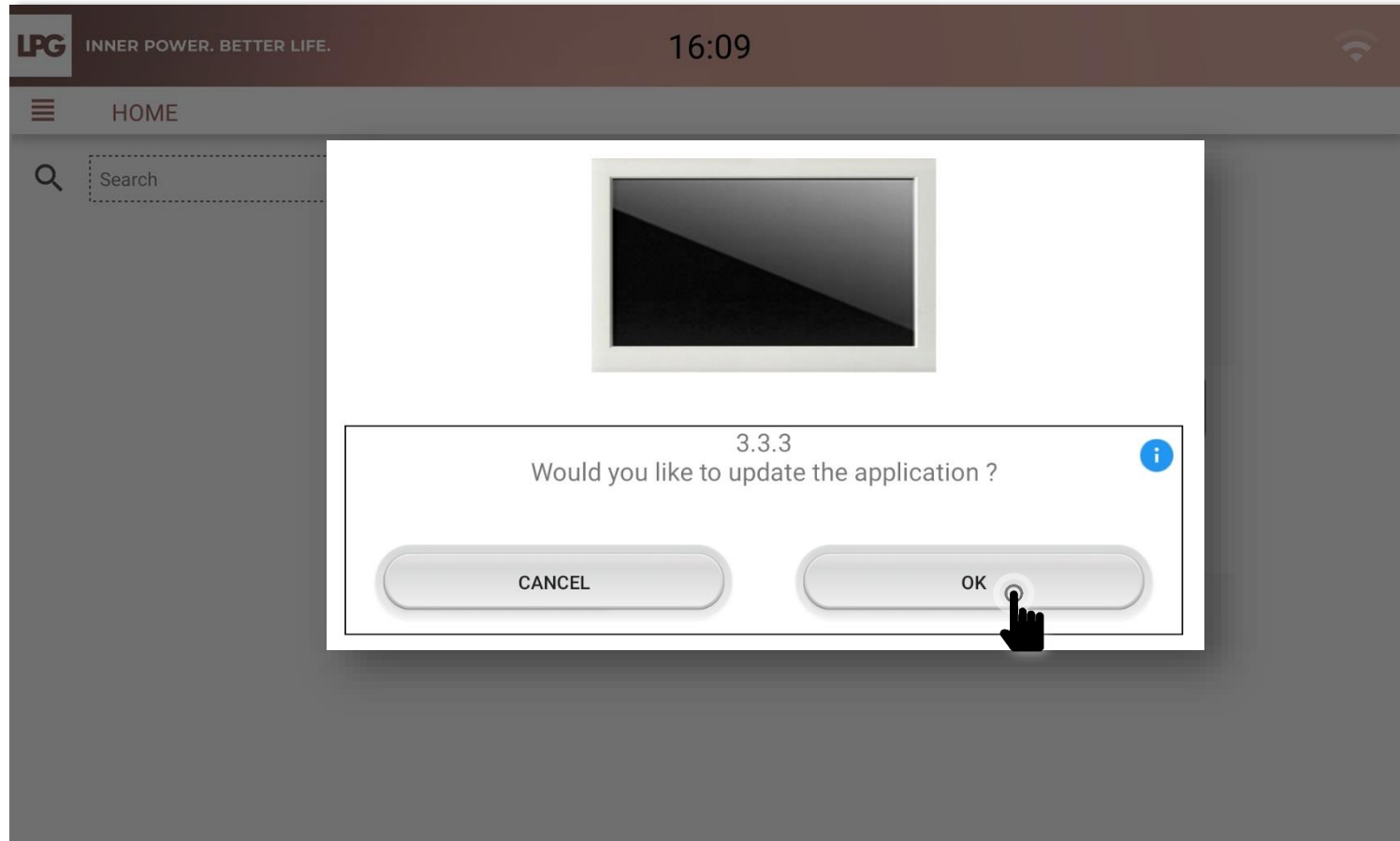


UPDATES

Please follow the instructions on the screen and continue by clicking on **OK**.

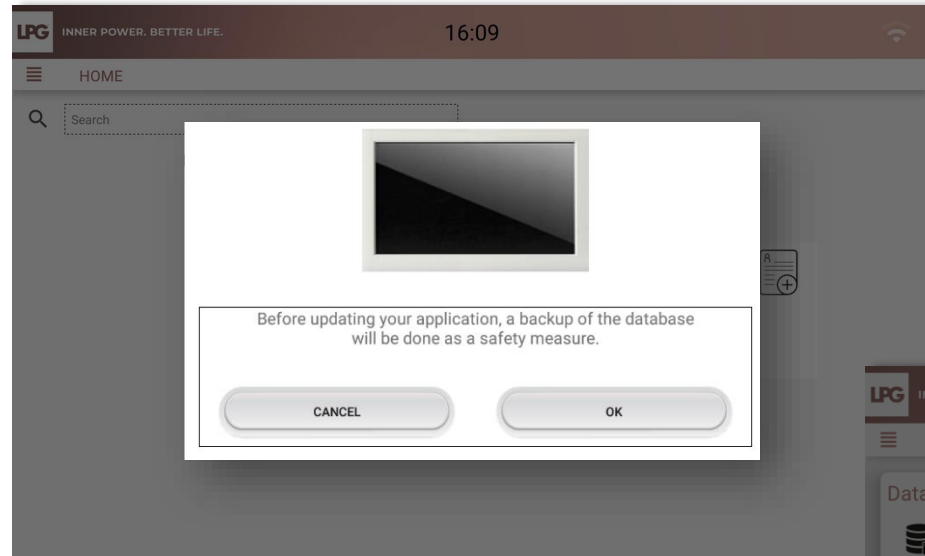

Enter your password for security protection. Please keep it safe because without it, it will be impossible to restore your backups. For privacy reasons, LPG SYSTEMS does not have access to your password and cannot ask for it.


LPG SYSTEMS can not be held responsible for any loss of your data.

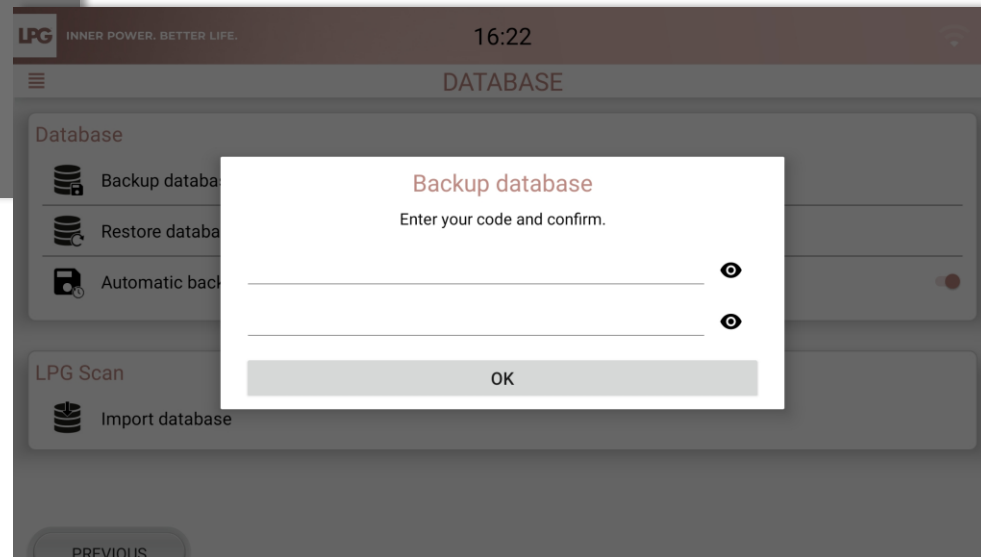

INNER POWER.
BETTER LIFE.

UPDATES

At the end of the backup, your tablet can display this message. Click on **Settings**.
Allow the installation of unknown sources with the cursor.

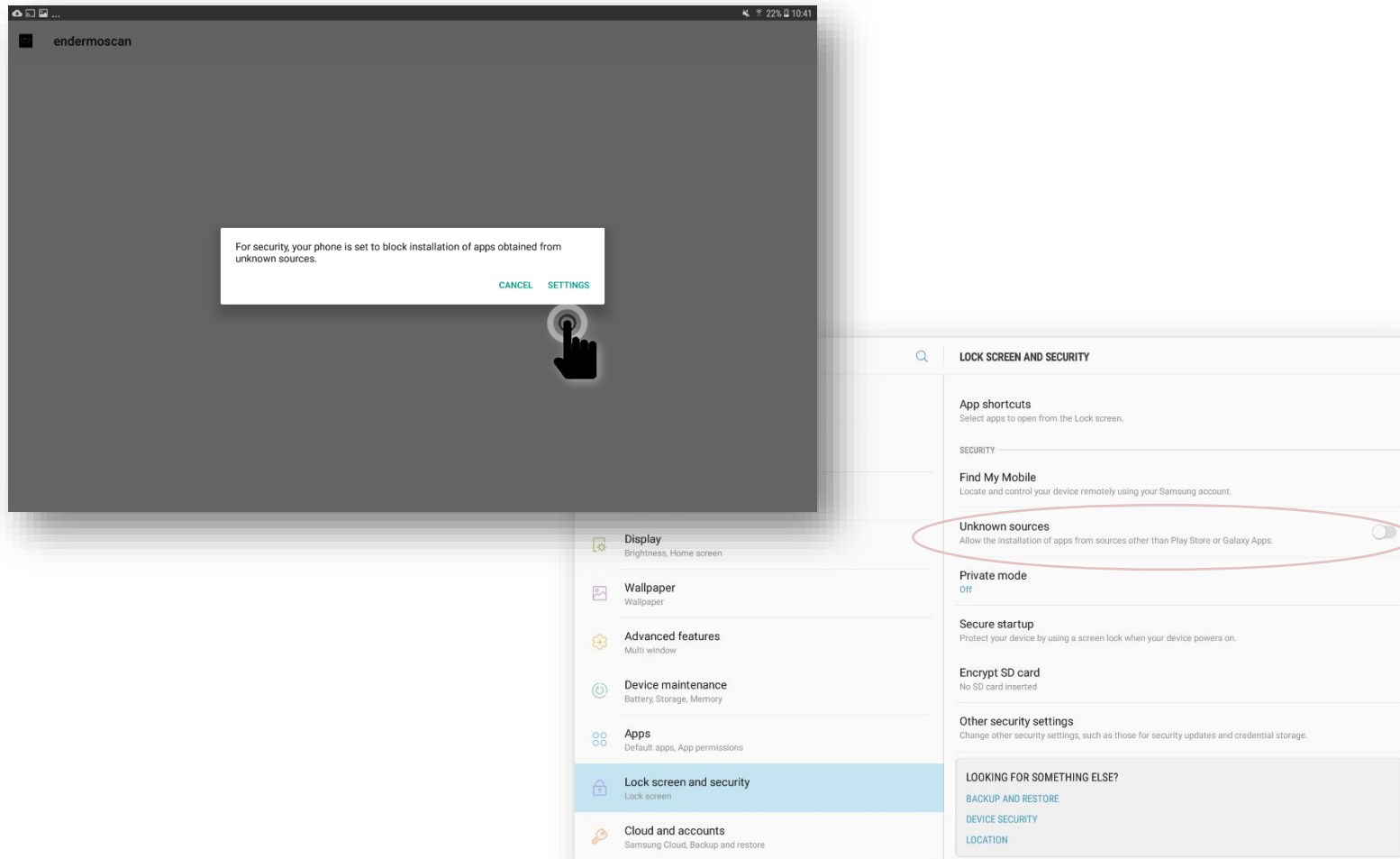

INNER POWER.
BETTER LIFE.



DATA IMPORT (LPG SCAN tablet to the new one)





INNER POWER.
BETTER LIFE.

LPG SCAN DATABASE

Before importing, please save your database.

To import your LPG SCAN database (client profile, photos), please insert a micro SD card into the intended location on the tablet. Then click on the menu button on the upper left side of the screen and select **SETTINGS**.

In **DATABASE**, click on **IMPORT DATABASE** and choose a backup location.

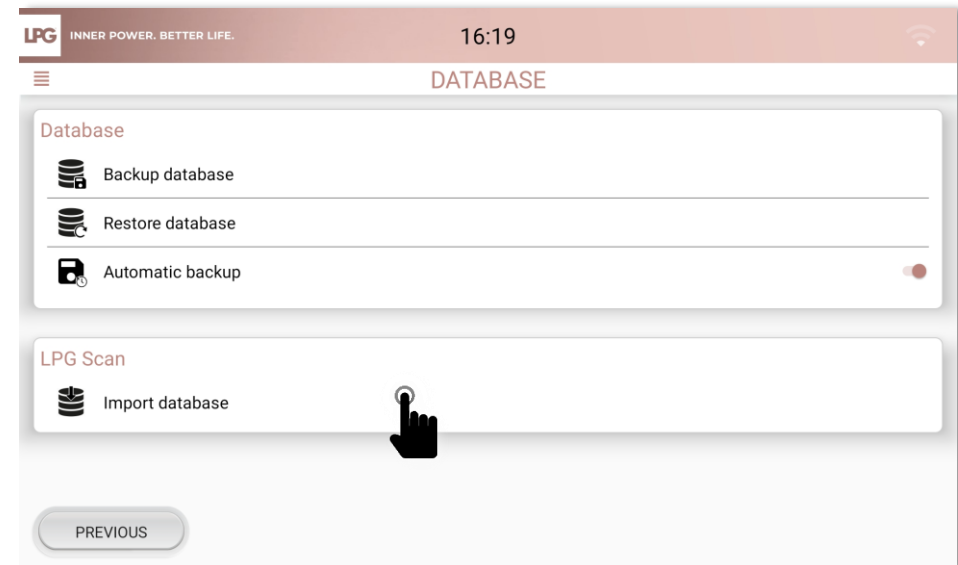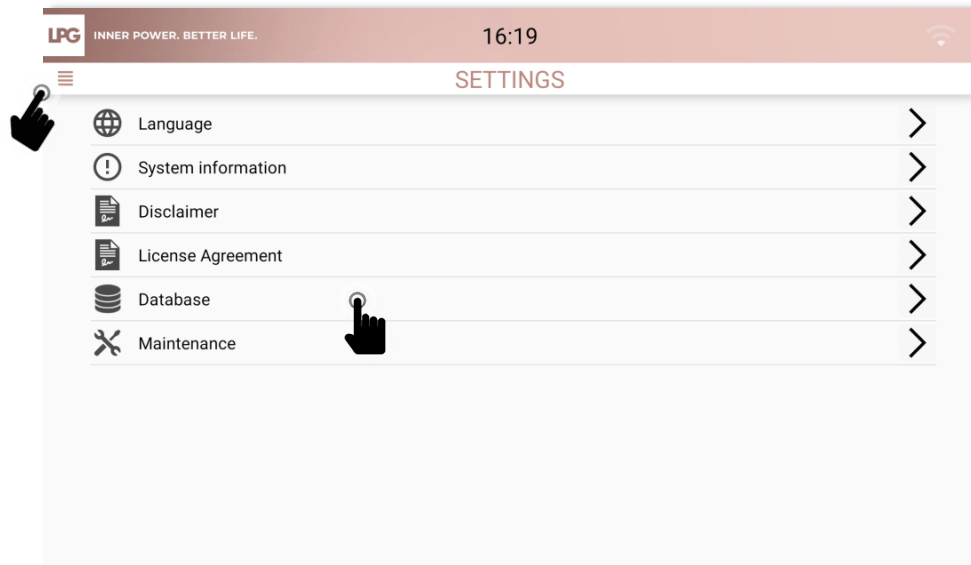




DATA EXPORT




INNER POWER.
BETTER LIFE.

DATA EXPORT

To access your tablet's settings, click on 

« **Disclaimer** » makes you aware of confidentiality and integrity of data if you decide to export our customer data (please check *the export of customer data*).

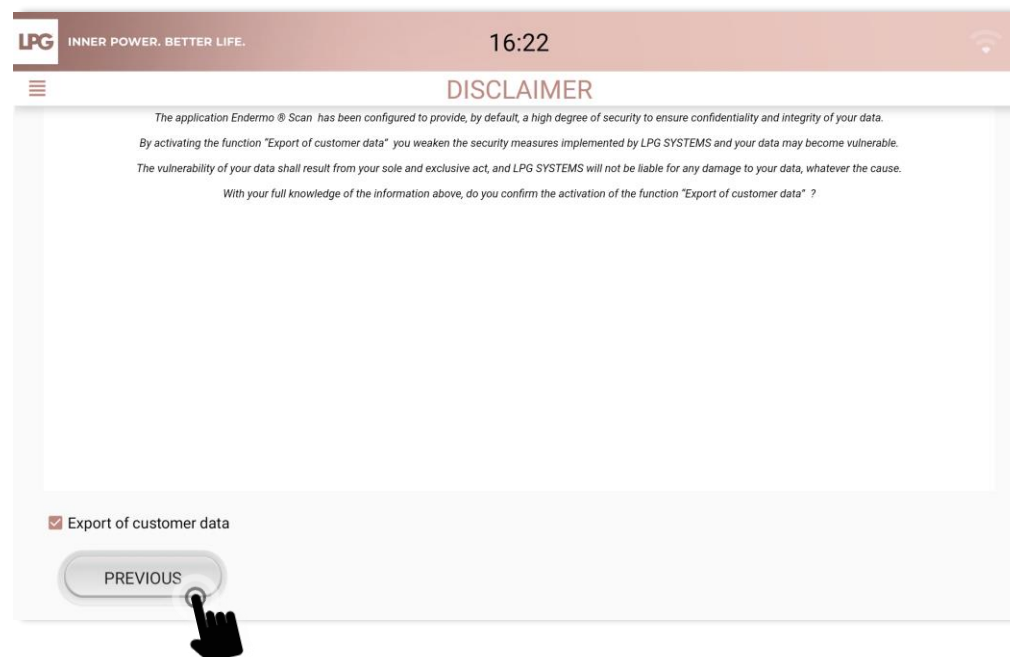

LPG is not responsible for the use of data outside of its application.



**INNER POWER.
BETTER LIFE.**