**INNER POWER.
BETTER LIFE.**

SOMMAIRE

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
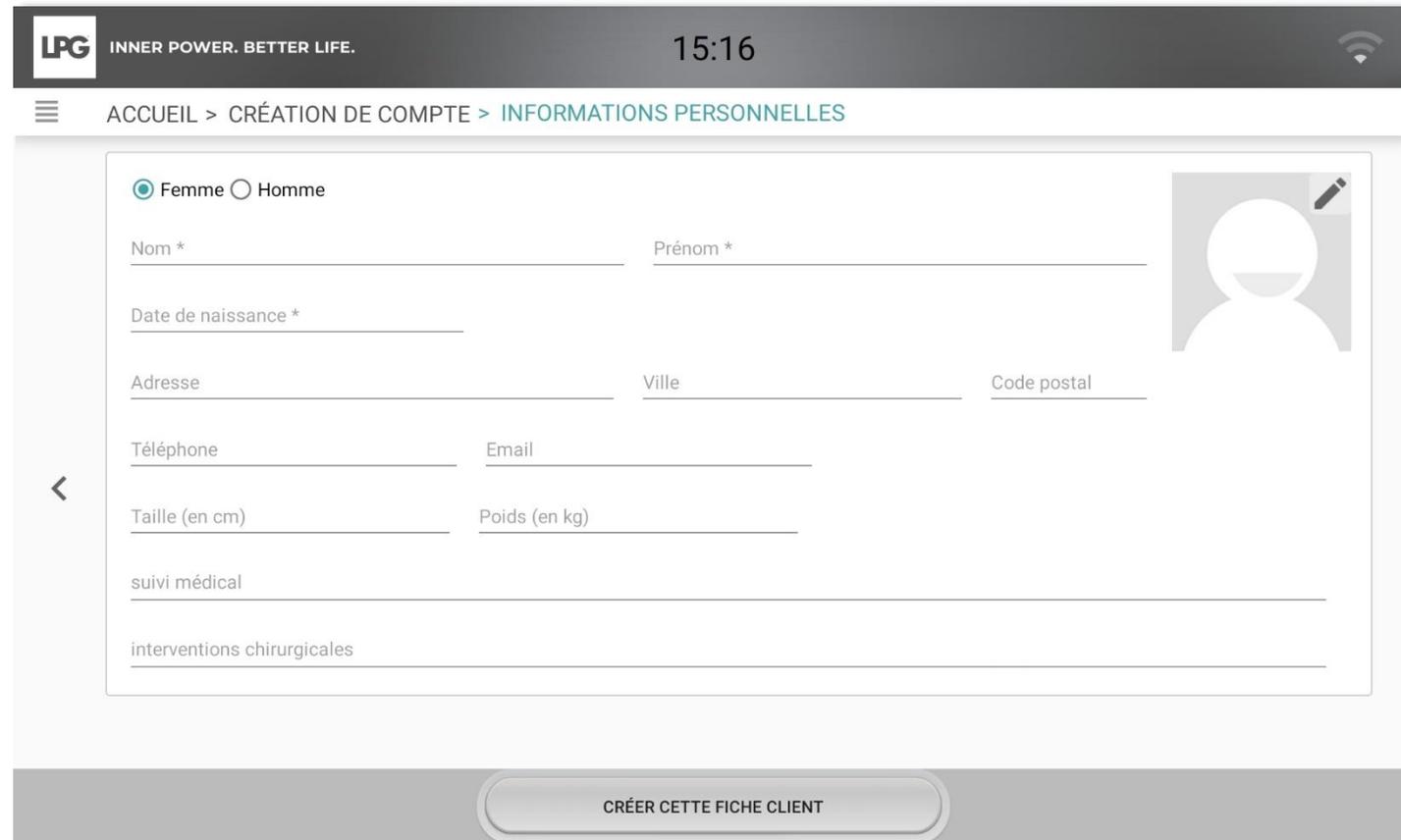
The screenshot shows the LPG mobile application interface for creating a client profile. The top bar displays the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "15:16", and a Wi-Fi signal icon. Below the top bar, a navigation breadcrumb reads "ACCUEIL > CRÉATION DE COMPTE > INFORMATIONS PERSONNELLES". The main form area includes a gender selection with "Femme" selected and "Homme" as an option. There are input fields for "Nom *", "Prénom *", "Date de naissance *", "Adresse", "Ville", "Code postal", "Téléphone", "Email", "Taille (en cm)", and "Poids (en kg)". There are also text areas for "suivi médical" and "interventions chirurgicales". A profile picture placeholder with a pencil icon is located on the right side of the form. A back arrow is visible on the left side of the form. At the bottom of the screen, there is a button labeled "CRÉER CETTE FICHE CLIENT".

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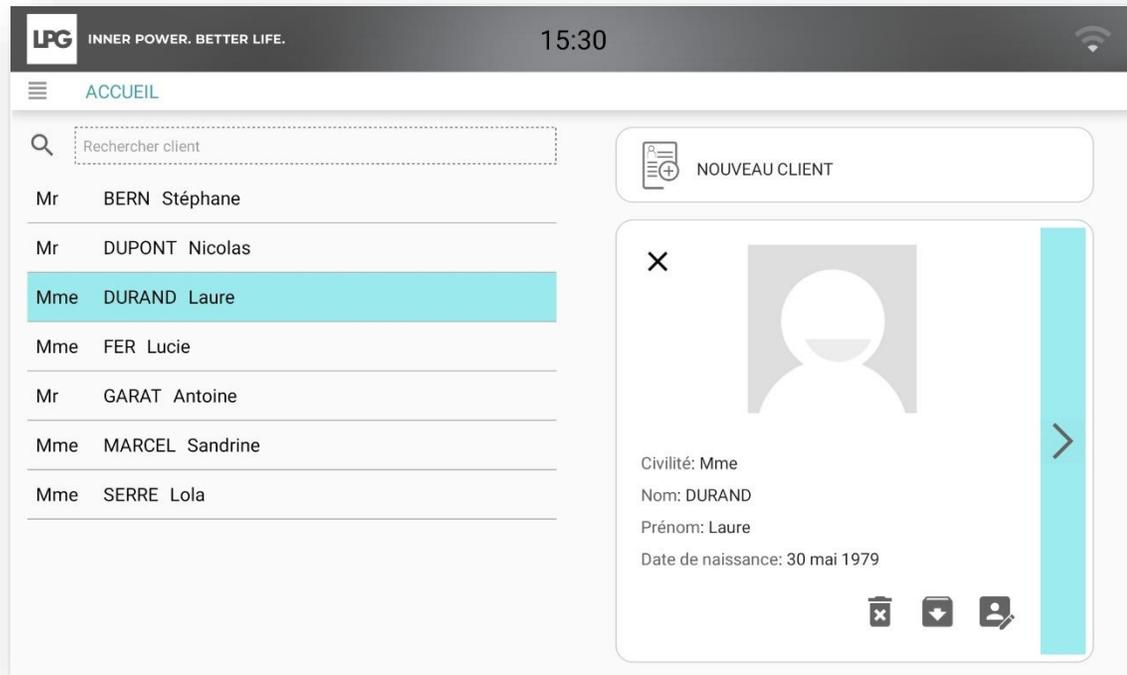
The screenshot shows the 'ACCUEIL' (Home) screen of the LPG application. At the top, the logo 'LPG INNER POWER. BETTER LIFE.' and the time '15:30' are visible. A search bar labeled 'Rechercher client' is present. Below it, a list of clients is displayed, with 'Mme DURAND Laure' highlighted in blue. To the right, a 'NOUVEAU CLIENT' button is visible. A detailed view of the selected client is shown, including a profile picture placeholder, a close button (X), and fields for 'Civilité: Mme', 'Nom: DURAND', 'Prénom: Laure', and 'Date de naissance: 30 mai 1979'. At the bottom of this view are icons for delete, archive, and edit.

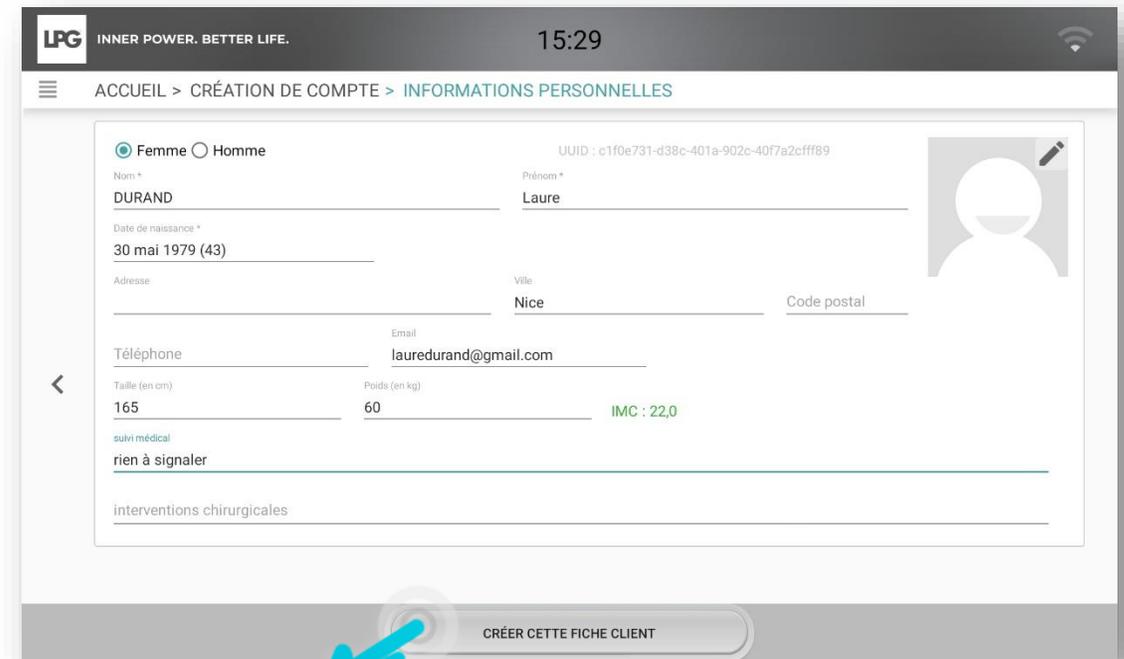The screenshot shows the 'INFORMATIONS PERSONNELLES' (Personal Information) form for creating a new client. The top bar shows 'LPG INNER POWER. BETTER LIFE.' and the time '15:29'. The navigation path is 'ACCUEIL > CRÉATION DE COMPTE > INFORMATIONS PERSONNELLES'. The form includes fields for gender (Femme selected), name (DURAND), first name (Laure), date of birth (30 mai 1979), address (Nice), and email (lauredurand@gmail.com). It also displays physical characteristics: height (165 cm), weight (60 kg), and BMI (22,0). There are sections for 'suivi médical' (medical follow-up) with 'rien à signaler' (nothing to report) and 'interventions chirurgicales' (surgical interventions). A blue hand icon points to the 'CRÉER CETTE FICHE CLIENT' button at the bottom.

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
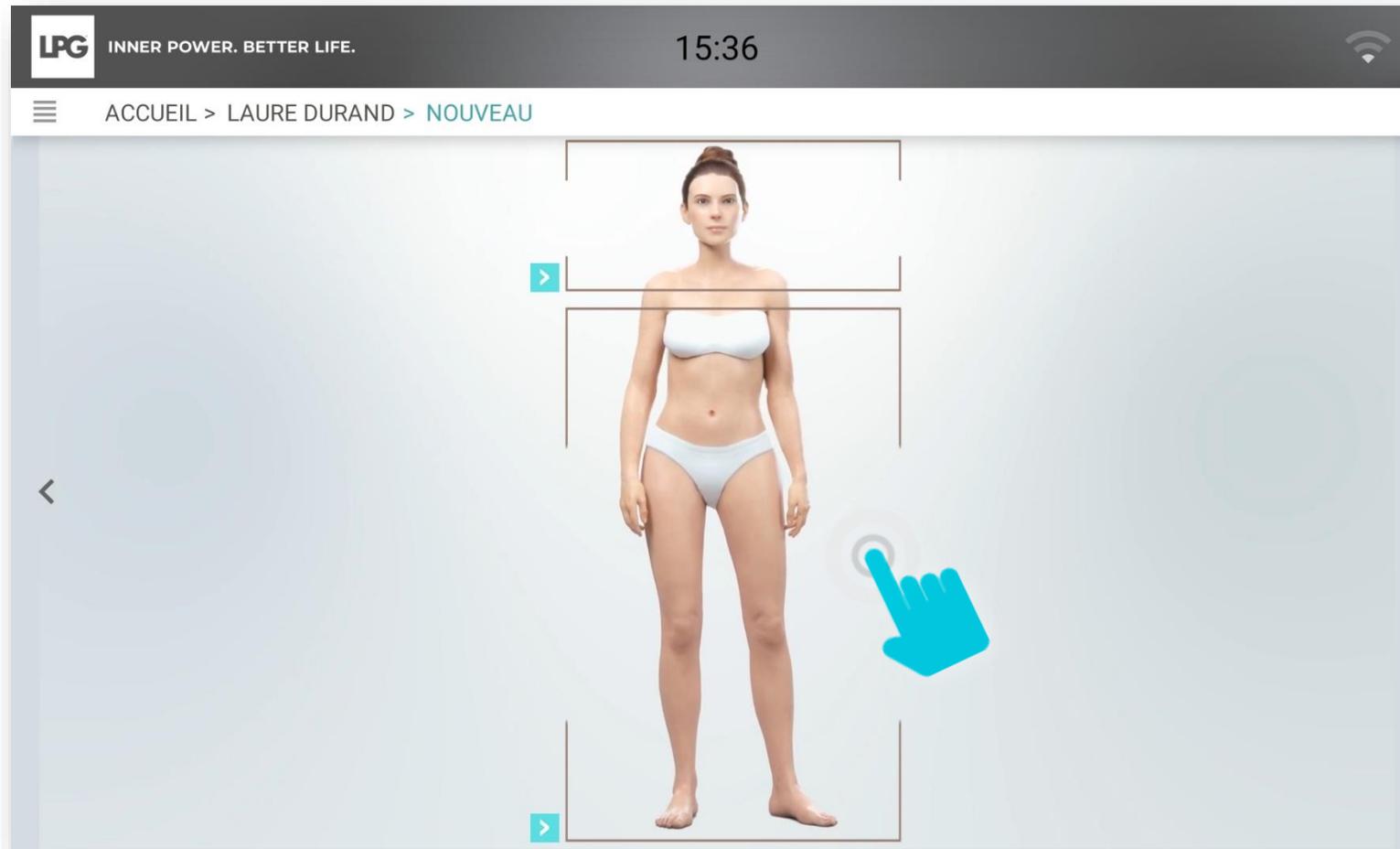
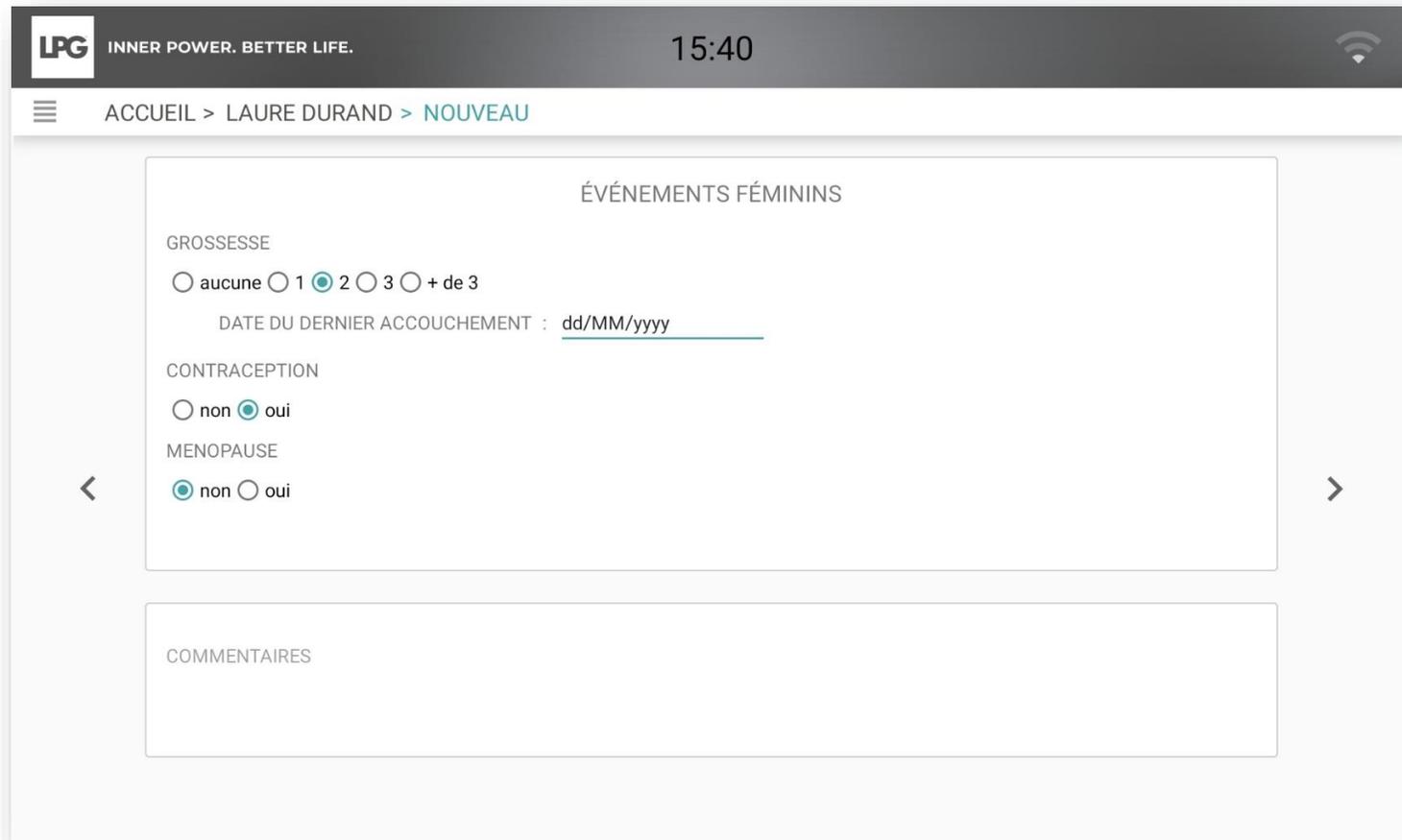
The screenshot shows a mobile application interface for a body evaluation form. At the top, there is a dark header with the LPG logo and the tagline "INNER POWER. BETTER LIFE." on the left, the time "15:40" in the center, and a Wi-Fi icon on the right. Below the header, a breadcrumb trail reads "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is titled "ÉVÉNEMENTS FÉMININS" and contains three sections: "GROSSESSE" with radio buttons for "aucune", "1", "2", "3", and "+ de 3" (where "2" is selected), a date input field for "DATE DU DERNIER ACCOUCHEMENT" with the placeholder "dd/MM/yyyy"; "CONTRACEPTION" with radio buttons for "non" and "oui" (where "oui" is selected); and "MENOPAUSE" with radio buttons for "non" and "oui" (where "non" is selected). Navigation arrows are visible on the left and right sides of the form. Below the form is a "COMMENTAIRES" section with a text input field.

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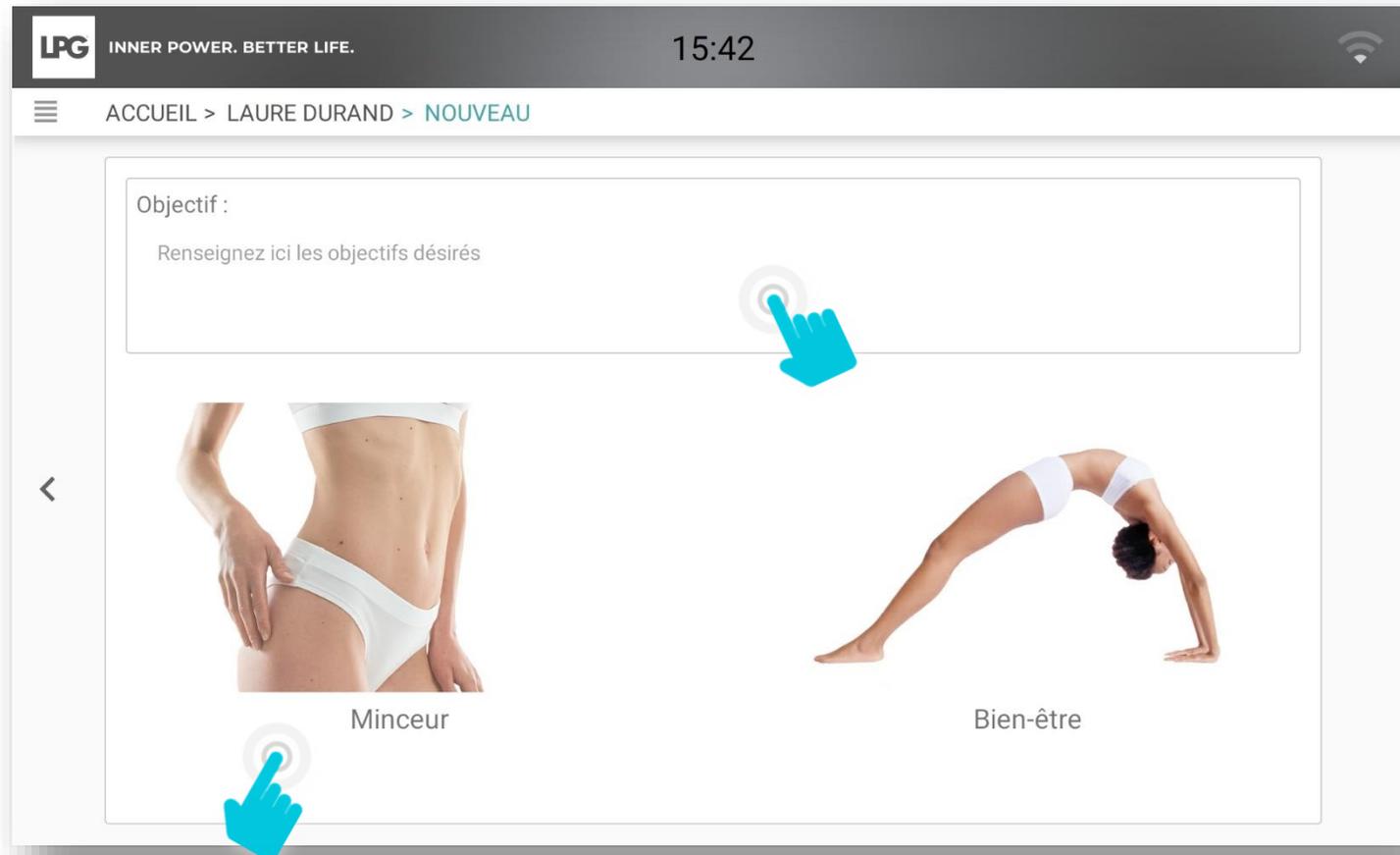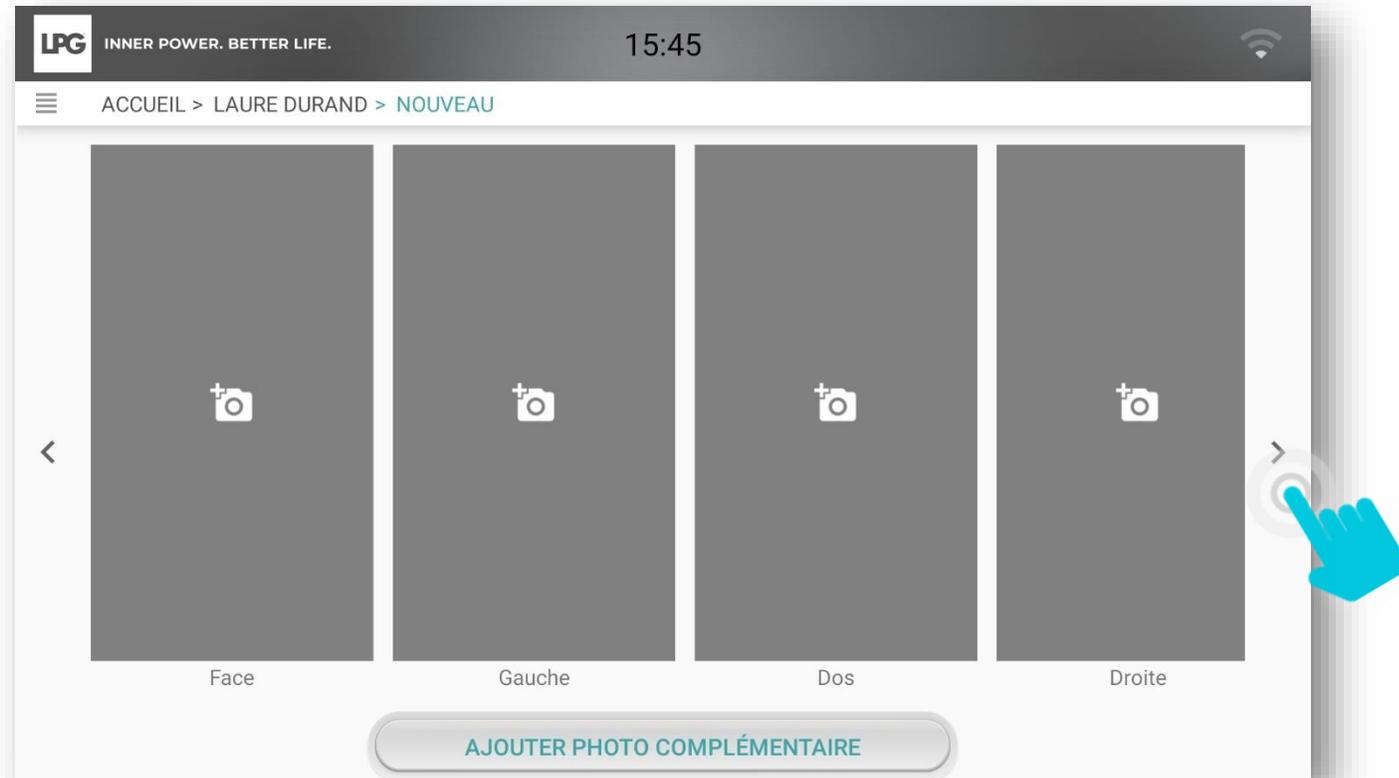
The screenshot shows a mobile application interface for a client named Laure Durand. The top bar displays the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time 15:47, and a Wi-Fi icon. Below the bar, a navigation path reads "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area is titled "Questionnaire rétention d'eau" and contains five questions with checkboxes for answers. The questions are:

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
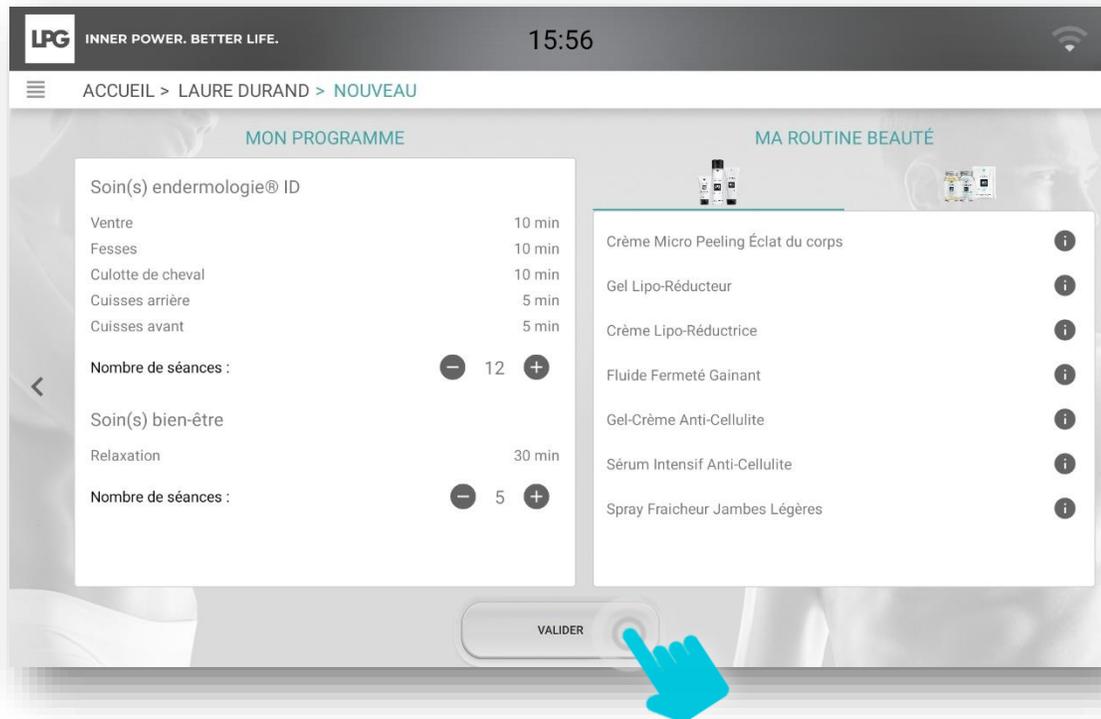
The screenshot shows a mobile application interface. At the top, there is a dark grey header with the LPG logo and the text "INNER POWER. BETTER LIFE." on the left, the time "16:08" in the center, and a Wi-Fi icon on the right. Below the header is a white navigation bar with a hamburger menu icon on the left and the text "ACCUEIL > LAURE DURAND > NOUVEAU" in the center. The main content area is a large white box with a thin border. Inside this box, the text "Objectif :" is followed by a large, empty text input field. Below the input field, the text "Renseignez ici les objectifs désirés" is displayed. On the left and right sides of the main content area, there are grey vertical bars with white left and right arrow navigation icons, respectively.

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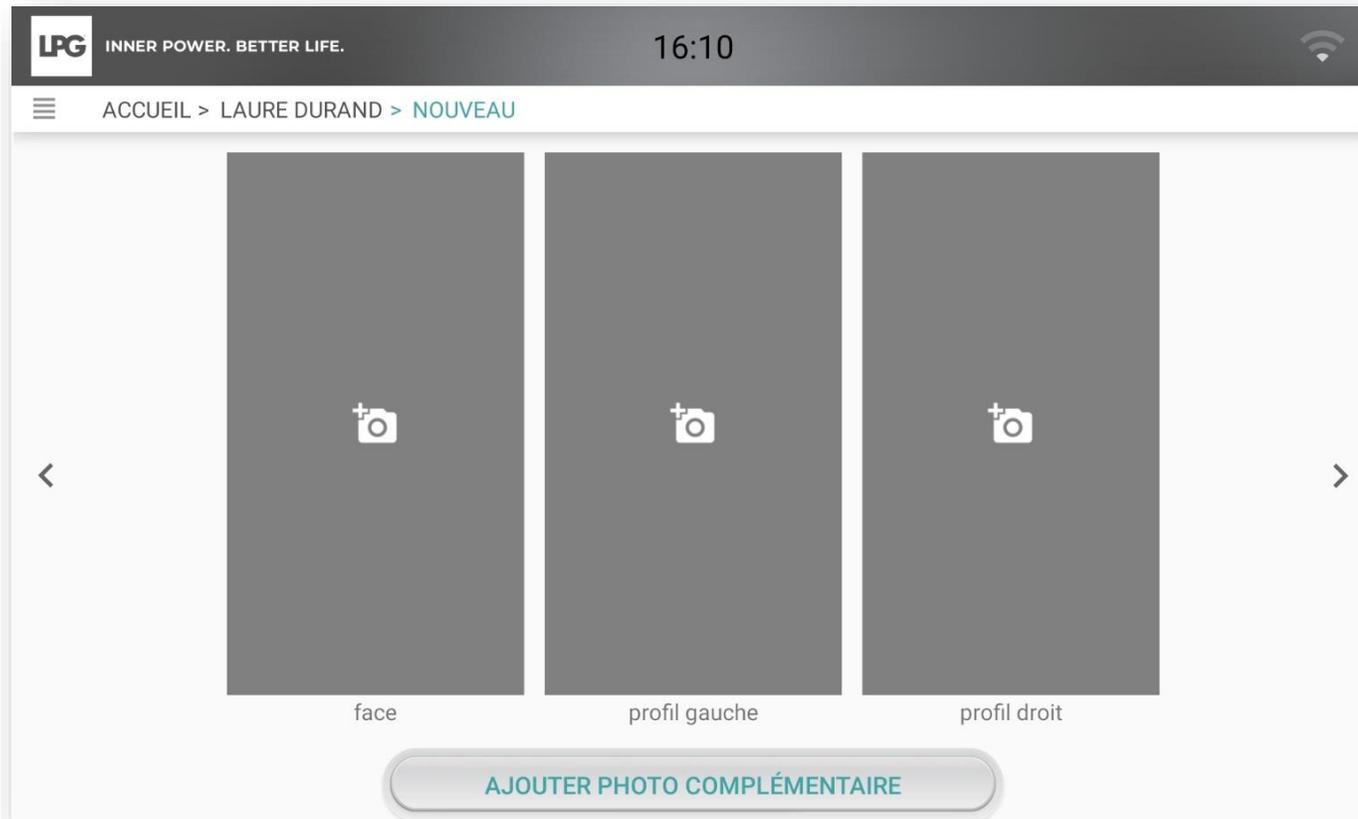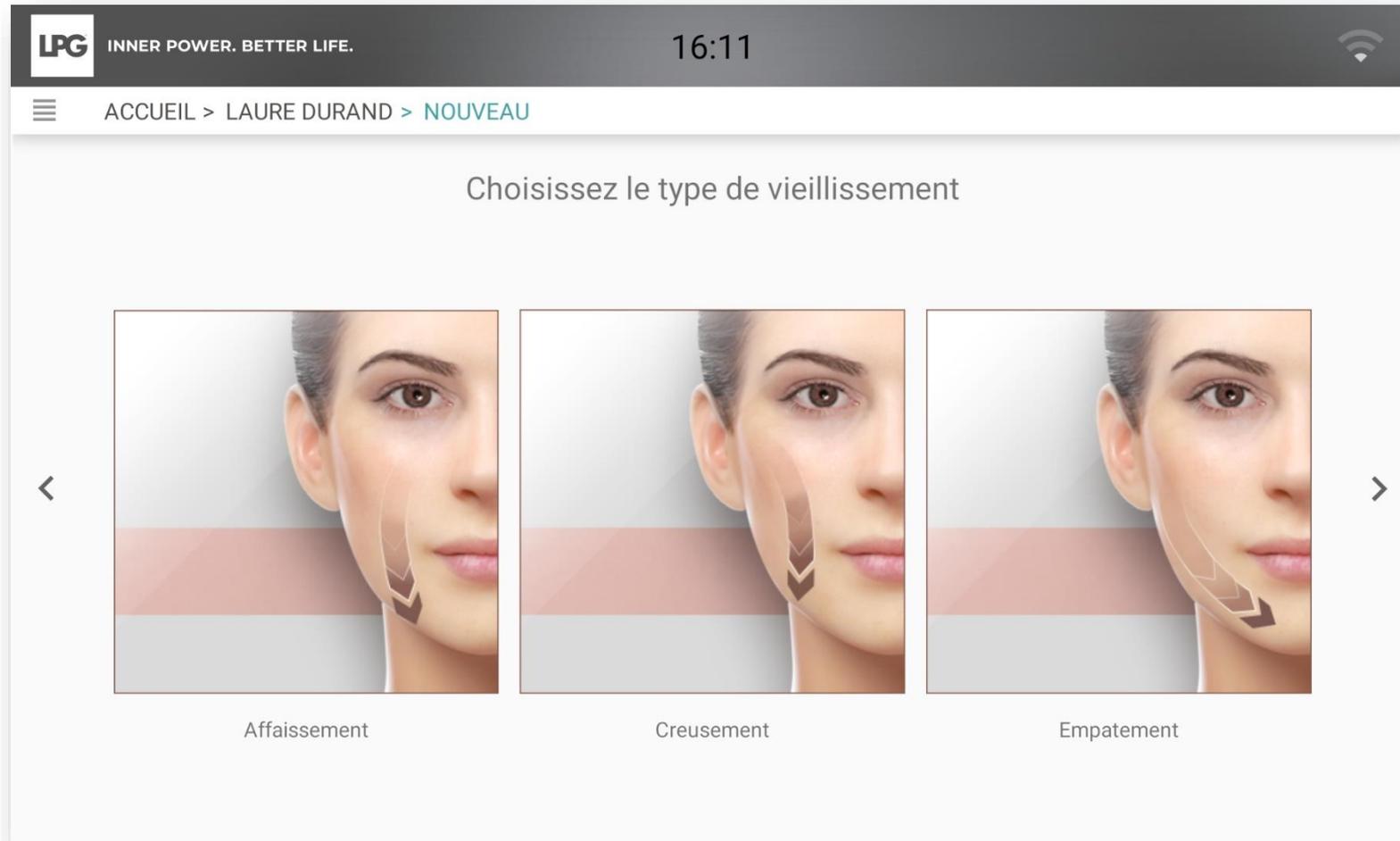
The screenshot shows a mobile application interface for skin aging evaluation. At the top, there is a dark grey header with the LPG logo and the tagline "INNER POWER. BETTER LIFE." on the left, the time "16:11" in the center, and a Wi-Fi icon on the right. Below the header is a white navigation bar with a menu icon and the text "ACCUEIL > LAURE DURAND > NOUVEAU". The main content area has a light grey background with the text "Choisissez le type de vieillissement" centered. Below this text are three square images of a woman's face, each showing a different skin aging sign: "Affaissement" (sagging), "Creusement" (wrinkles), and "Empatement" (swelling). Each image has a red horizontal bar at the bottom and a brown arrow pointing to the specific aging sign. Navigation arrows are visible on the left and right sides of the images.

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Liste des programmes

Date de l'évaluation initiale

The screenshot shows the LPG mobile application interface. At the top, there is a header with the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "16:31", and a navigation bar with "ACCUEIL > LAURE DURAND". Below the header, there are two program cards. The first card is for "VISAGE" dated "4 NOVEMBRE 2022". It features a list of treatments: "Regard poches & cernes - zone péri-buccale" and "Sublime regard & lèvres". It includes two photos of a woman's face (front and profile), a calendar icon, a camera icon, and a set of cosmetic products. The second card is for "CORPS" dated "4 NOVEMBRE 2022". It features a list of treatments: "Ventre - fesses - culotte de cheval - cuisses arrière - cuisses avant" and "Relaxation". It includes two photos of a woman's body (front and side), a calendar icon, a camera icon, and a set of LPG products. At the bottom of the screen, there is a grey bar with a plus sign icon and three action icons: a share icon, a pencil icon, and a trash icon.

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The screenshot shows the LPG app interface. At the top, there's a header with the LPG logo and the slogan 'INNER POWER. BETTER LIFE.' The time is 16:49. Below the header, there's a navigation bar with 'ACCUEIL > LAURE DURAND > MES SÉANCES'. The main content area shows a list of sessions. The first session is expanded, showing a list of treatments with their durations. The session is labeled '1/12' and has a total duration of 40 minutes. The treatments are: 'Ventre - souple' (10 min), 'Fesses - Fibreux' (10 min), 'Culotte de cheval - dense' (10 min), 'Cuisses arrière - souple' (5 min), and 'Cuisses avant - souple' (5 min). There are icons for validation, comments, and deletion for each session.

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The screenshot displays the LPG mobile application interface. At the top, the header includes the LPG logo, the slogan "INNER POWER. BETTER LIFE.", the time "10:40", and a Wi-Fi signal icon. Below the header, a navigation bar shows the path "ACCUEIL > LAURE DURAND > MA ROUTINE BEAUTÉ". The main content area is divided into two columns. The left column features a category "REGARD POCHES & CERNES - ZONE PÉRI-BUCCALE" with an upward arrow. Underneath, four product items are listed: "EAU MICELLAIRE PRÉPARATRICE" (highlighted with a teal background), "BAUME YEUX", "CRÈME LISSANTE REPULPANTE", and "GEL-CRÈME LISSANT REPULPANT". At the bottom of this list is an "ANTI-ÂGE REPULPANT" category with a downward arrow. The right column is dedicated to the selected product, "EAU MICELLAIRE PRÉPARATRICE", featuring a product image, a list of benefits, and usage instructions. The benefits listed are: "Démaquille et nettoie les impuretés" and "Hydrate et apaise". The usage instructions state: "Imbibez un coton et nettoyez le visage, les yeux et le cou, matin et soir. Séchez la peau délicatement sans rincer."

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The screenshot shows the LPG mobile application interface. At the top, the LPG logo and tagline 'INNER POWER. BETTER LIFE.' are on the left, the time '15:25' is in the center, and a Wi-Fi icon is on the right. Below the header, a breadcrumb trail reads 'ACCUEIL > LAURE DURAND > MA ROUTINE BEAUTÉ'. The main content area is divided into two columns. The left column is a vertical list of product categories: 'REGARD ANTI-RIDES - DOUBLE-MENTON - COU', 'BOOSTER DE VITALITÉ' (highlighted in a teal box), 'ACIDE HYALURONIQUE', 'COLLAGÈNE', 'OMÉGA', and 'HYDRATATION ECLAT'. The right column features the 'BOOSTER DE VITALITÉ' product details, including a product image, a list of benefits, and usage instructions.

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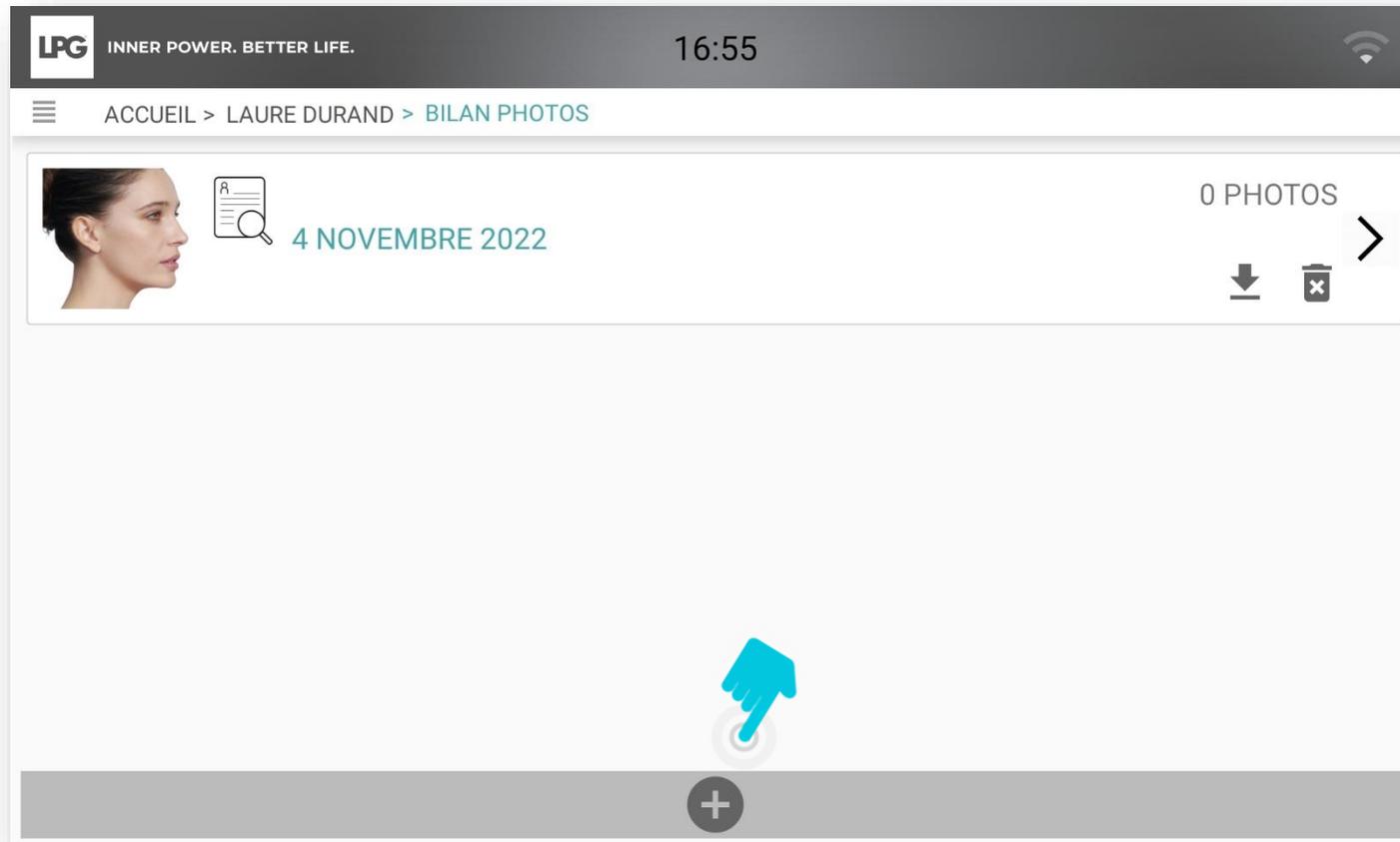
The screenshot shows a mobile application interface. At the top, there is a dark header with the LPG logo and the slogan 'INNER POWER. BETTER LIFE.' on the left, the time '17:02' in the center, and a Wi-Fi icon on the right. Below the header is a breadcrumb trail: 'ACCUEIL > LAURE DURAND > BILAN PHOTOS'. The main content area contains two text input fields. The first field is labeled 'Objectif :'. The second field is larger and labeled 'Ajouter votre commentaire ici'. There are left and right arrow navigation icons on the sides of the comment field.

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The image shows three white bottles of LPG skincare products. The two bottles in the foreground are 'CRÈME MICRO-PEELING RESURFACING CREAM' with 400 ML / 13.5 FL. OZ. The bottle in the background is 'PRÉPARATEUR DE SOIN CORPS SKINCARE BODY PREPARE' with 400 ML / 13.5 FL. OZ. All bottles feature the LPG logo and the text 'endermostim' and 'AHA, Antioxydants, Vitamines'. A large right-pointing arrow is visible on the right side of the product image.

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