



HUBER 360[®] EVOLUTION APP



Tablet application
USER GUIDE



HUBER 360[®] EVOLUTION APP



SUMMARY

1. USE HUBER 360[®] APP	p 03		
1.a GENERAL MENU	p 04		
1.b THERAPISTS	p 05		
1.b.1 Create a therapist profile	p 06		
1.c PATIENTS	p 07		
1.c.1 Create a patient profile	p 08		
1.c.2 Modify and archive a patient profile	p 09		
1.c.3 Access to general information	p 10		
1.c.4 Questionnaire, medical history, personal data	p 12		
1.d TREATMENT PROGRAMS	p 16		
1.d.1 Create a treatment program	p 17		
1.d.2 Session launch	p 19		
1.d.3 Viewing sessions	p 20		
1.d.4 Modify sessions	p 21		
1.d.5 Viewing treatment programs	p 22		
1.d.6 Viewing progression	p 23		
1.d.7 Free sessions	p 24		
1.d.8 Orphan sessions	p 25		
		1.e ASSESSMENTS	p 26
		1.e.1 Make a Medical assessment	p 26
		1.e.2 Make a Fitness assessment	p 32
		1.e.3 View assessments	p 33
		1.e.4 Send an assessment	p 37
		1.e.5 Orphan assessment	p 39
		1.f CUSTOMIZED EXERCISES	p 40
		1.f.1 General Menu	p 41
		1.f.2 Create a customized exercise	p 42
		1.f.3 Recover a modified exercise	p 47
		1.f.4 Export a customized exercise	p 50
		1.f.5 Import a customized exercise	p 51
		2. SEND TREATMENT PROGRAM	p 52
		3. DATA IMPORT	p 57
		3.a Data back-up	p 58
		3.b Data recovery	p 61
		4. DATA EXPORT	p 68



HUBER 360[®] EVOLUTION APP



1. USE HUBER 360[®] EVOLUTION APP



HUBER 360[®] EVOLUTION APP



1.a GENERAL MENU



SETTINGS : To access different functions (language choice, maintenance, software version...)

« **Disclaimer** » makes you aware of confidentiality and integrity of data if you decide to export our customer data.

THERAPISTS: To create a therapist profile.

CUSTOMIZED EXERCISE: To create your own exercises.

PATIENT FILE BACK-UP: To access patients who finished their treatment program.

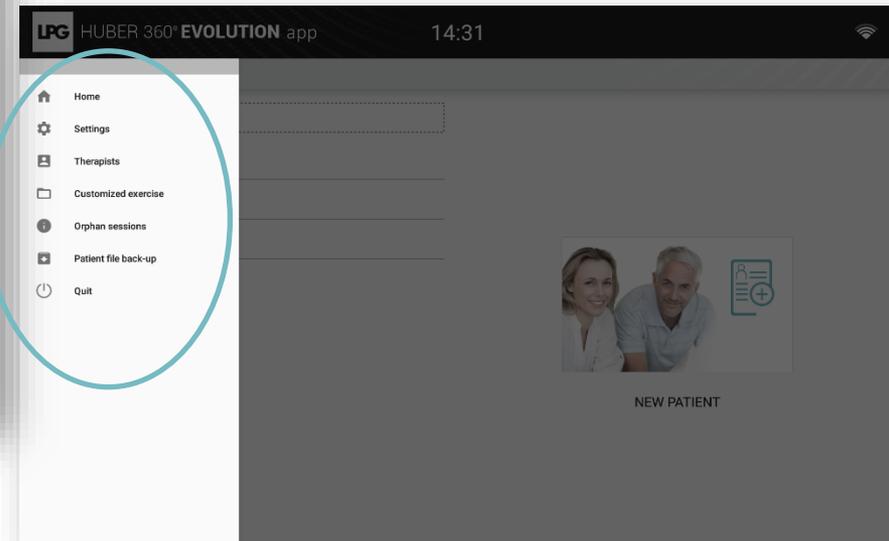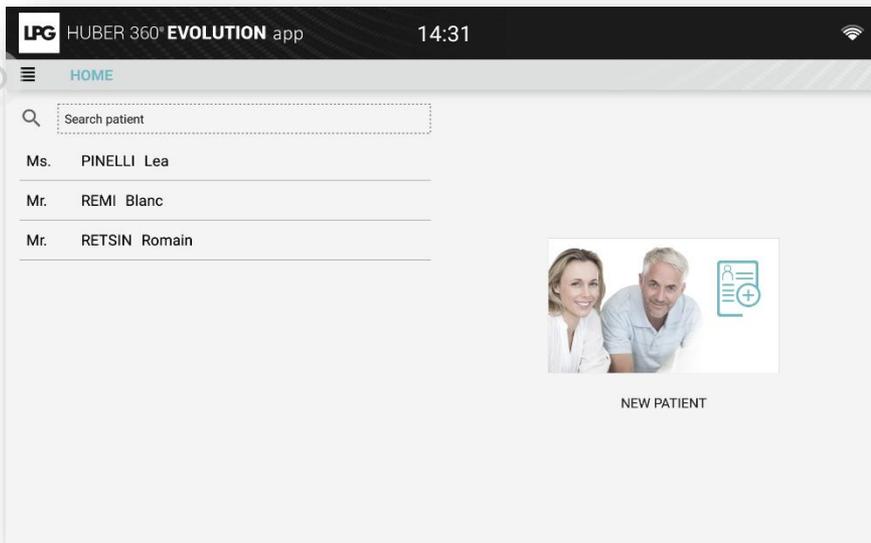





HUBER 360[®] EVOLUTION APP



1.b THERAPISTS

1.b.1 CREATE THERAPIST ACCOUNT

Click on  and select **THERAPISTS**.

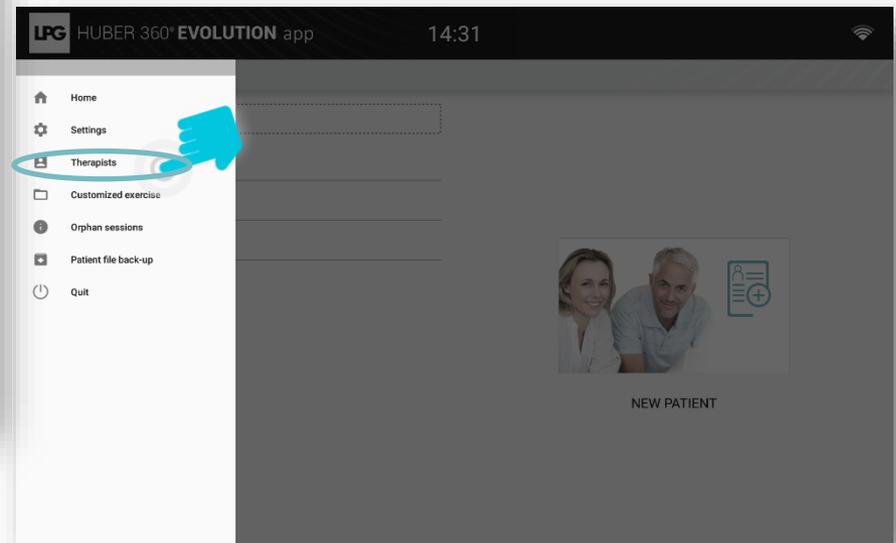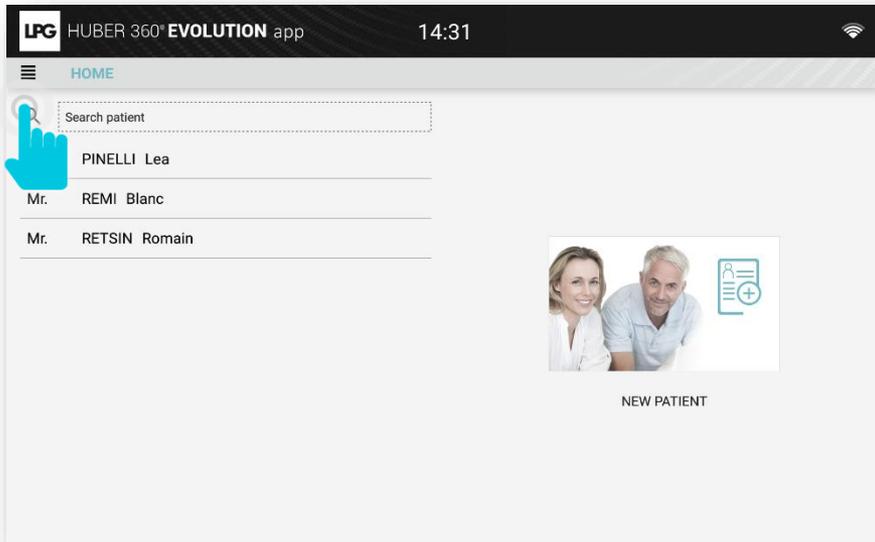





HUBER 360[®] EVOLUTION APP



1.b THERAPISTS

1.b.1 CREATE THERAPIST ACCOUNT

Click on NEW THERAPIST.
Fill in information. You can add a picture.
Then, click on CONFIRM to validate the profile.

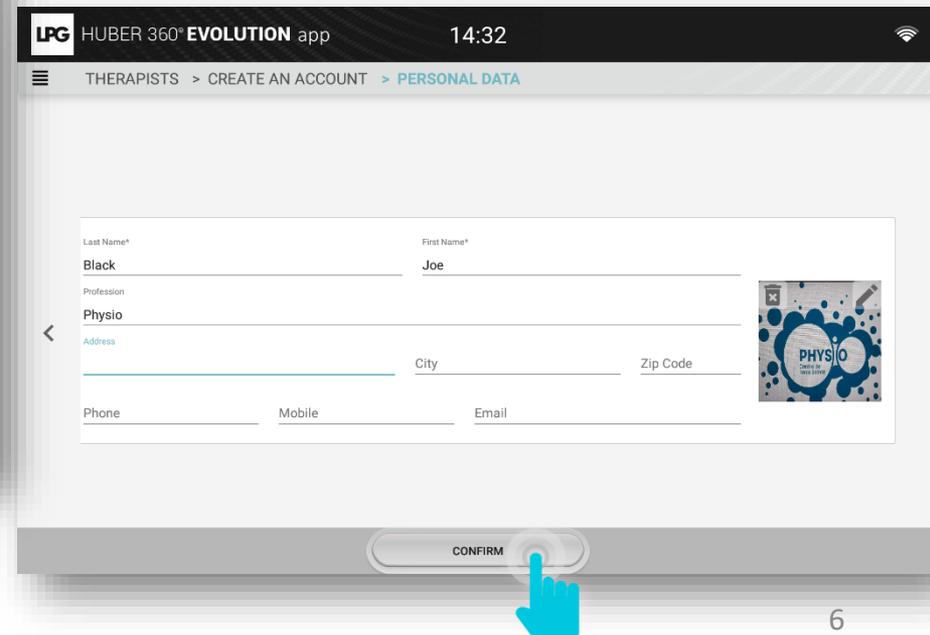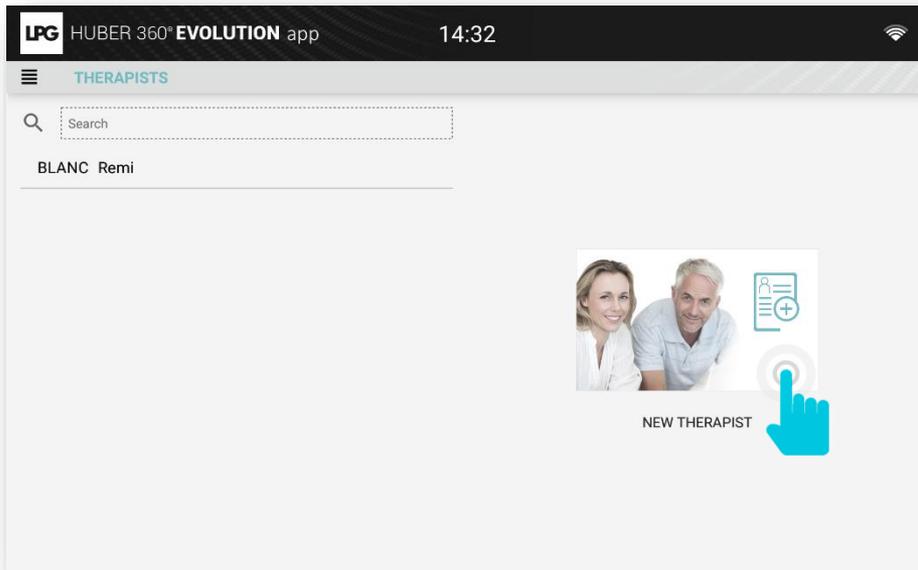




1.c PATIENTS

1.c.1 CREATE PATIENT ACCOUNT

Select NEW PATIENT on the home screen.

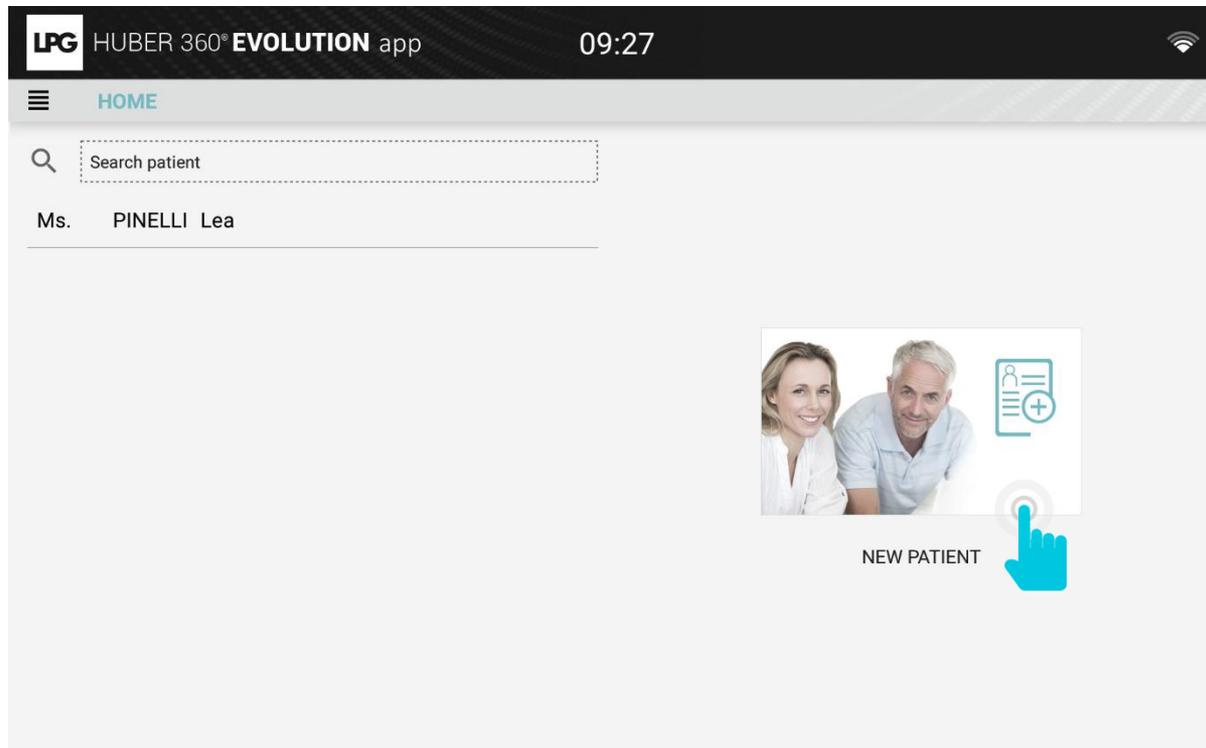



HUBER 360[®] EVOLUTION APP



1.c PATIENTS

1.c.1 CREATE PATIENT ACCOUNT

Fill in patient's information.

You can add a picture and choose a therapist (after creating the account).

The BMI is calculated automatically after filling in the height and the weight.

Click on CREATE THIS PATIENT PROFILE.

LPG HUBER 360[®] EVOLUTION app 14:33

HOME > CREATE AN ACCOUNT > PERSONAL DATA

Woman Man Login* Password*

Last Name* First Name*

Date of Birth Profession

Address City Zip Code

Phone Mobile Email

Height (in cm) Weight (in kg) Laterality : Right Left

Therapist : ▼

Doctor Email SSN

Medical follow-up

CREATE THIS PATIENT PROFILE

Please fill in a login and a password for each of your patients in order to identify them on your device and to access to their customized treatment programs.




HUBER 360[®] EVOLUTION APP



1.c PATIENTS

1.c.2 MODIFY AND ARCHIVE A PATIENT PROFILE

- To edit a patient profile, select your patient's name and click on . Don't forget to click on **CONFIRM** to save your modifications.
- To archive a patient profile, click on .

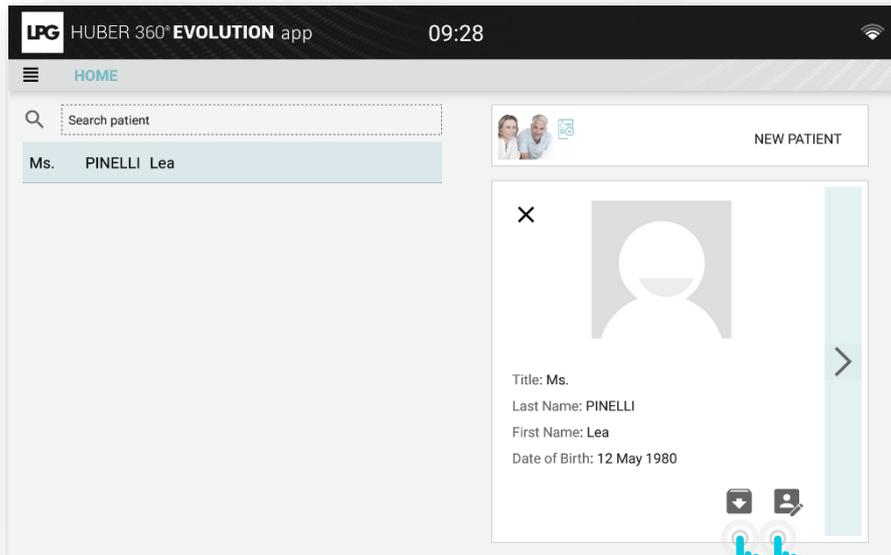

HUBER 360[®] EVOLUTION app 09:28


HOME

Search patient

Ms. PINELLI Lea

NEW PATIENT

Title: Ms.
Last Name: PINELLI
First Name: Lea
Date of Birth: 12 May 1980

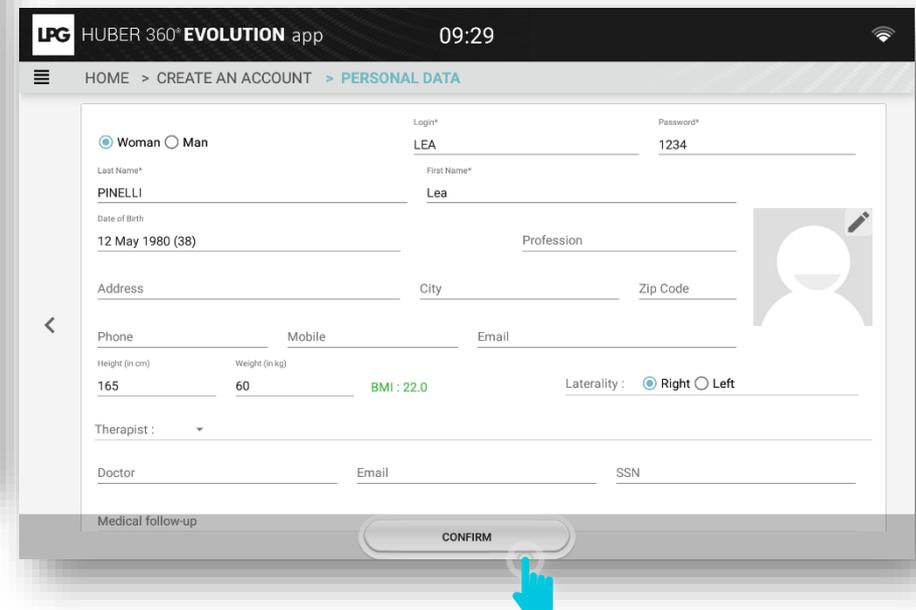

HUBER 360[®] EVOLUTION app 09:29

HOME > CREATE AN ACCOUNT > PERSONAL DATA

Woman Man

Login* LEA Password* 1234

Last Name* PINELLI First Name* Lea

Date of Birth 12 May 1980 (38) Profession

Address City Zip Code

Phone Mobile Email

Height (in cm) 165 Weight (in kg) 60 BMI : 22.0 Laterality : Right Left

Therapist :

Doctor Email SSN

Medical follow-up


CONFIRM



1.c PATIENTS

1.c.3 ACCESS GENERAL INFORMATION

On the home screen, select the patient concerned and click on the arrow below.

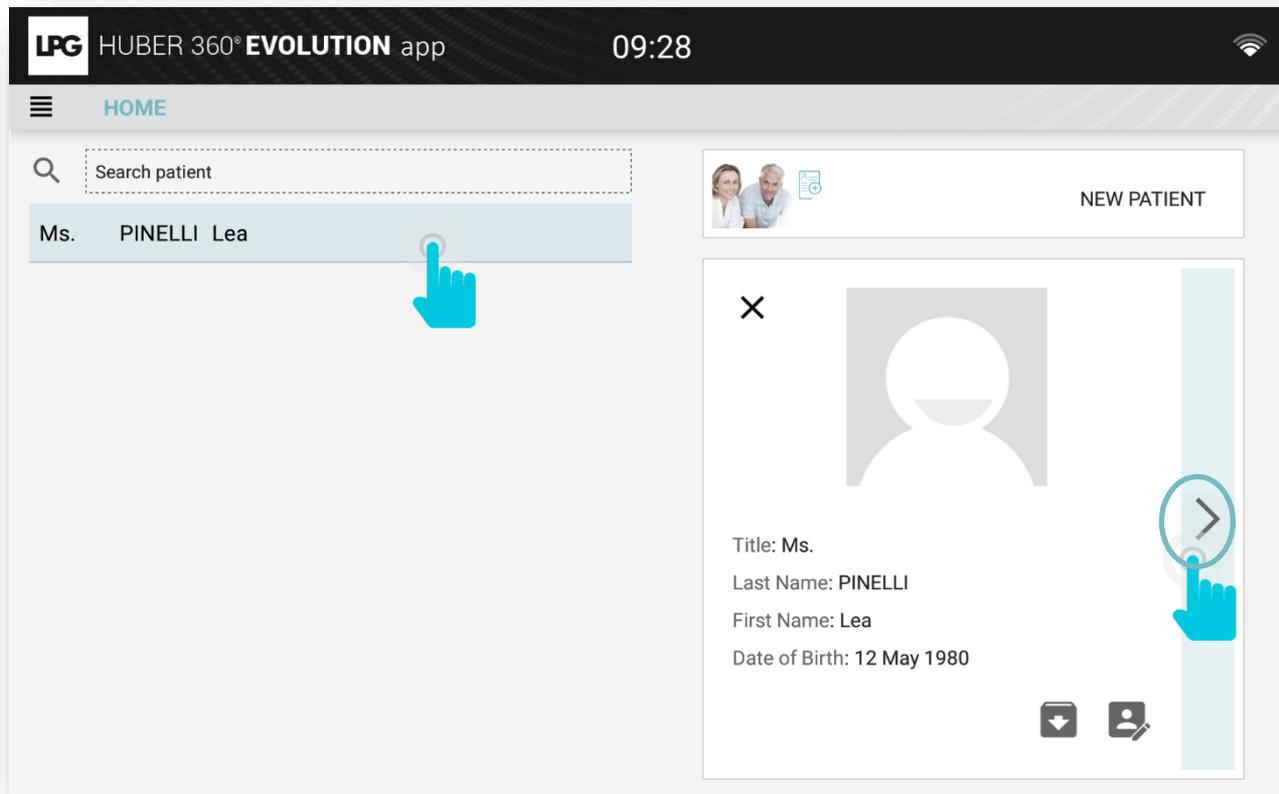



HUBER 360[®] EVOLUTION APP




1.c PATIENTS

1.c.3 ACCESS GENERAL INFORMATION

MY PROGRAMS: Current programs of your patient

MY ASSESSMENTS: Patient's assessments

MY PROFILE: Patient's information → quality of life questionnaire, medical history, personal data

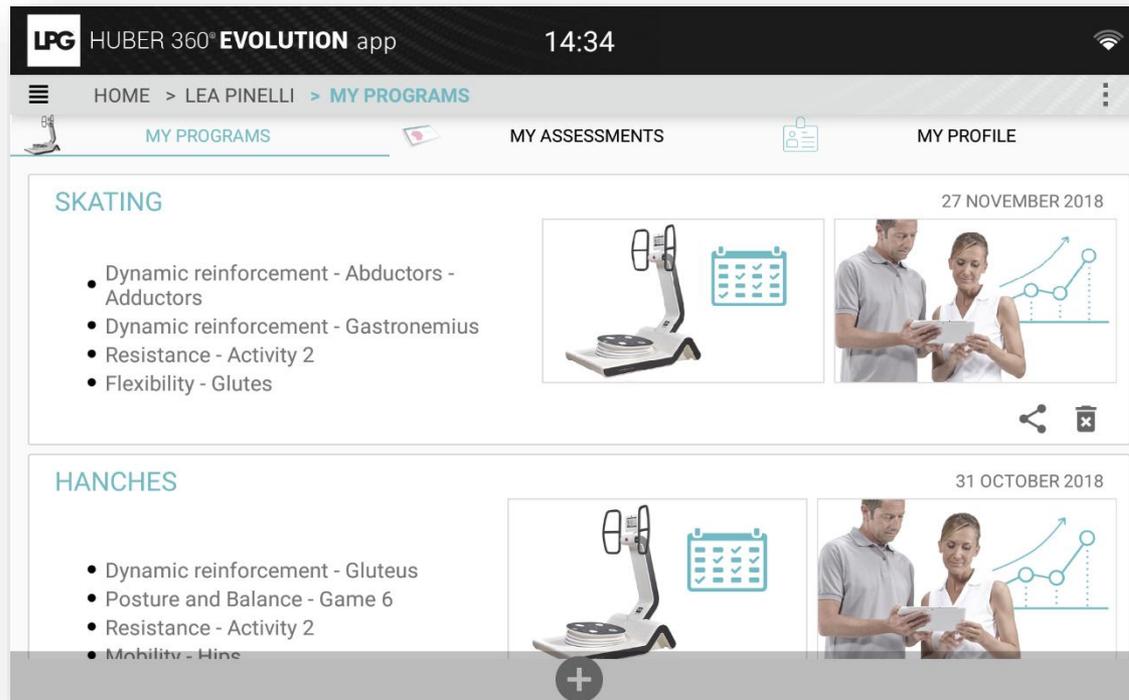




1.c PATIENTS

1.c.4 Questionnaire, medical history, personal data

To complete patient's information, click on MY PROFILE.

- **QUESTIONNAIRE:** Quality of life questionnaire.
- **MEDICAL HISTORY**
- **PERSONAL DATA:** Contact information.

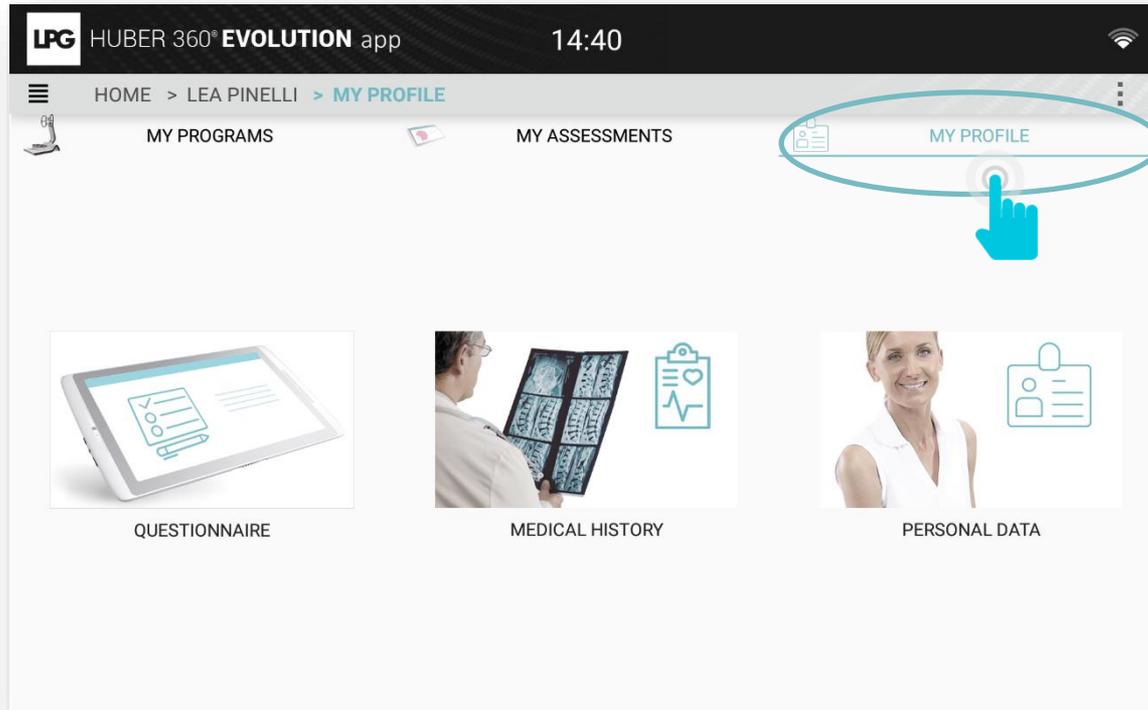



HUBER 360[®] EVOLUTION APP




1.c PATIENTS

1.c.4 Questionnaire, medical history, personal data



To carry out your patient's quality of life questionnaire, click on  Select answers and click on OK to save the questionnaire.

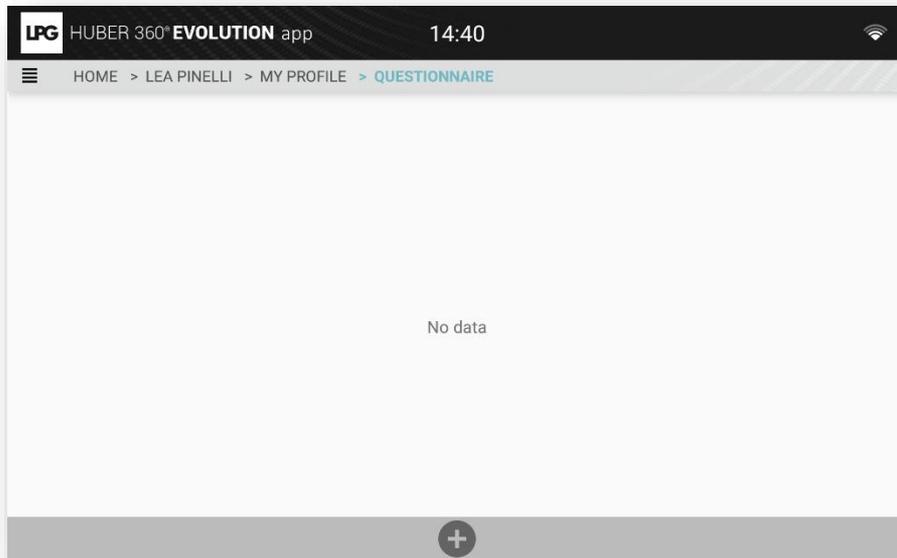

HUBER 360[®] EVOLUTION app 14:40

HOME > LEA PINELLI > MY PROFILE > QUESTIONNAIRE

New questionnaire

You sleep:

Very well Well Poorly Very poorly

You are tired when you wake up:

Never Rarely Often Very often

Your work generates pain:

Not at all A little bit Quite A lot

You feel stressed:

Not at all A little bit Quite A lot

You exercise:

Very often Often Rarely Never

You drink water:

Very often Often Rarely Never

You smoke per day:

0 cigarette 2-5 cigarettes 5-10 cigarettes More than 10 cigarettes

Your diet is balanced:

Very often Often Rarely Never

Do you feel physically limited in your daily activities?

Not at all A little bit Quite Very much

CANCEL **OK**





HUBER 360[®] EVOLUTION APP




1.c PATIENTS


1.c.4 Questionnaire, *medical history*, personal data



To carry out your patient's medical history, click on 
After filling in information, click on SAVE.

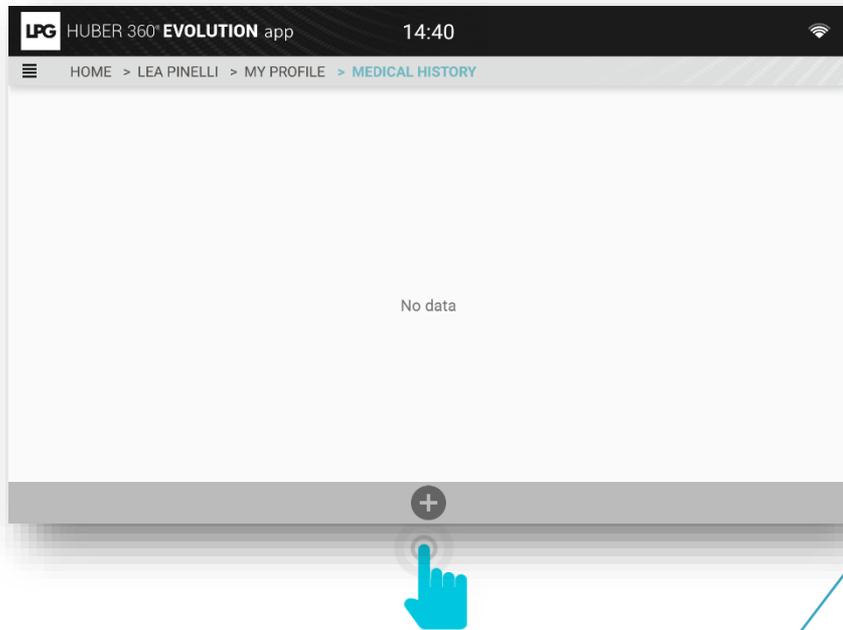

To add comments

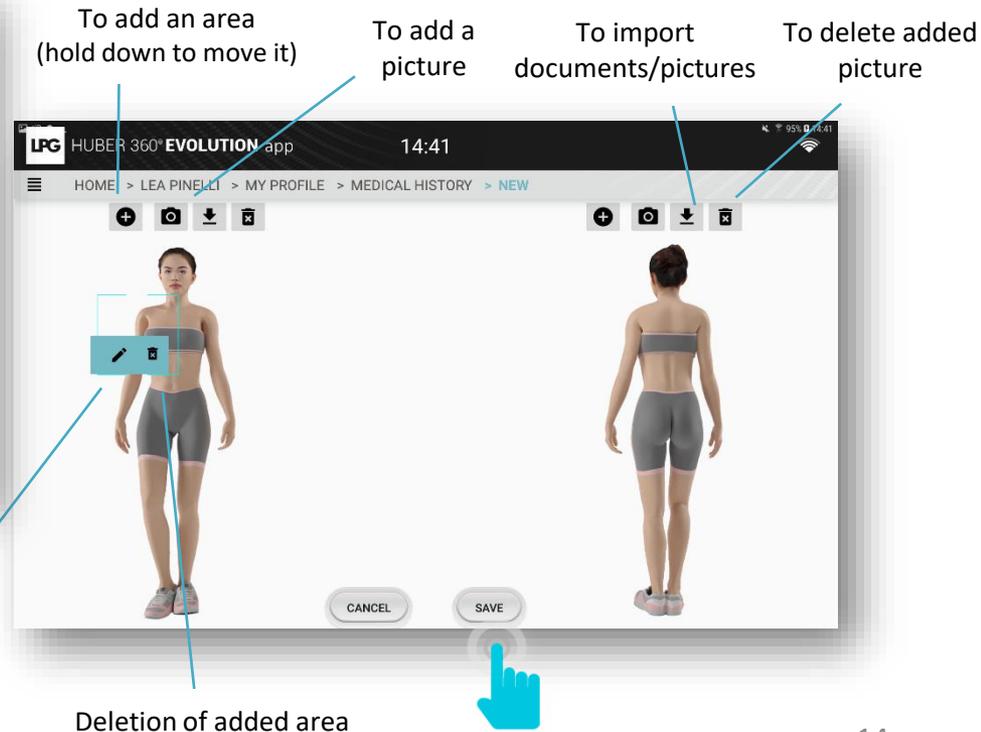

To add an area
(hold down to move it)

To add a picture

To import documents/pictures

To delete added picture



Deletion of added area






1.d TREATMENT PROGRAMS

1.d.1 CREATE NEW TREATMENT PROGRAM

Click on MY PROGRAMS section, then click on  .

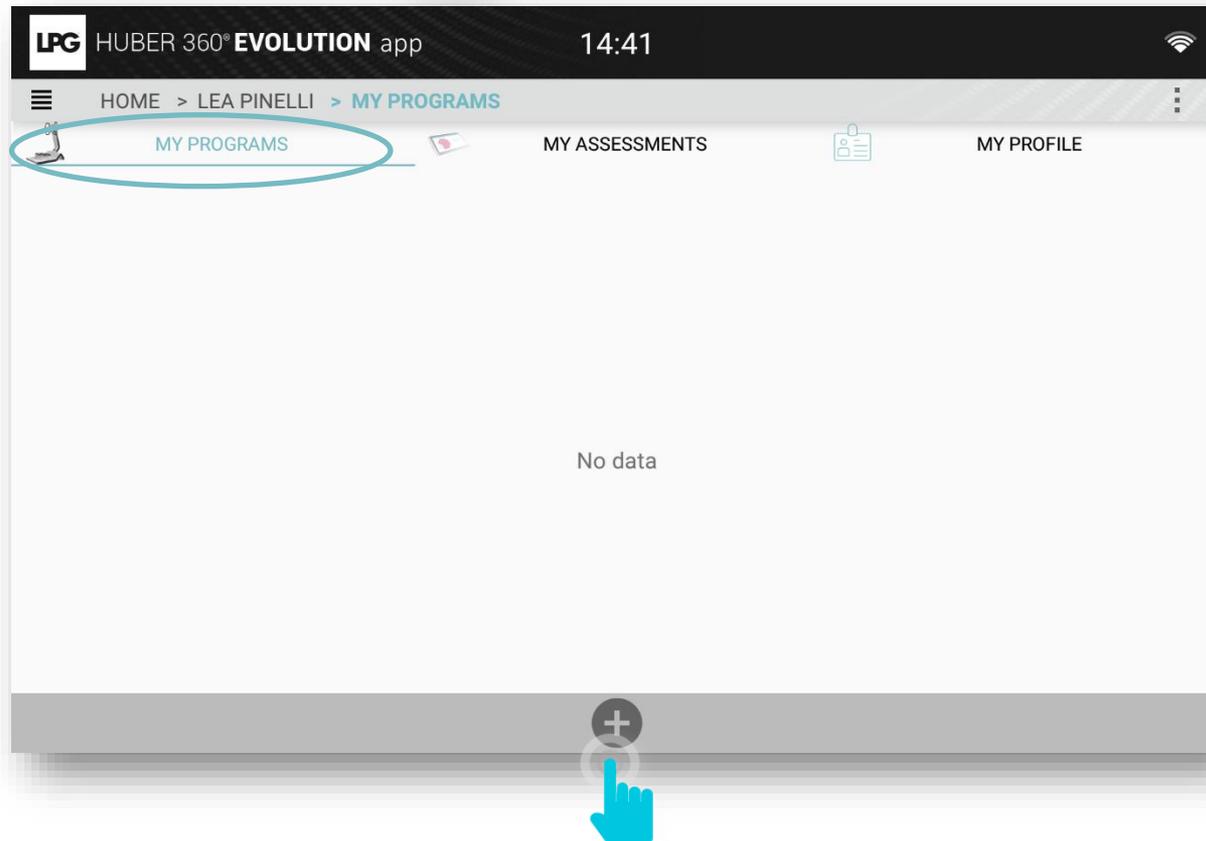




1.d TREATMENT PROGRAMS

1.d.1 CREATE NEW TREATMENT PROGRAM

Select the required menu.

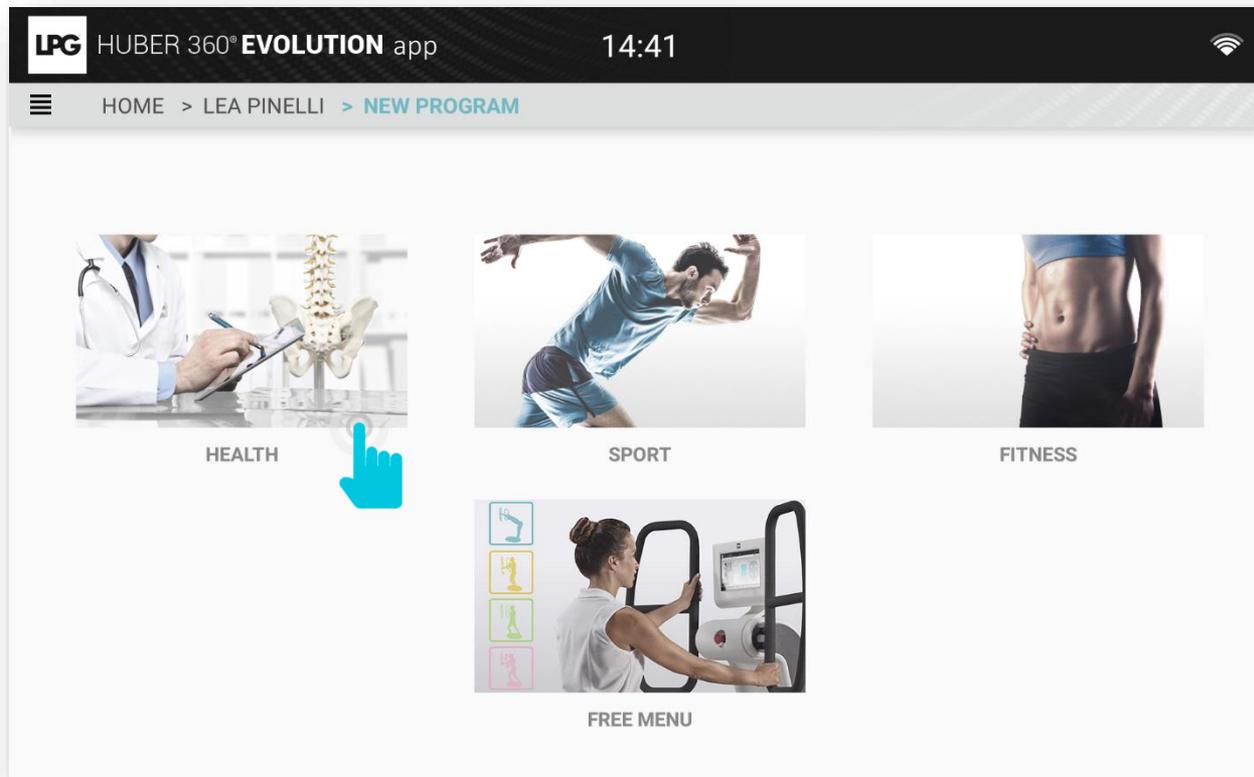




1.d TREATMENT PROGRAMS

1.d.1 CREATE NEW TREATMENT PROGRAM

Next, select the required theme.

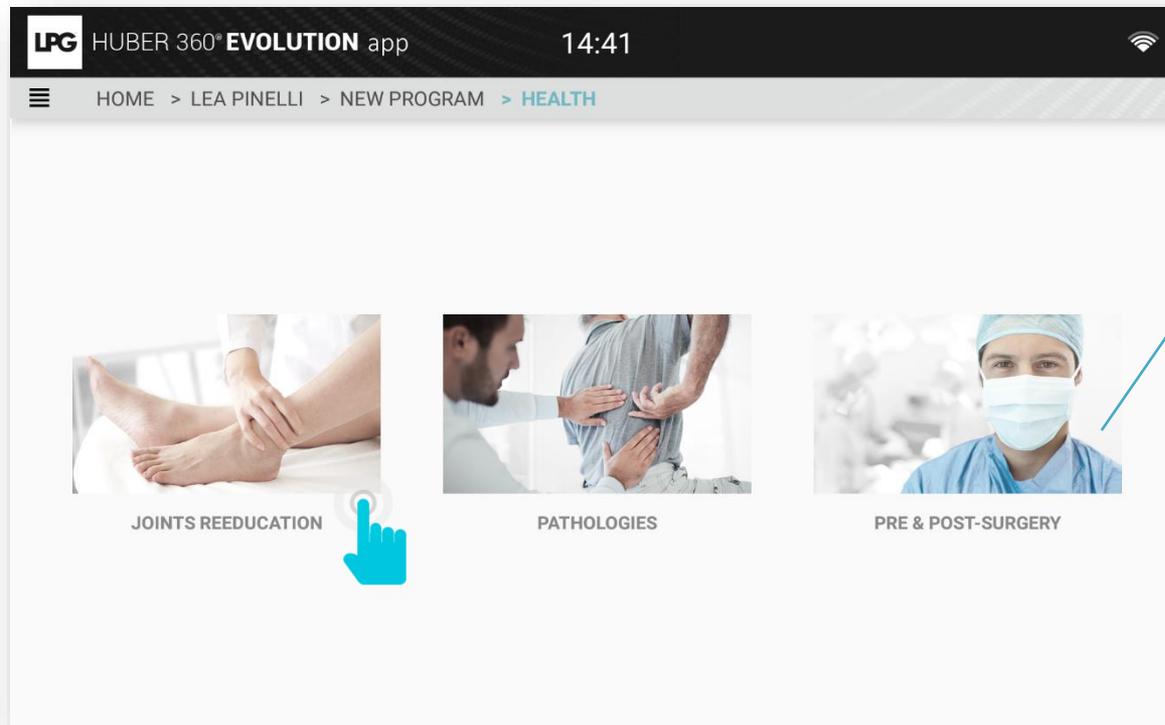


AVAILABLE IN EXPERT
MODE



1.d TREATMENT PROGRAMS

1.d.1 CREATE NEW TREATMENT PROGRAM

After selecting the area, select the level, the number of sessions and click on OK to validate the treatment program creation.


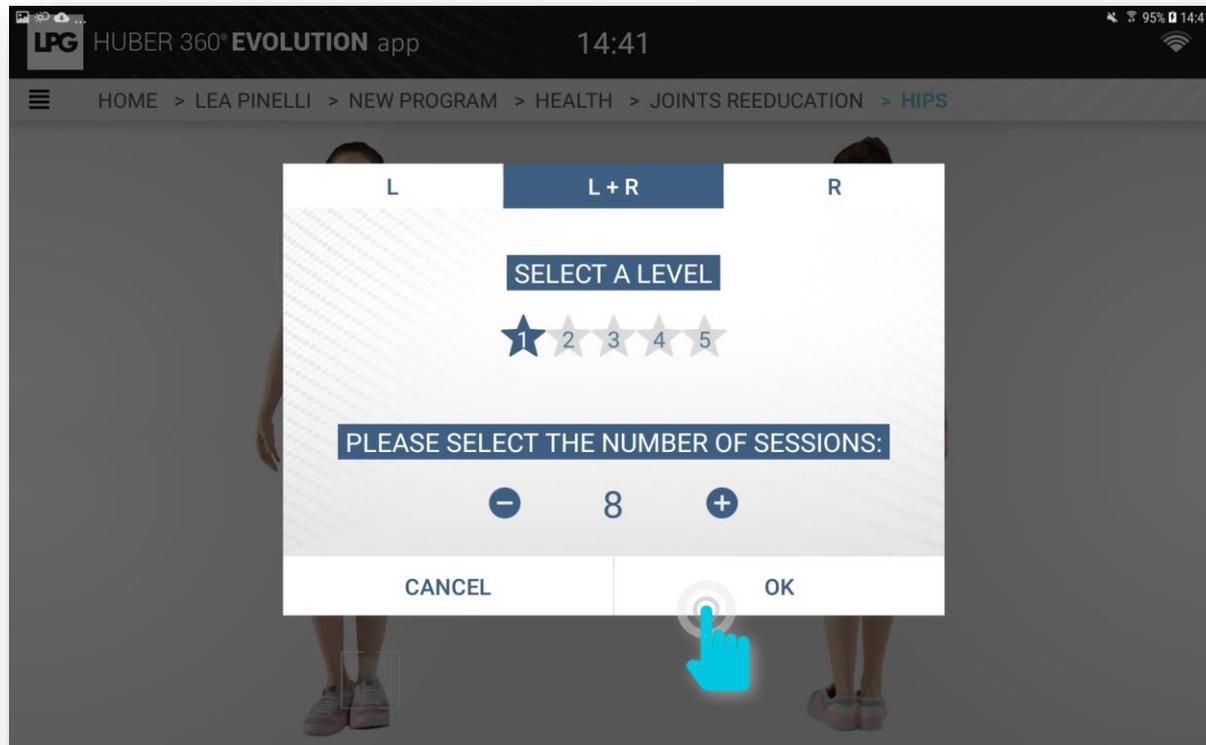




1.d TREATMENT PROGRAMS

1.d.2 SESSION LAUNCH

Click on START, select the device to associate and click on OK.
The session can start!

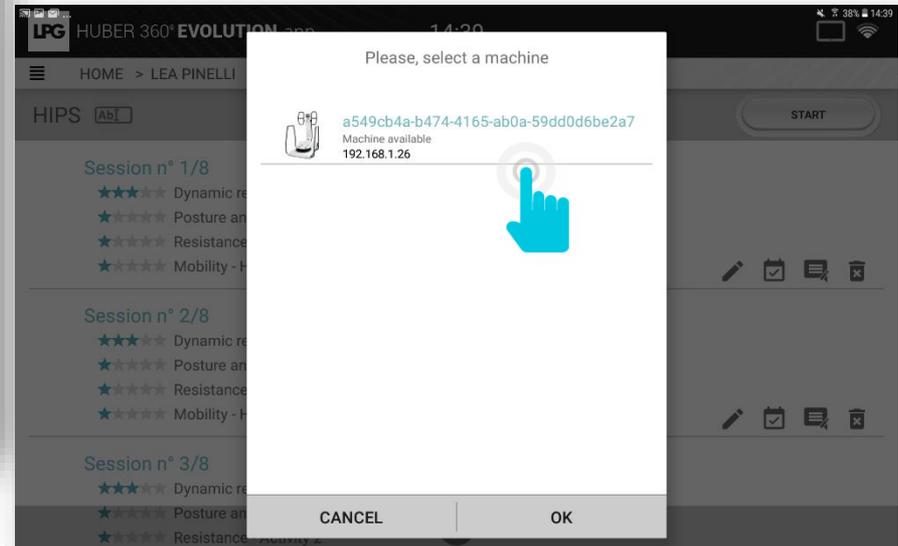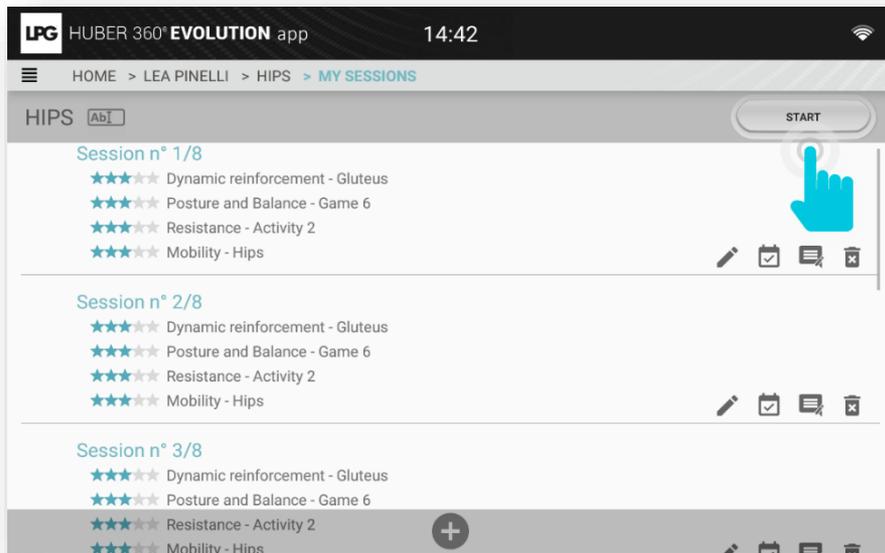



1.d TREATMENT PROGRAMS

1.d.3 VIEWING SESSIONS



: To modify a session.



: To confirm a session in case of connection loss with your HUBER 360[®] EVOLUTION.



: To comment a session (settings used...)




: To delete a session.



: To add a session.



: To modify the treatment program name.

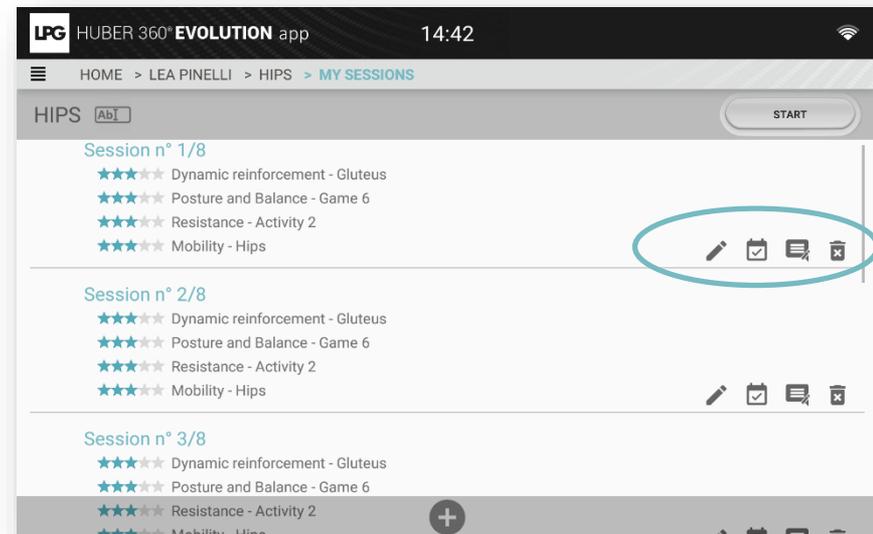



1.d TREATMENT PROGRAMS

1.d.4 MODIFY SESSIONS

You can choose to modify the pre-set exercises, delete them or add new exercises. Then click MODIFY so that your request is taken into account.



: To change the order of exercises.



: To delete an exercise.




: To modify the level.



: To add your customized exercises.

BE CAREFUL !

The modification applies to all sessions of the program.

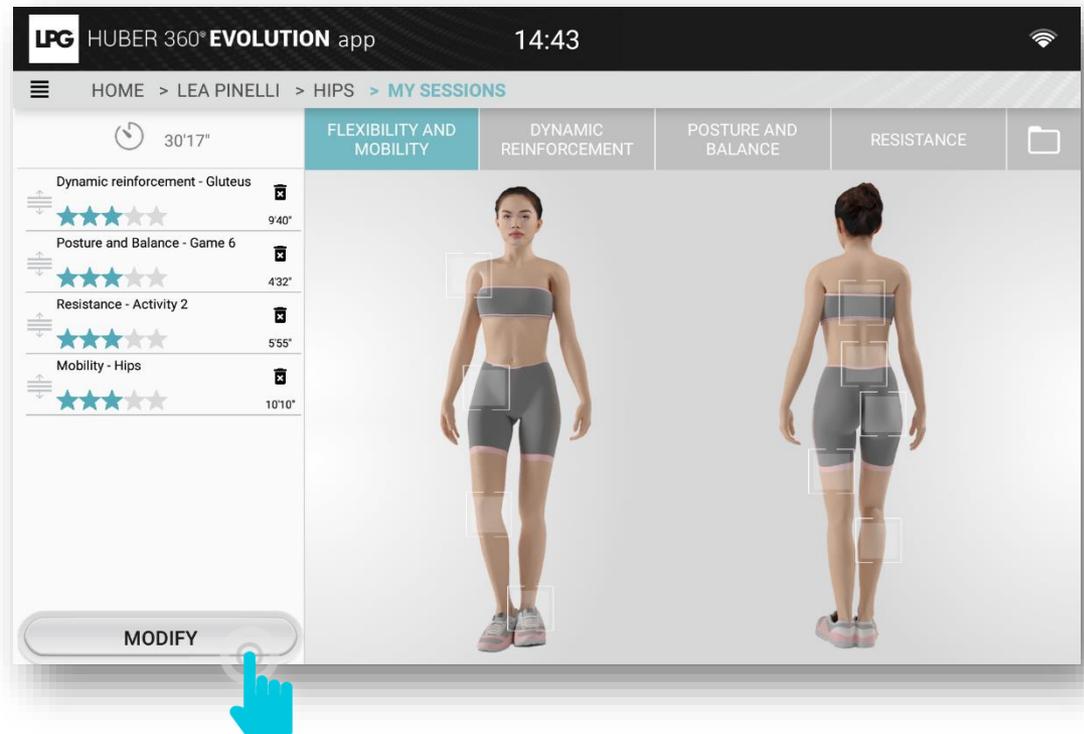



HUBER 360[®] EVOLUTION APP



1.d TREATMENT PROGRAMS

1.d.5 VIEWING PROGRAMS


SESSIONS

To view the completed HUBER 360[®] EVOLUTION sessions (protocol description, date). This will enable a precise treatment program follow-up.

PROGRESSION

To view your patient's progression

List of exercises

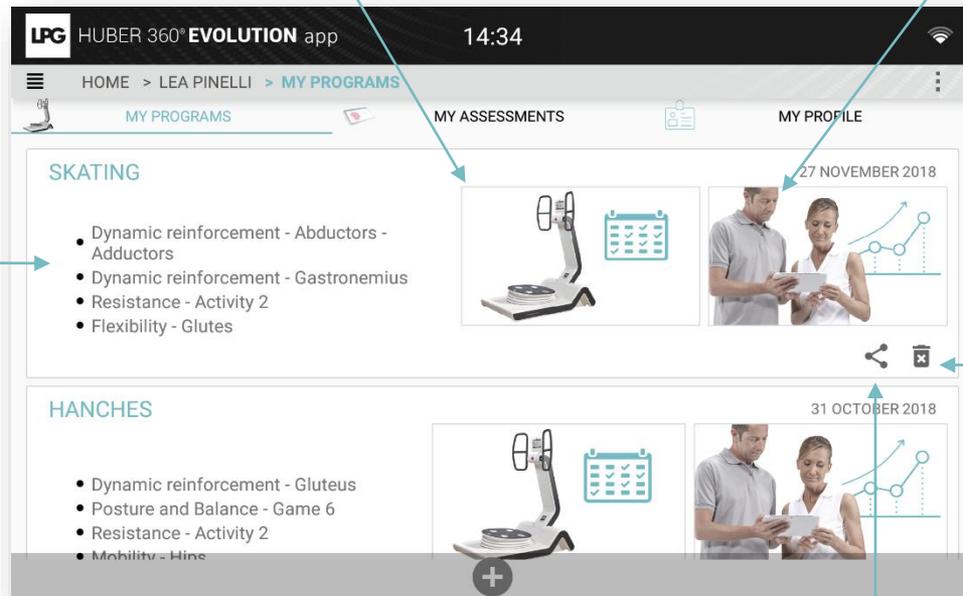

Treatment program deletion

To create new treatment program

To send your patient's treatment program by email or to save it on your tablet.



HUBER 360[®] EVOLUTION APP





1.d TREATMENT PROGRAMS

1.d.6 VIEWING PROGRESSION



You can view the progression of your patient session by session.

You can import this information in the PDF document when you send an email or save it on your tablet.

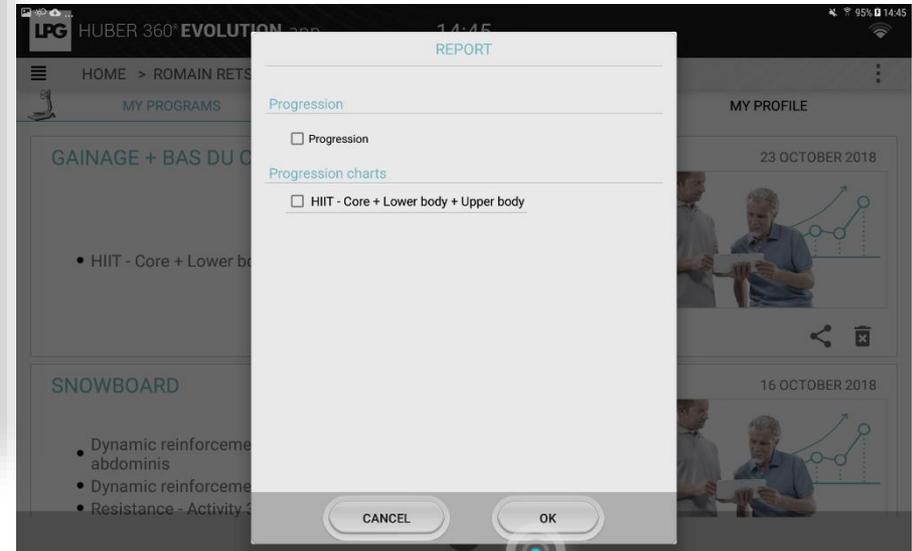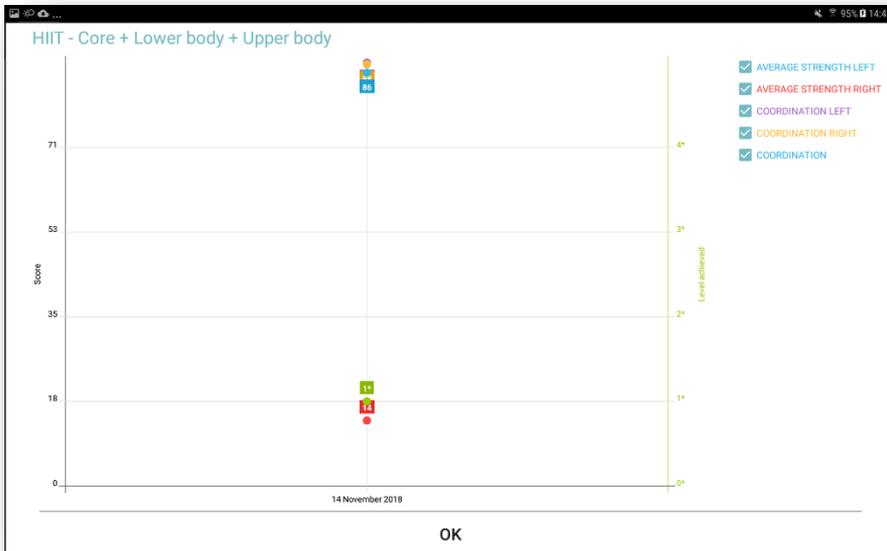



HUBER 360[®] EVOLUTION APP





1.d TREATMENT PROGRAMS

1.d.7 FREE SESSIONS



To access your patient's free sessions (sessions not included in a treatment program) , click on  Next, click on **MY FREE SESSIONS**.

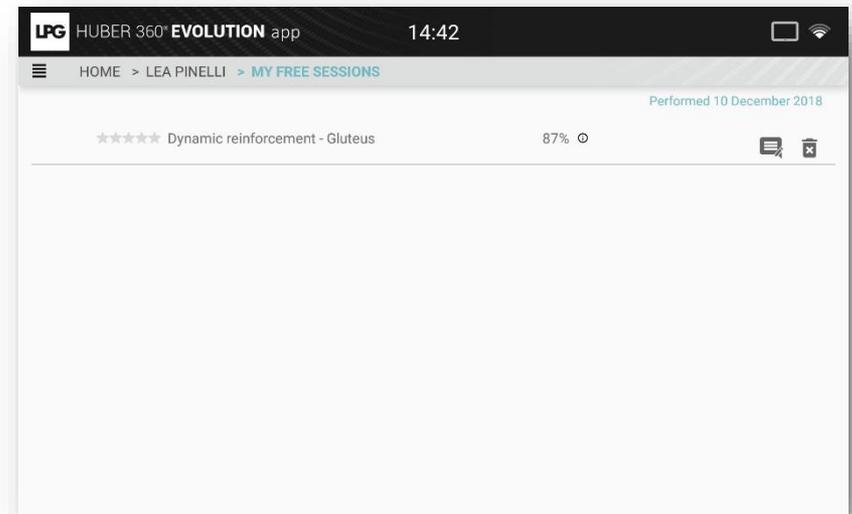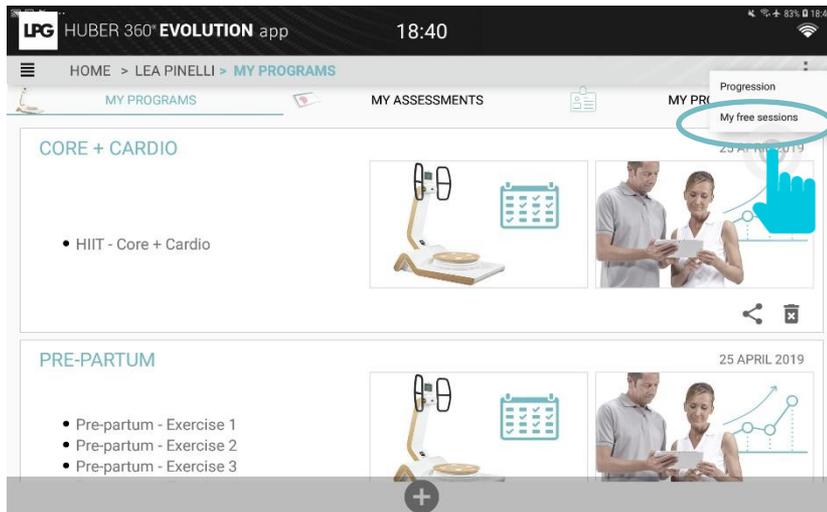




HUBER 360[®] EVOLUTION APP



1.d TREATMENT PROGRAMS

1.d.8 ORPHANS SESSIONS

To assign an orphan session * to a patient, go to the settings of the application then click on **ORPHAN SESSIONS**. Then click on the following pictogram  and select the patient concerned.

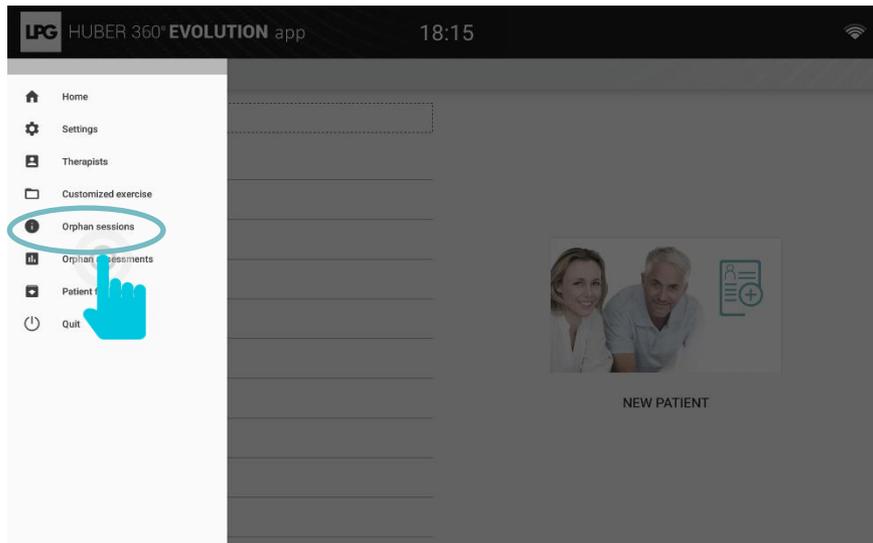

Session Name	Completion Percentage	Date
Flexibility - Lumbar	46%	Performed 14 May 2019
Mobility - Lumbar	100%	Performed 14 May 2019
Mobility - Lumbar	77%	Performed 14 May 2019
Posture and Balance - Game 6	82%	Performed 14 May 2019
Posture and Balance - Game 1	45%	Performed 14 May 2019
Resistance - Activity 1	84%	Performed 14 May 2019


*Session performed directly on the device, without tablet connectivity..



1.e ASSESSMENTS

1.e.1 MAKE AN ASSESSMENT

Click on MY ASSESSMENTS section

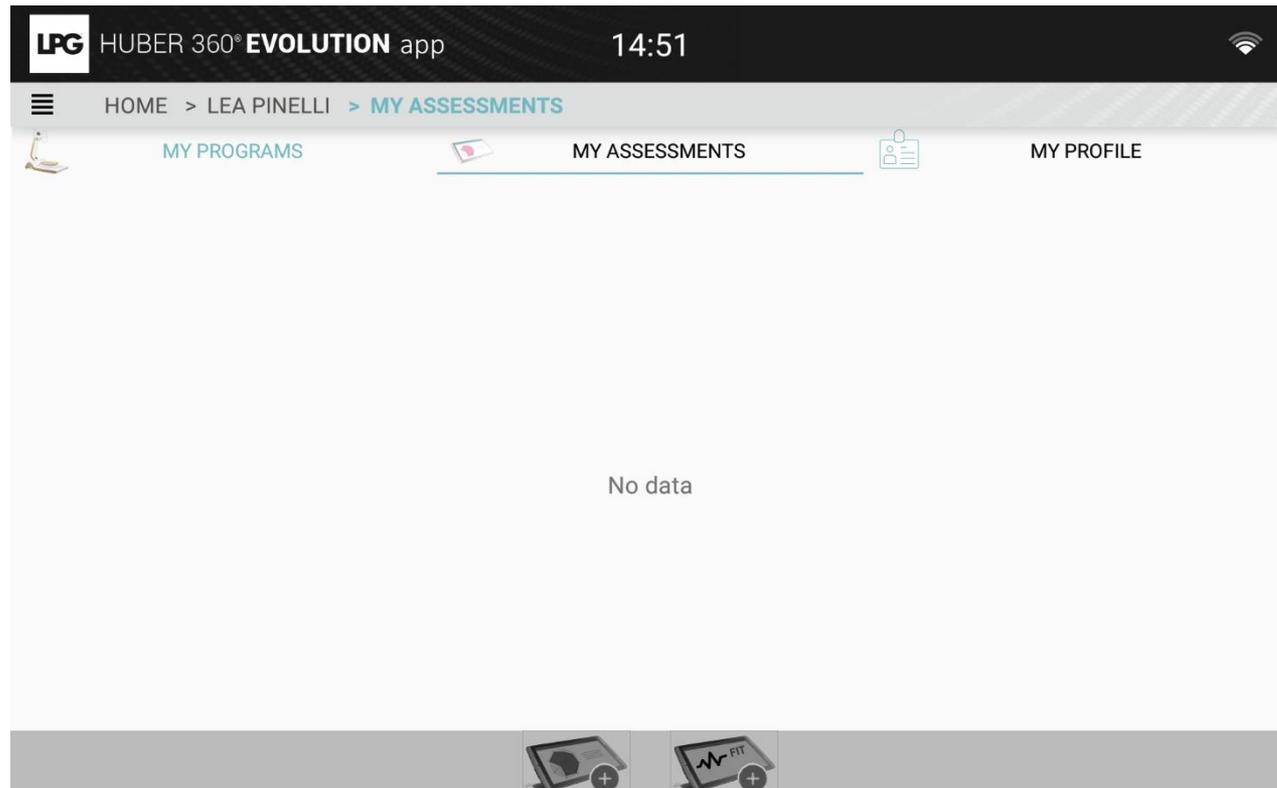




1.e ASSESSMENTS

1.e.1 MAKE A MEDICAL ASSESSMENT

Click on  for Medical Assessment

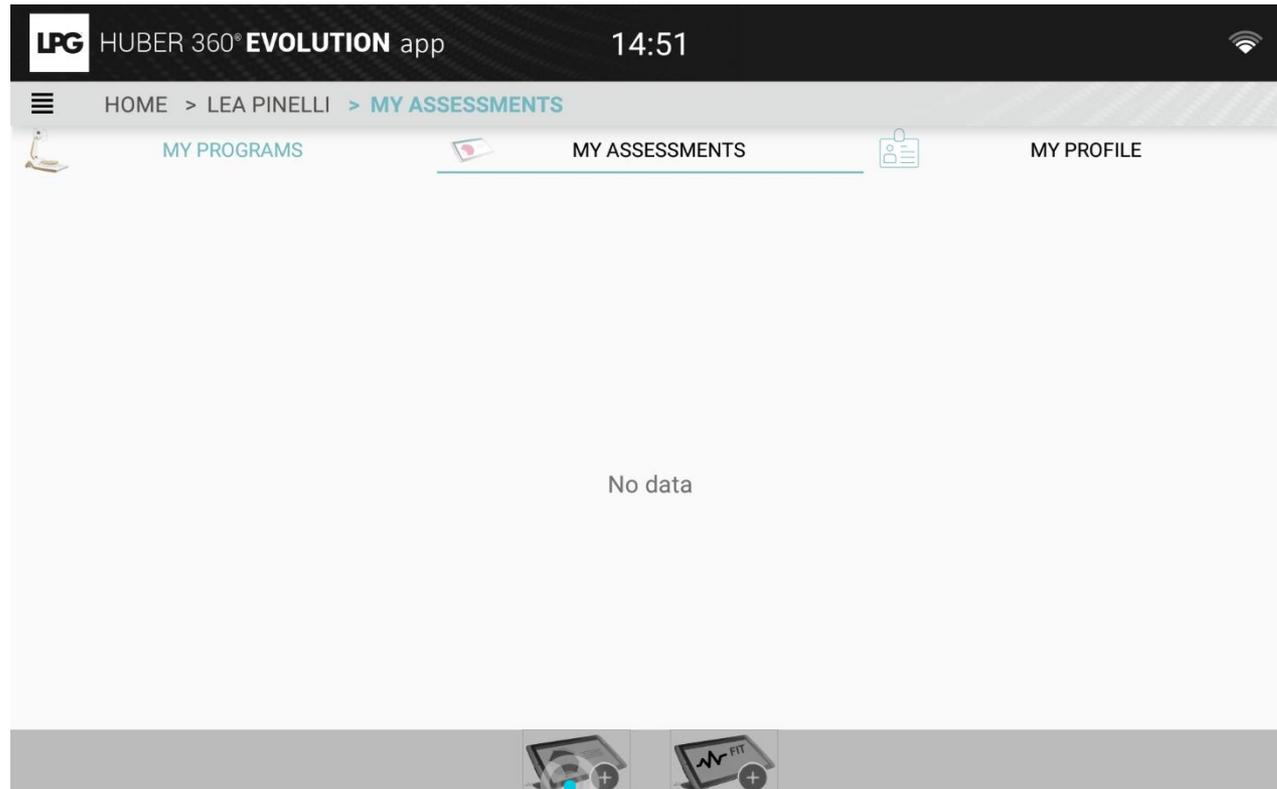



1.e ASSESSMENTS

1.e.1 MAKE A MEDICAL ASSESSMENT

Select the device to associate and click on OK. The assessment can start!

You can choose not to do all tests.

The strenght test is necessary to do the coordination test. You should respect this order.





1.e ASSESSMENTS

1.e.1 MAKE A MEDICAL ASSESSMENT

Stability test

There are 2 steps.

- Step 1 : eyes opened (50 sec)
During this step, a weight measurement is carried out.
Unselect the REPOSITIONING BLOCK PRESENCE button if you remove it during the assessment.
During this step, a heart rate measurement is made.
- Step 2: closed (50 sec)

For each step, launch the test using the PLAY button on the tablet.

Unipodal test

There are 2 steps.

- Step 1: stand on the left leg (automatic stop at 30 sec)
- Step 2: stand on the right leg (automatic stop at 30 sec)

For each step, launch the test using the PLAY button on the tablet.



1.e ASSESSMENTS

1.e.1 MAKE A MEDICAL ASSESSMENT

Walking test

There is one step: Take the position as indicated and walk at a steady pace.

The expected frequency is around 96 steps in 50 seconds

Stability limits test

Stability limits test is composed of 8 steps: for each of the 8 angles (0°;45°;90°;135°;180°;225°;270°;315°), the patient has to push the cursor as far as possible in the direction shown.

Ask the patient to be placed themselves in the center before starting.

For each step, launch the test using the PLAY button on the tablet.

Mobility restriction test

Mobility restriction test is composed of 8 steps.

- The platform tilts at 8 specific angles (0°;45°;90°;135°;180°;225°;270°;315°)
- The patient start to feel pain if there is one
- Stop the platform with the STOP button each time the patient feels pain.
- If there is no pain at 0 to 10° inclination, press on STOP button to launch the new step



1.e ASSESSMENTS

1.e.1 MAKE A MEDICAL ASSESSMENT

Maximum strength test

Maximum strength test is composed of two steps.

- The first step is a PUSH test.
- The second step is a PULL test.

These test values are used for the coordination test.
These two tests must be carried out in this order.

Coordination test

The coordination test is composed of one single step.

It consists in validating each level by remaining 3'' simultaneously withing the LEFT and RIGHT targets.

The test as 30 levels with increasing difficulty.

The time spent to reach the last level is saved.


The test stops if the current level is not reached in 30''.



1.e ASSESSMENTS

1.e.2 MAKE A FITNESS ASSESSMENT

Click on  for Fitness Assessment

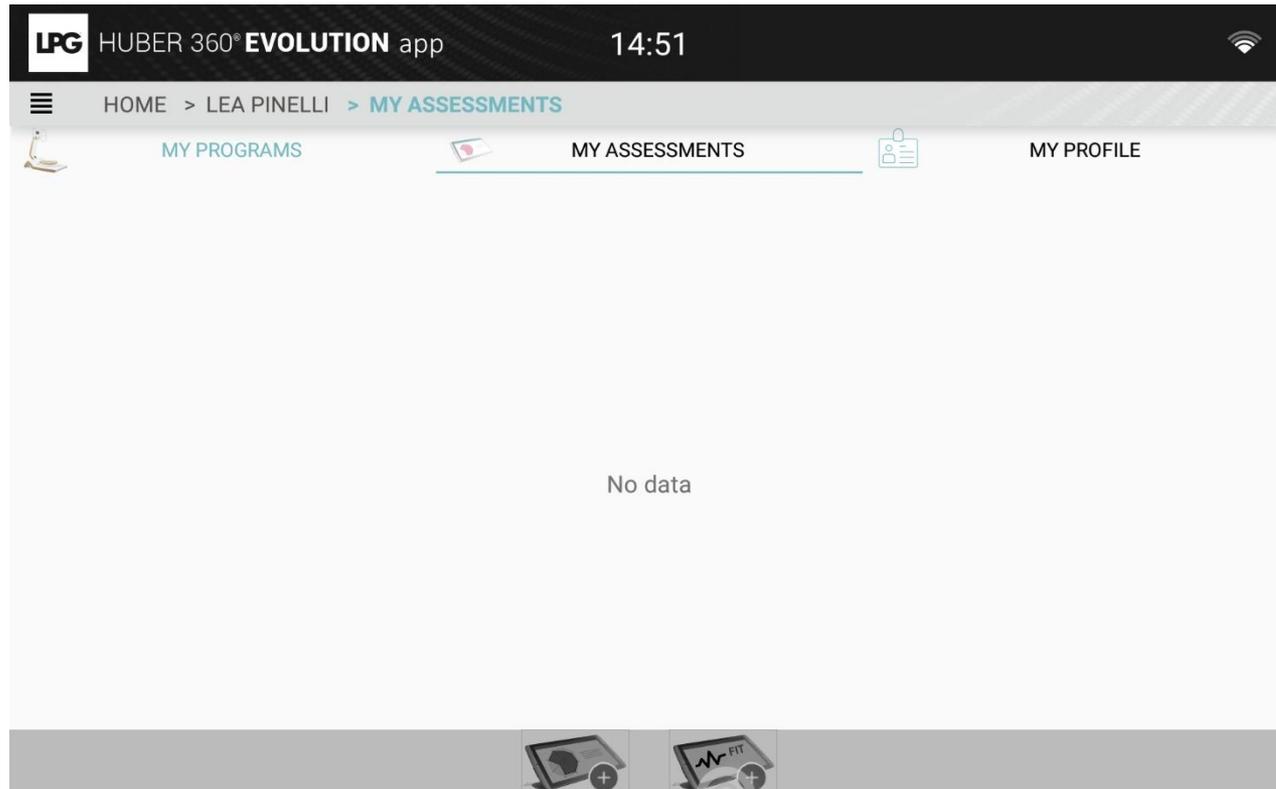




1.e ASSESSMENTS

1.e.2 MAKE A FITNESS ASSESSMENT

Select the device to associate and click on OK. The assessment can start!

You can choose not to do all tests.

The strenght test is necessary to do the coordination test. You should respect this order.

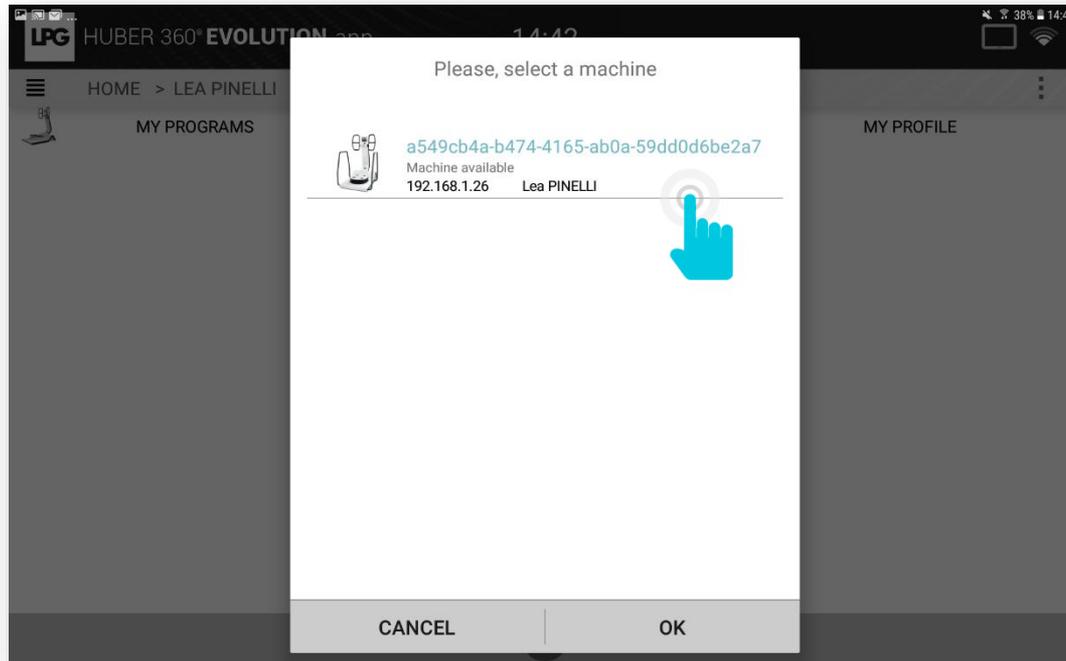



1.e ASSESSMENTS

1.e.3 VIEWING ASSESSMENT

Click on the arrow to view your patient's results for each test.

The screenshot displays the 'MY ASSESSMENTS' screen in the Huber 360 Evolution app. The header shows the time as 12:51 and the user's name as ROMAIN RETSIN. The main content area features a date and time stamp: 18 MARCH 2021 14:41. Below this, there is a row of icons for actions: a CSV file icon, an envelope icon, a person icon, a refresh icon, a person icon with a plus sign, and a trash can icon. A blue hand icon points to a right-pointing arrow icon. Annotations with arrows point to these icons:

- To create an Excel file (points to the CSV icon)
- To create and send a PDF report simplified to the patient (points to the envelope icon)
- To create and send a full pdf report for the prescribers (points to the person icon)
- To replay tests (points to the refresh icon)
- To link this assessment to an existing/other patient (points to the person icon with a plus sign)
- To delete your patient's assessment (points to the trash can icon)

The bottom of the screen shows two icons for additional actions: a plus sign in a square and a plus sign in a circle.



HUBER 360[®] EVOLUTION APP



1.e ASSESSMENTS

1.e.3 VIEWING MEDICAL ASSESSMENT

HUBER 360[®] EVOLUTION app 14:48

HOME > LEA PINELLI > MY ASSESSMENTS

STABILITY UNIPODAL WALKING STABILITY LIMITS MOBILITY RESTRICTION STRENGTH COORDINATION

EYES OPEN	
Center	9.26; -22.56
Length	1024.66 mm
Area	205.46 mm ²
Speed	20.49 mm/s

EYES CLOSED	
Center	17.16; -31.11
Length	
Area	
Speed	

Stability quotient: 0.

HUBER 360[®] EVOLUTION app 14:48

HOME > LEA PINELLI > MY ASSESSMENTS

STABILITY UNIPODAL WALKING STABILITY LIMITS MOBILITY RESTRICTION STRENGTH COORDINATION

Maximum amplitudes (mm)

Direction	Amplitude (mm)
Top	127
Top-Right	167
Right	171
Bottom-Right	137
Bottom	91
Bottom-Left	153
Left	156
Top-Left	139

Area: 57554.25 mm²

Comments





HUBER 360[®] EVOLUTION APP



1.e ASSESSMENTS

1.e.4 SEND AN ASSESSEMENT

Click on the icon below.
Click on SEND.

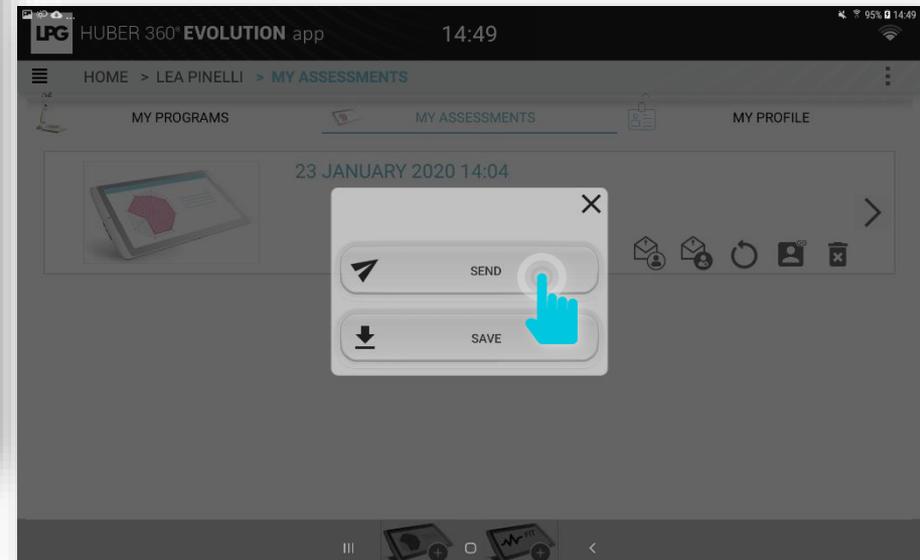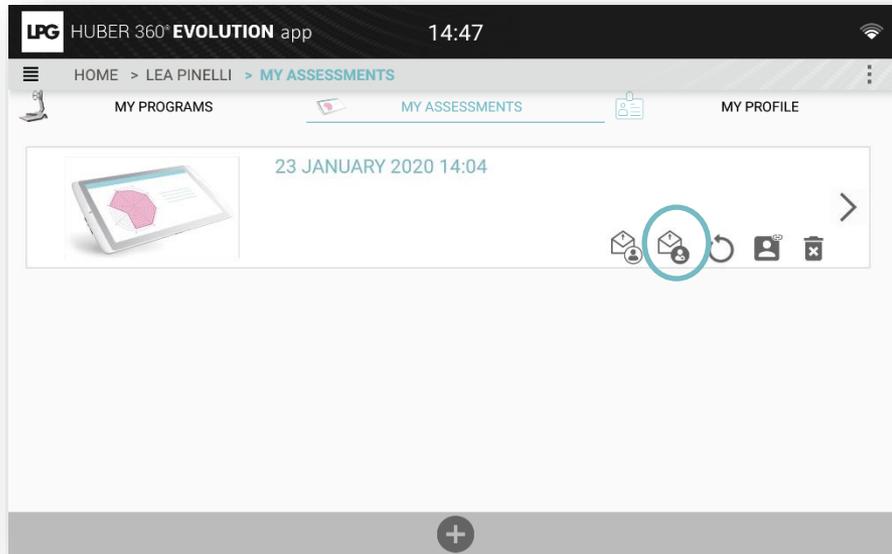




1.e ASSESSMENTS

1.e.4 SEND AN ASSESSMENT

You can customize the message which will be sent.

The program (PDF document) is automatically enclosed with your email. Next, click on SEND.

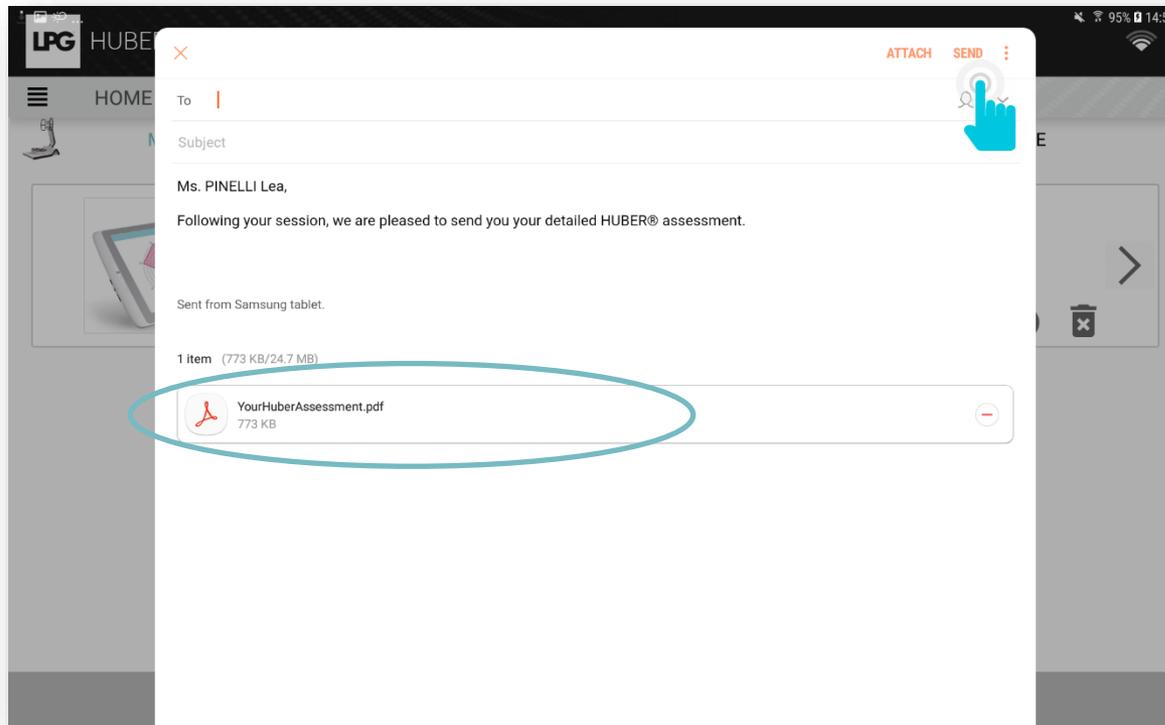




1.e ASSESSMENTS

1.e.4 SEND AN ASSESSMENT

You can customize the message which will be sent.

The program (PDF document) is automatically enclosed with your email. Next, click on SEND.


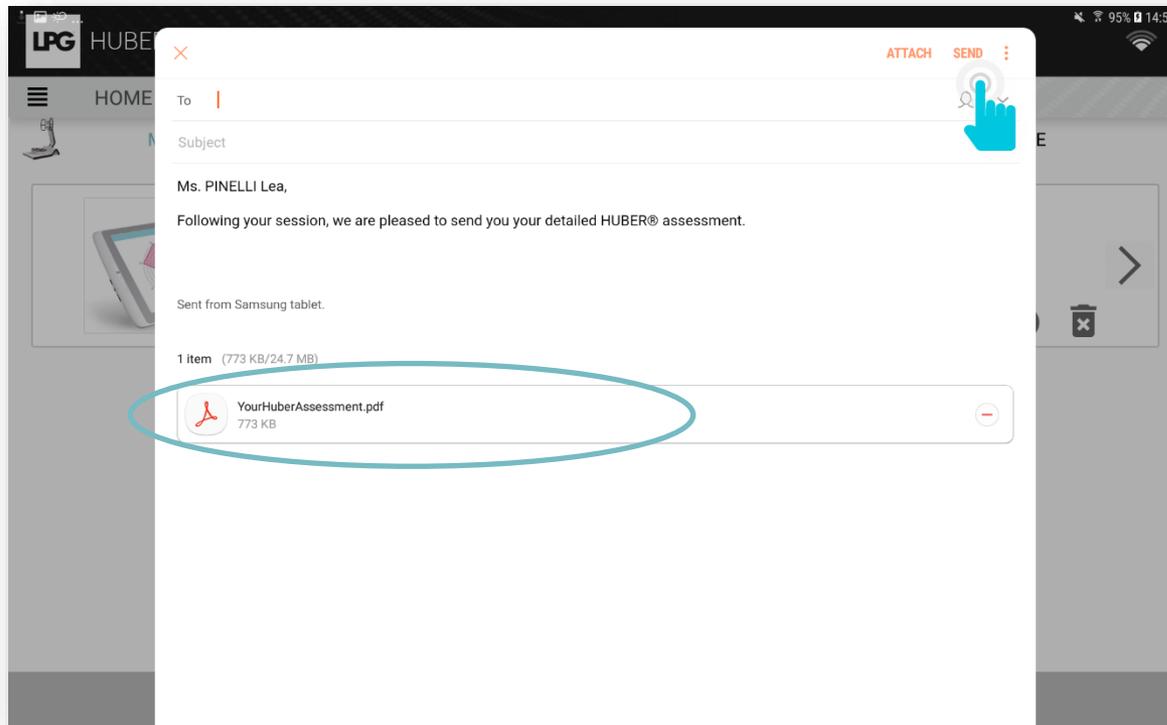




1.e ASSESSMENT MANAGEMENT

1.e.5 ORPHAN ASSESSMENTS

To assign to a patient an assessment made in guest mode *, go to the settings of the application then click on **ORPHAN ASSESSMENTS**. Then click on the following pictogram  and select the patient concerned

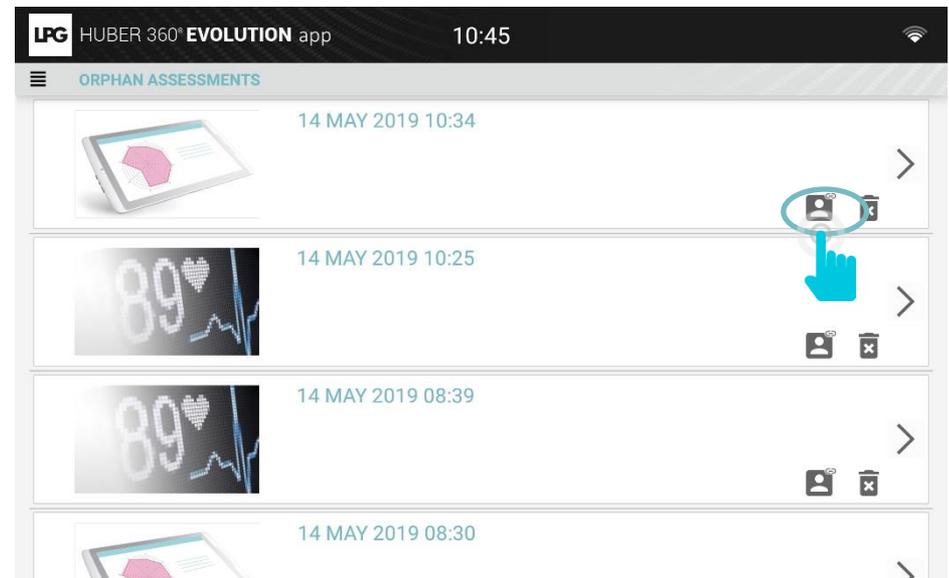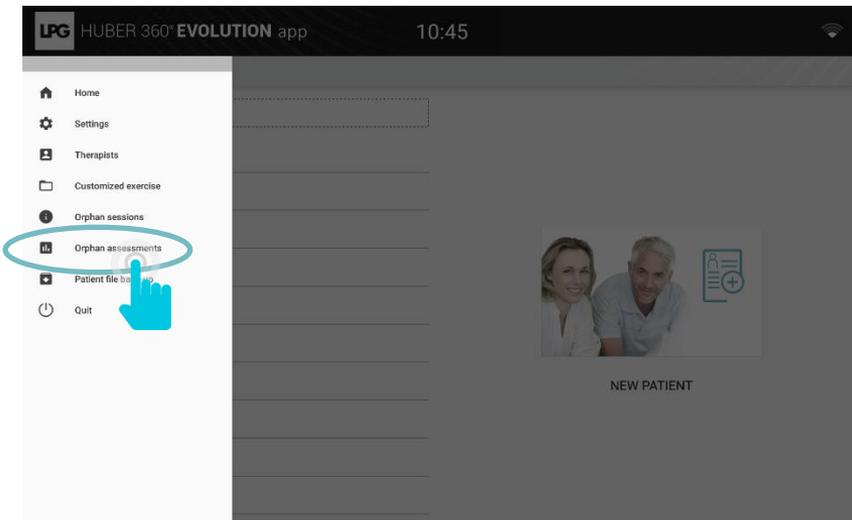

* Assessment done directly on the device, without tablet connectivity.





HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

To create your own exercises, click on  and on CUSTOMIZED EXERCISE.

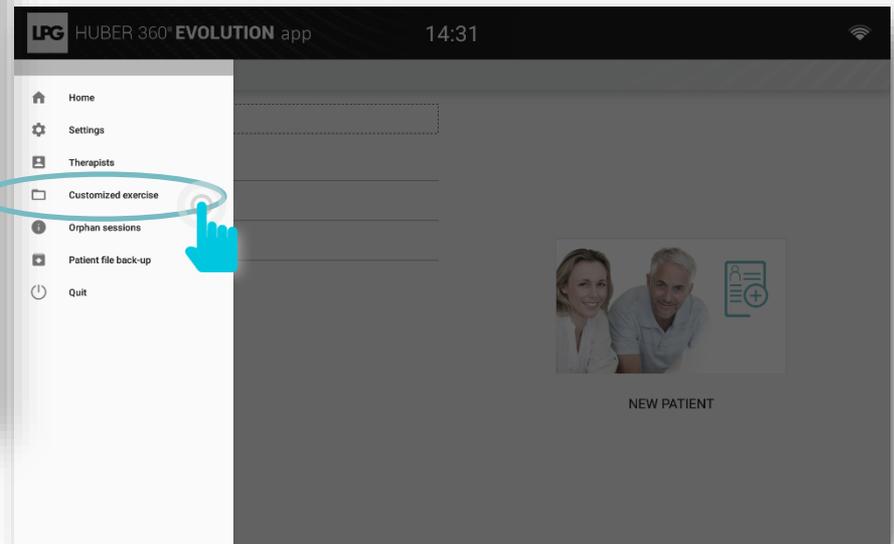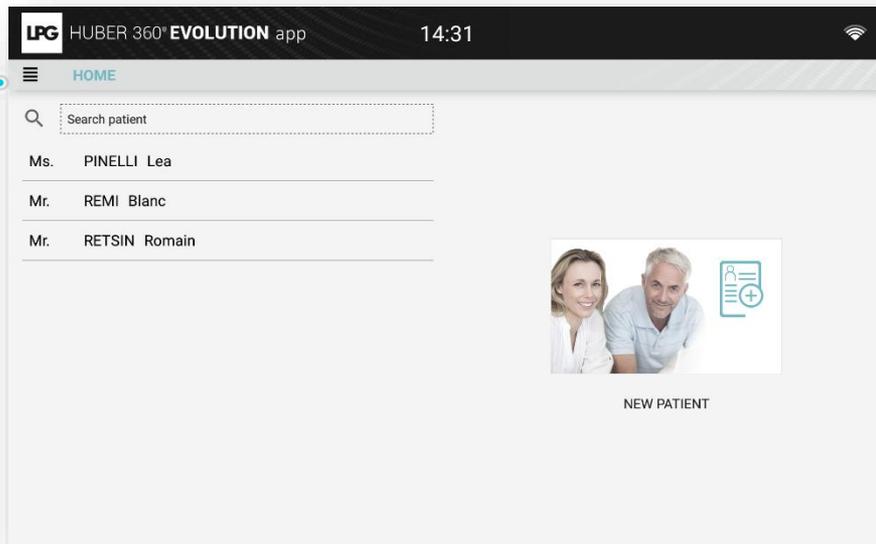




1.f CUSTOMIZED EXERCISES

1.f.1 GENERAL MENU

-  : To create a folder.
-  : To view folders.
-  : To download exercises saved on your device.
-  : To modify the name of exercises.
-  : To delete an exercise.
-  : To modify an exercise.
-  : To add an exercise.

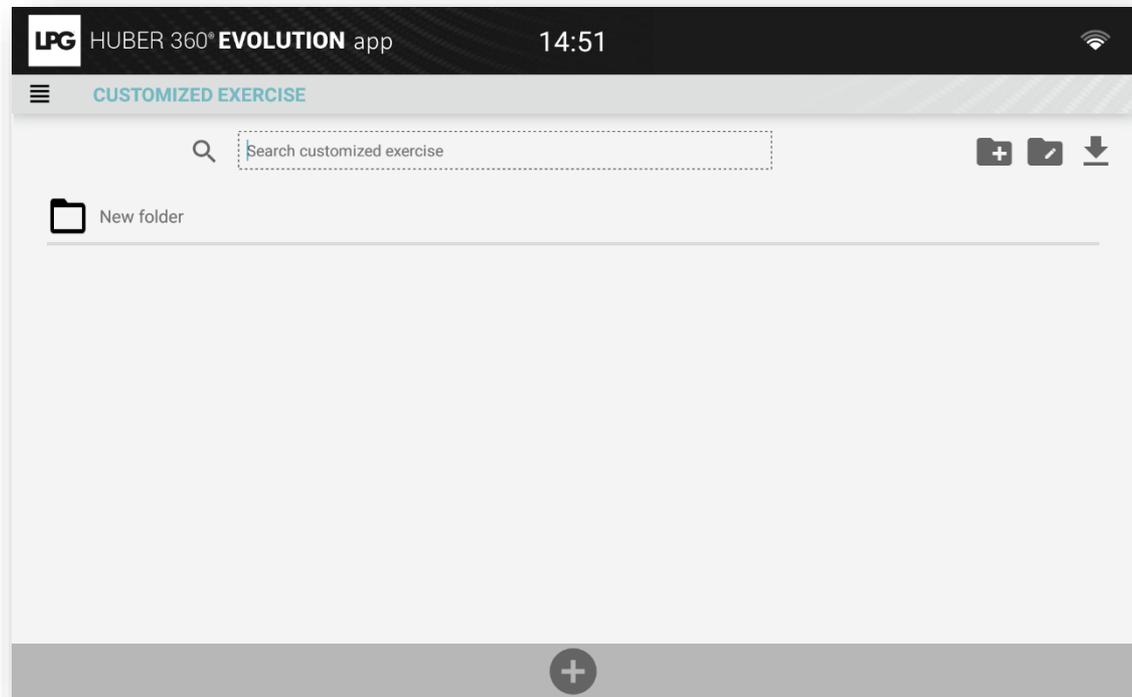




1.f CUSTOMIZED EXERCISES

1.f.2 CREATE A CUSTOMIZED EXERCISE

After clicking on , fill in a name of exercise and associate it with the previously created folder. Click on SAVE. You can start to create your customized exercise.

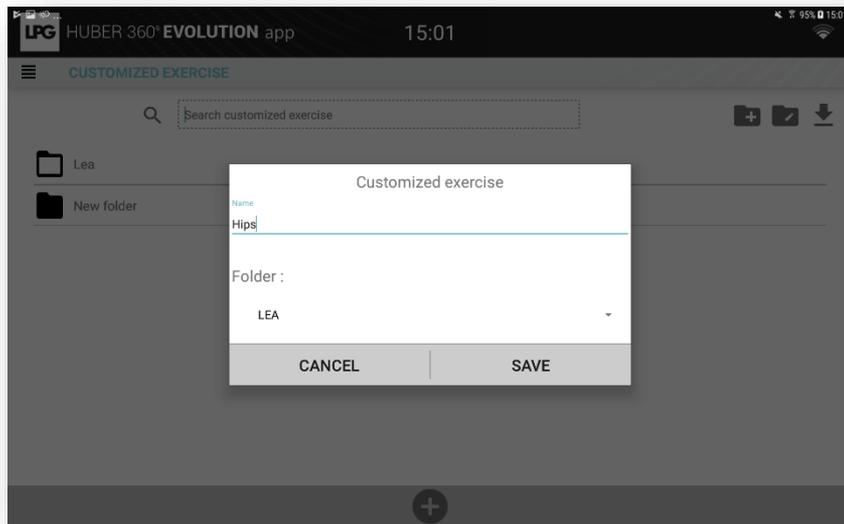



HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.2 CREATE A CUSTOMIZED EXERCISE

Handles target Platform target Platform trajectory Position

Session duration

Step deletion

Addition of exercise

Choice : repeat of position or do a different position

Modification of session duration

Parameter	Value	Unit	Control
Force (Left)	-5	daN	+
Force (Right)	-5	daN	+
%Fmax	100	%	+
Repeats	0	%	+



HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.2 CREATE A CUSTOMIZED EXERCISE

Target platform

The screenshot displays the 'CUSTOMIZED EXERCISE > HIPS' screen in the HUBER 360 EVOLUTION app. The interface includes a top status bar with the LPG logo, app name, and time (15:55). Below the title bar, there are icons for a menu, a timer (10'10"), and a user profile. The main area is divided into two columns. The left column lists 'STEP 1' and 'STEP 2', each with a small illustration of a person performing a hip exercise. The right column contains adjustment controls for the exercise, including a central play button, a distance slider set to 170 mm, and three directional sliders (diagonal, horizontal, and vertical) with values of -80 mm, 10 mm, and 10 mm respectively. At the bottom left, there is a 'SUBMIT' button. On the right side, a large circular diagram shows the layout of the exercise platform with numbered markers (1, 2, 3) and a pink circle highlighting a specific area. The LPG logo and 'HUBER 360 EVOLUTION' text are visible at the bottom of the diagram, along with a '1/8' indicator.



HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.2 CREATE A CUSTOMIZED EXERCISE

Platform trajectory

The screenshot displays the 'CUSTOMIZED EXERCISE > HIPS' screen in the Huber 360 Evolution app. The interface includes a timer set to 10'10", a list of two steps (STEP 1 and STEP 2) with associated icons, and a central control panel with various trajectory icons and numerical values. A large circular grid on the right shows a blue trajectory path with a point labeled 'A'.

Icon	Value
Target	4°
Angle	180°
Horizontal Arrow	4°
Vertical Arrow	4°
Diagonal Arrow	0°
Radial Arrow	20
Curved Arrow	1



HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.2 CREATE A CUSTOMIZED EXERCISE

Position

The screenshot displays the HUBER 360 EVOLUTION app interface. At the top, the LPG logo and app name are visible, along with the time 15:55 and a Wi-Fi signal icon. Below the header, the text "CUSTOMIZED EXERCISE > HIPS" is shown. The main area features a large 3D illustration of a man in a black athletic outfit standing on a circular mat, holding two black frames. To the left of this illustration is a list of steps: "STEP 1" and "STEP 2", each with a small icon and a delete button. Below the steps is a plus sign (+) and a timer icon. At the bottom left, there is a "SUBMIT" button. To the right of the main illustration is a grid of 20 smaller 3D illustrations showing the man performing the exercise from various angles, with blue arrows indicating movement directions. The word "Instruction6" is written above the grid.




1.f CUSTOMIZED EXERCISES

1.f.3 RECOVER A MODIFIED EXERCISE

To recover a modified exercise on your device, click on 

You must first save the protocol via the "floppy disk" button on the screen of your device: 


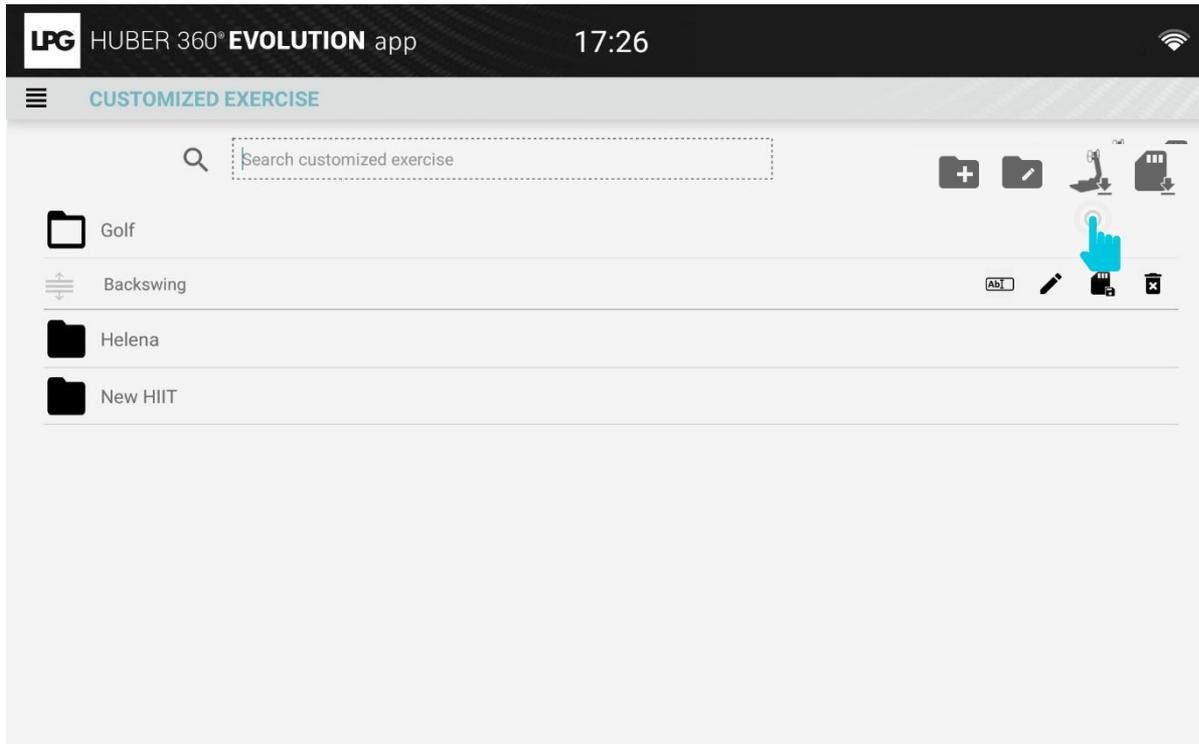




1.f CUSTOMIZED EXERCISES

1.f.3 RECOVER A MODIFIED EXERCISE

Select a device and choose the exercises you want to recover.

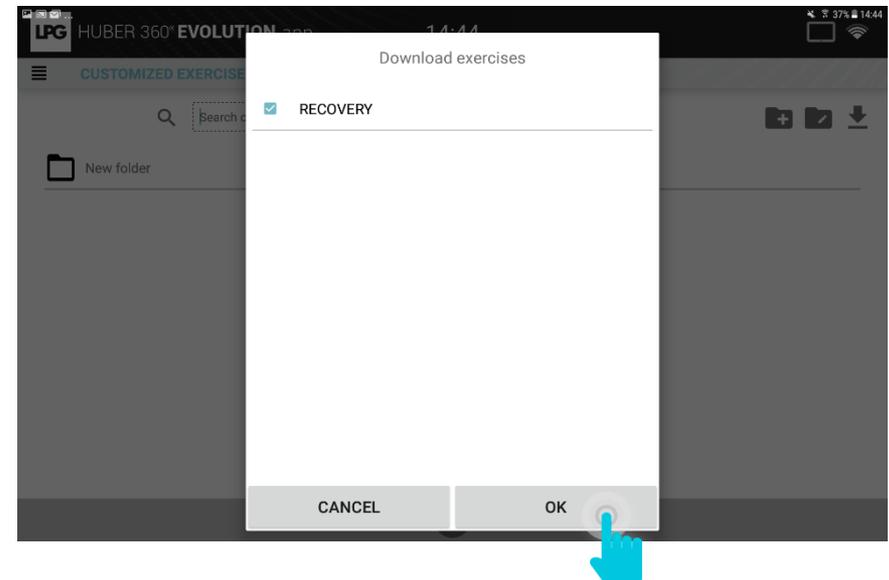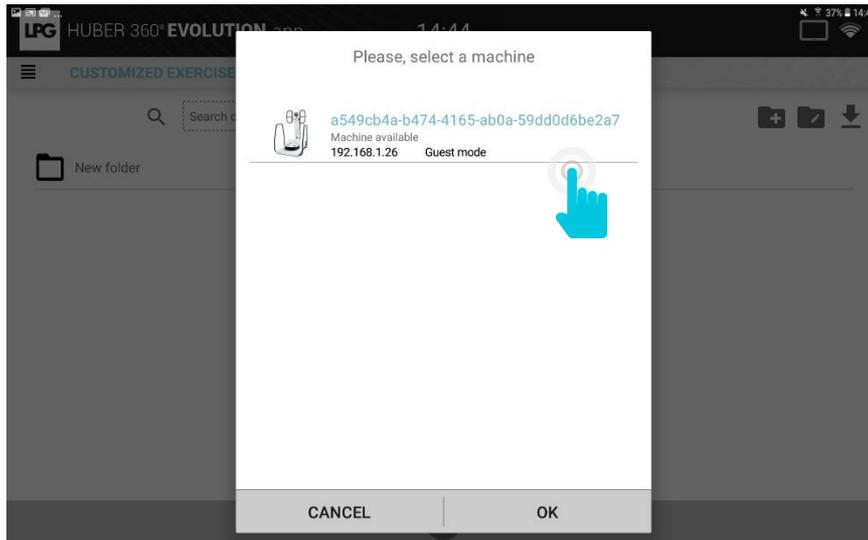




HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.3 RECOVER A MODIFIED EXERCISE

Select the folder to associate by using

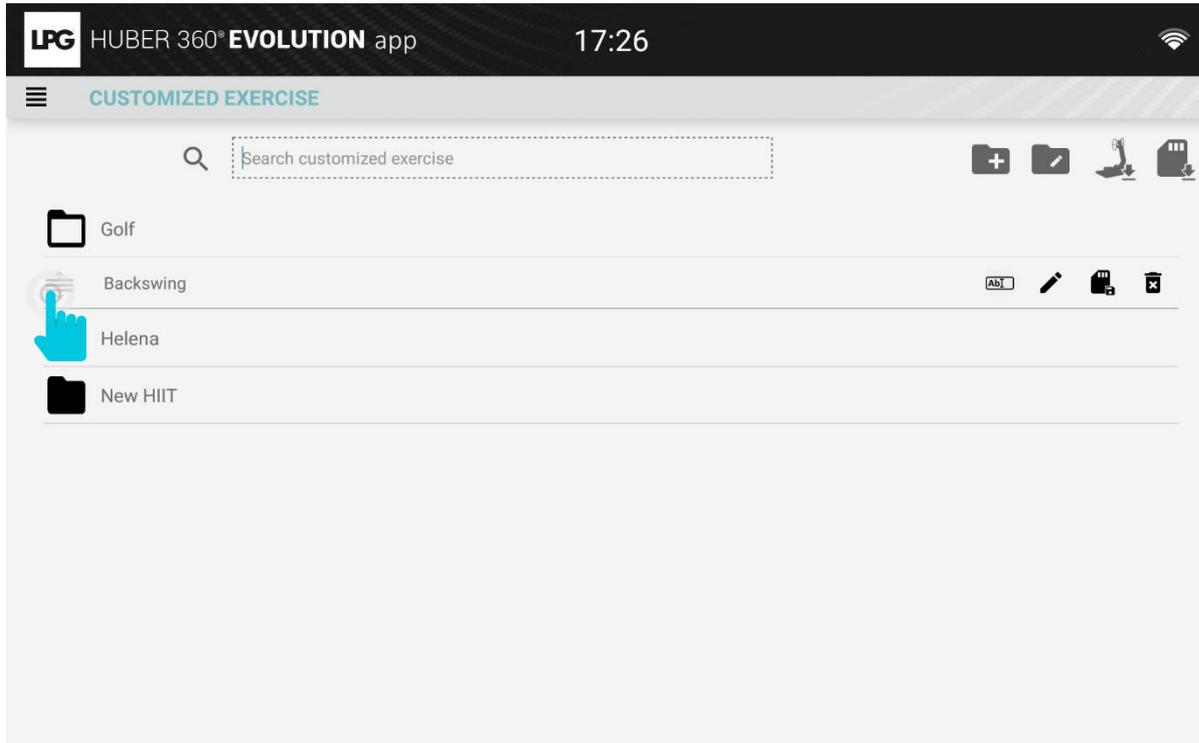



HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.4 EXPORT A CUSTOMIZED EXERCISE

To export a customized exercise on your device, click on






HUBER 360[®] EVOLUTION APP



1.f CUSTOMIZED EXERCISES

1.f.5 IMPORT A CUSTOMIZED EXERCISE

To import a customized exercise on your device, click on

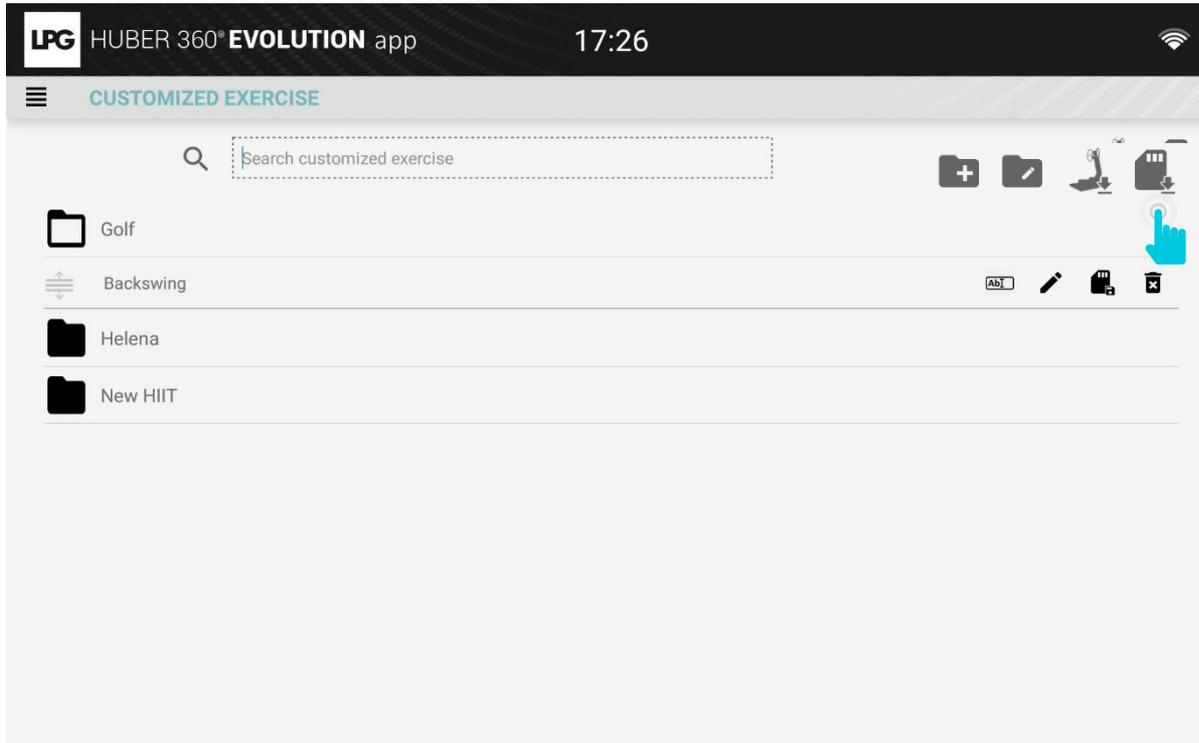



HUBER 360[®] EVOLUTION APP



2. SEND TREATMENT PROGRAMS TO YOUR HUBER 360[®] EVOLUTION



2. TREATMENT PROGRAM LAUNCH

After connecting your tablet to your device, your patient types his username and his password.
The username and the password are chosen during the creation of the patient profile.

The screenshot shows the app's login interface. At the top, a black status bar contains the LPG logo, the text 'HUBER 360[®] EVOLUTION', the time '15:58', and a Wi-Fi icon. Below this is a white navigation bar with a hamburger menu icon on the left and a heart icon with a slash, a timer showing '00:00', and a clock icon on the right. The main content area is a light gray rectangle with two input fields: 'Username' and 'Password'. Below these fields is a large 'OK' button.




HUBER 360[®] EVOLUTION APP



2. TREATMENT PROGRAM LAUNCH

Your patient clicks on MY PROGRAMS.

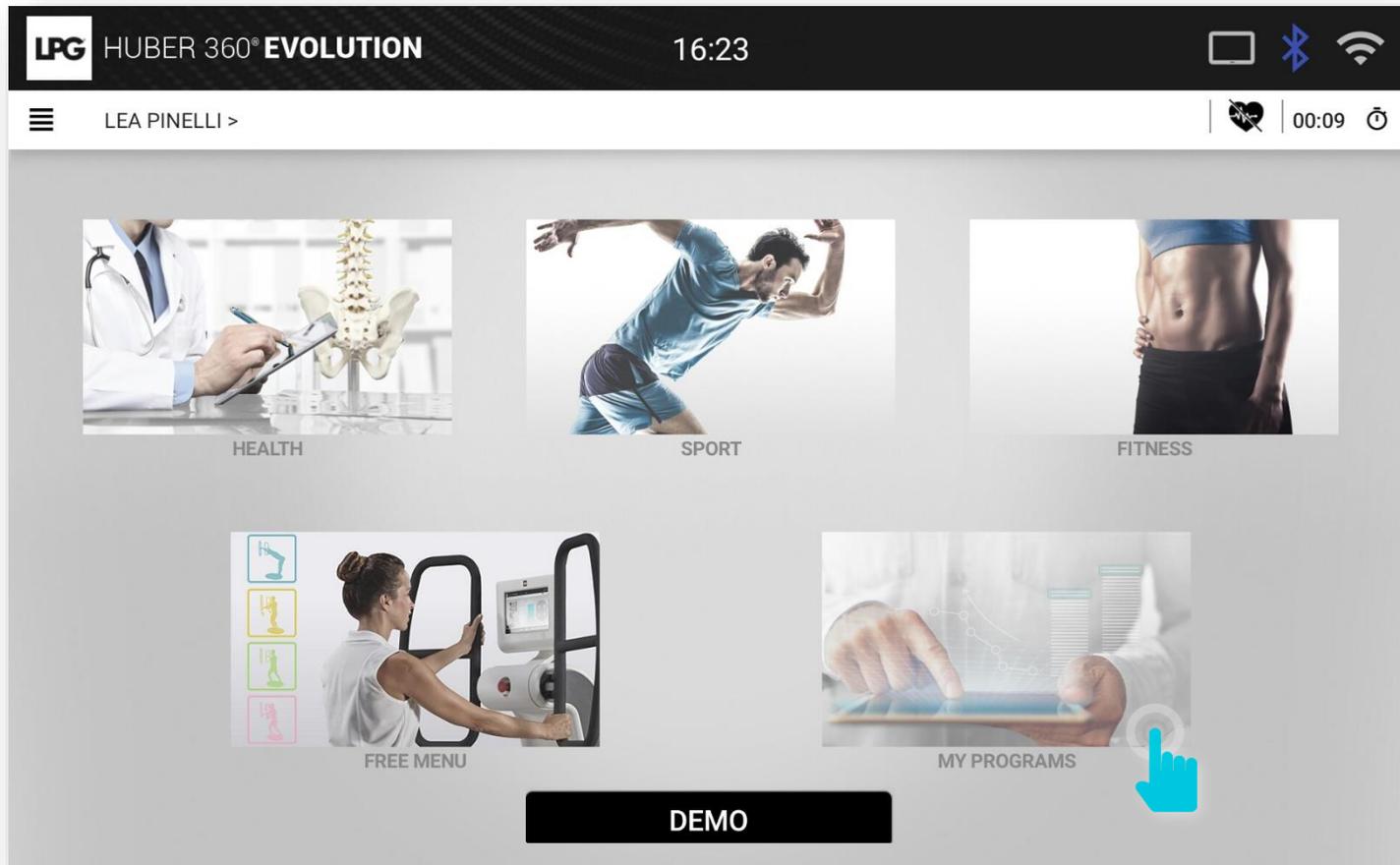




2. TREATMENT PROGRAM LAUNCH

Select the requested program.

In case of connection loss, possibility to update manually the sessions on the machine validated on the tablet () by clicking on 

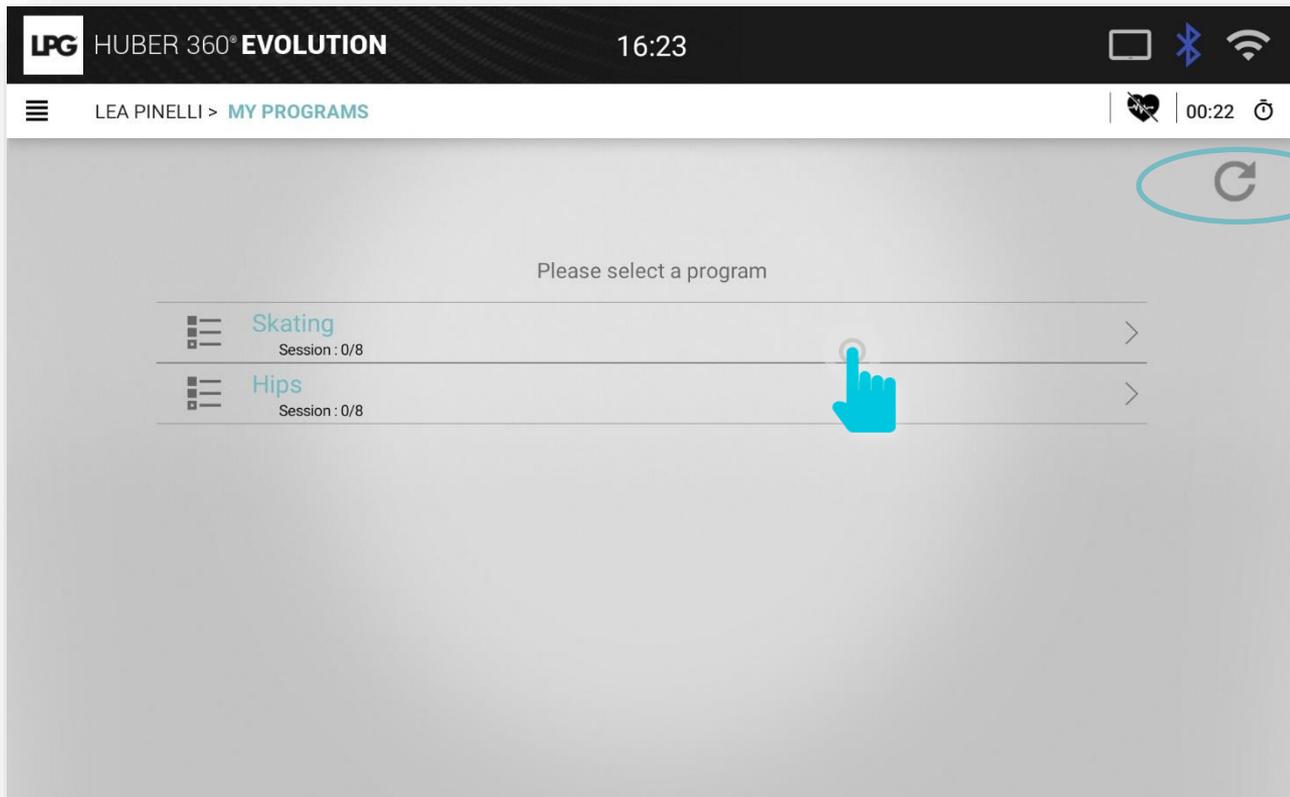




HUBER 360[®] EVOLUTION APP



2. TREATMENT PROGRAM LAUNCH

The session can start!

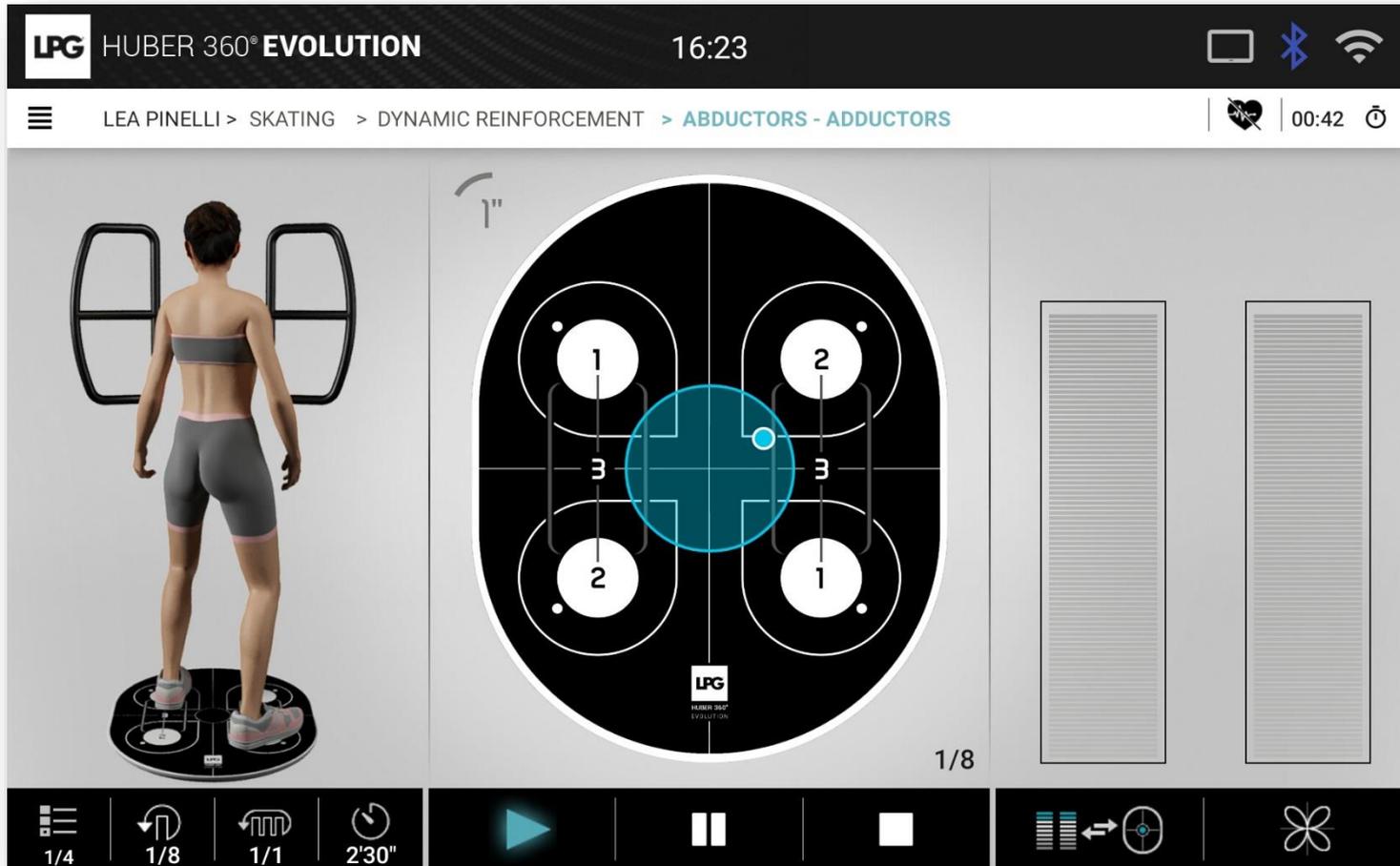



HUBER 360[®] EVOLUTION APP



3. DATA IMPORT (old tablet to the new one)




HUBER 360[®] EVOLUTION APP




3. DATA IMPORT

3.a DATA BACK-UP

1. In the old HUBER 360[®] application, select SETTINGS.

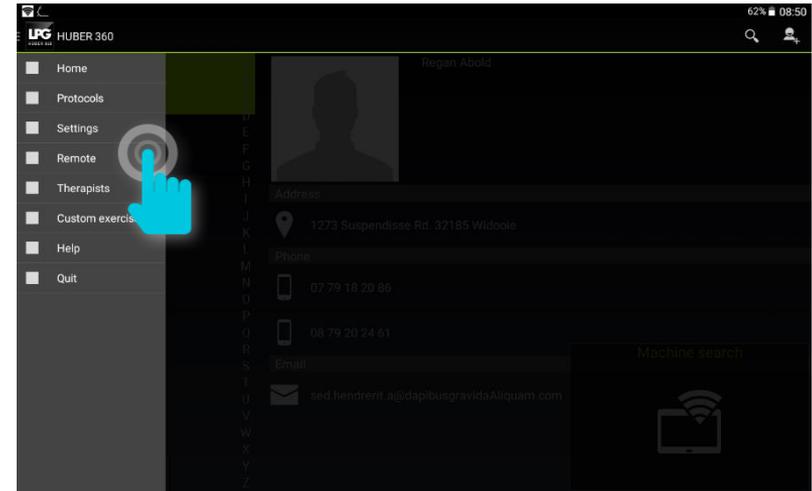

2. Click on DATABASE and EXPORT.

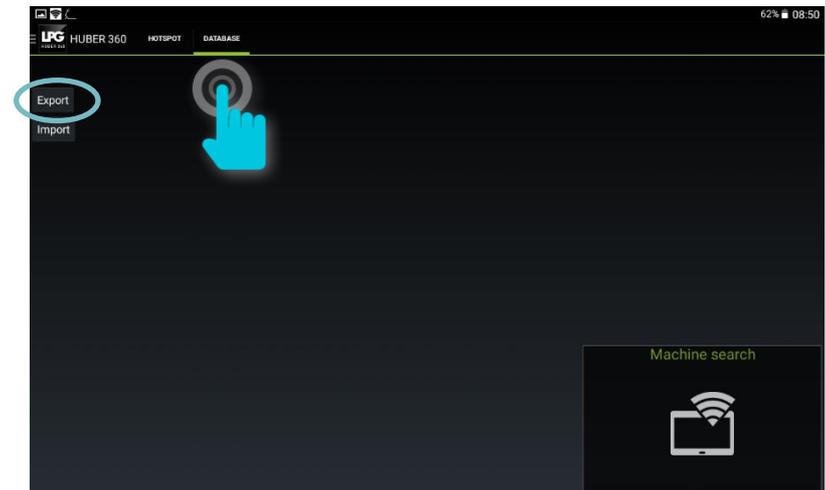




3. DATA IMPORT

3.a DATA BACK-UP

3. Click on DEVICE STORAGE.
4. Select **Huber360.zip**.
5. Click on COPY icon.

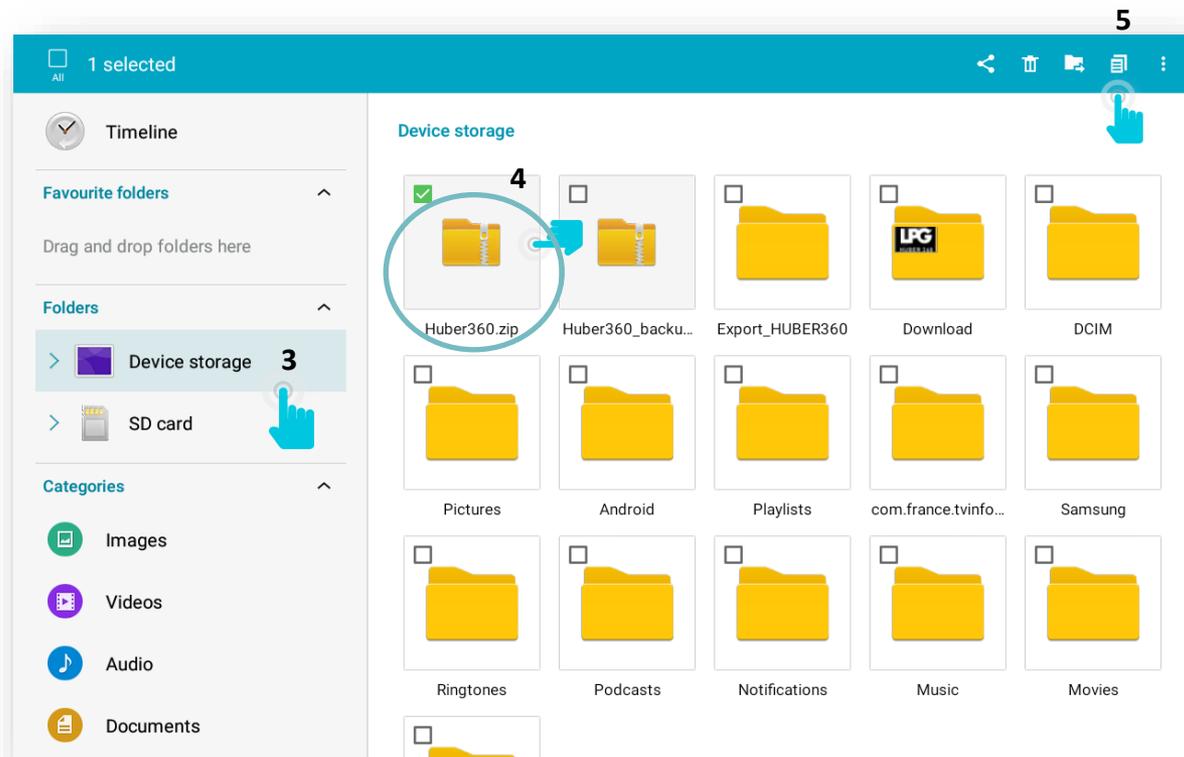



HUBER 360[®] EVOLUTION APP





3. DATA IMPORT

3.a DATA BACK-UP

6. Select SD CARD.

7. Click on PASTE HERE.

You can also copy this folder to your PC, external hard drive...

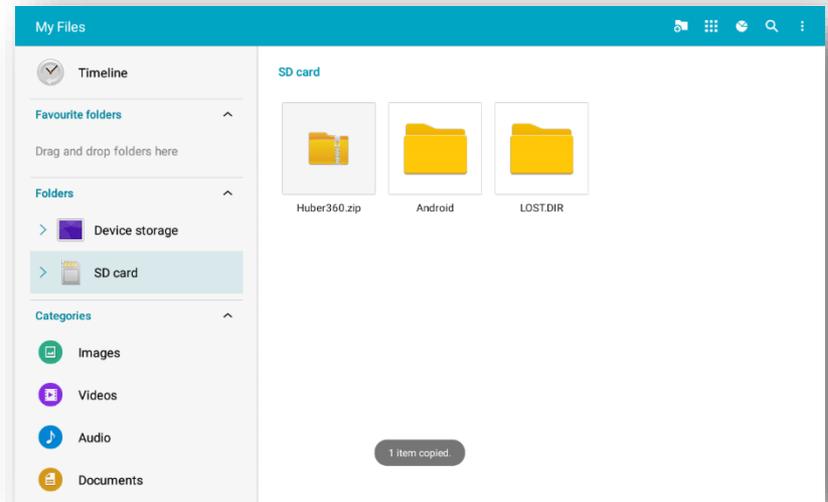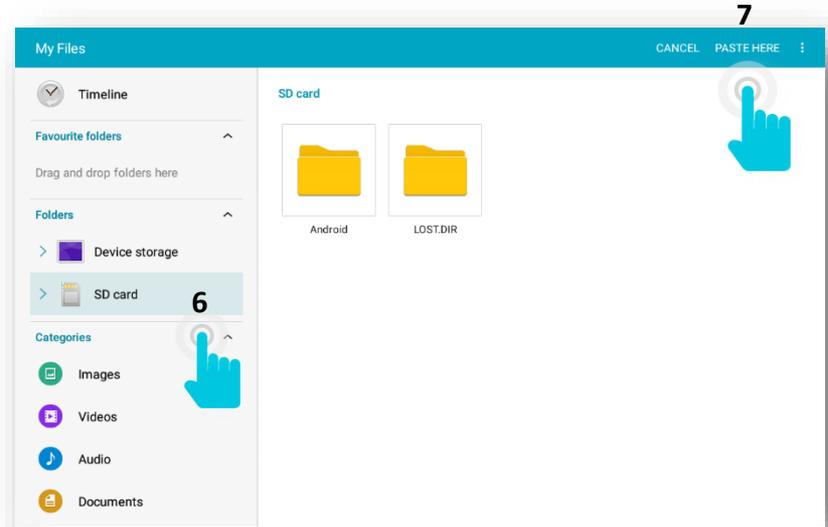



HUBER 360[®] EVOLUTION APP




3. DATA IMPORT

3.b DATA RECOVERY

Before importing, please save your database.

To import your database (old tablet), please insert a micro SD card into the intended location on the tablet.

Then open **HUBER 360 APP** application.

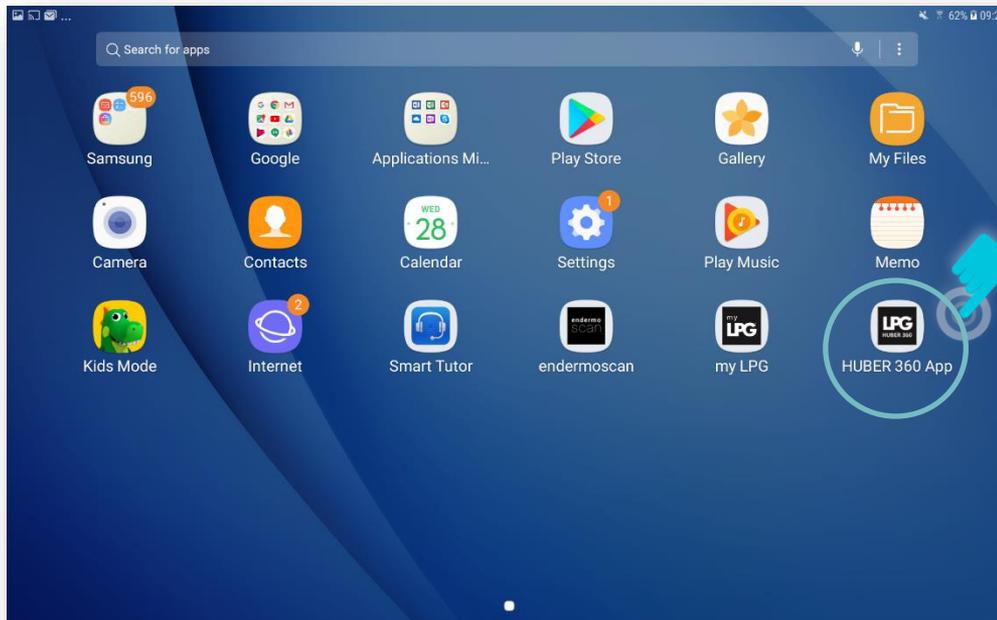

« HUBER 360 App » must be up to date.



HUBER 360[®] EVOLUTION APP





3. DATA IMPORT

5.b DATA RECOVERY

Once the HUBER 360[®] APP is opened, click on icon  at the top left on the screen.

Next, click on **SETTINGS**.

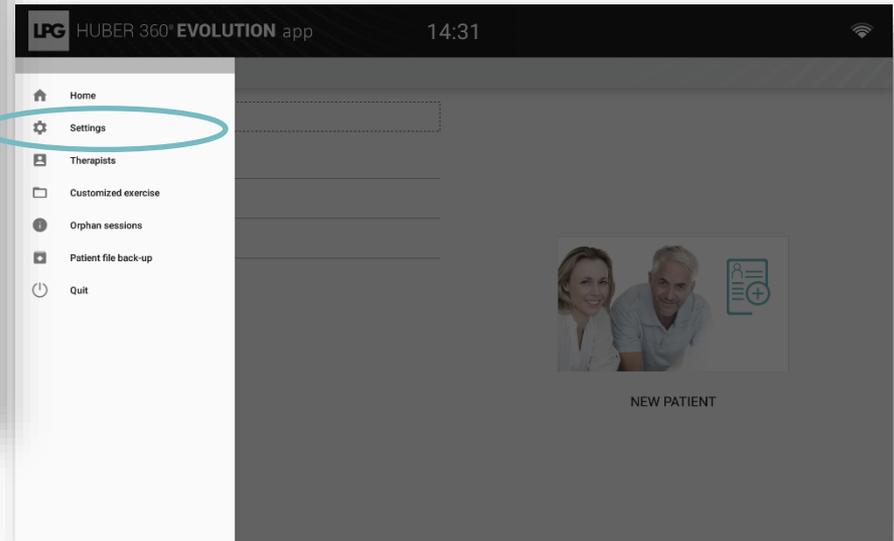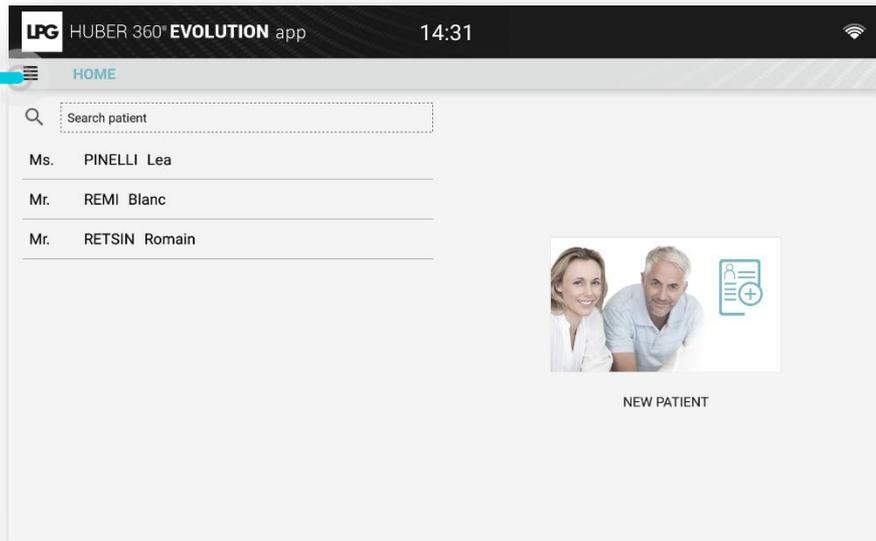




3. DATA IMPORT

3.b DATA RECOVERY

Click on DATABASE.

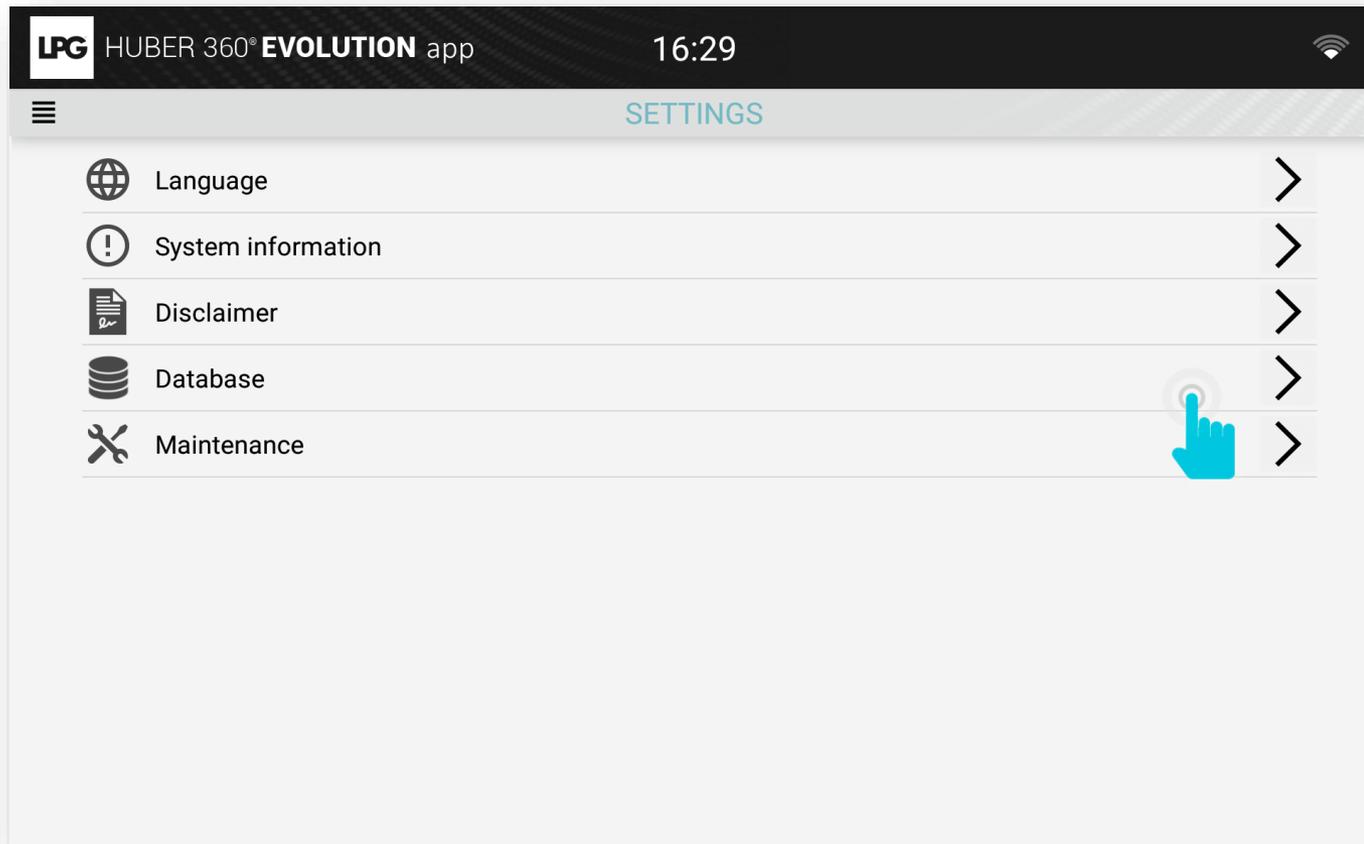




3. DATA IMPORT

3.b DATA RECOVERY

Click on the visual below.

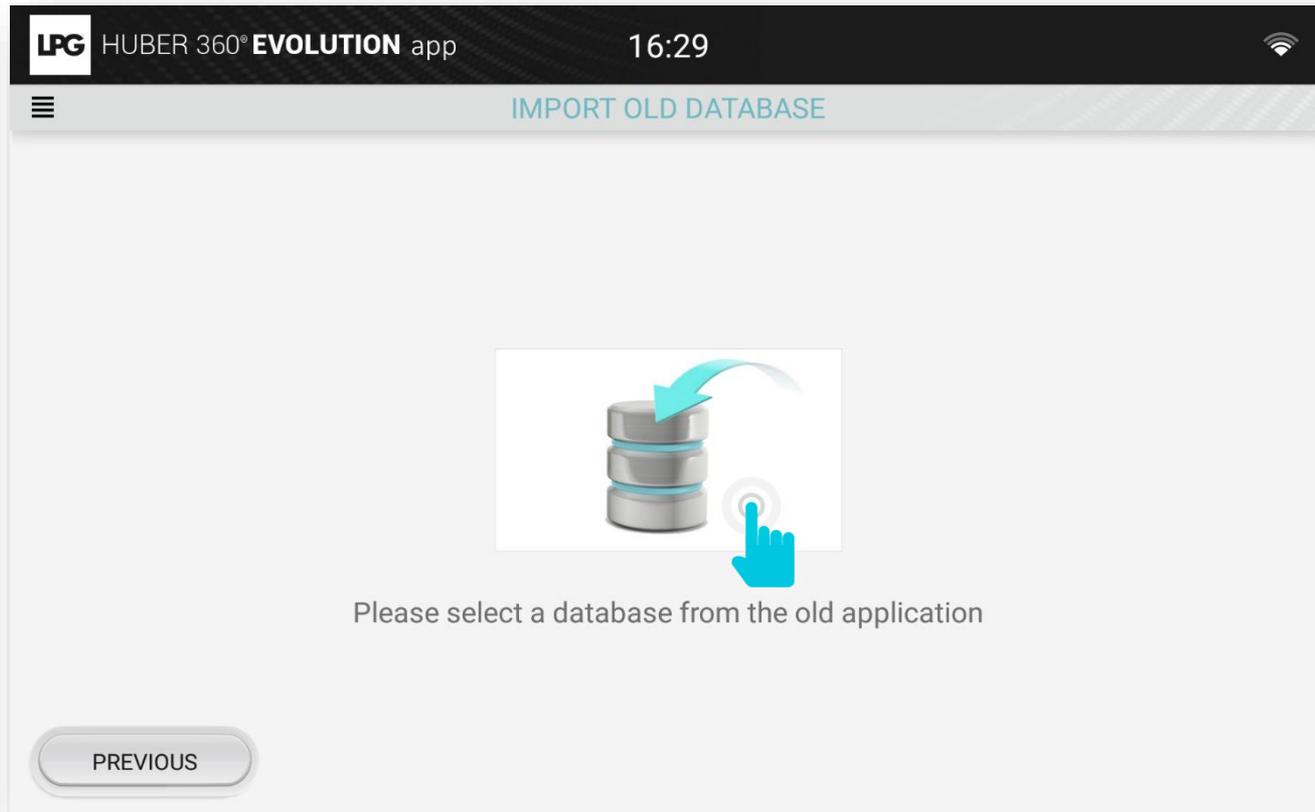




3. DATA IMPORT

3.b DATA RECOVERY

Select the container of your data back-up (SD card or USB key, or other ...)

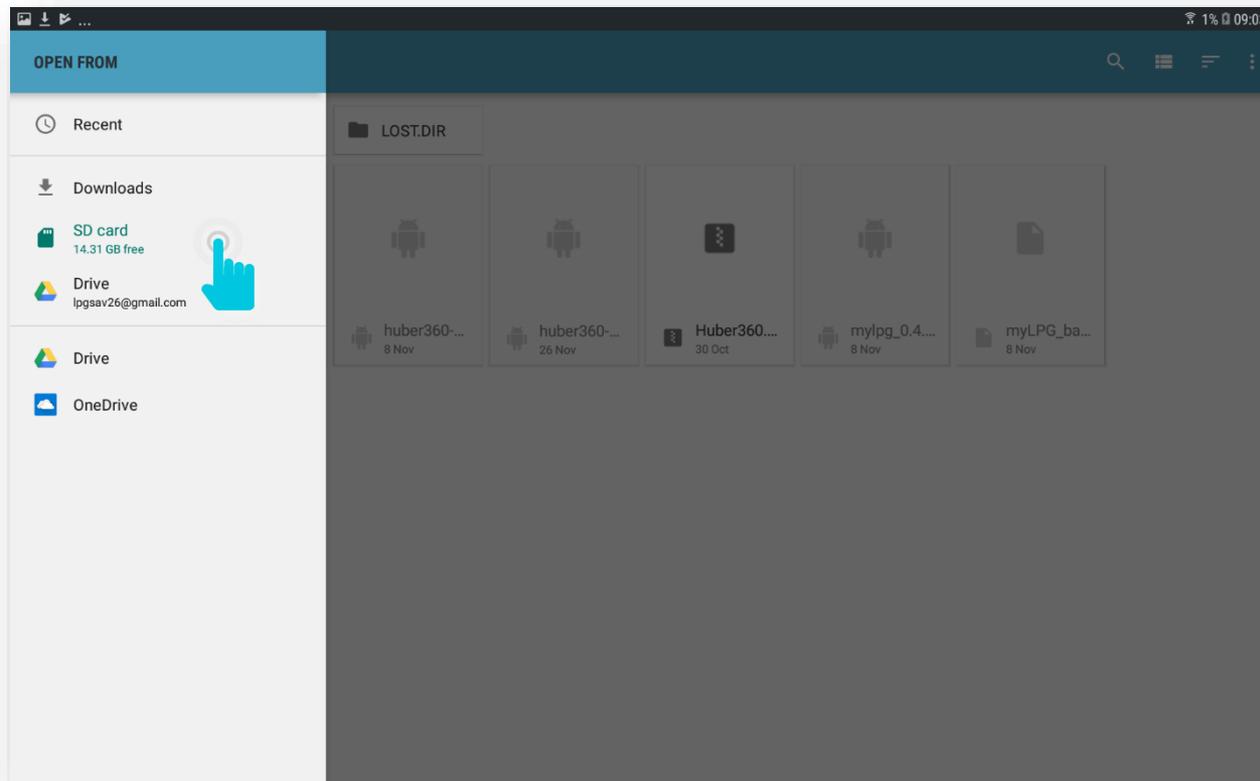





HUBER 360[®] EVOLUTION APP



3. DATA IMPORT

3.b DATA RECOVERY

Select the file corresponding to your backup « **Huber360.....zip** » with a long tap.
Click on OPEN.

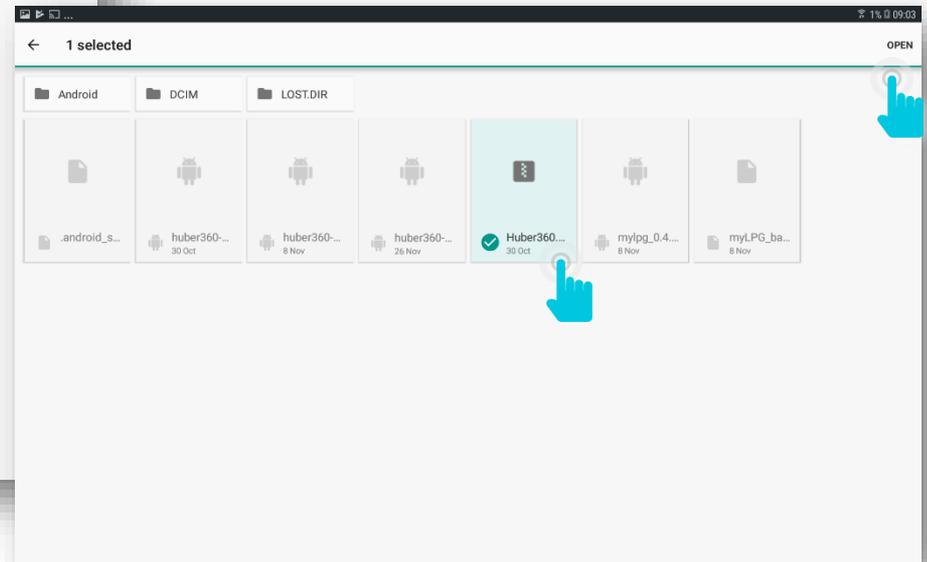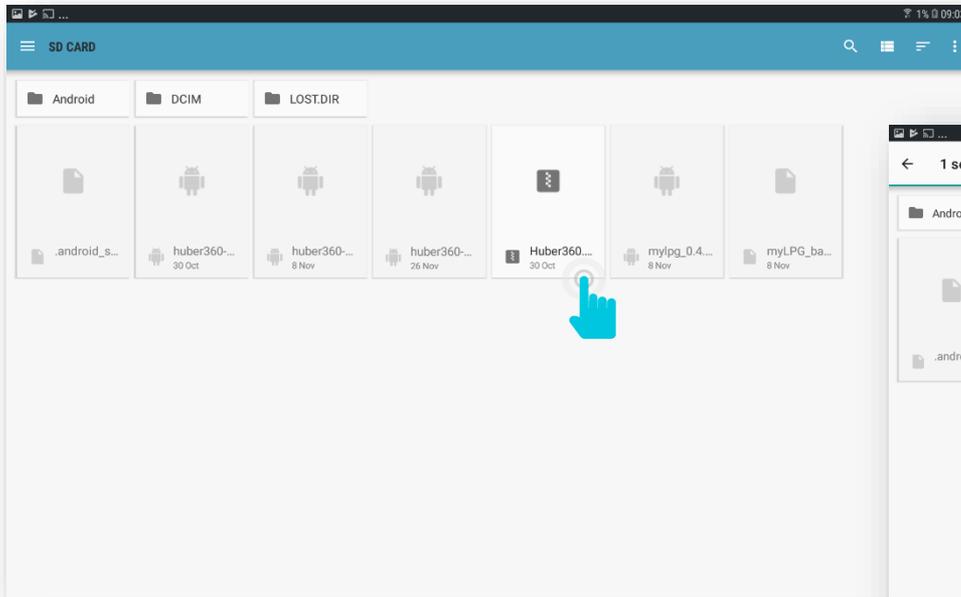




3. DATA IMPORT

3.b DATA RECOVERY

The data import is done.

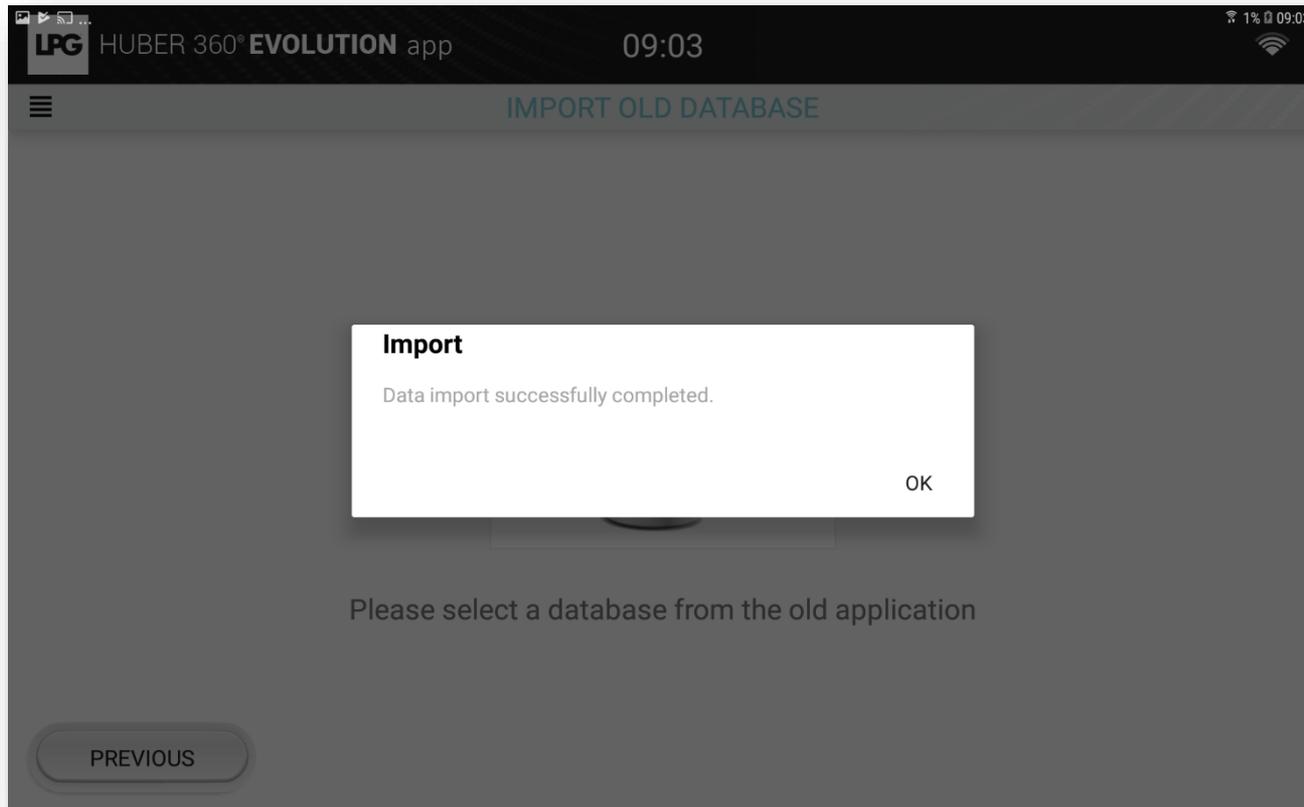



HUBER 360[®] EVOLUTION APP



4. DATA EXPORT



HUBER 360[®] EVOLUTION APP



4. DATA EXPORT

To access your tablet settings, click on  then Settings.

The section « **Disclaimer** » raises awareness about the protection and security of customer data that can be questioned if you choose to export this information. If you want to export this data, please select "**export of customer data**".

