



Registered trademark

OPERATOR'S MANUAL

Mobilift M6[®] CONNECT

**Please read the complete manual carefully
before using your equipment**

© LPG Systems 2024. LPG[®], endermologie[®] and Mobilift M6[®] are registered trademarks of LPG Systems.
Reproduction, even in part, is strictly prohibited.

GU1303-EN_indC1
edition dated 01/24



Congratulations on the purchase of your Mobilift M6[®] Connect. This model is the result of numerous years of experience in the design and production of cutaneous tissue care systems. You will fully experience the technical performance and reliability of LPG Systems, leader in this sector.

With the aim to continuously provide satisfaction, your device is equipped with a specific software devised to ensure connection with the LPG[®] dedicated server. In this case, your device is a connected device that, subject to your prior consent, will be able to send and receive information. The data collected through this software will allow LPG[®] to give you better services in respect of support and maintenance. The information that your device will receive, will allow you to benefit from all its features. This operator's manual contains the operating description, basic maintenance instructions to be performed periodically and the safety instructions. Your device is intended for use in the care of connective tissue. It should be used only by a professional who has attended the manufacturer's training provided by LPG Systems, its subsidiaries or an approved provider, if you live outside France. Should you have any doubts whatsoever concerning the operation or maintenance of your equipment, please do not hesitate to contact LPG Systems via the Customer Service Department of your distributor.

ATTENTION

In order to respond better to customer requirements and expectations, LPG Systems is continuously researching ways of improving the design and quality of its products. This will explain the few minor differences between your equipment and the item described in this guide.

→ PACKAGE CONTENTS

- One Mobilift M6® Connect unit
- One main Ergolift head
- Two Ergolift chambers (Lift 20 and Lift10)
- One electrical power cord
- One operator's manual
- One POS marketing set


→ TABLE OF CONTENTS

1. DESCRIPTION OF THE UNIT	4
2. DESCRIPTION OF THE CONTROLS	6
3. PRECAUTIONS OF USE	7
4. MAINTENANCE	11
5. ERGOLIFT HEAD	16
6. TROUBLESHOOTING	20
7. TECHNICAL SPECIFICATIONS	21
8. WARRANTY	22
GENERAL WARRANTY CONDITIONS	22
WARRANTY ACTIVATION	25


ATTENTION

Unpacking instructions to be retained.
The manufacturer reserves the right to amend the product technical specifications without prior notice. Any reproduction – even in part – is prohibited. All the illustrations in this operator's manual are non-binding.

→ MOBILIFT M6® CONNECT

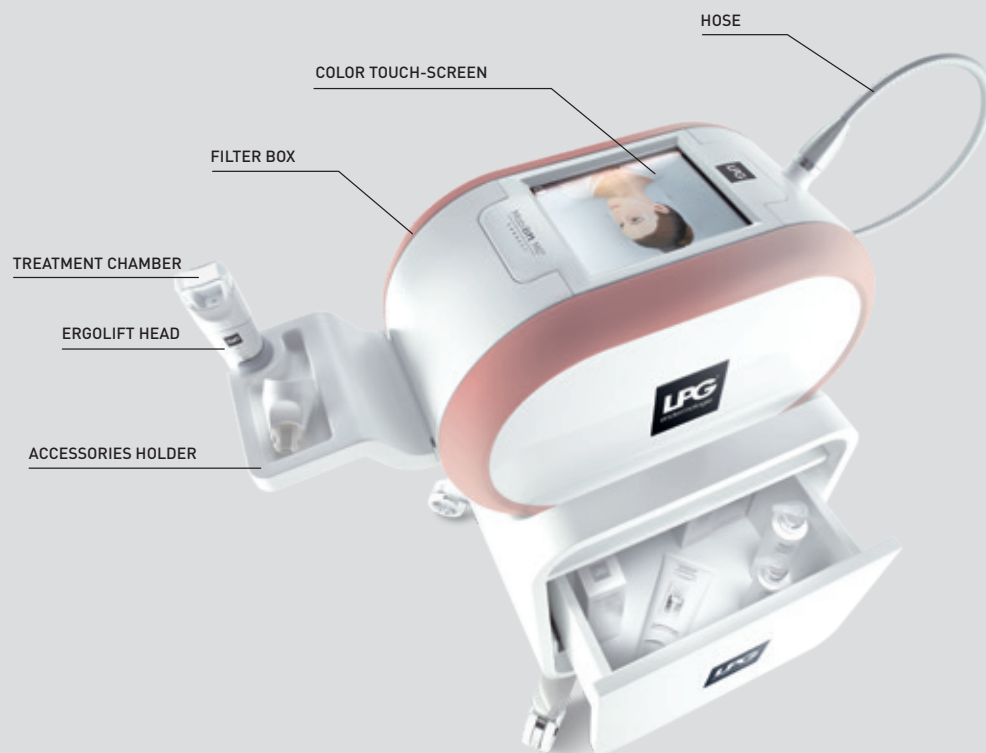

→ MOBILIFT M6® CONNECT (CONT'D)

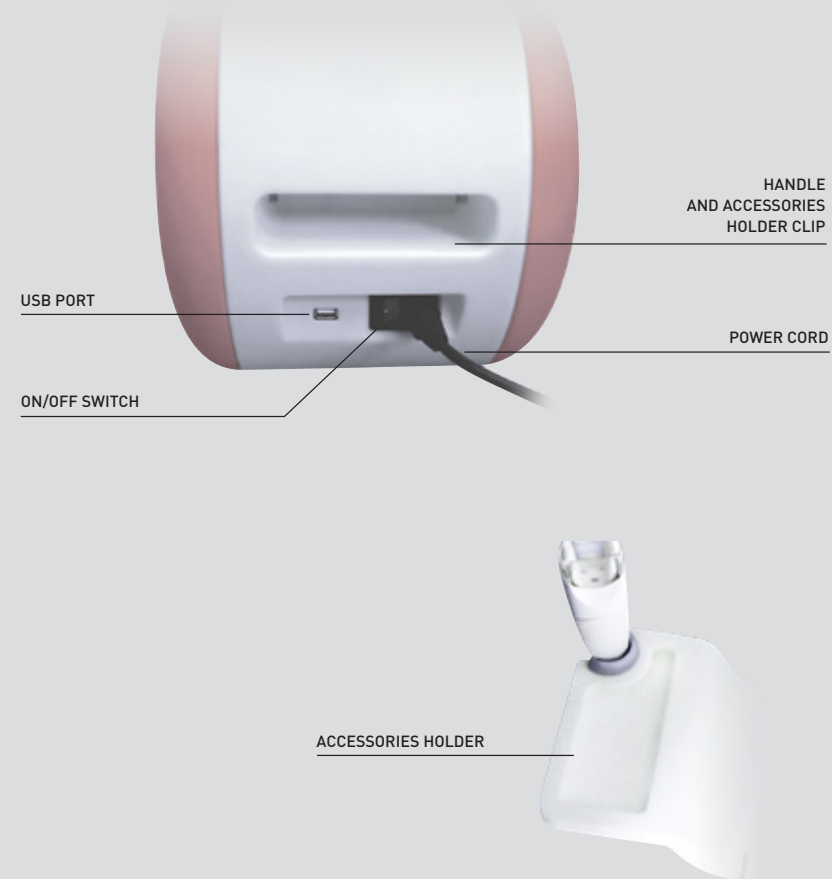

⚠ ATTENTION

The unit can only operate if it is connected to the mains by its power cord and provided the ON switch has been activated and the green voltage light is on.
After switching on the unit, please wait a few seconds for the screen to display information.

→ CONTROL TOUCH-SCREEN



TOUCH-SCREEN

→ ERGOLIFT HEAD

ERGOLIFT HEAD

TREATMENT CHAMBER

FLAPS



→ IMPORTANT SAFETY INFORMATION

All safety precautions must be observed whilst using electrical equipment. Please read all safety notices and precautions prior to use of the equipment.

DANGER - TO MINIMIZE THE RISK OF ELECTRICAL SHOCK:

- Always disconnect the equipment from the electrical supply outlet after use and before cleaning and maintenance.
- Check that the supply voltage of the unit indicated on the data plate complies with the mains voltage.
- The unit must be connected by the power cord¹ supplied to a grounded outlet in accordance with current electrical standards. Electrical adapters must not be used with this equipment.

→ WARNING

- TO MINIMIZE THE RISK OF BURNS, FIRE, ELECTRICAL SHOCK OR INJURY

- The equipment must not be left unattended whilst connected to the electrical supply.
- Disconnect the device when not used over a long period of time or before inserting or removing parts. This device can be used by children 8 years of age and older and those with reduced physical, sensory or mental abilities or lack of experience and knowledge if they have been instructed or instructed to use the hearing aid, safe way and understand the dangers involved. Children should not play with this device. Cleaning and maintenance should not be performed by children without monitoring.
- Never use the unit for purposes other than those recommended by LPG Systems. Only use the treatment heads supplied with your unit or those recommended by LPG Systems.
- Never use the equipment if:
 - the electrical supply cable or outlet is damaged.
 - the equipment does not function correctly.
 - the equipment is damaged or has fallen or been dropped.
 - the equipment has been exposed to excessive humidity.
 In such cases, return the equipment to an approved LPG service centre.
- Do not move the unit by pulling on the electrical supply cable or hose.
- Do not move the device by holding on to the accessories holder. It is recommended to remove it everytime you move the device and to use the handle provided.
- Fully unwind the electrical power cord and keep it away from warm surfaces.
- Never use the equipment if the ventilation ports are obstructed. Ensure that the ventilation ports are kept clear of dust or other contaminants.
- Do not allow solid debris, liquid or other foreign bodies to fall or be sucked into the unit, as these could cause damage.
- Never use the equipment on a dusty, uneven floor, or in a moist atmosphere.
- The device must be positioned on a flat and rigid surface.
- Never use the equipment in the presence of aerosols or oxygen.
- Before disconnecting the unit from the electrical supply, set all controls to the 'off' position and unplug the unit.
- It is prohibited to modify this equipment without prior authorisation from the manufacturer.

⚠ ATTENTION

¹Europe VIIG-H05VVF3G0,75-C13; Italy I/3/16-H05VVF3G0,75-C13; Switzerland 12G-H05VVF3G0,75-C13; UK BS13/13-H05VVF3G0,75-C13; Japan 498GJ-VCTF3X1,25-C13; USA, Canada, Mexico N5/15-SJT3X18AWG-C13

→ PRECAUTIONS OF USE

ATTENTION: KEEP THESE INSTRUCTIONS

Your device should be used on clean, healthy skin. It is important to read and observe the following precautions and contraindications before using your device.

- Never touch the client and the device's unprotected cables or connectors simultaneously.
- Do not use the Ergolift treatment head for scalp treatments.
- Only use treatment heads supplied with your unit or recommended by LPG Systems.
- LPG Systems will not be liable for any inappropriate use of the equipment.
- The operator must be particularly attentive to the sensations felt by the person undergoing treatment.
- The operator must ensure that the parameters (intensity, sequentiality, differential...) are always adapted to the cutaneous tissue being cared.
- Do not use the USB connection during treatment.
- Do not operate the unit in unsuitable environmental conditions (see technical specifications).

→ ELECTROMAGNETIC COMPATIBILITY

- The Mobilift M6® Connect uses RF energy only for its internal function. Therefore its RF emissions are very low and not likely to cause any interference to nearby electronic equipment.
- The Mobilift M6® Connect is suitable for use in all establishments including domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
- The Mobilift M6® Connect must be used on a wooden, concrete or ceramic flooring. If the floor covering is of synthetic material the relative humidity must be at least 30%.
- The Mobilift M6® Connect must be connected to an electrical supply equivalent in quality to typical commercial or hospital supplies.
- If the operator of the Mobilift M6® Connect requires continuous function during power supply network failure, it is recommended that an alternative failsafe electrical supply or battery power supply be provided.
- Portable or mobile RF communication devices must not be used in close proximity to the Mobilift M6® Connect or its cables.
- Interference may be encountered in proximity to devices marked with the following symbol:



→ CONTRAINDICATIONS

- Do not treat open wounds, the eyes, intra-cavity areas, mucous membranes, the genitals or nipples.
- Do not treat a client with an infectious disease, a growing tumour, a phlebitis, a wound, or an infected area.
- Do not treat a client with skin cancer, a visible tumor, or other cancerous lesions. Consult with a doctor in cases where the client has a case history of tumours or is in remission.
- Do not treat any swollen or inflamed areas without seeking medical advice and without having had training in specific LPG® techniques in this particular area.
- Do not treat people with circulatory problems without first consulting their doctor.
- Do not treat a client with unexplained and persistent pain without seeking medical advice or training in the LPG® technique in the particular area.
- Stop treatment immediately if the client experiences pain and consult a doctor.
- This device should not be used on skin rashes, herpes, inflamed or infected acne, or vitiligo.
- To avoid bruising, exercise caution when determining a client's level of sensitivity and avoid use on clients taking anti-coagulant drugs.
- For a more detailed list of the indications and contra-indications of endermologie®, please refer to the training manuals.
- As this list is not exhaustive, always seek medical advice in the event of doubt.
- Because of the risk of interference, it is important that the professional ensures the client is not equipped with a personal medical device such as a pacemaker. If this is the case, the professional should obtain details about the device in question to ensure that any interference will not affect the correct use of the equipment.

⚠ ATTENTION


This unit contains programs to help the operator obtain the best anticipated results for each case undergoing care. Under no circumstances may these programs be construed as a guarantee of successful care, which varies, depending on health, morphology, physiology and eating behaviour of each client.


→ IDENTIFICATION RATING PLATE


Your unit is identified by a serial number shown on the rating plate.


The rating plate also shows the permitted supply voltage for the unit and the serial number that enables to identify your device. If you need to contact LPG Systems because of a technical problem, please indicate the serial number of your Mobilift M6® Connect. This serial number provides information on the year and month of manufacture of your unit.


The letter indicates the year the device was manufactured. Z=2009, A=2010, B=2011, etc. The two digits indicate the production month: 01=January; 02=February; 03=March; etc.

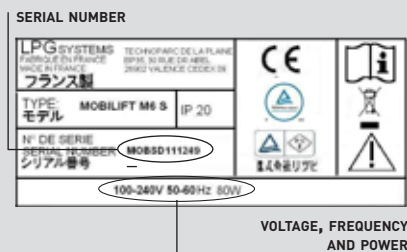



 This icon indicates that the unit was sold after August 13, 2006. In conformity with the 2002/96/CE directive, it cannot be thrown away with standard household waste but must be disposed of by means of recycling. When your device reaches the end of its useful life, it must be brought to an appropriate recycling center or returned to your dealer. By doing so, you help the environment by contributing to the conservation of natural resources and the protection of human health.


 This icon indicates that some specific warnings or precautions associated with this device are not on the label.

 This symbol means always consult the accompanying documents before using your device.

 This symbol means store the device somewhere protected from inclement weather.

 This symbol indicates temperature limits.

 This symbol indicates relative humidity limits.

 This symbol means "Danger: High Voltage" (US only)

→ CLEANING THE UNIT

It is recommended that you clean your unit as often as possible, not only for reasons of hygiene and aesthetics but also because cleaning the unit will help keep it in a good working order and extend its useful life.

Using a moist sponge, clean the following sections:

- All the external hoods.
- Hoses.
- The electrical supply cable.

Using a cloth soaked with a small amount of alcohol-free domestic cleaning product, clean the following parts:

- Control screen and control panel.

⚠ ATTENTION

Identification rating plates may vary. The approved one is one on your machine.

⚠ ATTENTION

Do not use corrosive products such as acetone, trichloroethylene, or rubbing alcohol.

→ REPLACING THE FILTER CARTRIDGE

Your device contains a filter cartridge.
These components guarantee the efficiency of your unit and prolong its useful life.

Ensure that they are changed as soon as the command screen displays the following icon (fig. 1).

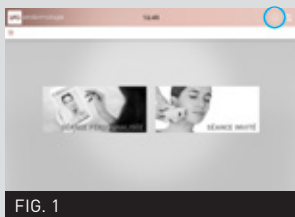


FIG. 1

Access the "filter change" menu as follows :

Select the "maintenance" menu by pressing the icon indicated (fig. 2).

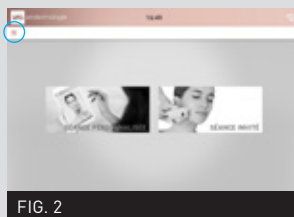


FIG. 2

Select the "filter" menu by pressing the icon indicated (fig. 3).



FIG. 3



FIG. 4

→ REPLACING THE FILTER CARTRIDGE (CONT'D)

Replace the filter cartridge with a new one. (fig. 6)
Once the filter cartridge is replaced, the filter meter should be reset by pressing the icon as indicated: (fig. 7).

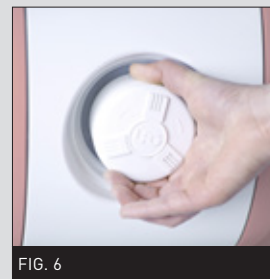


FIG. 6

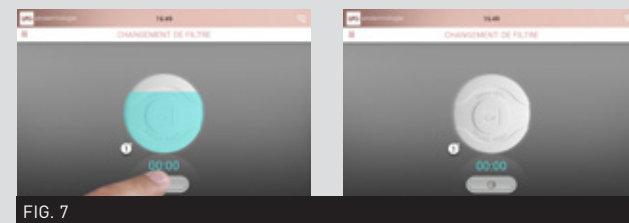


FIG. 7

→ INSTRUCTIONS FOR CONNECTING THE HOSE

To connect the hose, proceed as indicated below:

- Position the locking ring in the locked position (fig. 1).
- Position the end of the hose so that the hose key is lined-up with the key slot of the treatment head connection (fig. 2).
- Push the hose in the treatment head connection until it clicks into place.

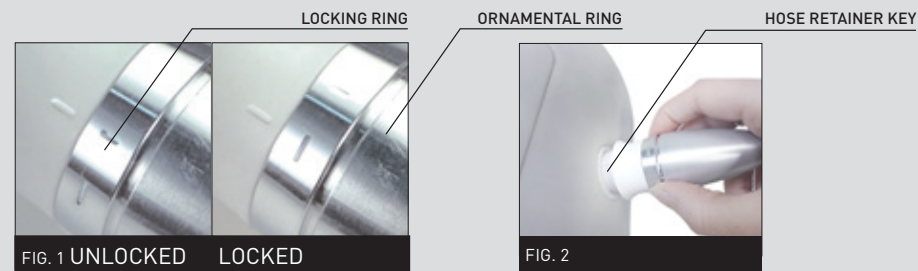


FIG. 1 UNLOCKED LOCKED


FIG. 2

➤ ATTENTION

Never use your device without a filter.
Turn the device off before replacing your filter cartridge.


→ INSTRUCTIONS FOR DISCONNECTING THE HOSE


- To disconnect the hose, follow the same steps in the reverse order:
- Unlock by turning the locking ring (fig. 3).
 - Lift the locking ring, then remove the hose gently by pulling on the white ring (fig. 4).

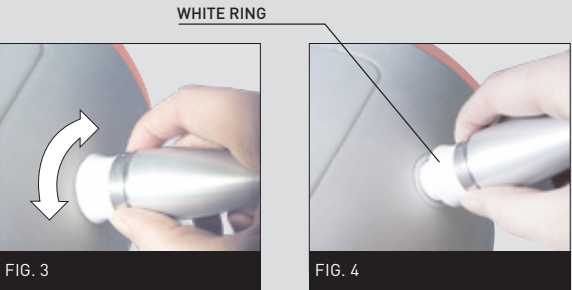

→ TRANSPORTATION/MOVING

- When moving your Mobilift M6® Connect, you should:
- Remove the accessories holder (fig. 1).
 - Carry the device by the handle (fig. 2).

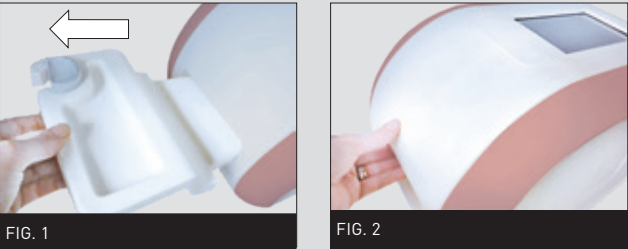

 **ATTENTION**
Never carry the device by the accessories holder.

→ REPLACING THE POWER CORD


If the power cord of your device is damaged, please contact LPG Systems Customer Service for a replacement via the Customer Service Department of your distributor.

→ MAINTENANCE LOG SHEET

- Replacing flaps: once the flaps no longer treat the skin properly, they must be replaced. They should be replaced after every 20 hours.
- A personal face endermologie® Kit for use by the client is available for sale.

DATE	NO. OF HOURS	OPERATIONS PERFORMED


→ ERGOLIFT HEAD DESCRIPTION

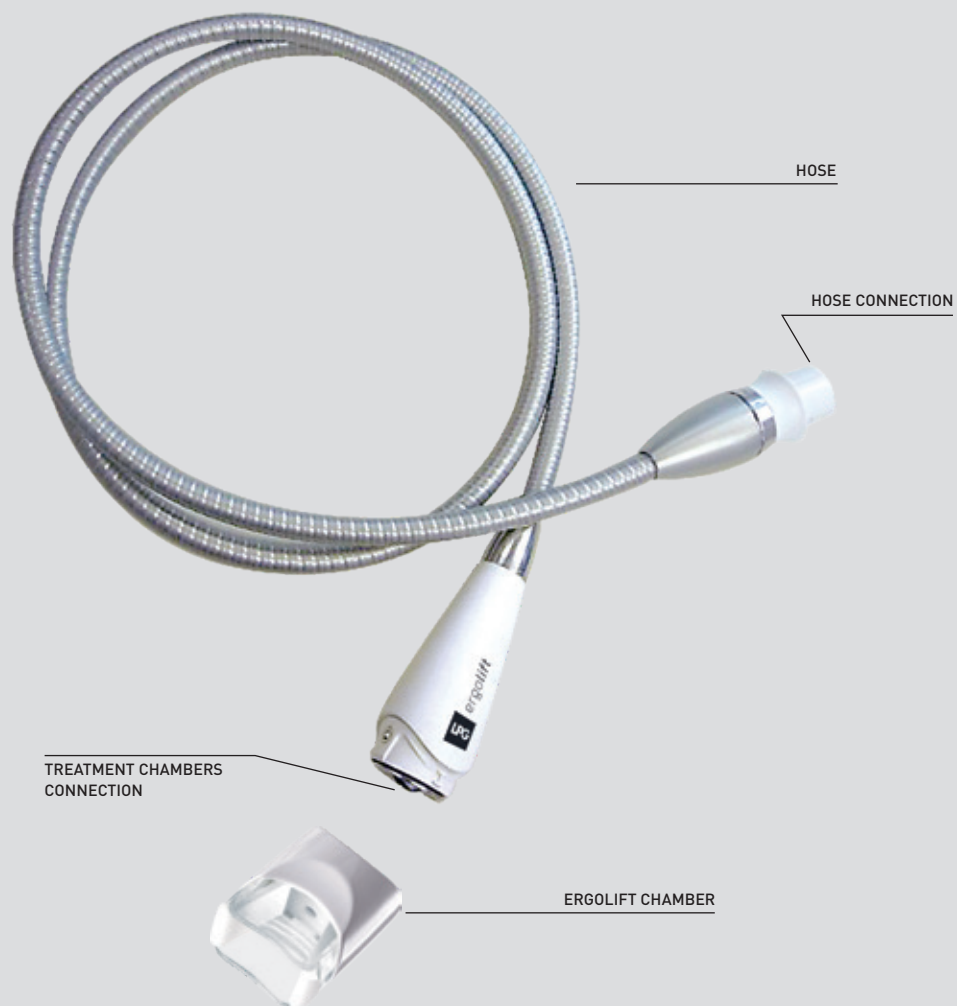

→ ERGOLIFT CHAMBERS




Lift 20
chamber
with removable
Lift 20 flaps

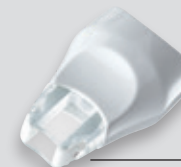

Lift 10
chamber
with removable
Lift 10 flaps

Only the Lift 20 and Lift 10 chambers can be connected to the Ergolift head. They can be connected and disconnected by a simple push-pull action.

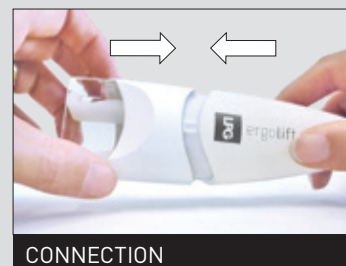

CONNECTION




→ CLEANING THE ERGOLIFT HEAD AND ERGOLIFT CHAMBERS


For reasons of hygiene, the disconnected heads should be cleaned before and after each use, using antiseptic wipes impregnated with a bactericidal and fungicidal solution. Special attention must be given to the cleanliness of the parts that are in contact with the client.

Before and after each use, clean the flaps and Ergolift chamber:

1. Disconnect the chamber from the Ergolift treatment head. (fig. 1)
2. Remove the flaps thanks to the dedicated tool. (fig.2)
3. Thoroughly clean the Ergolift chamber, the flaps and the tool for at least one minute with the wipes described above. (fig.3)
4. Put the flaps back in the Ergolift chamber by following the same steps in the reverse order (fig.4)

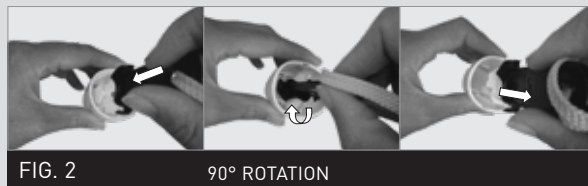

→ CLEANING THE ERGOLIFT HEAD AND ERGOLIFT CHAMBERS (CONT'D)

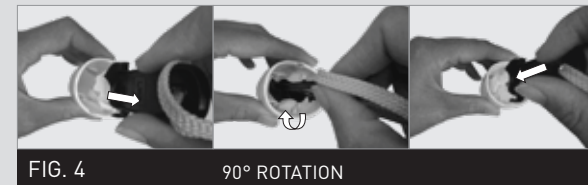

➤ ATTENTION

The use of aggressive products, such as acetone, trichloroethylene, or alcohol, and abrasive sponges, ultrasound or UV lamps is strictly prohibited. All cleaned and/or disinfected heads should be placed in the storage drawer to avoid any confusion.

→ WHAT IF I HAVE A PROBLEM?

If your unit is not working properly, proceed with the following checks before calling Customer Service:

- Is the unit properly connected to a mains plug?
- Is the mains plug live?
- Is the ON switch lit up?
- Is the filter cartridge clean and correctly placed?
- Is the hose properly connected?
- Is the hose clogged?
- Are the Ergolift chamber and flaps properly connected?

Once these checks have been carried out and if the malfunction persists, please contact LPG Systems Customer Service or the nearest authorized dealer, indicating the model of your unit and its serial number.


→ TECHNICAL SPECIFICATIONS

Dimensions: length x width x height:65 x 26 x31 cm
 Net weight: 10.5 kg
 Maximum set depression:35 kPa (350 mbar)
 Cooling:by mechanical ventilation incorporated into the pump
 Protection index: IP20
 Electrical protection class: 1
 Operating temperature: +10°C to +30°C
 Storage temperature: -20°C to +70°C
 Electrical features: 100-240V / 50-60Hz / 80 VA
 Wifi: 2.4Ghz b/g/n
 EIRP:100mW

Operating environment:
 Ambient temperature:..... 10 to 30° C for normal operation.
 Ambient relative humidity: 30 to 75% without condensation.
 Atmospheric pressure: no significant influence for operation.
 Max altitude: 2,500 m

Environmental characteristics of transport and storage:
 Ambient temperature: -20°C to +70°C
 Ambient relative humidity: 10 to 90% without condensation
 Atmospheric pressure: no significant influence of ambient conditions in operation

Unit fitted with patented Ergolift treatment head.
 Device designed for uninterrupted use.

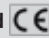The Mobilift M6® Connect is labeled  in accordance with legislation and standards set forth in directives EMC 2004/108/EC and low voltage 2006/95/EC.

→ GENERAL WARRANTY CONDITIONS

You have recently acquired an appliance distributed by LPG Systems or an LPG Systems approved distributor.

It is the purchaser/user's responsibility to find out from the local authorities the conditions and professional qualifications required before using the appliance. The purchase of this equipment implies the legal acceptance by the purchaser/ professional user of these general warranty conditions.

If the appliance was sold to you by an approved LPG Systems distributor, the purchaser/ user should refer to the supplier's warranty conditions. These may in no way increase the undertakings made by LPG Systems in these present warranty conditions. The warranty can only be implemented and is only valid if the warranty slip has been duly filled out and returned to LPG Systems within two weeks of delivery, irrespective of the country. Warranty slips that are only partially completed will be rejected.

The appliance is guaranteed against manufacturing flaws and defects in the raw materials.

The warranty extends for the shorter of the following two periods: one (1) year OR two thousand (2,000) hours of use from the invoice date. During this period, LPG Systems undertakes to exchange or repair free of charge, as quickly as possible, any part that LPG Systems acknowledges as defective, however LPG Systems does not undertake to replace the entire appliance. Traveling and living expenses for our technicians and transportation costs of the appliance or parts to and from the aftersales service workshop are not covered by this warranty.

Replacements and repairs performed within this warranty, with or without immobilization of the equipment, shall not have the effect of extending the warranty period. Replaced parts become the property of LPG Systems or the approved distributor. No compensation shall be paid for loss of use.

Subject to other conditions hereafter, warranty shall apply if the purchaser/professional user has allowed LPG to proceed to necessary repair works.

→ GENERAL WARRANTY CONDITIONS (CONT'D)

Warranty exclusions:

- Damage occurring during transportation. Transportation of this equipment and/or spare parts is at the recipient's own risk. Before taking delivery, it is the recipient's responsibility to verify the state of the goods and to make a claim against the transport company in the manner usual in the delivery country.
- Non-observance of the installation and operating instructions, failure to carry out maintenance and/or negligence in maintaining the appliance and/or its filter cartridges, connection to a faulty electricity supply or a non-earthed electricity supply or a power supply whose voltage is different to the one indicated on the appliance.
- If an appliance is sold before the end of the warranty period, the warranty is transferred to the purchaser for the remaining warranty period, on the condition that:
 - I The original invoice is provided.
 - II That the initial vendor is informed of the sale.
- Modification, mounting of accessories or dismantling of the equipment.
- Any operation and/or intervention not specified in the LPG Systems Operating Instructions and performed on the equipment by the purchaser/user and/or any party not approved by LPG Systems.
- Use of consumables, spare parts, inappropriate components or parts not supplied by LPG Systems.
- Blockage of the appliance through aspiration of a foreign body.
- Normal wear and tear of any of the equipment's parts resulting from normal usage.
- Damages or default resulting of accidental events (falls, impacts...).
- Damages or default resulting of natural disasters (lightening, water damages...), fire, negligence or abuses.

→ LIMITATION AND EXONERATION OF LIABILITY

The warranty is limited to the replacement of the components of the device which comply with conditions described above. Under no circumstances shall LPG Systems be liable for any loss or damage as a result or in connection with the device and/or its use, including any financial loss, loss of margin, loss of use, etc.

This clause shall apply under any and all legal basis.

Whenever the above restriction may not be applicable or enforceable, LPG Systems liability shall be limited to the price for the device and/or the service.

→ WARRANTY ACTIVATION

You can activate your warranty online by connecting to our warranty webpage:

www.lpgsystems.com/warranty

→ OR FILL IN THE FORM AND RETURN TO LPG SYSTEMS



Name*:

Address*:

Email:

Serial number **MOBILIFT M6® CONNECT***:

Date:

* Via the form above, LPG SYSTEMS operates, as controller, a personal data processing based on the contract of sale of your Mobilift M6® Connect device, in order to manage the guarantees that apply to it. LPG Systems keeps the data only for the time required for the operations for which it was collected and in compliance with the regulations in force.

As per the applicable regulation, you have a right to request from LPG Systems, access, rectification, objection and erasure of your personal data, restriction of processing or to object to processing as well as the right to data portability, to withdraw your consent at any time and to lodge a complaint with a supervisory authority.

You may exercise your rights by sending an e-mail to the address: HYPERLINK «mailto:dataprivacy@lpgsystems.com» dataprivacy@lpgsystems.com or a letter to the postal address: LPG SYSTEMS, Service Relation Clients, Technoparc de la Plaine, CS 900 35, 30 rue du Docteur Abel, 26902 Valence Cedex 9 FRANCE. Each request must include a proof of your identity containing your signature.

ATTENTION

To validate the warranty, detach this slip and return to the address below within 15 days following delivery of the unit:

**LPG SYSTEMS SAS - Technoparc de la Plaine
30, rue du docteur Abel - CS 90035 - 26902 VALENCE Cedex 9 - France**



HEADQUARTERS LPG SYSTEMS S.A.S.

TECHNOPARC DE LA PLAINE
30, RUE DU DR. ABEL - CS 900 35 - 26902 VALENCE CEDEX 09 - FRANCE
TÉL.: +33 (0)4 75 78 69 00 - FAX: +33 (0)4 75 42 80 85

INTERNATIONAL/MARKETING

ECOLUCIOLES - BAT A
955 RTE DES LUCIOLES - BP 243 - 06905 SOPHIA-ANTIPOLIS - FRANCE
TEL.: +33 (0)4 92 38 39 00 - FAX: +33 (0)4 92 96 09 65

