



You just purchased your Mobilift M6 Limited Edition

This machine is equipped with the TR30 treatment head. Here enclosed are the instructions.

Please read this manual carefully before installation and use of the Mobilift M6 Limited Edition.


If you have any doubts whatsoever concerning the operation or maintenance of your equipment, please do not hesitate to contact LPG via the Customer Service Department of your distributor.

## OPERATOR'S MANUAL


# Mobilift M6<sup>®</sup> *Limited Edition*

© LPG Systems 2024. LPG, Endermolift<sup>™</sup> and Mobilift M6<sup>®</sup> are registered trademarks of LPG<sup>®</sup>.  
Reproduction, even in part, is strictly prohibited.

## → ROLL HEAD HOSE

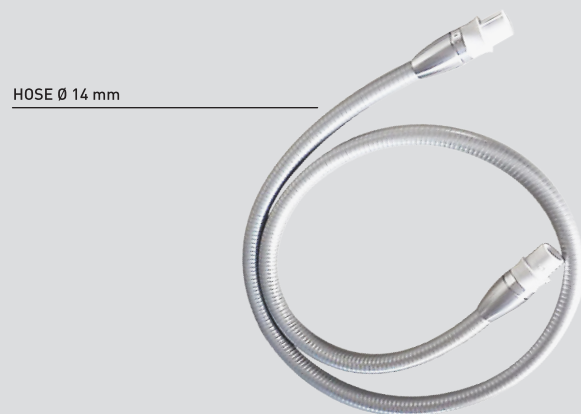

## → AUXILIARY HEAD TR30

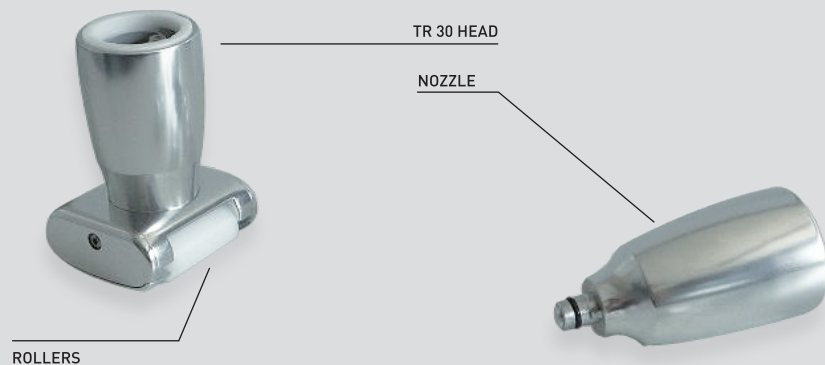

## → CLEANING

1. Disconnect the auxiliary head from the adapter.
2. Remove the two rollers from the head for effective and rapid cleaning (fig. 1 - 2).
3. Thoroughly clean for at least 1 minute the rollers, seal, treatment chamber, micro-heads, disassembly tool, and micro-nozzles with LPG wipes soaked in a bactericide and fungicide solution (fig. 3 - 4).
4. Refit the rollers and check they spin freely.





### ⚠ ATTENTION

The use of aggressive products, such as acetone, trichloroethylene, or alcohol at 90°, and abrasive sponges, ultrasound or UV lamps is strictly prohibited. All the procedures described in this section must be carried out with the machine turned off and the power cord unplugged.

## → INSTRUCTIONS FOR CONNECTING THE HOSE

- **To connect the hose**, proceed as indicated below :
- Position the locking ring in the locked position (**fig.1**).
- Position the end of the hose so that the hose key is lined-up with the key slot of the care head connection (**fig.2**).
- Push the hose into the care head connection until it clicks into place

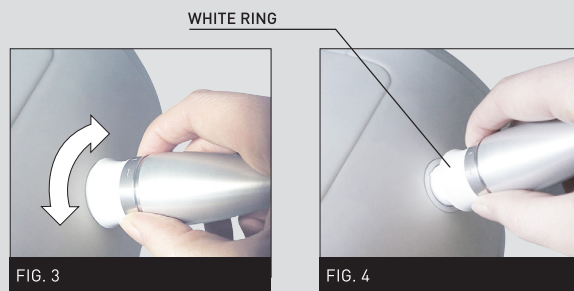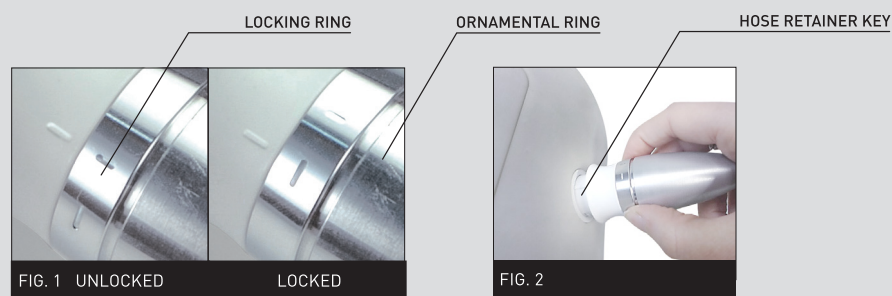

**To disconnect the hose**, follow the same steps in the reverse order :

- Unlock by turning the locking ring (**fig.3**).
- Lift the locking ring, then remove the hose gently by pulling on the white ring (**fig.4**).



### HEADQUARTERS: LPG SYSTEMS SAS

TECHNOPARC DE LA PLAINE  
30, RUE DU DR. ABEL - CS 90035 - 26902 VALENCE CEDEX 09 - FRANCE  
TEL.: +33 (0)4 75 78 69 00 - FAX: +33 (0)4 75 42 80 85

### INTERNATIONAL/MARKETING

ECOLUCIOLES - BAT A  
955 RTE DES LUCIOLES - BP 243 - 06905 SOPHIA-ANTIPOLIS - FRANCE  
TEL.: +33 (0)4 92 38 39 00 - FAX: +33 (0)4 92 96 09 65

