



OPERATOR'S MANUAL

Pack Endermolift

**Please read the complete manual carefully
before using your equipment**

© LPG Systems 2024. LPG, Cellu M6, Integral and Keymodule are registered trademarks of LPG.
Reproduction, even in part, is strictly prohibited.



You have just purchased an Endermolift pack. The Ergolift head and its care chambers contained in this pack can be connected to the following LPG devices: Cellu M6 Integral and Cellu M6 Endermolab. This document contains the operating instructions and explains basic maintenance procedures for your Ergolift head.


Your device is intended for use in the treatment of connective tissue. It should be used only by a professional who has attended the manufacturer's training provided by LPG Systems or an approved provider, if you live outside of France.

If you have any doubts whatsoever concerning the operation or maintenance of your equipment, please do not hesitate to contact LPG via the Customer Service Department of your distributor.

Customer Service LPG Systems

+33 (0)4 75 78 69 89


→ ERGOLIFT HEAD DESCRIPTION

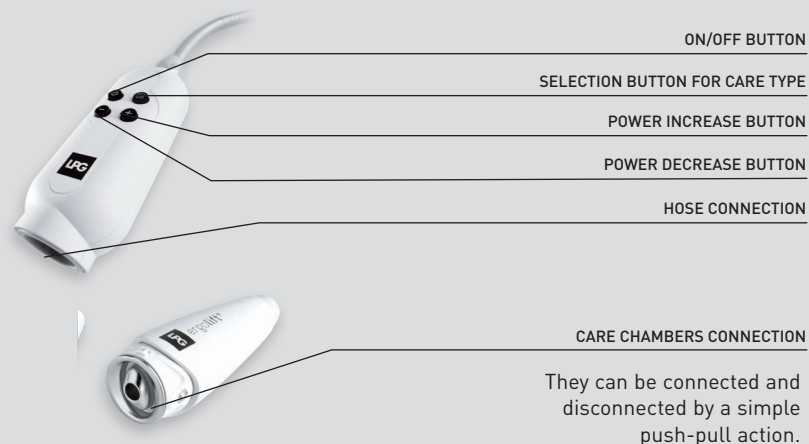

→ CARE CHAMBERS DESCRIPTION




Flap TML 20
Care chamber
TML20
removable flap

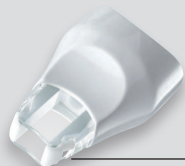

Flap TML 10
Care chamber
TML10
removable flap

Only the TML20 and TML10 care chambers can be connected to the Ergolift head. They can be connected and disconnected by a simple push-pull action.

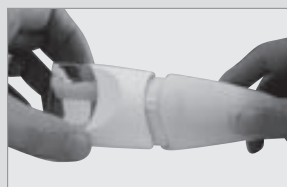

→ CLEANING THE ERGOLIFT HEAD AND CARE CHAMBERS

MAINTENANCE

For reasons of hygiene, the care heads should be cleaned after each use, using antiseptic wipes with a bactericidal and fungicidal solution.

Special attention must be given to the cleanliness of the parts that are in contact with the client.

CLEANING THE ERGOLIFT HEAD AND CARE CHAMBERS

For reasons of hygiene, the care heads should be cleaned after each use, using antiseptic wipes impregnated with a bactericidal and fungicidal solution. Special attention must be given to the cleanliness of the parts that are in contact with the client.

Before each use, clean the flap and care chamber.

1. Disconnect the Ergolift head care chamber. (fig. 1)

2. Remove the flaps. (fig. 2)

3. Meticulously clean the care chamber and flap for at least 1 minute with the aboved mentionned wipes. (fig. 3)

A personal Endermolift Kit for the client is available for sale. It can be used for approximately 20 treatment sessions.

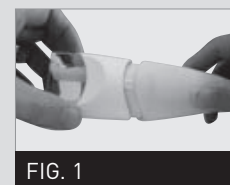


FIG. 1

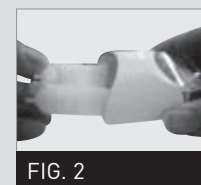


FIG. 2

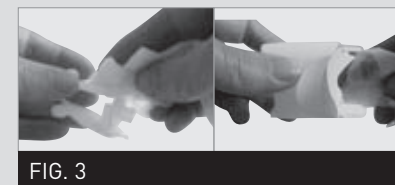


FIG. 3

DISINFECTING THE CARE CHAMBERS

The Ergolift head are in direct contact with client's skin. Under certain specific applications, it needs to be disinfected after each use:

1. Follow the cleaning procedure described above.
2. Soak the flap and care chamber in an OPA disinfectant for 12 minutes at 20° C, as recommended on the disinfectant packaging.
3. Carefully rinse the flap and the care chamber with sterile or drinking water for at least 1 minute using a large volume of water (approximately 8 liters). Repeat twice for a total of 3 rinses.
4. Dry the care chamber and flap.
5. Clean the storage drawer with antiseptic wipes, then place the care chamber and flap in it.

ATTENTION

The use of aggressive products, such as acetone, trichloroethylene, or alcohol at 90°, and abrasive sponges, ultrasound or UV lamps is strictly prohibited. All cleaned and/or disinfected heads should be placed in the storage drawer to avoid any confusion. Use a disinfectant whose active ingredient is ortho-phthalaldehyde (OPA). Before using the disinfectant, read and follow the recommendations, contraindications, and warnings associated with this product. Refer to the instructions for using this solution.



HEADQUARTERS LPG SYSTEMS S.A.S

TECHNOPARC DE LA PLAINE
30, RUE DU DR. ABEL - CS 900 35 - 26902 VALENCE CEDEX 09 - FRANCE
TEL.: +33 (0)4 75 78 69 00 - FAX: +33 (0)4 75 42 80 85

INTERNATIONAL/MARKETING

ECOLUCIOLES – BAT A
955 RTE DES LUCIOLES - BP 243 - 06905 SOPHIA-ANTIPOLIS - FRANCE
TEL.: +33 (0)4 92 38 39 00 - FAX: +33 (0)4 92 96 09 65

