TABLET QUICKSTART SNEL STARTEN MET DE TABLET GUIDA RAPIDA TABLET INICIO RÁPIDO TABLET TABLET SCHNELLSTART

INNER POWER. BETTER LIFE.

ENNL

SELECT THE DISPLAY LANGUAGE SELECTEER DE WEERGAVETAAL

The tablet you have received is set to the French language. To change the language, please follow the instructions below. Click on "**Paramètres**" application on your tablet's home screen. Select "**Gestion globale**" section and click on "**Langue et saisie**". Next, click on "**Langue**" to choose the desired language.

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Ni-Fi PG Machines	
Bluetooth Connect to nearby Bluetooth devices.	D
Flight mode turn off Wr Fi and Bluetoorh.	D I
Data usage	
More connection settings	
Looking for something else?	
Samsung Cloud	
Location	

Start **"Settings**" application on your tablet's home screen. Select **"Connections**" section and click on **"Wi-Fi**". Select your Wi-Fi network. Start de **«Instelligen**» toepassing, deze is aanwezig op het onthaalscherm. Kies de rubriek **«Verbindingen**», druk dan op **«WI-FI**». Selecteer uw WI-FI-netwerk.

ENDERMOSCAN™ APPLICATION LAUNCH START DE ENDERMOSCAN™ TOEPASSING

Click on the endermoscan[™] icon on your tablet's home screen to launch the application. Enter your LPG[®] account (email address connected to your LPG[®] account) and password (chosen by yourself). In case of forgotten password, please click on "**can't access your account?**" or on the following link to reset it: https://portal.lpgsystems.com/password

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	LICENSING AGREEMENT
OREWORD	
he activities of stended chiefly	the Company LPG SYSTEMS (hereafter inferred to as "LPG") include the design, production and inviketing on a worldwide basis of devices for the treatment of conjunctive tissue and functional rehabilitation for professionals from the medical, aeatheric and sports makers. The company rejops established based awareses at both a national and international level.
PG provides th unctions and fe	e Licensee with Software which can be used on the touchscreen Tablets supplied by LPG, enabling it to offer medical and sports care protocols adopted to the outcomens/patients and including off attares as described hereafter.
The Software is	intended for professionals having acquired one or several professional devices from LPG, either directly or via one of its distributors.
RTICLE 1 - CO	NOLUSION OF THE LICENCE
his Licence co umber B 335 1	institutes a legal agreement between the Licensee and LPG SYSTEMS, a simplified joint stock company constituted under French law, registered in the Trude & Company Register (RCS) of Romans under t ISI 816, being located on the Technoparc de la Plaine, 30 rue du docteur ABEL, Valence (2002) Gelex (9, FRANCE.
iny use of the S	offware presupposes that the Licence has been read and accepted.
The Licensee a Joence.	knowledges that he has been fully informed of the fact that his agreement regarding the content of the Licence does not require his handwritten signature but results simply from the confirmation of the
The Likensee In	a the option to backup or print this Licence using the functions provided in the Tablet's browser. The Licensee bears sole liability for this backup or printing.
ty clicking the "	confirm" buttor, the Licensee:
accept	the provisions of this Licence
agree to services.	the collection and transmission of technical and statistical data concerning the functioning of my device in order to optimize its use and the related
	te perforation of the Device

The licence agreement appears. Please scroll down to the bottom and tick the approval boxes. **Click on "Accept".**

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First of all, before connecting your tablet to your CELLU M6 ALLIANCE $\ensuremath{\mathbb{R}}$, make sure that:

1. Your tablet and your CELLU M6 ALLIANCE® are connected to the same Wi-Fi network.

2. Your endermoscan[™] application is launched.

Select "Tablet" item in the "Settings" menu of your CELLU M6 ALLIANCE®.

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CELLU M6 ALLIANCE[®].

Your CELLU M6 ALLIANCE[®] automatically detects your tablet. The "**tablet**" icon at the top right corner of your device screen and that of your tablet ensures that the connection is operational.

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ENNL

Back on the home screen of your CELLU M6 ALLIANCE[®], select «**Customized Session**» menu to access to the customized client's programs (previously created on the application).

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For more security, save all your data on an external support of your choice: micro SD card (available in supermarkets or in specialized stores), personal computer*, external hard drive...

At the time of saving", choose a password for security protection. Please keep it safe because without it, it will be impossible to restore your backups. For privacy reasons, LPG SYSTEMS does not have access to your password and cannot ask you for it. LPG SYSTEMS can not be held responsible for any loss of your data.

*Use the USB cable provided with the tablet to connect the tablet to your personal computer.

Go to the endermoscan[™] settings menu, "Settings**" and "**Database**" section.

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NEED HELP ? HULP NODIG ?

Discover our video tutorials for step-by-step guidance in the use of your device. Scan the QR Code below with your phone's or your tablet's camera. You will then be redirected to the corresponding playlist. If the QR Code reading does not work, simply download the free "**Unitag**" app from the Google Play Store or Apple Store.

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Should you have any questions, please contact your local LPG® distributor.

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CONECTE LA TABLET A SU RED WIFI CONNESSIONE DEL TABLET AL WI-FI

Image: Second s
V. P. P. Prove Manager Prove Prove Prove Image: State of the sta
Sile Sile O O N Calculator Celencer Disk VMacc Herit
Gallery Calculator Calendar Dlock YT Music Netfli
Soony Google TV endermodean HCBER 300 App

Wi-Fi LPG Machines	
Bluetooth Connect to nearby Bluetooth devices.	I D
Flight mode Turn off Wi-Fi and Bluetooth.	I D
Data usage	
More connection settings	
Looking for something else?	
Samsung Cloud Location	

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LANZAMIENTO DE LA APLICACION ENDERMOSCAN™ COME LANCIARE L'APP ENDERMOSCAN™

		INNER POWER. BETTER LIFE.	00.14	* 2% 8 09:14 ~
			someone@example.com	
	LPG INNER POWER. BETTER LIFE.		Sign in Carifi access your account?	
	_			
Control of the second se	(FG		«p+LPG Systeme	

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	09.19
	LICENSING AGREEMENT
FOREWORD	
The activities of the Comp intended chiefly for profes	rg LPG SYSTEMS (hereafter referred to as "LPU") include the design production and marketing on a workfulde basis of devices for the treatment of conjunctive tissue and functional rehabilitionals from the medical, aesthetic and sports markets. The company enjoys established based awareness at both a national and international level.
LPG provides the License functions and features as	with Schware which can be used on the touchspreen Tablets supplied by LPG, enabling it to offer medical and sports care protocols adapted to the outcomers/patients and including escabled hereafter.
The Software is intended f	r professionals having acquired one or several professional devices from LPO, either directly or via one of its distributors.
ARTICLE 1 - CONCLUSION	OF THE LICENCE
This Licence constitutes a number B 335 183 836, be	egal agreement between the Licensee and LPG SYSTEMS, a simplified joint stock company constituted under French law, registered in the Trade & Company Register (RCS) of Romans and gl located on the Technoparc de la Plane, 30 rue du docteur ABEL, Valence (26902) Cedex (9), FRANCE.
Any use of the Software pr	supposes that the Licence has been read and accepted.
The Licensee acknowledg Licence.	s that he has been fully informed of the fact that his agreement regarding the content of the Licence does not require his handwritten signature but results simply from the confirmation of
The Licensee has the optic	to backup or print this Licence using the functions provided in the Tablet's browser. The Licensee bears sole liability for this backup or printing.
By clicking the "confern" bo	don, the Licensee:
I accept the pr	ovisions of this Licence
agree to the coll services.	ction and transmission of technical and statistical data concerning the functioning of my device in order to optimize its use and the related
Laccent the neols	nation of the Device

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	R POWER. BETTER LIFE. 10:55	□
=	SETTINGS	
Ē	Date and time	>
0	Language	>
(†)	Update	>
•	Sound	>
Ø	Display	>
(î:	WiFi	>
(;;)	Head key config	>
	Tablet	
•	System information	>
-	Preferences	>

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LPG Machines	
Bluetooth Sciencet to nearby Bluetooth devices.	I D
Flight mode um off Wr FI and Bluetoorh.	I D
Data usage	
More connection settings	
Looking for something else? Samsung Cloud Location	

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	LICENSING AGREEMENT		
FOREWO	60		
The activities intended	ties of the Dampary LPG SYSTEMS (hereafter inferred to as 'LPG') include the design, production and marketing on a worldwide basis of devices for the treatment of conjunctive tissue and functional rehabilitation, chefy for professionals from the medical, andheric and sports markets. The company explays established based avaecees as both a national and international level.		
PFi provides the Licensee with Software which can be used on the louchscreen Tablets supplied by LPG, enabling it to offer medical and sports care protocels adupted to the outcomenipatients and including other unctions and features as described hereafter.			
The Software is intended for protessionals having acquired one or several professional devices from UPG, either directly or vis one of its distributors.			
ARTICLE	1 - CONCLUSION OF THE LICENCE		
This Lice number	rore constitutes a legal agreement between the Licensee and LPG SYSTEMS, a simplified joint stock company constituted under French law, registered in the Trude & Company Register (RCS) of Romans under the 3335 103 035, being located on the Technopart de la Plaine, 30 rue du docteur ABEL, Valence (20002) Cedex (9, FRANCE.		
Any use	of the Seffware presupposes that the Licence has been read and accepted.		
The Lice Licence	nsee acknowledges that he has been fully informed of the fact that his agreement regarding the content of the Licence does not require his handwritten signature but results simply from the confirmation of this		
The Lice	race has the option to backup or print this Licence using the functions provided in the Tablet's browser. The Licensee bears sole liability for this backup or printing.		
By clická	g the "confirm" button, the Licensee:		
lac	cept the provisions of this Licence		
	ee to the collection and transmission of technical and statistical data concerning the functioning of my device in order to optimize its use and the related ces.		
-			

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CELLU M6 ALLIANCE[®] - TABLETKOPPLUNG

LPG INNER POWER. BETTER LIFE.	10:55	□ \$
=	SETTINGS	
Co Date and time		>
Language		>
Update		>
Sound		>
Display		>
🔶 WiFi		>
(;;) Head hey config		>
Tablet		
Svetan information		>
Preferences		>

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